Instruction for use

IMM1050_IFU.Version10.2023-02-14

Absorbing Underpad

imedia
by Etac

Index

User manual – English3

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HMI-er 47685 fra hmi-basen.dk

HMI 47685

HMI-er 47689 fra hmi-basen.dk

HMI 47689

HMI-er 47690 fra hmi-basen.dk

HMI 47690

HMI-er 47692 fra hmi-basen.dk

HMI 47692

General

Thank you for choosing an Etac product.

To avoid accidents and injury when moving and handling the products, please read this manual carefully.

The person described as “the user” in this “Instruction for Use” is the person who is lying or sitting on the product. The carers are the people who manoeuvre the product.

This symbol appears alongside the text in the manual. It draws the reader's attention to points at which there may be a risk to the health and safety of the user or carer.

These products comply with the standards applicable for Class 1 products in the Medical Device Regulation (EU) 2017/745 on medical devices.

At Etac we strive to continually improve our products and therefore we reserve the right to make changes to products without prior warning. All measurements given on illustrations and similar material are for guidance only and Etac cannot be held liable for errors and defects.

The information given in this manual, including recommendations, combinations and sizing, does not apply to special orders and modifications. If the customer makes adjustments, repairs or uses combinations not predetermined by Etac, the Etac CE certification and Etac warranty will not be valid. If in doubt, please contact Etac.

Warranty: Two-year warranty on material and manufacturing defects, provided that the product is used correctly.

Expected lifetime: The device has an expected service life of 1 to 2 years under normal use. The service life of the device varies depending on usage frequency, loads and how often and how it is washed.

For further information on the Etac Patient handling, see www.etac.com.

In case of an adverse event occurred in relation to the device, incidents should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

Symbols in general

Washing

Manual Cleaning

Warning

Manufacturer

Do not bleach

Never leave on the floor

Production year, month and day
YYYY-MM-DD

CE-marking

Tumbledry

Risk of sliding down

Batch no./Lot no.

Keep dry

Iron

Keep out of the sun

User mass limit = maximum rated load
max 150 kg max 330 lbs

Non sterile

Do not dryclean

Read the user manual

The product can be scrapped according to national regulations

Article number

Medical device

Do not add fabric conditioner

UKCA marked

Intended use

Absorbing Underpads of various sizes can be used to protect the bed, chair, car seat, etc. since they can lead urine, blood and perspiration away from the user and into the underpad.

Absorbing Underpads are also suitable for changing nappies and bed washing. Suitable as draw sheet during transfer/positioning.

Intended environment

Long-term care, Home care

Practical handling

Positioning

Absorbing Underpad is placed under the user.

The white side on the cotton/polyester models faces upward while on the microfiber models the bright green side faces

upward, ensuring the absorbing side of the underpad is turned upward and the waterproof (blue) barrier downward.

Use

Absorbing Underpads can be used as a draw sheet for user transfers. It is recommended not to pull the flaps, only use them for fixing under the bed mattress.

Notice

Special features

Always check the product before use and after washing.

Never use a defective product. If the product shows signs of wear-and-tear, it must be scrapped.

Read these instructions carefully.

It is important that carers receive instruction in manual handling. Etac offers advice and training for carers. For further information, contact Etac.

Always use the correct manual handling techniques.

Encourage the user to assist where possible.

To ensure that the user feels safe and that every manual handling is smooth, always plan the manual handling in advance.

Never leave the product on the floor.

Always conduct a risk assessment, and ensure that the assistive product can be used for the individual user and, in combination with other devices, that it is safe for the user and carers.

Do not use the product as a sling.

If in any doubt - please contact Etac.

Material and cleaning

Material:

Absorbing Underpads:

Top side: 80% cotton/20% polyester (white) or 100% polyester microfibre (green)

Middle layer: 80% polyester, 20% viscose

Barrier: Polyurethane

Down side: Polyester, blue.

The product should be checked regularly, preferably each time it is used, and especially after washing.

Check that there is no damage to seams or fabric.

Never try to repair a product yourself.

Never use a defective product.

Washing instructions:

Do not add fabric conditioner.

This product can be cleaned with a solvent-free cleaning agent with a pH level between 5 and 9, or a 70% disinfection solution.

Products and accessories

Article no.	Product Description	Size (mm)
IM8590/M	Absorbing Underpad, green/blue	W850xL900
IM8590/BF	Absorbing Underpad, with wings, white/blue	W850xL900
IM110160/B	Absorbing Underpad, white/blue	W1100xL1600
IM110160/M	Absorbing Underpad, green/blue	W1100xL1600

Service information

Inspection

First inspection:

- Is the packaging intact?
- Read the label on the packaging and check the article no. and product description
- Check that Short Instruction is enclosed – updated User Manual/Instruction for use can be downloaded from www.etac.com or contact your Etac Customer Service or the local dealer.
- Check the label on the product - does it include article no., product description, lot/batch no., cleaning instructions and supplier name?

Periodic Inspection:

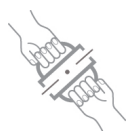
- Make sure that your Manual transfer system always is in a perfect condition.
- After wash – always check materials, stitching, handles, buckles.
- If the product shows signs of wear and tear, it must be removed from service immediately.

Visual inspection/Check the product:
Ensure that the materials, seams, stitching, handles, buckles are intact/faultless.

Stability test:
Try to bend the product verify that it feels firm, stable and solid.

Mechanical load or stress/Test handles:
Draw hard in the handles in opposite directions and control the material, seams and stitching.
Test buckles:
Lock the buckle and draw in opposite directions, control the buckle and the stitching.

Test low/high friction:
Place the product on a firm surface or a bed, place your hands on the material and test the friction by pushing your hands down into the product.
Low friction – slides effortlessly
High friction – no sliding/moving

Check:

Manual transfer aids with handles, straps and buckles:

Manual transfer aids with for sitting and lying transfer and with high or low friction:

Production year, month and day

The product can be scrapped in accordance with national regulations

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Etac Ltd, Unit 60
Etac Immedia A/S