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**Table of contents**

<b>Package content</b>	<b>13</b>
<b>Functions</b>	<b>14</b>
<b>Setup</b>	<b>15</b>
Wireless setup for mobile telephone	15
Installing neck loop	15
<b>Operation</b>	<b>16</b>
Indicators	16
Charging	16
Adjusting volume	16
Accepting an incoming call	17
Making a mobile telephone call	17
Rejecting an incoming call	17
<b>Warranty</b>	<b>18</b>
<b>Test and approval</b>	<b>19</b>
<b>Warnings</b>	<b>20</b>
<b>Declaration of conformity</b>	<b>103</b>

**Package content**

Heartt Mobile


Power supply

Congratulations and thank you for purchasing Heartt Mobile.

Heartt Mobile is a hands-free, neck-loop solution for amplifying sound from your Bluetooth mobile phone. Heartt Mobile has been specially designed for people with impaired hearing who want to use a mobile phone hands-free – and without any compromise in sound quality.

- R1:** On/off switch, answering button, and Bluetooth indicator
- R2:** Volume button +/- (up/down)
- R3:** Bluetooth and volume level 1 indicator
- R4:** Volume level 2 indicator
- R5:** Battery and volume level 3 indicator
- R6:** Volume level 4 indicator
- R7:** Volume level 5 indicator
- R8:** Headphones jack
- R9:** Neck loop jack
- R10:** Neck loop
- R11:** Charging jack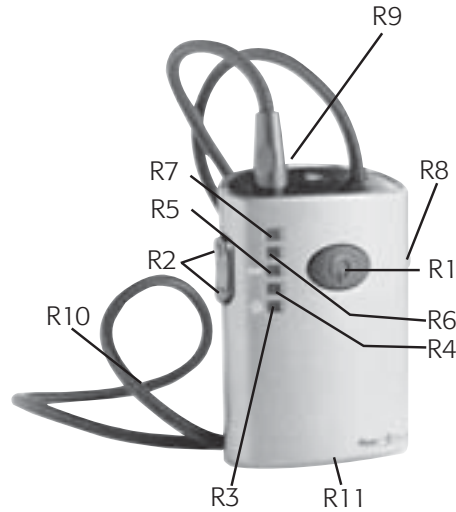


Button/diode/jack	Functions
R1	<ul style="list-style-type: none"> <li>• Power on (hold for up to 5 seconds)</li> <li>• Power off (hold for up to 5 seconds)</li> <li>• Answering and ending mobile telephone calls</li> <li>• Blue light: Bluetooth active</li> <li>• Blue blink: Incoming call</li> </ul>
R2+ (up)	<ul style="list-style-type: none"> <li>• Volume up</li> </ul>
R2- (down)	<ul style="list-style-type: none"> <li>• Volume down</li> <li>• Reject mobile telephone call (hold for 2 seconds)</li> </ul>
R3	<ul style="list-style-type: none"> <li>• Blue blink: Bluetooth not connected</li> <li>• Green light: Volume level 1</li> </ul>
R4	<ul style="list-style-type: none"> <li>• Green light: Volume level 2</li> </ul>
R5	<p><b>When in use</b></p> <ul style="list-style-type: none"> <li>• Red blink: Low battery</li> <li>• Green light: Volume level 3</li> </ul> <p><b>When charging</b></p> <ul style="list-style-type: none"> <li>• Red light: Charging</li> <li>• Green light: Fully charged</li> </ul>
R6	<ul style="list-style-type: none"> <li>• Green light: Volume level 4</li> </ul>
R7	<ul style="list-style-type: none"> <li>• Green light: Volume level 5</li> </ul>
R8	<ul style="list-style-type: none"> <li>• Jack for headphones</li> </ul>
R9	<ul style="list-style-type: none"> <li>• Jack for neck loop</li> </ul>
R10	<ul style="list-style-type: none"> <li>• Neck loop</li> </ul>
R11	<ul style="list-style-type: none"> <li>• Jack for charger</li> </ul>

## Wireless setup for mobile telephone

### Establishing a wireless connection between the Bluetooth mobile telephone and the Hearit Mobile

1. Turn Hearit Mobile on by holding down the receiver button **R1** for up to five seconds
1. Enter the mobile telephone menu to allow the mobile telephone to search for available Bluetooth devices.
2. Select the Bluetooth entry "**Hearit Mobile**".
3. Enter the pass code into the mobile telephone (**0000**). Select connect. A short beep will be heard in the hearing aid or in the headphones.
4. Once the wireless connection is established, receiver indicator **R3** stops blinking blue.

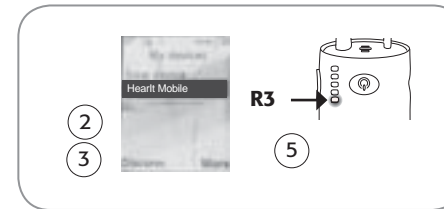**Please note:**

- Once a wireless connection between Hearit Mobile and a Bluetooth unit has been established the units will automatically connect when they are within range of each other.
- During telephone conversations, the receiver indicator **R1** will illuminate blue indicating that the wireless connection is active.

## Installing neck loop

### Neck loop and T/MT function on HA

In order to use Hearit Mobile with hearing aids in T-position (telecoil), the hearing aid must be equipped with a telecoil fitted by the hearing care professional.

### Installing neck loop

1. Detach neck loop **R10** from receiver jack **R9** by pulling the loop wire.
2. Place the Hearit Mobile on the chest and place loop wire around the neck.
3. Reattach neck loop **R10** to receiver jack **R9**.
4. Switch hearing aid to T (telecoil only) or MT (microphone and telecoil simultaneously) position.

**Please note:**

- Connecting headphones to the headphones jack **R8** disconnects the signal to the neck loop.

## Indicators


Below you can see an overview of the light indicators on HearIt Mobile

Flashing blue (R1):	Incoming call
Blue (R1):	Wireless Bluetooth call activated
Flashing blue (R3):	No Bluetooth connection
Green (R3-R7):	Volume level
Flashing red (R5):	Battery low

## Charging

If the HearIt Mobile light indicator **R5** flashes red during operation, or if the unit cannot be turned on, HearIt Mobile requires recharging.

1. Connect the power supply wire to HearIt Mobile's jack **R11**.
2. Connect the power supply to a power source and turn on.
3. The light indicator **R5** lights continuously red during recharging.
4. When HearIt Mobile is fully recharged, light indicator **R5** lights continuously green.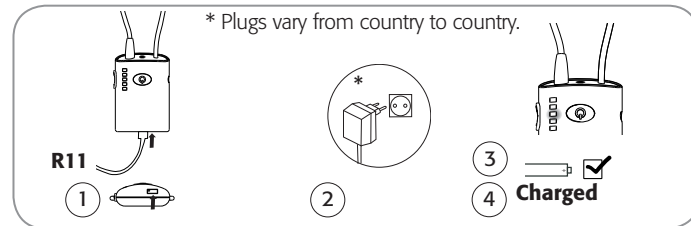


### Please note:

- We recommend full charging batteries before use, however, partly charging the units does not inflict damage to them.

## Adjusting volume

1. Press receiver button **R2+** (volume up) or **R2-** (volume down) to set comfortable listening level.
2. The receiver indicators **R3** to **R7** illuminate depending on volume level.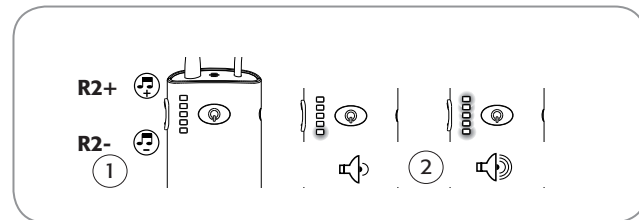


### Please note:

- The maximum volume level has been reached if all five indicators **R3** to **R7** are illuminated. If the signal is still too weak adjust the volume on your mobile telephone or contact your hearing care professional for adjustment of the telecoil in the hearing aid.

## Accepting an incoming call

1. When a call comes in a ringing tone will be heard in the hearing aid or headphones and the receiver indicator **R1** starts blinking blue.
2. Press the button **R1** to accept the call.
3. Press the button **R1** to end the call.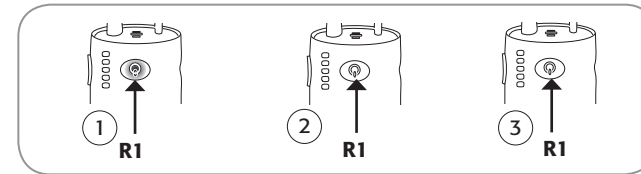

### Please note:


- Only make a single short push to accept or end a call. It may take a couple of seconds before a call is completely connected or disconnected.
- The ringing tone of the mobile telephone call may be the same sound as heard from the mobile telephone itself depending on the mobile telephone used.
- If HearIt Mobile is turned off when a mobile telephone call comes in, that specific call cannot be answered or ended using HearIt Mobile.

## Making a mobile telephone call

1. Turn on HearIt Mobile by pressing the button **R1** for up to five seconds.
2. Enter the dial number on the mobile telephone and pick up the handset.
3. The call is automatically transferred to HearIt Mobile.
4. To end the call press the button **R1** or end the call on the mobile telephone.

## Rejecting an incoming call

1. When a call comes in a ringing tone will be heard in the hearing aid or headphones and the receiver indicator **R1** starts blinking blue.
2. Press receiver button **R2-** (volume down) for 2 seconds to reject the call.

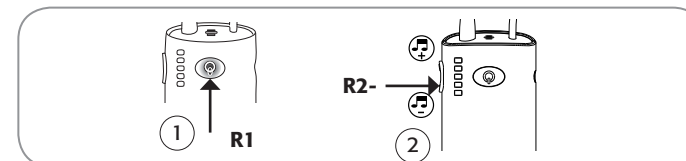A Two-year limited warranty is issued to the original consumer of this product by Phonic Ear A/S.

### Time period of warranty

This warranty will come into effect upon the date of the original purchase of this equipment, and will remain in effect for two calendar years from that date as long as the instrument remains the property of the original consumer.

### What is covered by this warranty

Any electronic component which, because of workmanship, manufacturing, or design defects, fails to function properly under normal use during the lifetime of this warranty will be replaced or repaired at no charge for parts or labour when returned to the factory service centre. Transportation cost are to be met by the customer. If it is determined that repair is not feasible, the entire unit may be replaced with an equivalent unit upon mutual agreement of the manufacturer and customer.

### What is not covered by this warranty

- 1 Malfunctions resulting from abuse, neglect, or accident.
- 2 Peripheral accessories as itemized within the product brochure, when such items are returned after 90 days from the original purchase.
- 3 Batteries.
- 4 Instruments connected, installed, used, or adjusted in a manner contrary to instructions provided by the manufacturer.
- 5 Consequential damages and damages resulting from delay or loss of this instrument. The exclusive remedy under this warranty is strictly limited to repair or replacement as herein provided.
- 6 Products damaged in transit unless investigated by the shipper and returned to the warrantor with the investigation report.

Phonic Ear A/S reserves the right to make changes to the design or construction of any of its instruments at any time without incurring any obligation to make any changes whatsoever on units previously purchased. This warranty is in lieu of all other expressed warranties. All expressed and implied warranties will terminate upon the expiration of this written warranty. No representative or person is authorized to represent or assume for us any liability in connection with the sale or use of our products other than as set forth above. The above warranty does not affect any legal rights that you might have under applicable national legislation governing sale of consumer goods. Your distributor may have issued a warranty that goes beyond the clauses of this limited warranty. Please consult your distributor for further information.

### What to do if you need service

If you require service under the terms of this warranty, carefully package the instrument to prevent damage and return it to your distributor (see information on back cover). Please provide your receipt, a detailed description of the problem, your full name, billing address, and telephone number.

Phonic Ear A/S, Kongebakken 9, 2765 Smørum, Denmark

Electrical and electronic equipment (EEE) contains materials, components and substances, that could be dangerous or detrimental to human health and the environment if waste electrical and electronic equipment (WEEE) is not disposed of correctly.

Products marked with the crossed-out dustbin shown above are electrical and electronic equipment. The crossed-out dustbin indicates that waste electrical and electronic equipment may not be disposed of with unsorted household waste and that it must be collected separately.

In developing and manufacturing your Phonic Ear product, we used high-quality materials and components that can be reused. Thus, when disposing of your waste electrical and electronic equipment, you must use a collection system designated for this purpose, regardless of whether you dispose of your waste at your local civic amenity site or whether your waste is collected from your household. Please contact your local authority for further information.

How to remove batteries

1. Unscrew two screws in the top corners on the rear of unit
2. Open top cover by lifting cover up in the top
3. Remove top cover completely
4. Cut battery away from the printed circuit board
5. Cut wires one at a time

The socket outlet must be installed near to the equipment and must be easily accessible. Only use power supplies listed in the user guide.

The following switch mode power supply can be used with the HearIt Mobile. Manufacture: Fuhua, MODEL No.: UE15WCP-060100SPA and PART No.: UE070904HKCN1-R.

You should familiarize yourself fully with the following general warnings and the entire contents of this booklet before using your Heartl Mobile.

Misuse can result in sudden and permanent hearing loss.

Heartl Mobile is not a toy and therefore should be kept out of the reach of children and anyone else e.g. people with learning difficulties who might swallow parts or otherwise cause injury to themselves. Special attention should be paid to the smaller components in order to prevent children from swallowing them and choking. If a part is swallowed, see a doctor immediately.

Always make sure to set the volume controls correctly according to your hearing. By using Heartl Mobile with a sound pressure level that is too high, you may risk impairing your hearing.

Never allow others to wear or use your Heartl Mobile as incorrect/wrongful usage could cause permanent damage to their hearing.

### Dysfunction in Heartl Mobile

Heartl Mobile may stop functioning, for instance if the batteries are discharged. You should be aware of this possibility, in particular when you are in traffic or otherwise dependent on warning sounds.

### Interference

Your Heartl Mobile has been thoroughly tested for interference, according to the most stringent international standards. But new technical developments constantly bring new products into society, and some may emit electromagnetic radiation, leading to unforeseen interference in Heartl Mobile. Examples include induction cooking appliances, shop alarm systems, mobile telephones, fax machines, personal computer devices, X-rays, computer tomography etc.

Your hearing instruments are designed to comply with the most stringent Standards of International Electromagnetic Compatibility. However Heartl Mobile might cause interference with other medical devices. Such interference can also be caused by radio signals, power line disturbances, airport metal detectors, electromagnetic fields from other medical devices and electrostatic discharges.

### Possible side effects

The otherwise non-allergenic materials used in hearing instruments may in rare cases cause a skin irritation. Consult a doctor if you experience any side effects.

### Use on Aircrafts

The Heartl Mobile should not be used on board on aircrafts unless specifically permitted by the flight personnel.

### Implantable devices

Heartl Mobile is designed to comply with the most stringent Standards of International Electromagnetic Compatibility. However, Heartl Mobile might cause interference with other medical devices such as defibrillators and pacemakers. Please, contact the manufacturer of your implantable device for information about the risk of disturbance. Interference can also be caused by power line disturbances, airport metal detectors, electromagnetic fields from other medical devices, radio signals and electrostatic discharges.

Do never exposure Heartl Mobile to great heat by putting it in the oven, in the micro wave oven or by burning it. There is a risk that it will explode and cause serious injury.

The safety of the use of Heartl Mobile with auxillary input cable is determined by external signal source. When the input cable is connected to mains-operated equipment, this equipment must comply with IEC-60065, IEC-60601, IEC-60950 or equivalent safety standards.

The emission power from the Heartl Mobile is below international emission limits for Human Exposure. For comparison, the radiation of the Heartl Mobile is equal to that of a Bluetooth headset for a mobile telephone. Heartl Mobile complies with international standards concerning electromagnetic Compatibility.

### Cleaning and maintenance

Any dirt, moisture or grease should be cleaned of by using a dry or damp cloth. Heartl Mobile must never be washed or immersed in water or other liquids. Make sure not to drop Heartl Mobile onto hard surfaces.

### Avoid heat, humidity and chemicals

Your Heartl Mobile must never be exposed to extreme heat e.g. left inside a parked car in the sun. It must never be exposed to a lot of moisture e.g. steam baths, showers or heavy rain. Nor must it be dried in micro wave ovens or other ovens. Heartl Mobile must not be exposed to or cleaned with cleaning chemicals or alcohols.

### Use in cars

Using the Heartl Mobile with a mobile phone while driving can distract you. If driving conditions demand it, pull of the road and park before making and answering calls.

### Use of the neck loop

If by accident the neck strap should get caught in something while wearing it, the neck loop is constructed to open automatically. Therefore, never try to modify the neck loop in an unauthorized way. The neck loop should not be used for children who weighs less than 11 kg.

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
**FCC RF Radiation Exposure Statement:**

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. End users must follow the specific operation instructions for satisfying RF exposure compliance. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.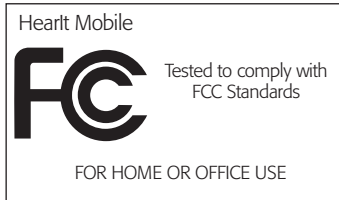

This device complies with Part 15 of the FCC rules and RS-210 of Industry Canada.

Operation is subject to the following two conditions:

- This device may not cause harmful interference
- This device must accept any interference received, including interference that may cause undesired operation

Changes or modifications not expressly approved by Phonic Ear could void the user's authority to operate the equipment.

**NOTE:**

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is

encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

## Declaration of conformity

We

**Phonic Ear A/S  
Kongebakken 9  
DK-2765 Smørum**

declare under our own responsibility that the product

**Heart Mobile**

to which this declaration refers conforms with the relevant standards or other standardising documents

**EN 300 328 v. 1.6.1:2004**  
Electromagnetic compatibility and Radio spectrum Matters (ERM); Wideband transmission systems; Data transmission equipment operating in the 2.4 GHz ISM band and using wide band modulation techniques; Harmonized EN covering essential requirements under article 3.2 of the R&TTE Directive

**EN 301 489-1 v. 1.5.1:2004/3 v. 1.4.1:2003/-17 v. 1.2.1:2004**  
Electromagnetic compatibility and Radio spectrum Matters (ERM); Electro Magnetic Compatibility (EMC) standard for radio equipment and services; Part 1: Common technical requirements; Part 3: Specific conditions for Short-Range Devices (SRD) operating in frequencies between 9 MHz and 40 GHz; Part 17: Specific conditions for 2.4 GHz wideband transmission systems and 5 GHz high performance WLAN equipment

**IEC 60950-1:2006**  
Information technology equipment - Safety - Part 1: General requirements

According to the regulations in

**Radio Equipment and Telecommunications Terminal Equipment (R&TTE) Directive  
1999/5/EC**

Smørum the 22<sup>nd</sup> of September 2008

  
 Karen Wibling Solgård, General Manager

**Phonic Ear A/S**  
 Kongebakken 9, 2765 Smørum, Denmark  
 tel: +45 3977 7101  
 fax: +45 3927 7900

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