Index

Illustrations	3
User manual - English	4
Bruksanvisning - Svenska.....	9
Bruksanvisning - Norsk.....	14
Brugervejledning - Dansk.....	19
Käyttöohjeet - Suomi.....	24
Gebrauchsanweisung - Deutsch	29
Gebruiksaanwijzing - Nederlands.....	34
Instruction d'utilisation - Français.....	39
Istruzioni per l'uso - Italiano.....	44
Instrucciones de uso - Español	49


See www.etac.com


Illustrations

1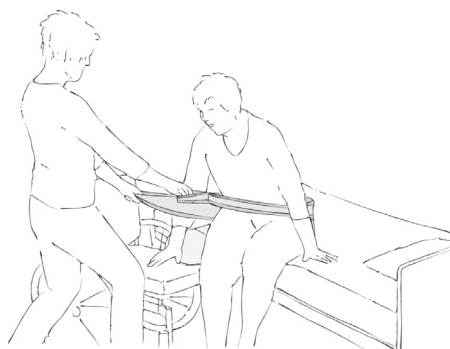

2

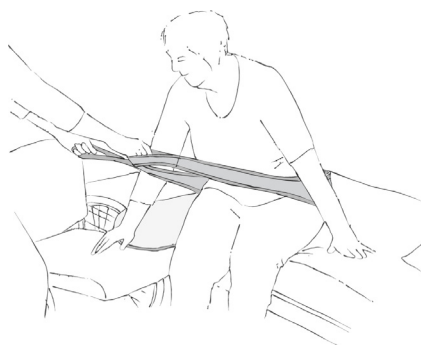3

ILL.

GB

SV

NO

DA

FI

DE

NL

FR

IT

ES

General

Thank you for choosing an Etac product.

To avoid accidents and injury when moving and handling the products, please read this manual carefully.

The person described as “the user” in this “Instruction for Use” is the person who is lying or sitting on the product. The carers are the people who manoeuvre the product.

This symbol appears alongside the text in the manual. It draws the reader's attention to points at which there may be a risk to the health and safety of the user or carer.

These products comply with the standards applicable for Class 1 products in the Medical Device Regulation (EU) 2017/745 on medical devices.

At Etac we strive to continually improve our products and therefore we reserve the right to make changes to products without prior warning. All measurements given on illustrations and similar material are for guidance only and Etac cannot be held liable for errors and defects.

The information given in this manual, including recommendations, combinations and sizing, does not apply to special orders and modifications. If the customer makes adjustments, repairs or uses combinations not predetermined by Etac, the Etac CE certification and Etac warranty will not be valid. If in doubt, please contact Etac.

Warranty: Two-year warranty on material and manufacturing defects, provided that the product is used correctly.

Expected lifetime: The device has an expected service life of 1 to 2 years under normal use. The service life of the device varies depending on usage frequency, loads and how often and how it is washed.

For further information on the Etac Patient handling, see www.etac.com.

In case of an adverse event occurred in relation to the device, incidents should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

Symbols in general

Washing	Manual Cleaning	Warning	Manufacturer
Do not bleach	Never leave on the floor	Production year, month and day YYYY-MM-DD	CE-marking
Tumbledry	Risk of sliding down	Batch no./Lot no.	Keep dry
Iron	Keep out of the sun	User mass limit = maximum rated load	Non sterile
Do not dryclean	Read the user manual	The product can be scrapped according to national regulations	Article number
Medical device	Do not add fabric conditioner	UKCA marked	

Intended use

Sling is ideal for innumerable forms of manual handling, e.g. moving the user higher up in the bed, back in a wheelchair, between two surfaces (seated), moving legs into bed, etc.

Intended environment

Acute care, Long-term care, Home care

Practical handling

Positioning

Sling can best be described as an extra arm or arms for the carer. Sling can be positioned where it has the greatest effect.

Use

Moving the user higher up in the bed:

Position Sling under the user's pelvis so that the non-slip surface is facing the user and the nylon back is against the underlying surface. The carers stand at either side of the bed, one foot forward and pull the user slowly higher up using weight transference (Illustration 1).

To reduce friction on the shoulder blades, we recommend using MultiGlide or MultiGlide Glove.

To glide back in the chair:

Position Sling under the user's thighs so that the non-slip surface is facing the user and the nylon back faces the seat. If the wheelchair is low – the carer stands behind the chair – grasp Sling's handles with arms outstretched and prompt a "thigh-walk" movement by gently rolling and pulling the user backwards.

If the wheelchair is high – two carers stand at either side of the wheelchair – Sling is then crossed behind the user's back – this again prompts a "thigh-walk" movement – the carers take turns to roll and pull the user backwards.

In cases where the user has poor balance when leaning forward, we recommend an extra carer.

Manual handling between two surfaces (seated):

Sling can also be used for manual handling in a low seated position. Sling is positioned under the user's pelvis. The carer grasps Sling close to the body and the user is raised to the standing position using weight transference.

Alternatively, the carer can pull the user up from e.g. a bed to a wheelchair, using a glide board (if necessary) (Illustrations 2 and 3).

Legs in/out of bed:

Position Sling around the user's ankles. The carer can now lift the user's legs up into the bed. Alternatively, the functional handles can be hung onto the hoist. When the lift is raised, the legs are lifted with it and can then easily be pushed into position on the bed.

Notice

Special features

Always check the product before use and after washing.

Never use a defective product. If the product shows signs of wear-and-tear, it must be scrapped.

Read these instructions carefully.

It is important that carers receive instruction in manual handling. Etac offers advice and training for carers. For further information, contact Etac.

Always use the correct manual handling techniques.

Encourage the user to assist where possible.

To ensure that the user feels safe and that every manual handling is smooth, always plan the manual handling in advance.

Never leave the product on the floor.

Always conduct a risk assessment, and ensure that the assistive product can be used for the individual user and, in combination with other devices, that it is safe for the user and carers.

Do not use the product as a full body sling.

If in any doubt - please contact Etac.

Material and cleaning

Material:

Nylon, non-slip, polyester, PE-Foam.

The product should be checked regularly, preferably each time it is used.

Check that seams and fabric are not damaged.

Washing instructions:

Do not use fabric softener.

Recommended to wash the product before use.

This product can be cleaned with a solvent-free cleaning agent with a pH level between 5 and 9, or a 70% disinfection solution.

**Never try to repair a product yourself.
Never use a defective product.**

Products and accessories

Article no.	Product Description	Size (mm)
IM428	Sling - long	W160xL1900
IM428K	Sling - short	W160xL1200

ILL.

GB

SV

NO

DA

FI

DE

NL

FR

IT

ES

Service information

Inspection

First inspection:

- Is the packaging intact?
- Read the label on the packaging and check the article no. and product description
- Check that Short Instruction is enclosed – updated User Manual/Instruction for use can be downloaded from www.etac.com or contact your Etac Customer Service or the local dealer.
- Check the label on the product - does it include article no., product description, lot/batch no., cleaning instructions and supplier name?

Periodic Inspection:

- Make sure that your Manual transfer system always is in a perfect condition.
- After wash – always check materials, stitching, handles, buckles.
- If the product shows signs of wear and tear, it must be removed from service immediately.


Visual inspection/Check the product:

Ensure that the materials, seams, stitching, handles, buckles are intact/faultless.

Stability test:

Try to bend the product verify that it feels firm, stable and solid.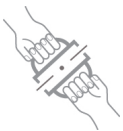

Mechanical load or stress/Test handles:

Draw hard in the handles in opposite directions and control the material, seams and stitching.
Test buckles:

Lock the buckle and draw in opposite directions, control the buckle and the stitching.

Test low/high friction:

Place the product on a firm surface or a bed, place your hands on the material and test the friction by pushing your hands down into the product.
Low friction – slides effortlessly
High friction – no sliding/moving

Check:

Manual transfer aids with handles, straps and buckles:

Manual transfer aids with for sitting and lying transfer and with high or low friction:

Production year, month and day

The product can be scrapped in accordance with national regulations

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Etac Ltd, Unit 60
Etac Immedia A/S