Vision High-Low

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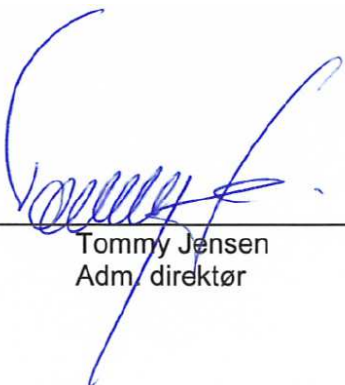
www.ropox.dk

Vision High-Low User Manual

Keep always this folder with product!

Contents:

1.	INTRODUCTION	12
2.	COMPLIANCE WITH EU-DIRECTIVES	12
3.	APPLICATION	12
4.	ASSEMBLY	12
5.	USER GUIDE	13
6.	TECHNICAL DATA	13
7.	PERFORMANCE TEST	15
8.	SAFETY IN USE	15
9.	CLEANING/ MAINTENANCE	15
	9.1 <i>Frame</i>	15
	9.2 <i>Tabletop, laminated surfaces</i>	15
	9.3 <i>Maintenance</i>	16
10.	TROUBLE SHOOTING	16
	10.1 <i>The table seems loose or unstable</i>	16
	10.2 <i>The table cannot be adjusted in height or tilt adjustment seems very difficult</i>	16
	10.3 <i>The table does not move straight</i>	16
11.	CE-MARKING	16
12.	COMPLAINTS	18

1. Introduction

Vision High-Low is the new range of modern height-adjustable Ropox tables. In our opinion a product must be functional and beautiful – no matter who is going to use it. That's what we call Vision.

Vision High-Low meets all needs and ensures ergonomically correct working positions for people with or without disability.

This document should always accompany the product and be read by and be available to the user. Correct use, operation and inspection are decisive factors for efficient and safe performance.

As this product is adjustable in height there is a risk of squeezing. The product must therefore always be operated by or under the guidance of an experienced adult, who has read and understood the importance of section 8 "Safety in use".

It is very important that there are no obstacles between the tilted tabletop and the frame to prevent any change of entrapment.

2. Compliance with EU-Directives

This product has CE-marking according to the current Directives and thus complies with the basic safety requirements. See separate CE-declaration, section 11.

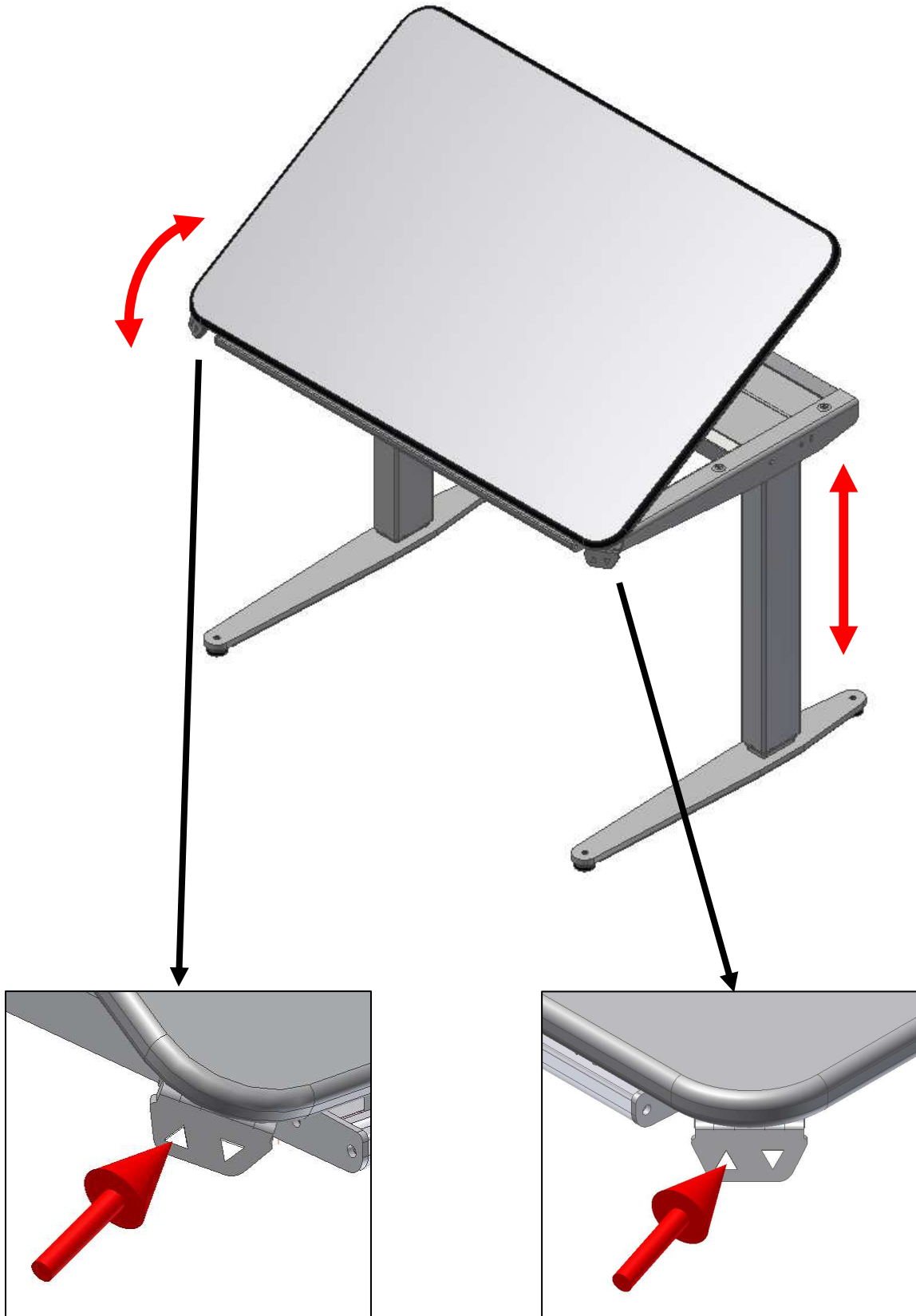
3. Application

Vision High-Low is designed to obtain optimum working height for the user. It is an activity table not intended for lifting people or objects. Vision should be used indoor, at temperatures and humidity as described in section 6.

4. Assembly

The table is delivered fully assembled.

5. User guide

6. Technical data

Product:	Vision High-Low
Item numbers:	Electrical: Vision High-Low, 90x60 cm, A 20-60800 Vision High-Low, 90x60 cm, B 20-60801 Vision High-Low, 120x60 cm, A 20-60810 Vision High-Low, 120x60 cm, B 20-60811 Vision High-Low, 90x90 cm, A 20-60812
Height range:	62–127 cm incl. tabletop
Frame material:	Steel St. 37 Various plastic components
Surface treatment:	Powder coating, various colours Chromating
Tabletop:	Etronit-M laminate
Max. load on the table (static):	100 kg evenly distributed over the entire tabletop
Max. load during lifting (dynamic):	80 kg
Max. load on the tabletop tilted (static):	30 kg
Max. load during tilting the tabletop (dynamic):	4 kg
Power supply:	230VAC / 2.5A / 50 Hz - Alternatively 100VAC / 50/60 Hz
Standby primary:	≤0,1W
Control voltage:	24VDC
Duty circle:	Max. 10 % conform to 2 min. active / 18 min. pause
Speed:	Approx. 38 ^{mm} / _{sec}
Temperature:	5-45°C
Air humidity:	5-85% (non-condensing)
Complaints:	See complaints, section 12
Manufacturer:	Ropox A/S, DK-4700 Naestved, Tlf.: +45 55 75 05 00 E-mail: info@ropox.dk, www.ropox.com

7. Performance test

After installation and prior to use, all functions of the table must be tested. Subsequently this test must be carried out by competent personnel at least once a year:

1. Check that there is no load on the table.
2. Check that there is nothing preventing the table from moving freely within the height adjustment range.
3. Push the "DOWN" button and move the table to the bottom position. Check that the movement is even and smooth.
4. Now move the table to the top position. Check that the movement is even and smooth.

If Tilt fitting is mounted:

5. Push the "UP / DOWN" button for the tilt and tilt the tabletop to the upright and down position. Check that the movement is even and smooth.

When the above mentioned items have been checked, the table is ready for use. See "Safety in use" section 8.

8. Safety in use

- The table should only be used by people who have read and understood these instructions.
- The table is not intended for lifting people or objects.
- Always use the table in a manner precluding the risk of damage to persons or property. The person, who operates the table, is responsible for avoiding damage or injury.
- Children and people with reduced observational ability must only be allowed to operate the table under supervision.
- When the table is used in publicly accessible locations where children or other people with reduced observational capability may get close to it, the person operating the table must pay utmost attention in order to prevent dangerous situations.
- Make sure that there is free space above and below the table to allow height adjustment.
- Do not operate the table in the case of errors or damage.
- Do not overload the table and make sure that the load distribution is correct.
- In connection with inspections, service or repairs make sure that the table is not loaded.
- Any modification of the table, which may influence its operation or construction, is forbidden.
- Only use Ropox original spare parts as replacement parts. If other spare parts are used, the guarantee may become void.

9. Cleaning/ Maintenance

9.1 Frame

Clean the frame with a damp cloth using lukewarm water and ordinary cleaning agents. Do not use corrosive/ abrasive liquids or grinding cloths, brushes or sponges. Dry the frame after cleaning.

9.2 Tabletop, laminated surfaces

Clean the Etronit-M surface with a soft cloth firmly wrung in hot water with a little liquid detergent. The detergent should be without sulphonic soap as this may result in a greasy surface. Stains that cannot be removed with water can often be removed with a mixture of equal parts chlorine bleaching and water or with acetone. When using these products see that the room is sufficiently ventilated and note that these products should not be used on the frame.

Stains from felt-tip pencils can be removed using methylated spirit and pencil strokes can be removed with a piece of eraser. In case of doubt, we recommend you to experiment on a spot that cannot be seen as certain products may damage the laminate, e.g. sulphuric acid etc.

Some chemicals, ink and strongly colouring fruit juices may cause discoloration if they remain on the table for a longer period of time as the pigmentation sticks to the surface.

Do not use steel wool, scouring powder and other abrasive products or detergents leaving a film on the surface, e.g. yellow soap.

After cleaning, wash the table with a soft cloth firmly wrung in clean water and dry with a cloth.

9.3 Maintenance

Inspections, service and repairs must be carried out by competent personnel.

After power failures always reset the table before use.

The frame is maintenance-free and the moving parts have been lubricated for life. For reasons of safety and liability we recommend inspection of the frame once a year:

- Check that all bolts have been tightened.
- Check that the table moves freely and without problems from bottom to top position.
- **Resetting the frame:** Press the DOWN button and adjust the frame to bottom position. Press the DOWN button once more and keep it depressed for 5 sec. to reset the control unit. Check that the movement is smooth and even. Make sure that the mains cable moves freely.
- Check that all cables have been mounted correctly and are undamaged.

Only use Ropox original spare parts as replacement parts. If other spare parts are used, the guarantee may become void.

10. Trouble shooting

10.1 The table seems loose or unstable

- The screws assembling the frame have not been securely tightened.
- Tighten all screws of the frame. See assembly instructions.

10.2 The table cannot be adjusted in height or tilt adjustment seems very difficult

- Check mains voltage to control unit and that the power has been switched on.
- Check cables and plug-in connections between control unit and motor.
- Check cable and plug-in connection between control unit and hand control switch.

10.3 The table does not move straight

- If the table does not move straight it must be reset. See Maintenance 9.3 "Resetting".

11. CE-marking

The undersigned hereby declares that the following products:

Tabletop 90x60 cm		Tabletop 120x60 cm	Tabletop 90x60 cm
<u>Type</u>	<u>Height 62-127 cm</u>	<u>Height 62-127 cm</u>	<u>Height 62-127 cm</u>
A	20-60800	20-60810	20-60812
B	20-60801	20-60811	

all belong to risk class I and conform to the Directives and Standards mentioned below:


DIRECTIVES

The Machinery Directive 2006/42/EC

STANDARDS

DS/EN 12182: 2012	Assistive products for functionally disabled persons - General requirements and test methods
DS/EN 14971: 2012	Medical devices – Application of risk management to medical devices Risk analyses according to the Machinery Directive 2006/42/EC
DS/EN 60335-1: 2012	Household and similar electrical appliances - Safety – Part 1: General requirements
DS/EN 9999: 2011:	Assistive products for functionally disabled persons - Classification and terminology

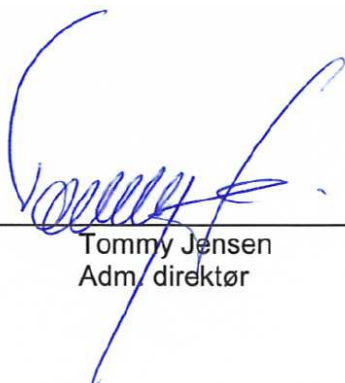Date: 01-10-2014

Tommy Jensen
Adm. direktør

12. Complaints

We refer to our general Terms of sale and delivery on our homepage www.ropox.com

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ROPOX A/S