- Ⓓ **Montageanleitung**
- ⒼⒷ **Installation instructions**
- Ⓕ **Montage**
- ⒼⓁ **Montage van**
- Ⓘ **Istruzioni di montaggio**
- ⒴Ⓛ **Instrukcja montażu**

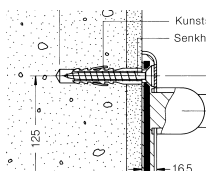
Artikel-Nr.
item number
No de référence
Type Nr.
Nr. Articolo
Numer produktu

3


Warnhinweise
Warning
Mises en garde
Let op
Avvertenze importanti
Ostrzeżenia

4 - 9


Befestigungsarten
Fixing types
Types de fixations
Bevestigingswijzen
Sistemi di fissaggio
Rodzaje mocowań

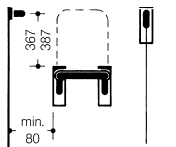10 - 16


Montagehinweis
Mounting information
Conseil de montage
Montage aanwijzing
Indicazioni per il montaggio
Wskazówka dotycząca montażu

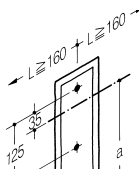18 - 21

Montageanleitungen
Installation instructions
Notice de montage
Montage van
Istruzioni di montaggio
Instrukcje montażu

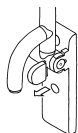22 - 30

Wartung
Maintenance
Entretien
Onderhoud
Manutenzione
Konserwacja

32 - 37

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WARNING!

Use of HEWI wall mounted seat with lift-up feature

The HEWI wall mounted seat with lift-up seat feature is designed to withstand a maximum static working load of 150 kg (vertical). Do not use this wall mounted seat for weights in excess of this. Any non-compliance with loading limits can cause permanent damage to the wall mounted seat or the joint. In such cases, the wall mounted seat must be examined by the company appointed to carry out maintenance work prior to the wall mounted seat being used again. If in doubt, contact HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300 for further assistance.

Only use the HEWI wall mounted seat with lift-up feature in the manner described in the instructions. Any improper use e.g. as a climbing aid can cause an accident with persons potentially suffering serious injury.

Assuming normal use, the wall mounted seat is of a flexible nature thanks to the materials used in its manufacture. This flexibility reduces the risk of injury if anyone bumps into the wall mounted seat. The fact that the wall mounted seat bends to a certain extent under load is normal. Should however the support become permanently deformed, you should immediately contact the company appointed to carry out maintenance work or HEWI's after-sales service team (sanitary) on tel. 01634 269599.

Installation of HEWI wall mounted seat with lift-up feature

The smooth and safe functioning of the HEWI wall mounted seat with lift-up feature largely depends on its correct fixing to the given wall, which is why this work should only be entrusted to a licensed contractor (sanitary fitter). Serious impairment to the wall mounted seat's proper functioning can occur if its installation is not carried out professionally, potentially resulting in accidents with persons suffering serious injury. This is why we advise you not to install the wall mounted seat yourself.

When the licensed contractor mounts the HEWI wall mounted seat with lift-up feature, he should ensure that the entire area of the wall to which the wall mounting plate is to be fitted is flat (no projecting joints or tiles sticking out), that the structure of the wall is suitable for the installation of the wall mounted seat and has no weak points. Furthermore, the choice of the right fixing material kit for the job is important; HEWI offers various types to suit the nature of the wall concerned. If in doubt, contact HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300 for further assistance. The instructions accompanying the wall plugs should be observed in full.

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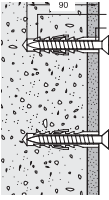
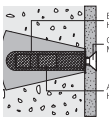
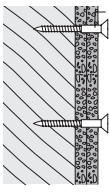
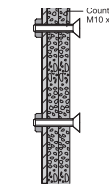
It is absolutely necessary to consider the appropriate assembly instructions and further referrals on page 10.

<p>Serie 801 Suitable for fixing to walls of:</p>	<p>Stationary support rail Hinged support rail Shower spray guard</p>	<p>Hinged seat</p>	<p>Back rest suitable for fixing on Support rail</p>
 <p>Plastic wall plug M10 Countersunk bolt 10 x 95</p> <p>- Beton (>B15)</p>	<p>BM10.2</p> <p>Supplied with: 2 bolts, 2 wall plugs</p>	<p>BM20.4</p> <p>Supplied with: 4 bolts 4 wall plugs</p>	<p>BM10.2.1</p> <p>Supplied with: 2 bolts 2 centering sleeves 2 wall plugs</p>
 <p>Basket sleeve HT-SC 22 x 85 Countersunk bolt M 10 x 30 DIN 7991 Anchor sleeve HT-IG M10</p> <p>- solid bricks of lightweight concrete - aircrete - sand-lime perforated bricks</p> <p>With basket sleeve</p> <ul style="list-style-type: none"> - vertically-perforated bricks - hollow blocks - sand-lime perforated bricks - other perforated bricks <p>Without basket sleeve</p> <ul style="list-style-type: none"> - concrete - solid bricks - solid pumice and other solid materials - natural stone - sand-lime solid bricks 	<p>BM11.2</p> <p>Supplied with: 2 bolts 2 basket sleeve 2 anchor sleeve</p>	<p>BM21.4</p> <p>Supplied with: 4 bolts 4 basket sleeve 4 anchor sleeve</p>	<p>BM11.2.1</p> <p>Supplied with: 2 bolts 2 centering sleeves</p>
 <p>Countersunk wood screw 10 x 80</p> <p>Suitable for fixings to light-weight partition walls with support frame of hardwood min. 20 mm thick.</p>	<p>BM12.2</p> <p>Supplied with: 2 bolts</p>	<p>BM22.4</p> <p>Supplied with: 4 bolts,</p>	<p>BM12.2.1</p> <p>Supplied with: 2 stainless steel locknut 2 centering sleeves</p>
 <p>Countersunk bolt M10 x 60 DIN 7991</p> <p>Suitable for fixings to light-weight partition walls with integrated steel plates.</p>	<p>BM13.2</p> <p>Supplied with: 2 bolts</p>	<p>BM23.4</p> <p>Supplied with: 4 bolts</p>	<p>BM13.2.1</p> <p>Supplied with: 2 bolts 2 centering sleeves</p>

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(GB) Position template and stick to the wall. Take into consideration the necessary freespace $L = 80$ mm, min. between centre line and wall. Installation height between upper floor level and centre to centre of seat level $a = 458$ mm, height of seat 480 mm.

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Fastening method BM20.4

Test structure of wall! Drill holes 4 mm \varnothing and remove template. Drive in plastic plugs and tighten seat. Snap in caps. Adjust level of seat by turning the buffers.

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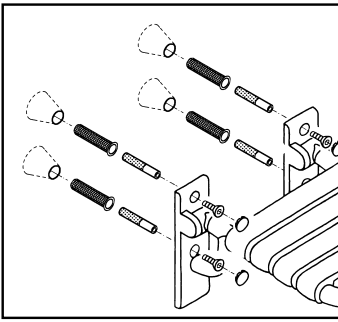
Fastening method BM21.4

Test structure of wall! Drill plug holes using 22 mm ø undercut taper drill, remove template. Install injection anchoring allow cement to set as per instruction (injection mortar not supp-

lied from HEWI). Remove protective sleeves and tighten seat (protect screws against corrosion with grease). Snap on caps. Adjust level of seat by turning the buffers.

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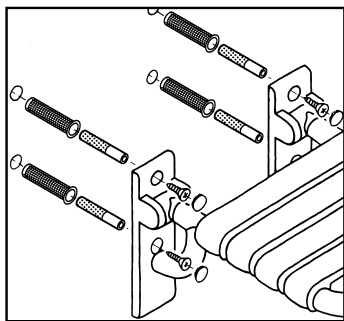
Fastening method BM21.4 Perforated brick

Test structure of wall! Drill plug holes 22 mm ϕ x 95 mm deep. Remove drill template. Insert filter sleeve and inject HIT-HY50 injection mortar from cartridge

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(from local supplier; observation of manufacturer's instructions vital). Insert anchor sleeve.

Important! Protect M10 internal thread. Observe manufacturer's recommended setting time and tighten seat (protect

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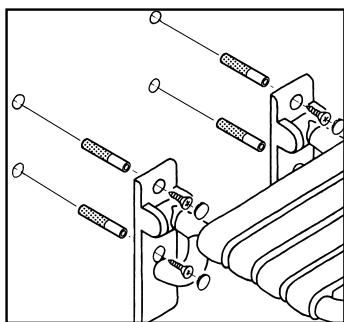
screws against corrosion with grease). Snap on caps. Adjust level of seat by turning the buffers.

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Fixing type BM21.4

Test structure of wall! Solid brick: Drill plug holes 18 mm \varnothing x 90 mm deep. Remove drill template. Ensure plug holes are clean and free of dust. Apply HIT-HY70 injection mortar or from

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cartridge (from local supplier; observation of manufacturer's instructions vital).

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Insert anchor sleeve. **Important!** Protect M 10 internal thread. Observe manufacturer's recommended setting time and tighten support (protect screws against corrosion with grease). Snap on caps.

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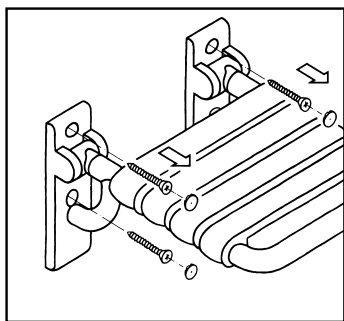
To remove

Pierce caps in the middle and prise off. Unscrew seat.

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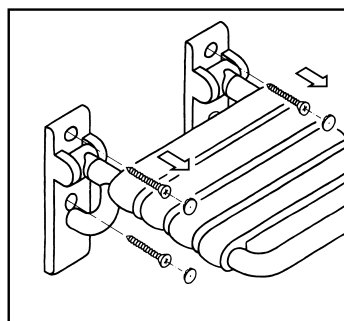
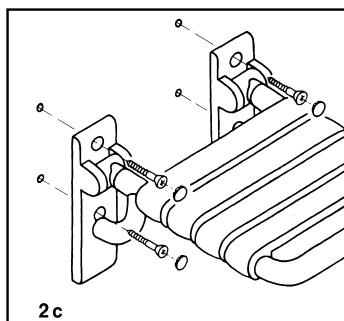
Fastening method BM22.4

Test structure of wall! Drill holes for screws 7 mm ø (if mounted on tiles 11 mm ø). Tighten seat by turning the buffers.

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Fastening method BM23.4

Fit M 10 countersunk screws into the existing holes and tighten to secure folding seat. Adjust level of seat by turning the buffers.

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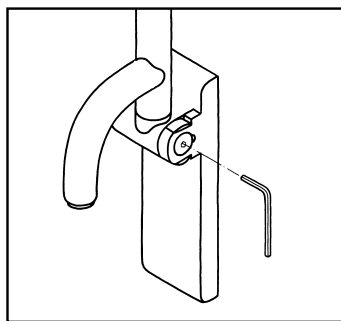
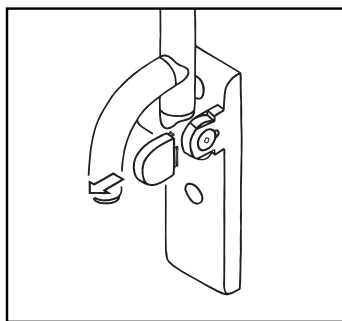
(GB) **Seat with lift-up**
Pull off right-hand cap.

Tighten or slacken the axial screw with
a size 4 Allen key until the desired braking
force is achieved.

Warning: do not turn the axial screw at
the slotted hole side.

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**Maintenance of seat with lift-up feature**

The HEWI seat with lift-up feature is not maintenance free! The correct functioning of the brake device that prevents the seat from dropping accidentally can be impaired by external influences (temperature, moisture). The seat ought to be inspected by the maintenance company at least once a month to ensure its correct functioning. The persons entrusted with the maintenance work must be familiar with the functionality of the seat and the correct setting of the brake device. The appropriate instructions and description are enclosed with every seat. If in doubt, contact HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300 for further assistance. Do not carry out the maintenance work yourself as this could lead to the correct functioning of the seat being severely impaired with the risk of persons potentially suffering severe injuries.

The maintenance work must include examination of the brake device to ensure the seat folds down slowly and smoothly. If necessary, the friction device should be re-adjusted. Observe these instructions. Under no circumstances should the brake discs or joint be greased or lubricated. Moreover, the wall fixing must be inspected to ensure everything is as it should be. Non-compliance with these instructions can cause an accident with persons potentially suffering serious injury.

Damage in transit

The seat left the factory in a perfect condition. When the customer takes possession of it, he/she should examine it for any damage that may have occurred in transit. Any defects have to be reported immediately to ensure that no potentially severe impairment to the seat functionality can subsequently occur, thus eliminating the risk of persons suffering serious injury. The surface of the seat is sensitive and can suffer scratching caused by dust or dirt on or near the installation site. Keep the seat in its packaging whenever possible. Once the seat has been installed, put the protective foil back around it until the support needs to be used.

Exclusion of liability

HEWI accepts no liability whatsoever for damage to persons or property that occurs or has occurred as a result of the improper installation, use or maintenance of the company's seat with lift-up feature, particularly if the safety instructions and other points to note listed above are not or have not been observed in full.

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Care tips

HEWI products are easy-care. Generally speaking, an occasional wipe with a damp cloth is enough. If, however, you feel you have to use detergents, you should take note of the following information.

Only use detergents with a ph-rating will do between 6 and 8.

Detergents containing active substances such as phosphates, soap and tensides may be used.

Detergents containing acids, alkalis, bleach or scouring agents may not be used. Nor should utensils with a scouring effect such as brushes etc. be used.

Usage of any detergent requires full compliance with the manufacturer's instructions (concentration, soaking time etc.).

If this is not done, the surface finishes of the HEWI products may suffer as a result and sustain damage. HEWI accepts no liability whatsoever for damage occurring from incorrect handling.

For further information please contact HEWI Technical Support, tel. +49 5691 82-300.

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HEWI Heinrich Wilke GmbH