Monokular

Bedienungsanleitung
User Manual
Mode d'emploi
Istruzioni d'uso
Manual de instrucciones
Gebruiksaanwijzing
Betjeningsvejledning
Instrukcja obsługi
Provozní návod
Инструкция

 **ESCHENBACH**

Deutsch	3
English.....	7
Français.....	11
Italiano.....	15
Español.....	19
Nederlands	23
Dansk	27
Polski.....	31
Česky	35
Русский	39

Deutsch

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English

Safety Instructions

- ▶ Never look through the monocular at the sun or a very bright light source. This can lead to long-lasting damage to your eyes! Also tell this to children and other people who want to use the device.
- ▶ If you leave the device out in sun, this can lead to the ignition of flammable materials in the area due to the burning glass effect. You should therefore always put the safety caps on when you are not using the device. This also protects the lenses.
- ▶ Do not open the device. There are no parts inside the device that have to be serviced by the user and opening the device renders your warranty invalid.

Correct Use

This device is used to magnify distant objects for human sight. It is not intended for use in commercial or industrial fields. We accept no liability for damage that arises due to incorrect use of the device.

We recommend that you read these instructions through carefully in order to get the best possible use out of your new monocular.

Scope Of Supply

- ▶ monocular
- ▶ 2 x safety caps
- ▶ hand loop
- ▶ belt bag
- ▶ lens-cleaning cloth
- ▶ operating instructions

Using The Monocular

- ❶ focussing drive
- ❷ eyepiece shell
- ❸ eyepiece

1. Remove the device from the packaging.
2. Take off the two safety caps.
3. Look through the monocular with the eyepiece to your eye.
4. Aim at an object that you would like to look at.
5. Look through the monocular with one eye and then the other in order to find out which one you can see through the monocular best with.
6. Turn the focussing drive in the + and — direction until the image is sharp.
7. If you are using the monocular while wearing sun(glasses), leave the eyepiece shell in its initial position. If you are using the device without glasses, please turn the eyepiece shell anti-clockwise so that it moves outward somewhat. This ensures that your eye is always the correct distance from the eyepiece, with or without glasses.
8. Before you put the safety caps back on after use, twist the eyepiece shell back again if necessary.
9. We recommend that you attach the enclosed hand loop to the eyelets on the device and safety caps, which are there for this purpose. Then always place the loop around your wrist when you are holding the device in your hand. This prevents you from accidentally dropping the device or losing the safety caps.
10. If required, fasten the enclosed belt bag to your belt and carry your monocular in it. Make sure however that the hook and loop fastener of the bag is safely closed, so that you do not lose the monocular.

Cleaning and Maintenance

- ▶ Protect your monocular against knocks and impacts and keep it in a dry place.
- ▶ Remove dirt and fingerprints from the lenses with the accompanying lens-cleaning cloth or a soft brush.
- ▶ If the device becomes dirty it can be cleaned with a damp cloth and mild cleaning agents. Never use any kind of liquid or corrosive cleaning agents. Do not dip the device in water!

Warranty

In the framework of the legal regulations, we guarantee the functioning of the product described in these instructions, with respect to deficiencies that are to be attributed to manufacturing defects or materials defects. No warranty claim exists in the case of damage due to incorrect use or due to a fall or impact. Warranty requires proof of purchase with sales slip!

Disposal

- ▶ At the end of its useful life, the device should be disposed of responsibly in accordance to the local regulations.

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