# H2O kørestolspude H2O wheelchair cushion

**Anvendelse**  
Use

40-120 / 120-270 kg

**Brugsanvisning**  
User manual

## DK

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08/2018/JP

## CAUTION

**This product should not be used without proper instruction from a healthcare professional.**

**Using the product for individuals weighing more than the maximum weight-bearing capacity may pose a risk for damaging the product.**

**Cobi Rehab assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.**

**If components are damaged or missing, contact Cobi Rehab immediately. Do NOT use substitute parts.**

**Cobi Rehab cannot be held liable for faults or accidents occurring after repairs by service staff without written authorisation from Cobi Rehab.**

** To ensure proper use of the H2O wheelchair cushion, please read these instructions thoroughly before using the product. In doing so, pay particular attention to the warning notes marked with .**

**Also remember that in order to prevent the risk of injury to the user, the cushion may be operated only by trained caregivers or aides or by individuals who have been instructed in its proper use.**

## Safety notes

### Open fire

 The pillow satisfies current standards regarding fire safety. However, the pillow should never be left close to an open fire, hot ashes or intense heat.

### Warnings

 Never use sharp objects or electric blankets on or under the pillow.

 Only use the original pillow cover, water cell, foam and insert/wedge parts accompanying the pillow and use them as instructed. There is no guarantee the product will work correctly if an incorrect combination of pillow, cover and accessories are used or if other types of pillow cover, other type of covers, sliding pieces, anti-slip material or the like are used.

 Store the pillow indoors at room temperature (approx. 20° C). If the pillow is exposed to extreme cold (under 0° C), we recommend waiting until it has reached room temperature before using the pillow.

 Do not open the valves or change the amount of water in the water cells, as it is carefully measured and should only be changed by specially trained staff.

 Do not use the pillow if it is damaged. If, for some reason, the pillow becomes damaged, contact your therapist or dealer for further information.

## Specifications

Maximum weight capacity is	40-120 / 120-270 kg
Incontinence cover	Yes
Stretchable and breathable cover	Yes
Cleaning	Comfort cover, washable at 65°C. Incontinence cover, washable at 90°C.
ISO - classification	ISO 04 33 03 - 08

## Use

H2O wheelchair cushion has a perfect combination of water and pressure relief foam.

The H2O wheelchair cushion has a documented high pressure relief and is suitable for prevention and treatment of pressure sores up to grade 4. The cushion is useful for users with a weight of 40-120 kg, as well as bariatric users with a weight of 120-170 kg.

 If users are weighing more than 40-120 / 120-270 kg there is a risk that the cushion will break or take damage.

## Description

H2O wheelchair cushion is specially designed to provide maximum pressure relief for high risk users developing pressure ulcers, or to users with existing tissue damage. The elastic cover has a high degree of adaptability and also contributes to high comfort for the user.

H2O wheelchair cushion combines the best of two worlds, namely water and pressure relieving foam. The pressure relieving wheelchair cushion has two water cells that ensure high pressure relief with good pressure distribution and stability. Water technology helps the temperature regulating effect and increases blood flow in the tissue. Thus, the user achieves better circulation, better healing and increased comfort.



The cushion is ergonomically correct for an optimal sitting position and consists of two quality foams combined with 2 incorporated water cells with foam inside that ensure both high comfort and optimal pressure-relief. The height of the water cells is individually adjustable by means of two wedges, allowing an ergonomically correct seating position - even in case of imbalanced pelvic where the pelvis obtains a neutral position. It is possible to adjust the height of the "well". The foam is anatomically shaped, which contributes to a stable positioning. To ensure that the cushion retains its stability and temperates, a thin foam cap is placed on top of the cushion.

The cushion's elastic cover with high adaptability contributes to high comfort.

The cushion is furthermore fitted with a hygienically correct zipper opening, which allows you to easily insert and remove the accompanying wedges to correct the sitting position.

The amount of water is carefully matched to the size of the cushion. The weight of the elements varies depending on the size of the cushion. The weight range for the elements: seat width 40-45 cm is 0.9 - 1.35 kg. From seat width 50 cm and above 1.58 kg. This ensures optimum seating comfort and pressure distribution without compromising stability.

#### Overview of cushion parts – seen from below and above:

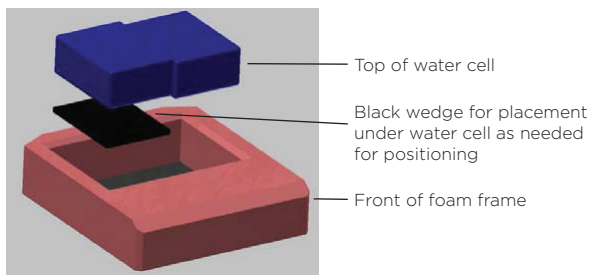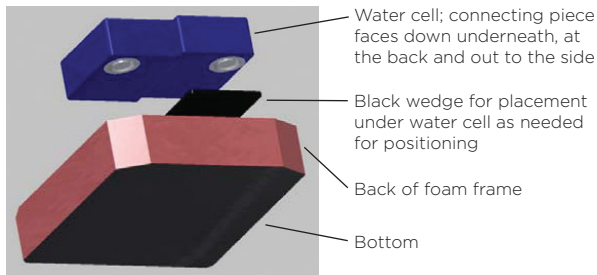Daily maintenance is minimal.

Developed and produced in Denmark based on experience from various positioning experts.

**⚠ NOTE:** A label and washing instructions are sewn within the cushion cover.

## Description of cover

The cushion is available with either a comfort case or an incontinence cover. Both pillow covers have a base consisting of a slip-proof material and both are multi-stretch.

## Before use

### Inspection of cushion and wheelchair

When you sit down for longer period of time, the pressure is highest on your sit bones. Here you run the greatest risk of developing pressure sores. Therefore, the cushion consists of a fitted foam base with a hole in the back in which the water cells can be placed at the spot where the sit bones sink down. This distributes pressure across the entire sitting surface, and the water cells are also designed to offset the pressure as much as possible. Therefore, it is important that the cushion size is correct and that it fits the wheelchair. The cushion must be placed correctly in the chair. The wheelchair's footrests and armrests are adjusted if necessary, so that the user's legs are resting on the front edge of the pillow. This distributes the pressure to the entire cushion, thus reducing the pressure on the sit bones.

Furthermore, always ensure that the cushion is properly placed in the wheelchair. The cushion is placed on the wheelchair's seat with the water cells facing up and back and with the foam base facing forward.

**⚠** Check that the cushion has the proper size and that it fits the wheelchair.

### Cushion cover

Remember to make sure the cover is properly fitted on the cushion according to the instructions found on the cover. Always make sure that the cushion is facing correctly in relation to the indication sticker on the cover. Place the cover on the cushion so the slip-proof material is on the bottom facing the wheelchair and that the zipper opening is facing backwards.

With normal use, we recommend you replace the outer cushion cover once a year. Intensive or incorrect use may significantly shorten the service life of the cushion cover.

**⚠** Do not place any further layers or padding between the cushion and the user as this may reduce the effect of the cushion.

**⚠ Note:** Use the cover together with the cushion. For optimal function, the cushion should always be fitted with an outer cushion cover.

## During use

### Observation of the user's skin during use of a new cushion

When starting with a new cushion, it is important to keep a close eye on the user's skin the first couple of days. If redness develops in the skin, the user should stop using the cushion and contact a therapist or dealer for further instructions and help.

### If the cushion has been dropped

The cushion should be checked regularly for damage to the case and the content of the cushion. It should especially be checked if it has been dropped, as the cushion may have been damaged or the water cells may have been displaced.

**⚠** If the cushion has been dropped, check the position of the water cells. Also check that the individual water cells are not deformed or displaced. If all parts are intact, put them back in place and make sure the surface of the water cells is flat.

### In case of leakage

In case of leakage, where liquid is leaking from the water cell and the water comes in contact with the user's skin, wash the user, change their clothes and watch for any skin irritation. Parts from the cushion must be washed and dried before being used again.

**⚠** Inspect the cushion on a regular basis to ensure that the cushion and the cover are properly oriented and inspect if the cushion is wet or damaged.

**⚠** Do not attempt to use the cushion when the user is sitting on it.

# Maintenance and cleaning

## Of the cushion

⚠ Never wash the foam base in the washing machine.

⚠ Do not autoclave the cushion.

⚠ Do not expose the cushion to extreme cold, heat or pressure.

## Of the cover

The incontinence cover is machine-washable at 90 °C and can be tumble dried on low temperature.

The comfort cover is machine-washable at 65 °C.

We recommend to machine-wash the cover inside out.

Both covers can be tumble dried at low temperature.

⚠ Do not use alcohol or phenolic cleansers.

⚠ If the cover is damaged do not use the cushion before the cover is replaced.

## Of water cells

The water cells should be inspected at least twice a year to ensure that the water volume in the water cells is intact, as defective water cells may reduce the pressure relief.

# Warranty

The H2O wheelchair cushion is covered by one year warranty from the invoice date for materials and workmanship. Products which have been used for purposes other than the intended uses; or which have been subject to negligence, abuse, improper storage or handling, improper operation, unauthorized modifications or damages beyond normal wear and tear as determined by Cobi Rehab, are not covered by this warranty.

If weight capacity is exceeded, the warranty will be void. Any unauthorized repairs to product / part, as well as tampering with any components, will void the warranty.

## MANUFACTURER

Akva Waterbeds A/S  
Boeletvej 25  
8680 Ry  
Danmark

## DEALER

Cobi Rehab  
Fuglebækvej 1D  
DK-2770 Kastrup  
Denmark  
Tel.: +45 70 25 25 22  
[www.cobirehab.com](http://www.cobirehab.com)

11/2018/JP

# COBI REHAB<sup>®</sup>

The Positioning Company

**COBI REHAB**  
Fuglebækvej 1D  
DK-2770 Kastrup  
Denmark  
Tel. +45 7025 2522  
[www.cobirehab.com](http://www.cobirehab.com)