



TR 9650

Mobile Bath Lift System

Hygiene Equipment
Designed for People

Operating manual

Serial Number:



Registered at Medical Products Agency, Sweden, as a MD Class 1 product, MDD 93/42EEC.

TR reserves the right to make design changes and/or improvements without notice.



Always study this manual carefully before operating the TR 9650. Operating instructions and recommended maintenance procedures are explained in the following pages.

If you have any questions regarding the operation or maintenance of your TR 9650 and cannot find the answer in this manual, please contact us

Only qualified staff, who understand all operating and safety procedures may operate this equipment.

The TR 9650 is designed for years of trouble free operation, with proper use and maintenance.

Head office


TR Equipment AB, Sweden
Phone: +46 140 38 50 60
info@trequipment.se
www.trequipment.com

Group Subsidiaries

TR Group Inc, US, Phone: +1 (800) 752 6900
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



TR EQUIPMENT

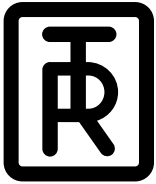

TR 9650

Mobile Bath Lift System

Hygiene Equipment
Designed for People

Table of Content

Introduction	3
Intended Use, Product Description and Technical Specifications	4
Safety and Control Instructions	5
Features and Controls for the Stretcher Lift - Combi Lift	6
Features and Controls for the Chair Lift	7
Features and Controls for the Mobile Lifts	8
Height adjustment	8
Overload protection	8
Autostop mechanism	9
Charging instructions for the Mobile Lifts	10
Stretcher side part(s) attachment/Castor brakes	10
Straight steering castors	11
Chair Backrest	11
Stretcher Backrest	11
Safety belts	12
Cleaning the Lift	13
Patient Handling - Procedures for the Stretcher	12
Transport on Stretcher	13
From Wheelchair on to Stretcher	14
From Bed onto Stretcher	15
On Stretcher into Bathtub	16
From Stretcher onto Bed	17
Patient Handling - Procedures for the Chair	18
Patient Transport on Chair	18
From Wheelchair to Chair	19
From Bed onto Chair	20
On Chair into Bathtub	21
From Chair into Bed	22
Toilet Transfer using Chair	23
Routine Maintenance	24
Labels	25
Maintenance record	26
Training record	26

Head office


TR Equipment AB, Sweden
Phone: +46 140 38 50 60
info@trequipment.se
www.trequipment.com

Group Subsidiaries

TR Group Inc, US, Phone: +1 (800) 752 6900
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



TR EQUIPMENT

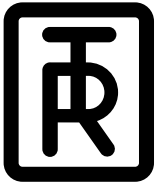

TR 9650

Mobile Bath Lift System

*Hygiene Equipment
Designed for People*

Introduction

Dear Customer,

Thank you for choosing one of our quality products. We hope you enjoy the TR 9650 Mobile bath lift.

This manual shall, in conjunction with staff training, ensure that the operator/user is always using and/or operating the TR 9650 correctly.

TR requires that each user has read and understands the entire operating manual and that they can apply the instructions as specified. This applies to anyone who was not present at the in-service.

This instruction manual should generally be kept easily accessible for every user. Copying the whole manual is encouraged since its sole purpose is to improve the standard of training and safety.

Please order another copy from us if the instruction manual is lost or unusable, or if it has become illegible over time.

If you require further information or have any questions please do not hesitate to contact TR or ask one of our qualified partners.

Please make sure you have read, understood and applied the information on the following pages.

Head office


TR Equipment AB, Sweden
Phone: +46 140 38 50 60
info@treqipment.se
www.treqipment.com

Group Subsidiaries

TR Group Inc, US, Phone: +1 (800) 752 6900
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



TR EQUIPMENT

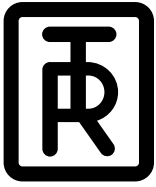

TR 9650

Mobile Bath Lift System

Hygiene Equipment
Designed for People

Intended Use, Product Description and Technical Specifications

Device Model(s):

- Chairlift
- Stretcherlift
- Combilift with chair
- Stretcher side parts for combilift

The device is not intended to be used with patients that exceed a weight of 150 kg.

The device is not intended to come in contact with injured skin.

The device “TR 9650” is a battery operated mobile bath lift.

The user is any adult person who is capable of reading and understanding the operating manual and is qualified to care for dependent persons/patients.

The device is intended to ease transferring and bathing of the patient. Typical situations where the device is used include transporting the patient to and from a bathing area, bathing and showering the patient and dressing the patient.

Before using the device, the user must read and understand the operating manual.

The patient is typically a person who is unable to take care of their own personal hygiene.

The device is intended to be used indoors in all types of care facilities and private homes, including hygiene areas.

Product Description:

- 1 - Art.no: 0542000 - Chair Lift
Battery operated, height adjustable with chair. Including battery charger and safety belts.
- 2 - Art.no: 0542010 - Stretcher Lift.
Battery operated, height adjustable with stretcher. Including battery charger and safety belts.
- 3 - Art.no: 0542020- Combi Lift
Battery operated, height adjustable with chair. Including battery charger and safety belts.
- 4 - Art.no: 0542030 - Stretcher Side Parts for Combi Lift. Including pillow and safety belts

Technical Specifications


Operating height range	200 - 965 mm/8” - 38”
Working Load Limit	150 kg/330 lbs
Structure	Powder coated, zinc plated and stainless steel
Hygiene seat and side parts	PUR (Poly Urethane)
Castors	Twin Ø100 mm castors, two with brakes and two with straight steering
Control	Foot control for height adjustment
Pillow	Flame retardant material
Battery	Rechargeable, 24 V

Head office
TR Equipment AB, Sweden
Phone: +46 140 38 50 60
info@trequipment.se
www.trequipment.com

Group Subsidiaries
TR Group Inc, US, Phone: +1 (800) 752 6900
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



TR EQUIPMENT

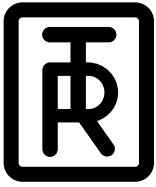

TR 9650

Mobile Bath Lift System

Hygiene Equipment
Designed for People

Safety and Control Instructions

If you have a question regarding the operation or maintenance of your Lift System and cannot find the answer in this manual, please contact us.

- Only qualified staff, who understand all operating and safety procedures may operate the equipment.
 - When used in Baths always physically check water temperature before and during bathing/showering.
 - Castor brakes must be securely locked on the Stretcher/Chair Lift before transferring patient.
 - The patient must be comfortable on Chair or Stretcher, with **safety belts securely attached**. Safety belts must be maintained in good condition.
 - Do not let the patient sit on the short ends on the Stretcher.
 - Never leave the patient unattended.
 - Make sure that all parts that come in contact with the patient are intact and without sharp edges.
 - Make sure the patients hands are placed on the proper supports.
 - To avoid pinching of arms or fingers during transfer, make sure that the patients arms are crossed over his/her body or are firmly grasped on the stabilizing bar or the armsupport.
 - Operate the Lift carefully when transporting patients through narrow corridors or over irregular surfaces.
 - Wipe beneath Stretcher/Chair to prevent water dripping on the floor after bathing/showering.
 - Equipment must be cleaned after each use.
 - Avoid skin or eye contact with detergent, which is highly corrosive. If contact is made, rinse immediately with excess water. If irritation occurs, consult, a doctor. Always read and follow detergent supplier instructions.
 - Never clean pads, or pillow in gas sterilization unit, the items may explode.
- It is recommended that the Lift batteries are charged daily or whenever Lift is not in use. Batteries can not be overcharged.
 - The TR battery charger is the only one approved for charging the TR Lift.
 - Do not use battery charger in wet or damp areas - potential shock hazard.
 - Before you attempt to attach or remove Stretcher side parts, castor brakes must be securely locked.
 - Stretchers, seats, all pads etc. must be kept in place, well maintained, and all screws should be tightened.
 - Make sure that all castors are clean, in place and free from damages.
 - Make sure that the safety belts are clean, in place and free from damages.
 - Drive belt must be inspected visually for signs of wear. Lifter should not be used if defects are noted. Prompt service is required. If drive belt fails, the Stretcher/Chair will stop on column.

Note: Do not use the Lifts in any way that varies from the safety precautions and operating procedures detailed in this manual. The Lifts must not be operated by anyone until the user has read and understands the instructions in this manual.

Head office


TR Equipment AB, Sweden
Phone: +46 140 38 50 60
info@trequipment.se
www.trequipment.com

Group Subsidiaries

TR Group Inc, US, Phone: +1 (800) 752 6900
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



TR EQUIPMENT

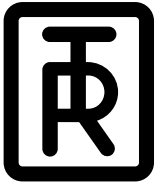

TR 9650


Mobile Bath Lift System

Hygiene Equipment
Designed for People

Features and Controls for the Stretcher Lift - Combi Lift

Position	Description
1	Support column
2	Chassis Base
3	Transfer handle
4	Chassis leg
5	Positive brake castors
6	Straight steering castors
7	Foot control for raising
8	Foot control for lowering
9	Stainless Steel column
10	Hygiene seat
11	Stretcher side parts

12	Hygiene slot
13	"TR" Protection cover
14	Stretcher attachment and position setting bars
15	Safety belts
16	Transport grip
17	Stabilizing bar
18	Arm/Back support
19	Drive belt
20	Low battery audible tone
21	Battery charger input
22	Emergency Stop

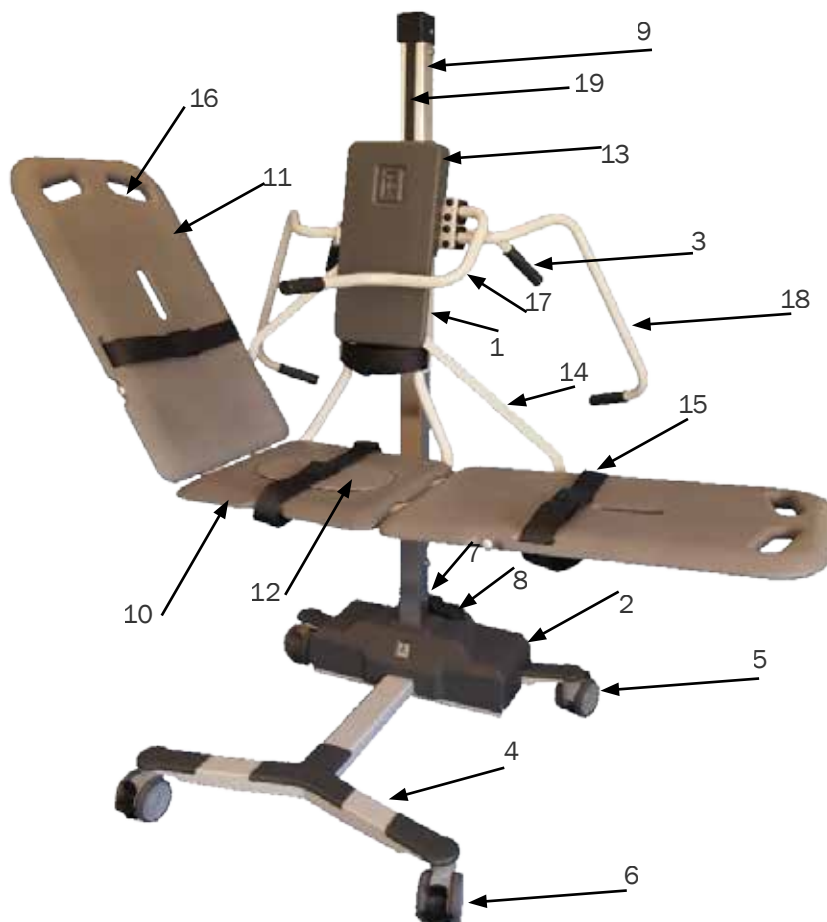


Head office
TR Equipment AB, Sweden
Phone: +46 140 38 50 60
info@trequipment.se
www.trequipment.com

Group Subsidiaries
TR Group Inc, US, Phone: +1 (800) 752 6900
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



TR EQUIPMENT

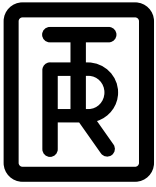

TR 9650



Mobile Bath Lift System

Hygiene Equipment
Designed for People

Features and Controls for the Chair Lift

Position	Description
1	Support column
2	Chassis Base
3	Transport handle
4	Chassis leg
5	Positive brake castors
6	Straight steering castors
7	Foot control for raising
8	Foot control for lowering
9	Stainless Steel column

10	Rotating arm/back supports
11	Hygiene seat
13	"TR" Protection pad
14	Backrest Pad
15	Safety belts
16	Grip
19	Drive belt
20	Low battery audible tone
21	Battery charger input
22	Emergency Stop

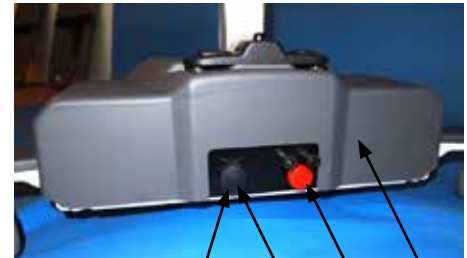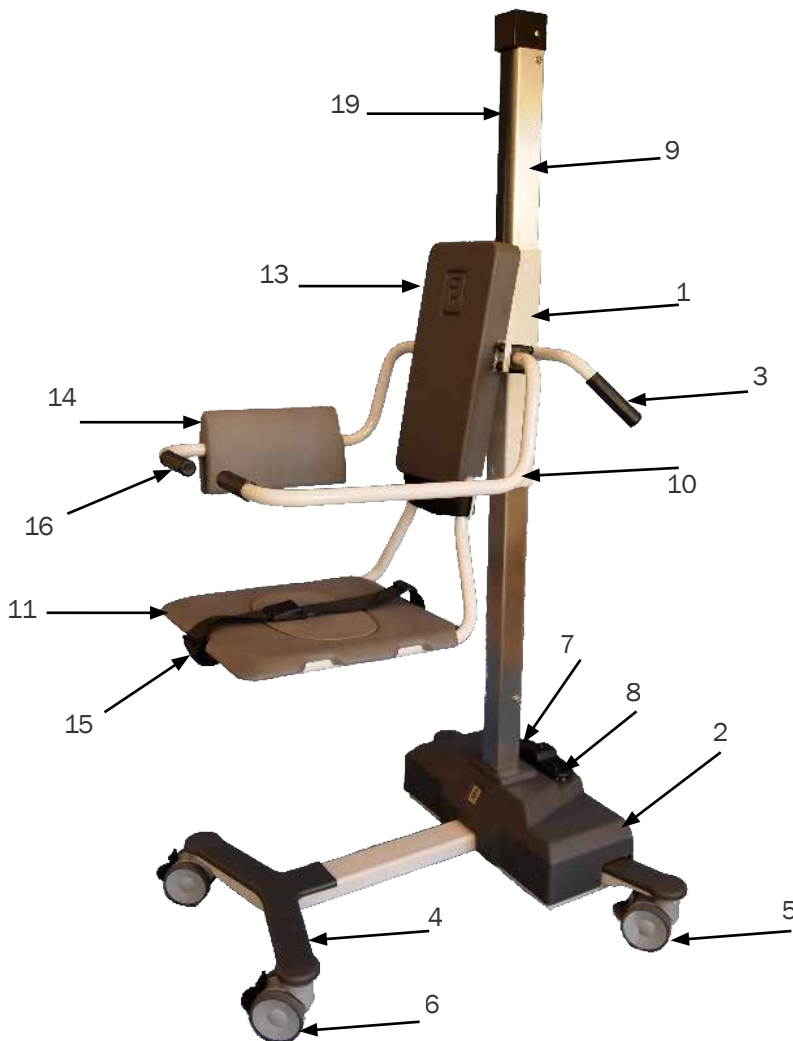

Open lid:




20 21 22

Head office
TR Equipment AB, Sweden
Phone: +46 140 38 50 60
info@trequipment.se
www.trequipment.com

Group Subsidiaries
TR Group Inc, US, Phone: +1 (800) 752 6900
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



TR EQUIPMENT

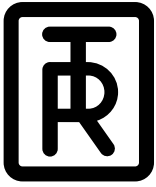

TR 9650

Mobile Bath Lift System

Hygiene Equipment
Designed for People

Features and Controls for the Mobile Lifts

Height adjustment

Raise and lower the chair/stretcher with the foot control arrow symbols.

Raise by pressing the "up" right arrow symbol (7).

Lower by pressing the "down" left arrow symbol (8).


The chair/stretcher raises and lowers smoothly while the foot arrow symbols are activated and stops immediately when the foot arrow symbols are released.

When the chair or stretcher hits the top position an audible tone sounds. Press the down foot control (8) to lower.

Overload protection

Working Load Limit 150 kg/330 Lbs.

The circuit board is protected from overload. When overloaded an audible tone sounds. Reduce the load. Press the "down" foot control (8) and lower.

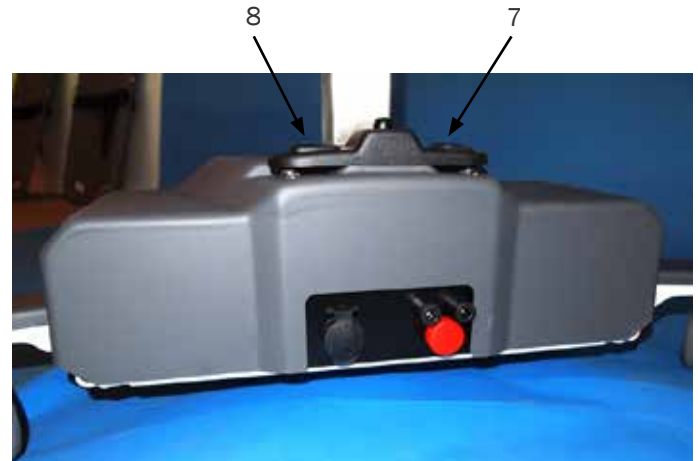

Autostop mechanism

If the chair or the stretcher hits an obstacle when lowering, the Lift will automatically stop.

To release it from this position just press the "up" right arrow symbol (7).

When the drive belt for some reason is slacking press the "up" foot control (7), until a "click" sounds.

If drive belt (19) fails, chair/stretcher will automatically stop/lock on the column. Call for technical assistance.



Head office


TR Equipment AB, Sweden
Phone: +46 140 38 50 60
info@trequipment.se
www.trequipment.com

Group Subsidiaries

TR Group Inc, US, Phone: +1 (800) 752 6900
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



TR EQUIPMENT

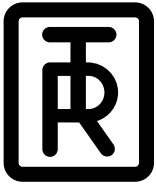

TR 9650

Mobile Bath Lift System

Hygiene Equipment
Designed for People

Features and Controls for the Mobile Lifts

Charging instructions for the Mobile Lifts

Charge the lift daily or when the low battery audible tone sounds, when pressing the “up” foot control arrow. The lift batteries can never be overcharged.

- Use only the 24 V DC battery charger model delivered by TR.
- **Note:** Connect the battery charger to the lift before it is connected to the mains. Insert the battery charger magnetic plug into chassis base (21). Make sure that the plug is completely in place.
- Plug the battery charger into the mains.
- When the battery charger is properly connected, an indicator on the battery charger will be lit. If not, recheck the connections.
- The indicator informs the charging cycles, yellow while charging and green when the batteries are fully charged.
- Maximum charging time is 8 hours. The lift batteries can never be overcharged.
- When the charging is completed, disconnect the battery charger from the mains, before you disconnect it from the lift.
- A flat fuse (15 A) is connected between the batteries under the cover.
- Emergency Stop button (22).



Open lid:



21 22

Batteries

If replacing the lead batteries, deposit in accordance with local regulations.



Head office

TR Equipment AB, Sweden
Phone: +46 140 38 50 60
info@treqipment.se
www.treqipment.com

Group Subsidiaries

TR Group Inc, US, Phone: +1 (800) 752 6900
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



TR EQUIPMENT



TR 9650

Mobile Bath Lift System

Hygiene Equipment
Designed for People

Features and Controls for the Mobile Lifts

Attaching Stretcher side part(s) on to the Combilift Chair. No tools required.

Lock the castors and open up the Chairs chest safety belt. Remove the "TR" protection cover (13) by pulling it straight out at the top. Lower the protection cover slightly, as it is secured at the bottom and remove it.

Raise the Combilift Chair, with the foot control, so it is appr. 90 cm (36") above floor for easy stretcher attachment. Swing away the Chair's Arm/Back support arm(s) (18).



Attach (hang) the Stretcher side part(s) in the side brackets on the Chair. Raise the Stretcher side part(s) so that the stretcher setting bar(s) (14) can be positioned into the bracket track(s) on the Combichair column plate.

Attach the "TR" protection cover again. Start at the bottom so it overlaps the lower steel part on Chair plate. Snap on the protection cover at the top.

Buckle up the chest safety belt again and release castor brakes and the Combilift with Stretcher is ready for use.

To remove Stretcher parts, do it in a reverse way.

Important: Never use the Lift with Stretcher side parts attached without the "TR" protection cover (13) securely in place. This to avoid pinch injuries and the possibilities for the stretcher attachment setting bars (14) to drop out from bracket tracks on the column plate.

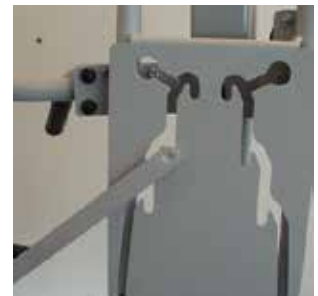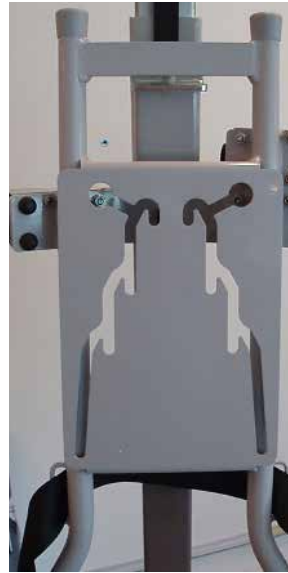

Castor Brakes

To lock the positive, foot-operated brakes on the two twincastors (5), lock the grey brake levers with your foot.

To release, press on the raised part on the grey lever.

To increase the brake stability also lock the two grey straight steering castor (6) black levers.

Straight Steering Castors

Lock the straight steering castor facing the direction you are transporting in.

To lock the straight steering, press down the black locking lever on castor until it clicks into position.

To release the straight steering press the raised black locking lever.

Chair lift chassis:



Stretcher and Combi lift chassis:



Head office


TR Equipment AB, Sweden
Phone: +46 140 38 50 60
info@trequipment.se
www.trequipment.com

Group Subsidiaries

TR Group Inc, US, Phone: +1 (800) 752 6900
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



TR EQUIPMENT

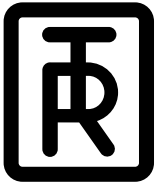

TR 9650

Mobile Bath Lift System

Hygiene Equipment
Designed for People

Features and Controls for the Mobile Lifts

Chair Back rest

For more versatility in patient transfer, patients can be oriented in either direction on the Chair. Simply switch the backrest pad (14) from one rotation arm support (10) over to the other.


The Chair has a "TR" protection cover (13) which temporary can be used as a backrest and enables the patient to easily be positioned for transport.

Stretcher Back/Leg rest

For more versatility in patient transfer, patients can be oriented either way on the Stretcher. Simply switch the head pillow from one end to the other, and note that a hinged back/leg pad operates either way on the Stretcher.

Raise either backrest by lifting the stretcher slightly and lift the setting bar (14). The setting bar (14) automatically latches into either angled positions.

Lower either backrest stretcher by slightly lifting the stretcher. Grasp and lift the setting bar (14) and lower the stretcher to desired position.

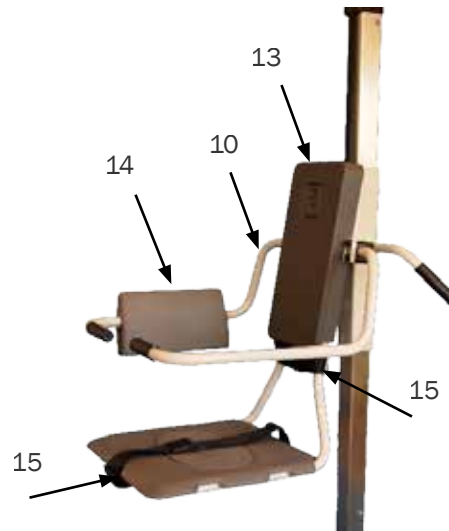


Important. Ensure that the setting bar (14) is fully in place in either positions.

Safety belts

When patient is seated on the Chair, bring the adjustable safety belts (15) around the patient's chest and legs. Run the belt ends (15) through the guides on the Chair. Adjust the safety belts (15) to ensure patient's comfort and security, then tighten and fasten the safety belts securely.

When the patient is placed on the Stretcher bring the two adjustable safety belts (15) over the patients chest and legs. Adjust the safety belts (15) to ensure patient's comfort and security, then tighten and fasten the safety belts securely.

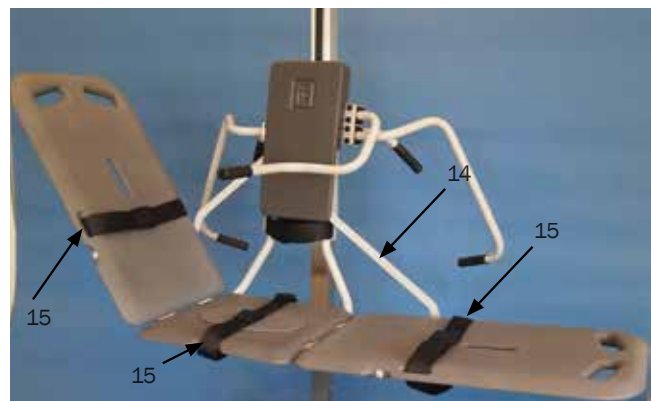

Head office


TR Equipment AB, Sweden
Phone: +46 140 38 50 60
info@trequipment.se
www.trequipment.com

Group Subsidiaries

TR Group Inc, US, Phone: +1 (800) 752 6900
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



TR EQUIPMENT

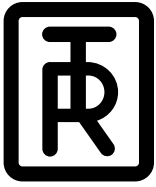

TR 9650

Mobile Bath Lift System

Hygiene Equipment
Designed for People

Features and Controls for the Mobile Lifts

Cleaning the Lift


Note: The User/operator is responsible for selecting the detergents and additives. Please read the detergent supplier's instruction manual and find out whether the product is suitable.

Do not use solvent based cleaning liquids, which can damage or stain the pad and pillow surfaces.
(See Routine Maintenance)

Chair Lift

For thorough cleaning, remove the backrest pad (14) and clean it in the tub.

Position the Chair over the tub and spray with the tub's cleaning system. Rinse Chair, backrest pad with fresh water from the patient shower after cleaning.
Wipe beneath Lift to prevent water dripping on floor.

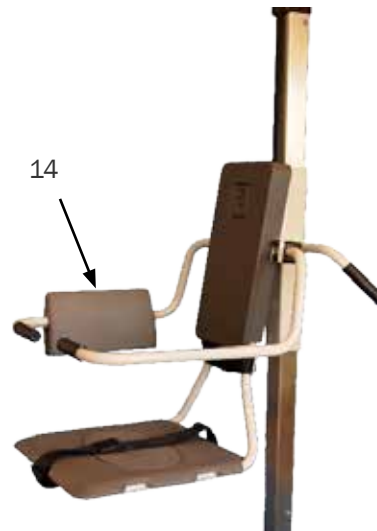


Stretcher Lift

The Stretcher has removable pads (11). The head pillow, made of PVC, is heat sealed against moisture and can be washed and cleaned.

For thorough cleaning, position the Stretcher over the tub and spray with the tub's cleaning system. Rinse the stretcher, pillow and pads thoroughly with fresh water from the patient shower after cleaning.

Wipe beneath Lift to prevent water dripping on floor.

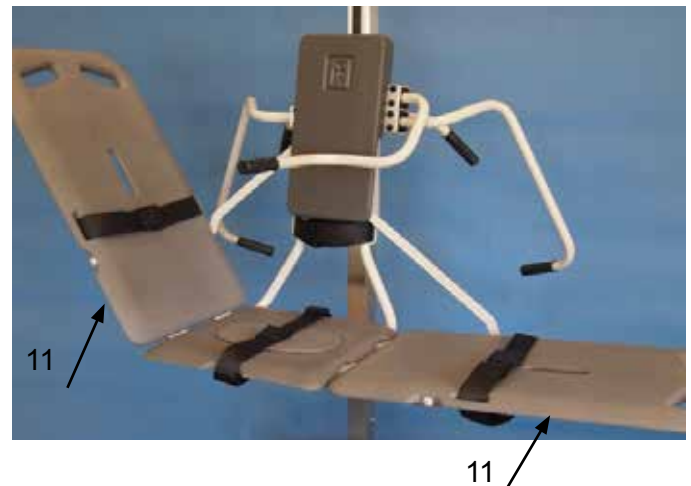

Warning: Never clean pads and/or pillow in gas sterilization unit.
Safety belts may be machine washed and dried.
Stretcher pillow velcro strap must be pressed together during washing and drying to preserve their gripping power.

Head office


TR Equipment AB, Sweden
Phone: +46 140 38 50 60
info@trequipment.se
www.trequipment.com

Group Subsidiaries

TR Group Inc, US, Phone: +1 (800) 752 6900
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



TR EQUIPMENT

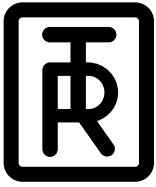

TR 9650

Mobile Bath Lift System

Hygiene Equipment
Designed for People

Patient Handling - Procedures for the Stretcher


Transport on Stretcher

To ensure safety, patients shall be transported on the Stretcher following the special recommended procedures.

The Stretcher can always be lowered all the way down to the lift legs.

Warning: Safety belts must be securely fastened around the patient at all times.

1. Lock the castor brakes.
2. Adjust stretcher side part to a comfortable position.
3. Support the patient's comfort on the Stretcher.
4. Adjust the chest and leg safety belts (15) and fasten them.
5. Adjust the stretcher to a comfortable transport height by using the "up" right arrow symbol (7) for raising and (8) "down" left arrow symbol (8) for lowering.
6. Allow the patient to firmly grasp the stabilizing bar for extra safety and support.
7. Unlock the castor brakes (5) and, if desired, activate the straight steering castor (6) when transporting.
8. When transporting patient on the Stretcher, push the Lift with one hand on the transfer handle on the column and your other hand in the stretcher side part opening (16), behind the patient.

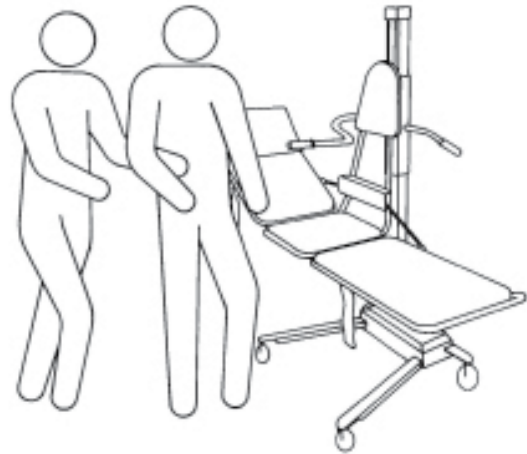

Head office


TR Equipment AB, Sweden
Phone: +46 140 38 50 60
info@trequipment.se
www.trequipment.com

Group Subsidiaries

TR Group Inc, US, Phone: +1 (800) 752 6900
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



TR EQUIPMENT

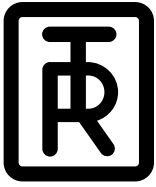

TR 9650

Mobile Bath Lift System

Hygiene Equipment
Designed for People


Patient Handling - Procedures for the Stretcher

From Wheelchair onto Stretcher

You can raise or lower the Stretcher to match wheelchair height and thus ease the transfer of wheelchair patients.

1. Lock the Lifts castor brakes (5) and raise stretcher backrest part to an upright position.
2. Position the wheelchair next to the Stretcher and lock the wheelchair castors.
3. Lower the Stretcher, using the "down" left arrow symbol (8) for lowering, to wheelchair seat height.
4. Remove the wheelchair armrest and remove or swing wheelchair footrest away on the transfer side.
5. Transfer patient laterally from wheelchair onto Stretcher, supporting patient under arms.
6. Position patient comfortably on the Stretcher, secure safety belts (15) and raise Stretcher, using the "up" right arrow symbol (7) for raising, to transport position.
7. Unlock the castor brakes (5) and, if desired, activate the straight steering castor (6) when transporting.

To return patient to wheelchair/bed reverse steps.

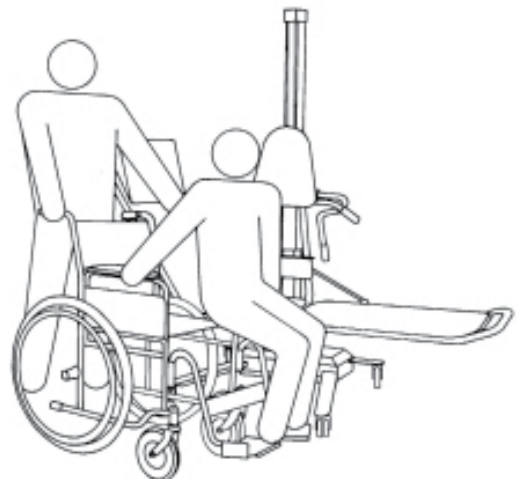

Head office


TR Equipment AB, Sweden
Phone: +46 140 38 50 60
info@trequipment.se
www.trequipment.com

Group Subsidiaries

TR Group Inc, US, Phone: +1 (800) 752 6900
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



TR EQUIPMENT

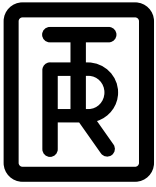

TR 9650

Mobile Bath Lift System



Hygiene Equipment
Designed for People


Patient Handling - Procedures for the Stretcher

From Bed onto Stretcher

Follow the recommended procedures. Do not omit any steps. Stay in sequence.

1. Make sure that the bed guard rail is raised and secured on the opposite side from the Stretcher.
2. Roll the patient onto his/her side, facing away from the Stretcher and parallel to the Stretcher.
Note: Position patients on the bed mattress so that the Stretcher can be placed close to the patient's back for transfer.
3. Position the Stretcher over the bed and lower it, using the "down" left arrow symbol (8) for lowering, onto the mattress. The lowering will automatically stop when Stretcher hits bed surface.
4. Lock the Lift castor brakes (5).
5. Roll the patient over, onto a comfortable position on the Stretcher.
6. Raise the Stretcher slightly, using the "up" right arrow symbol (7) for raising, and secure the safety belts (15) over the patients chest and legs.
7. Raise the stretcher side part to a comfortable position, using the end support and latch the setting bar (14).
8. Raise the Stretcher with the foot control.
9. Unlock the Stretcher's castor brakes (5).
10. Pull the Lift clear from the bed, using the transfer handle (3) and adjust the Stretcher to a comfortable transport height on column.
11. Push the Lift by using the end support opening and transfer handle.
12. If desired, activate a straight steering castor (6) when transporting.

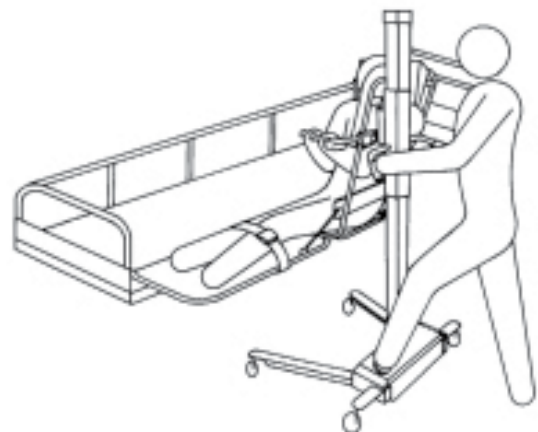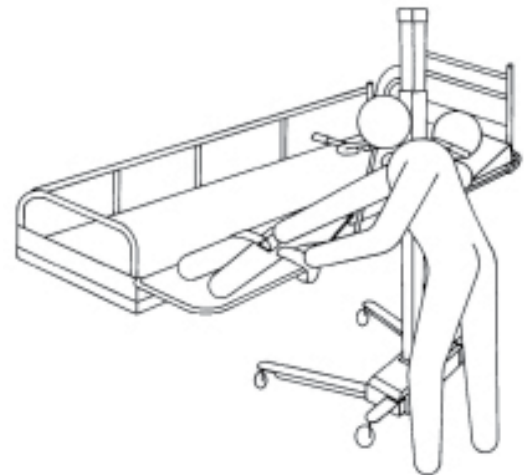

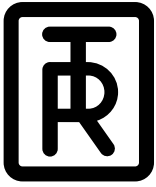

TR 9650

Mobile Bath Lift System

Hygiene Equipment
Designed for People

Patient Handling - Procedures for the Stretcher


On Stretcher into Bathtub

When bathing, the patient must at all times remain on the Stretcher with safety belts (15) securely fastened. Backrest must be in full upright position when Stretcher is lowered into the tub and the tub foot support removed.

Warning: Never leave patient unattended at any time.

1. Use the transfer handles (3) to place the Lift next to the prefilled bathtub.
2. Lower the bathtub and raise the Stretcher high enough to clear the tub rim.
Important: Make sure that patients hands firmly grasp the stabilizing bar (17) or that the patients arms are crossed over his/her body. This to avoid pinching arms or fingers between Stretcher frame and tub rim when Stretcher is lowered or tub is raised.
3. Position the Stretcher so it is centred over the tub.
4. Raise the bathtub to a comfortable working height before lowering the Stretcher and patient into the water.
5. After bathing, lower the bathtub before raising the Stretcher. You can use the patient shower system to rinse the patient in this position, allowing the water to drain into tub.
6. Raise Stretcher to clear tub rim and pull it away from the bathtub. Lower the stretcher to convenient transport height. The patient can then be dried and wrapped in a bath towel.
7. If desired, activate one of the straight steering castor (6) when transporting.

Note: Wipe beneath Stretcher to prevent water dripping on floor.

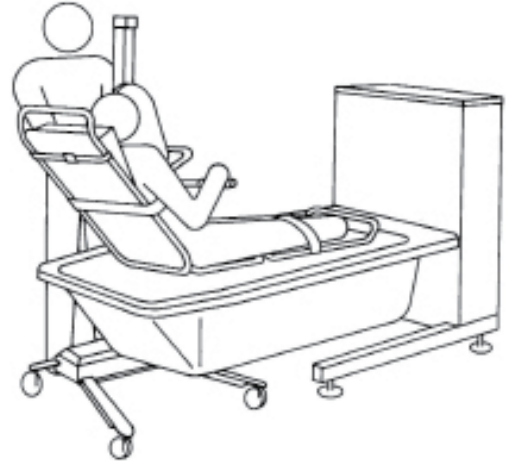

Head office


TR Equipment AB, Sweden
Phone: +46 140 38 50 60
info@trequipment.se
www.trequipment.com

Group Subsidiaries

TR Group Inc, US, Phone: +1 (800) 752 6900
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



TR EQUIPMENT

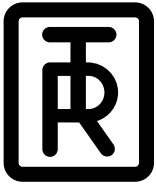

TR 9650


Mobile Bath Lift System

Hygiene Equipment
Designed for People

Patient Handling - Procedures for the Stretcher

From Stretcher onto Bed

1. Make sure that the bed rail is raised and secured on the opposite side from the Stretcher.
2. Position the Stretcher over the bed, lock the Lift castor brakes (5), and lower the Stretcher onto the mattress. The stretcher will automatically stop when hitting the bed surface.
3. Lower the stretcher backrest onto the bed mattress.
4. Loosen the safety belts.
5. Roll the patient gently onto the bed.
6. Raise the Stretcher slightly to clear the mattress. Unlock the Lift castor brakes (5), and pull the Lift away from the bed.
7. Roll patient back to a comfortable position on the bed.
8. Raise and lock the bed guard rail on transfer side.

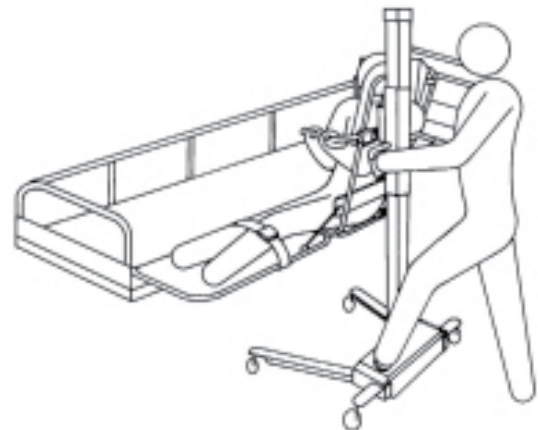



TR 9650

Mobile Bath Lift System

Hygiene Equipment
Designed for People

Patient Handling - Procedures for the Chair

Transport on Chair

To ensure safety, patients shall be transported on the Chair following the recommended procedures below.

Note: For the following special transfer procedures with the Chair, patients can be pivoted on the seat so they rest against the attached back protection pad (14).

Warning: Safety belts (15) must be securely fastened around patient at all times.

1. Lock the Chair Lifts castor brakes (5).
2. Adjust the backrest pad (14) for patient comfort.
3. Assist the patient to be seated comfortably on the Chair.
4. Adjust the safety belts (15) around the patients chest and legs and fasten them.
5. Raise the chair, using the "up" right arrow symbol (7) for raising, to a comfortable transport position.
6. Patient should grasp the grip on the arm support for extra safety and support.
7. Unlock the castor brakes (5), and if desired, activate one straight steering castor (6) when transporting.
8. When transporting push Lift with one hand on the transfer handle on the column (3) and your other hand on the rotating arm/back support (10), behind the patient.



Head office


TR Equipment AB, Sweden
Phone: +46 140 38 50 60
info@trequipment.se
www.trequipment.com

Group Subsidiaries

TR Group Inc, US, Phone: +1 (800) 752 6900
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



TR EQUIPMENT

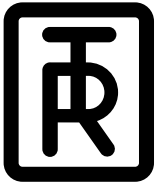

TR 9650

Mobile Bath Lift System

Hygiene Equipment
Designed for People

Patient Handling - Procedures for the Chair

From Wheelchair onto Chair

You can raise or lower the Chair to match wheelchair seat heights.

1. Lock the Chair castor brakes (5). Lift and rotate the armsupport to the rear position.
2. Position the wheelchair next to the Chair and lock the wheelchair wheels.
3. Lower the Chair seat to wheelchair seat height.
4. Remove the wheelchair armrest and remove or swing footrest away on the transfer side.
5. Transfer patient laterally from wheelchair to Chair, supporting patient under arms.
Note: Backrest pad (14) on rotating arm support (10) shall be in its regular transport position during the transfer.
6. Position patient comfortably on Chair with back against backrest pad (14).
7. Rotate the armsupport forward and secure safety belts (15) around patients chest and legs.
8. Raise Chair to transport position.
9. Unlock Chair Lift castor brakes (5), and if desired, activate one of the straight steering castors (6) when transporting.

To return patient to wheelchair reverse the steps.



Head office


TR Equipment AB, Sweden
Phone: +46 140 38 50 60
info@trequipment.se
www.trequipment.com

Group Subsidiaries

TR Group Inc, US, Phone: +1 (800) 752 6900
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



TR EQUIPMENT

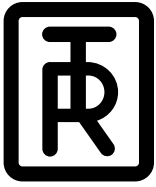

TR 9650

Mobile Bath Lift System

Hygiene Equipment
Designed for People

Patient Handling - Procedures for the Chair


From Bed onto Chair Lift

Follow the recommended procedures. Do not omit any steps. Stay in sequence.

1. Make sure that the bed guard rail is raised and secured on the opposite side from the Chair.
2. Rotate both arm back supports (10) to their rear positions.
3. Roll the patient onto his/her side, facing away from the Chair.
4. Position the Chair over the bed so that the seat is aligned with the patient buttocks.
5. Lower the Chair seat. The lowering will automatically stop when Chair hits bed surface.
6. Lock the Chair Lifts castor brakes (5).
7. Roll the patient over, onto a comfortable position on the Chair. Rotate the forward arm support (10) into its transport position in front of the patient.
8. Assist the patient to a sitting position and have him/her to firmly grasp the arm support (10). If the patient is in an electric bed, raise the bed head end to assist the patient. With the patient sitting, rotate the arm support (10) with backrest (14) forward and let patient rest against it.
9. Adjust safety belts (15) around patients chest and legs and fasten them.
10. Raise the Chair from the bed mattress and unlock the Lifts castor brakes (5).
11. Support the patients legs with one arm, roll the Chair Lift away from the bed, and then lower the patients legs.
12. Raise or lower the Chair to convenient height.

13. Push the Chair Lift by using the transfer handle (3) on the column and the arm support (10) behind the patient.

14. If desired, activate one straight steering castor (6) when transporting.

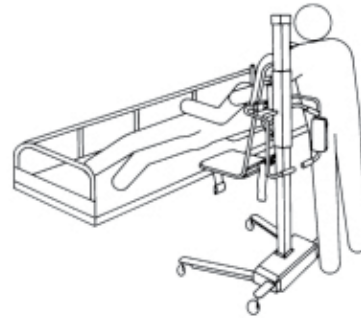

Head office


TR Equipment AB, Sweden
Phone: +46 140 38 50 60
info@trequipment.se
www.trequipment.com

Group Subsidiaries

TR Group Inc, US, Phone: +1 (800) 752 6900
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



TR EQUIPMENT

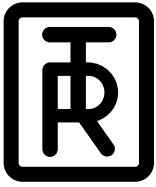

TR 9650

Mobile Bath Lift System

Hygiene Equipment
Designed for People

Patient Handling - Procedures for the Chair

On Chair into Bathtub

When bathing, the patient must remain on the Chair with safety belts (15) securely fastened at all times.

Warning: Never leave patient unattended at any time.

1. Position the Chair Lift next to the prefilled bathtub.
2. Lower the bathtub and raise the Chair high enough to clear the bathtub rim.
3. Support the patients legs clear of the bathtub rim with one arm and position Chair over the bathtub.
Note: Raise the Chair to ensure clearance of all body parts over the bathtub rim. Also, make sure the patients hands firmly grasp arm support (10) or cross arms over his/her body. This to avoid any possibility of the patient being pinched between the rim of the bathtub and Chair during the transfer.
4. Lower the patients legs into the bathtub.
5. Raise the bathtub to a comfortable working height and lower the Chair into the water until the Chair seat is resting on the bathtub bottom and lock the castor brakes.
6. When the patient is comfortable in the water, rotate one or both arm supports (10) over the bathtub rim to allow the patient to recline, if the patient is capable of selfsupport. The bathtub has an adjustable footrest for more comfort and security.
7. After bathing, rotate the forward arm support (10) into transport position in front of the patient.
8. Assist the patient to a sitting position and have him/her firmly grasp the arm support (10). Rotate the arm support (10) with the backrest pad (14) forward and let patient rest against it.

9. Lower the bathtub and raise the Chair above water level. You can use patient shower system to rinse the patient in this position, and the water will drain into the bathtub.
10. Raise Chair, to clear tub rim and support patients legs. Unlock the castor brakes and pull the Lift away from the bathtub.
11. Lower the Chair to a transport height. The patient can then be dried and wrapped in a bath towel.
Note: Wipe beneath Chair to prevent water dripping on the floor.
12. If desired, activate one of the straight steering castors when transporting.



Head office


TR Equipment AB, Sweden
Phone: +46 140 38 50 60
info@trequipment.se
www.trequipment.com

Group Subsidiaries

TR Group Inc, US, Phone: +1 (800) 752 6900
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



TR EQUIPMENT

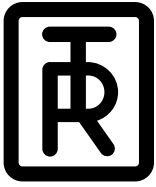

TR 9650



Mobile Bath Lift System

Hygiene Equipment
Designed for People

Patient Handling - Procedures for the Chair

From Chair into Bed

1. Make sure that the bed guard rail is raised and secured on the opposite side from the Chair.
2. Support the patients legs with one arm to clear the bed mattress as you position the Chair over the bed. Lock the castor brakes (5).
3. Lower the Chair. The seat will automatically stop when hitting the bed surface.
4. Make sure patient firmly grasps the arm support (10).
5. Loosen the safety belts (15).
6. Support patients back with one arm, and rotate rear arm support (10) with backrest pad (14) out of way. Lower patient back into bed.
7. Rotate forward arm support (10) behind Chair.
8. Roll the patient gently over in the bed.
9. Raise Chair seat. Unlock castor brakes (5) and move Chair Lift away from bed.
10. Roll patient to a comfortable position in the bed.
11. Raise and lock bed guard rail on transfer side.

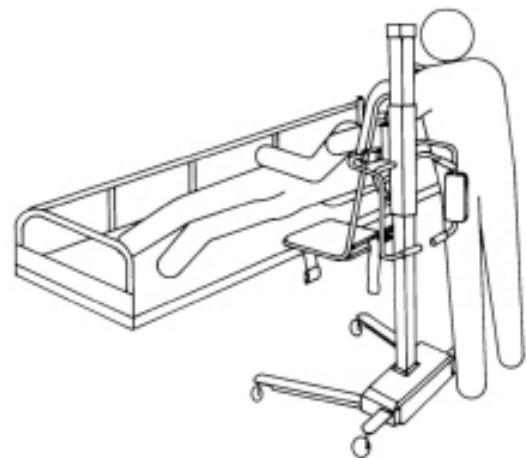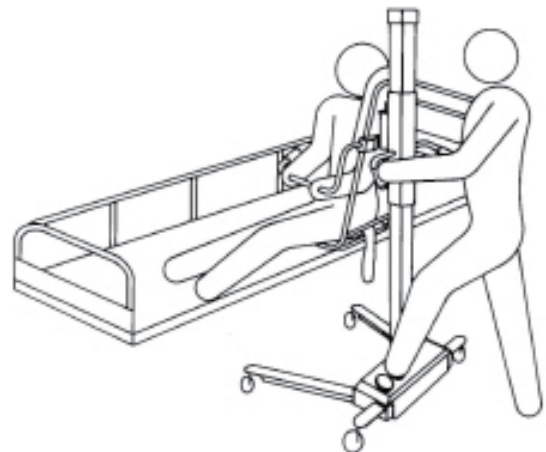

Head office


TR Equipment AB, Sweden
Phone: +46 140 38 50 60
info@trequipment.se
www.trequipment.com

Group Subsidiaries

TR Group Inc, US, Phone: +1 (800) 752 6900
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



TR EQUIPMENT

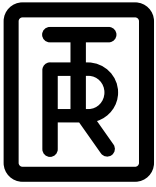

TR 9650

Mobile Bath Lift System

Hygiene Equipment
Designed for People

Patient Handling - Procedures for the Chair

Toilet Transfer using Chair

The Chair Lift can be used for either floor or wall mounted toilets. Remove the chair lid on the lift.

Wall Toilet

1. Lift the toilet seat and place the Chair Lift the same way as going into a bath, over the bowl.
2. Lock the castor brakes (5) and lower the Chair, using the "down" left arrow symbol (8) for lowering, to the toilet.

Floor Toilet

1. Lift the toilet seat and place the Chair Lift up to the toilet with the Lifts legs straddling the bowl.
2. Lock the castor brakes (5) and lower the Chair to the toilet.
3. When done, raise the Chair for easy access to the perineum area.



The hygiene seat (11) enables the patient to be washed in the perineum area.

Warning: Never leave patient unattended.

Head office


TR Equipment AB, Sweden
Phone: +46 140 38 50 60
info@trequipment.se
www.trequipment.com

Group Subsidiaries

TR Group Inc, US, Phone: +1 (800) 752 6900
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



TR EQUIPMENT

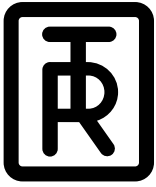

TR 9650

Mobile Bath Lift System

Hygiene Equipment
Designed for People

Routine Maintenance

The recommended frequency of these maintenance routine procedures will depend on how often the equipment is used.

It is good practice to establish these procedures and then adjust the frequency as your conditions indicate.

Daily Routine Maintenance

- Clean the Lift Chair/Stretcher, and pillow after each use with mixed, nonabrasive type of detergent, according to your detergent supplier instructions.
Note: Do not use solvent based cleaning liquids, which can damage or stain the pads and pillow.
- Use a sponge or brush, and promptly remove any soap or lime deposits, and faeces to avoid spots/stains. Rinse thoroughly with fresh water.
- Safety belts may be machine washed and dried. Velcro strips must be pressed together during washing and drying to preserve their gripping power.
- It is recommended that the Lift batteries are charged daily or whenever Lift is not in use. Batteries can not be overcharged.

Warning: Never clean any parts in gas sterilization unit.

Weekly Maintenance

- Visually inspect the Lift Chair/Stretcher to make sure there are no nicks or cracks which can scratch the patient.

Monthly Maintenance

- Check that all screws and fastenings are tight.

Annually Maintenance

- To keep the Lift running smoothly, we recommend you to keep the square column clean.
- Drive belt on the Lift column must be inspected visually for signs of wear.
- Lifts should not be used if defects are noted. Prompt service is required.

Autostop check-up

1. Check the autostop mechanism by lifting the chair/stretcher about 30 cm up on the column.
2. Let go, and push slightly and the chair/stretcher will lock on the column.
3. Drive belt will be slacking. Press the "up" right symbol (7) to straighten the drive belt and to release the autostop.

Batteries

If replacing the lead batteries, deposit in accordance with local requirements. We recommend that the batteries in the product are replaced every three (3) years.



Head office


TR Equipment AB, Sweden
Phone: +46 140 38 50 60
info@trequipment.se
www.trequipment.com

Group Subsidiaries

TR Group Inc, US, Phone: +1 (800) 752 6900
TR Equipment Ltd, UK, Phone: +44 (0) 844 335 8386
TRanspoRehaMed GmbH, Germany, Phone: +49 (0) 212 645850-0



TR EQUIPMENT

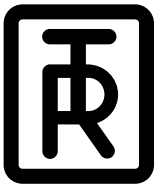






TR 9650

Mobile Bath Lift System

Hygiene Equipment
Designed for People

Labels on the TR 9650

1.		Maximum working load
2.		Mandatory to read the operating manual before use
3.		When using the mobile lift with stretcher. Do not sit on the side parts of the stretcher.
4.		TR Equipment AB Logotype
5.		The identification label states the product's model- and part number, serial number, date of manufacture, the manufacturing country and the name and address of the manufacturer. REF = Model- and part number CE = CE marking in accordance with the Medical Devices Directive 93/42/EEC as a Medical Device Class 1. SN = Serial number



