- GB** Mobilex transfer board
- D** Mobilex Transferbrett
- DK** Mobilex transferbrædt
- I** Mobilex tavola di trasferimento

*Item no.*      **240501, 240502 &**  
*Art. Nr.*      **240505**  
*Varenr.*  
*Codice*

## User manual

**MOBILEX A/S**  
Grønlandsvej 5  
DK-8660 Skanderborg  
www.mobilex.dk

## **GB** Mobilex transfer board

The Mobilex transfer board is made to help disabled persons with the transfer from one sitting surface to another. The models 240501 and 240502 are made of polypropylene and the model 240505 of PVC.

### **Safety tips**

Only use the product for the intended purpose. Do not step up on the device. It is not recommended to use the transfer board without previous training by a healthcare professional. Attempting a transfer without the tavolestance from a professional is not recommended. The boards should not be used to support more than the allowed max. user weight. Prior to use always inspect the board for signs of wear. Do not use the board if it appears weakened. Be sure the board is securely in place before putting weight on it. The open space between the two sitting surfaces (e.g. from wheelchair to bed) may not be larger than 20 cm.

### **Instructions for use**

Position one end of the board under the user's body and the other end on the destination surface. Check that the distance between the two sitting surfaces is not more than 20 cm and position the board so that it supports the full body weight during the transfer. The smooth surface of the board is best suited for clothed transfers. For bare-skin transfers a towel can be placed over the board for a smoother transfer.

### **Maintenance**

Normal dirt can be removed with standard cleaning agents and a soft cloth. Do not use solvents or abrasives.

<b>Technical data</b>	<b>240501</b>	<b>240502</b>	<b>240505</b>
<i>Dimension (L x W x H)</i>	650 x 245 x 6 mm	800 x 245 x 6 mm	800 x 245 x 6 mm
<i>Material</i>	Polypropylene	Polypropylene	PVC
<i>Weight</i>	0.81 kg	0.99 kg	1.54 kg
<i>Max. user weight</i>	130 kg	130 kg	130 k

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## Gurantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

## Declaration of Conformity (CE)

The company Mobilex A/S, Grønlandsvej 5, 8660 Skanderborg, Denmark, declares that the

### **CE Mobilex transfer board (item no. 240501, 240502 and 240505)**

is designed in accordance with the technical documentation and made of which conforms to the following basic requirements:


- **EU Council Directive 93/42/EEC for medical equipment**

Lars Nygaard  
Managing Director  
Mobilex A/S

Skanderborg, 3/5-2015

.....  
signature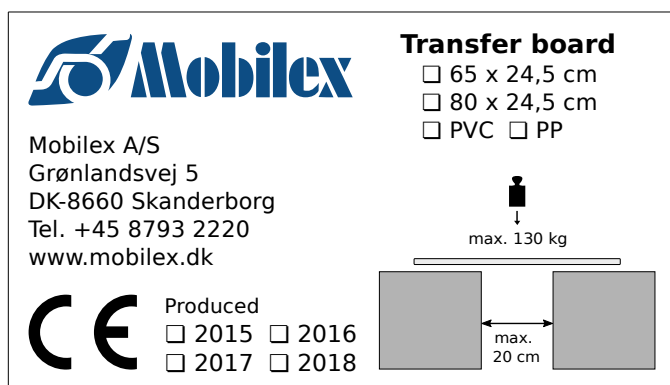

Example of the product label