- Ⓒ **GB Fixed toilet seat raiser**
- Ⓒ **D Fest montierte WC-Sitz Erhöhung**
- Ⓒ **DK Fastmonteret toiletforhøjer**
- Ⓒ **I Alza-water fisso**

Item no.
Art. Nr.
Varenr.
Codice

301703

User manual

CE

MOBILEX A/S
Grønlandsvej 5
DK-8660 Skanderborg
www.mobilex.dk

GB Fixed toilet seat raiser

Intended use of the device

The Mobilex Product is medical device which is intended to be used by elder or disabled persons only as a seating option and additional support for toilet. Any other use is prohibited.

Safety requirements

- ✓ Use the product only in perfect technical condition
- ✓ If you find any damage or defects to the product, please contact your dealer immediately
- ✓ Observe the details on the product label
- ✓ Please use the product only for the intended application
- ✓ The load capacity of 120 kg does not refer to the lid of the toilet seat. The lid is not suitable for sitting.
- ✓ Make sure you have a tight screw connection to the toilet bowl.
- ✓ Ensure that the height adjustments are evenly adjusted.
- ✓ Support the armrests evenly with both hands.

Product description


The toilet seat raiser 301703 can be mounted on most used toilet ceramics. The fixation takes place at the receiving holes on the toilet. The armrests can be swiveled and removed without tools. The seat can be removed for cleaning.

Assembly description

1. Remove the existing toilet seat and lid.
2. Adjust the width of the two brackets to the hole width on the toilet ceramic.
3. Fix the toilet seat raiser with the plastic mounting nuts.

Height adjustment

1. At the back of the toilet seat raiser remove the cover plugs from the holes you want to use.
2. On the rear side loosen the two hand screws off the support rod and screw the support rod to the desired position.
3. The front support rod is telescopic and can be adjusted to the desired length by removing the two adjusting clips. Then fix the length by re-placing the two clips. Now you have to fit the matching support rod into the desired support grid.
4. Make sure that all 4 settings are set to the same height.

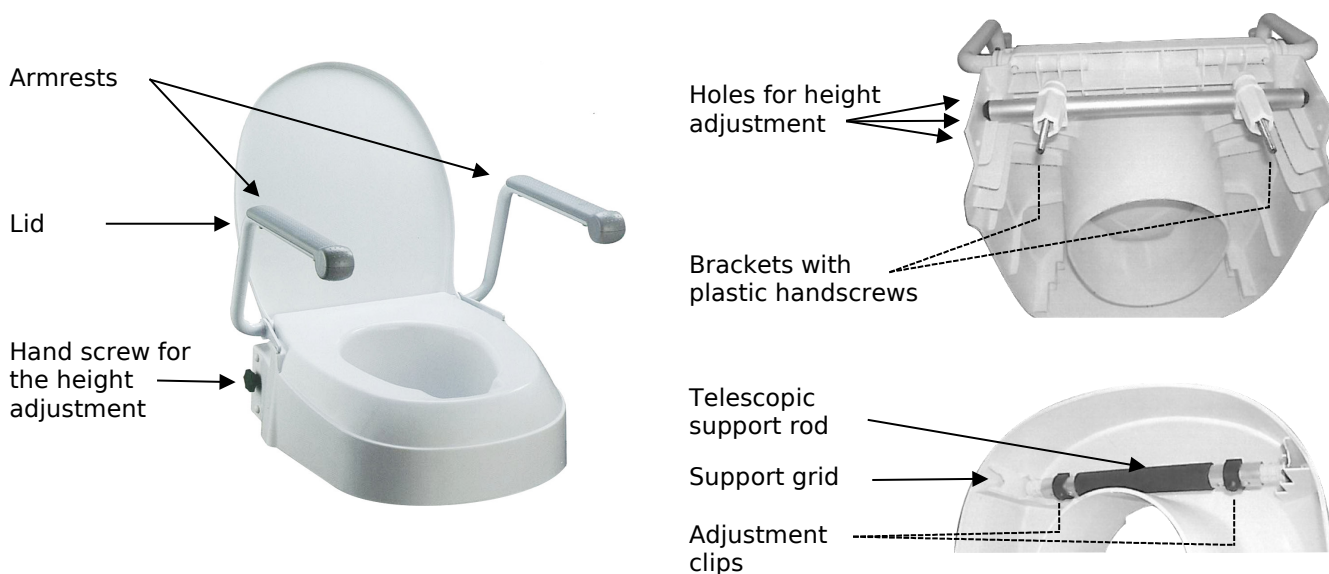Accessories

There are no accessories provided for this toilet seat raiser

Maintenance

Normal dirt on metal and plastic parts can be removed with standard household cleaning agents. Check the specific product information and follow it during cleaning. Use only commercial cleaning products which are suitable for cleaning and disinfection (no solvents or abrasives). Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to let check the product at regular intervals by a professional.

Disposal of the product

The product can not be disposed of with household waste but must be brought to the local recycling centre.

Most common problems and solutions

Symptoms	what to do?
<i>The armrests are unstable</i>	Check armrest assembly. Tighten the screws.
<i>The device isn't stable. It is moving.</i>	Check the fixing of the device to a toilet. Tighten the plastic nuts.

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Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

Declaration of Conformity (CE)

The company Mobilex A/S, Grønlandsvej 5, 8660 Skanderborg, Danmark, declares that the

 **Mobilex Toilet Seat Raiser**
Item nr. 301703

is designed in accordance with the technical documentation and made of which conforms to the following basic requirements:

- **EU Council Directive 93/42/EEC for medical equipment**

Lars Nygaard
Managing Director
Mobilex A/S

Skanderborg, 03/07/2017

.....
signature

 MOBILEX A/S Grønlandsvej 5 DK - 8660 Skanderborg Tel: +45 87 93 22 20 www.mobilex.dk	Fixed toilet seat raiser  ↓ Max. 120 kg
 Produced <input type="text"/>	Serial no. <input type="text"/>

Example of the product label