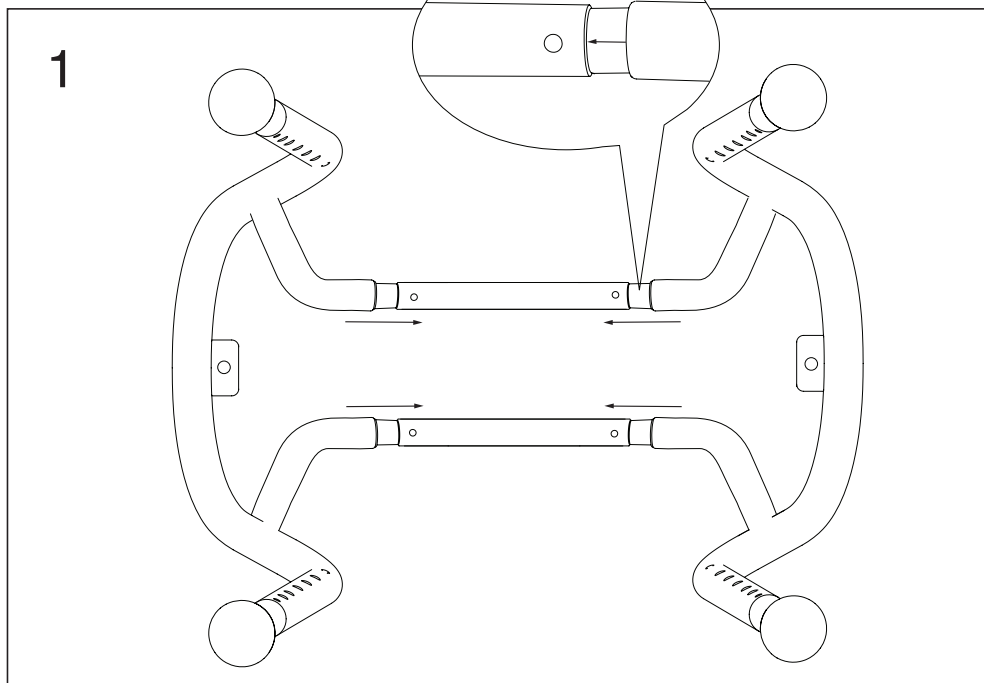
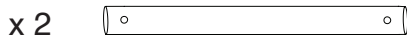
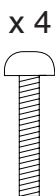
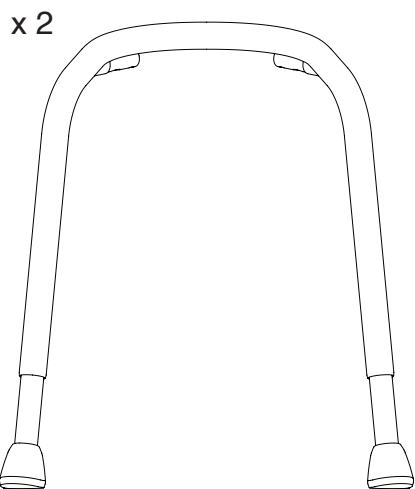
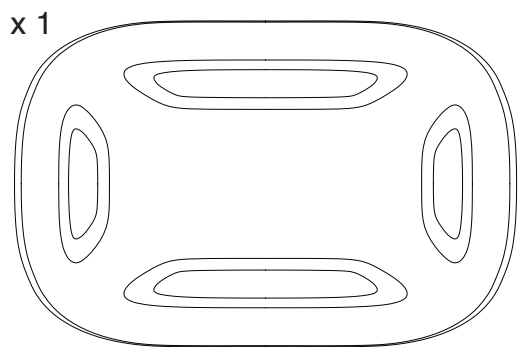
**TRUST**  
CARE





**LET'S SING**

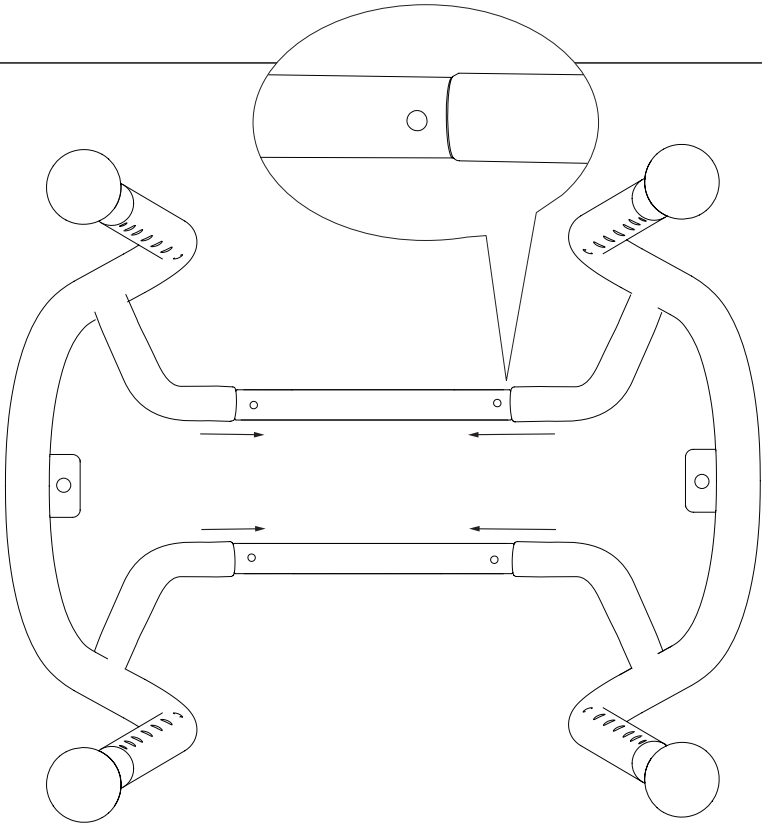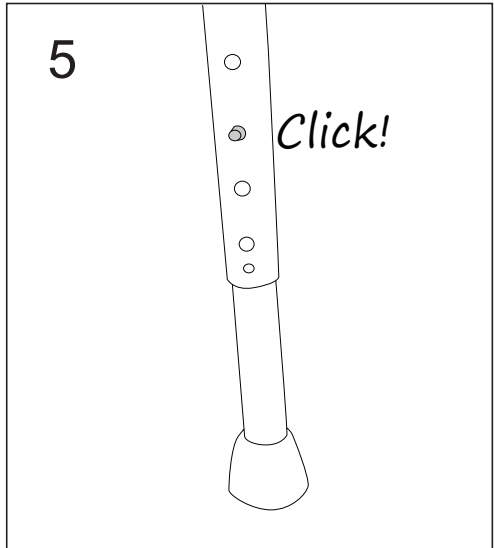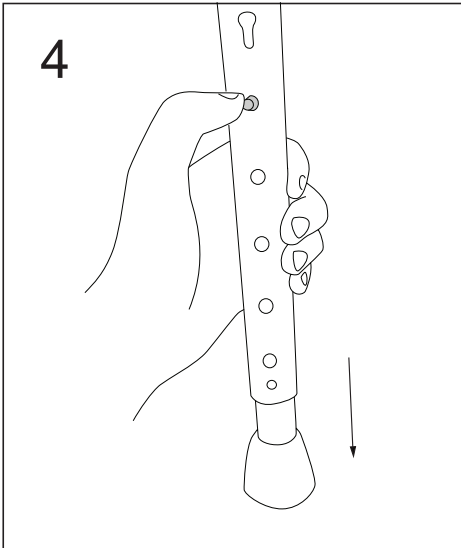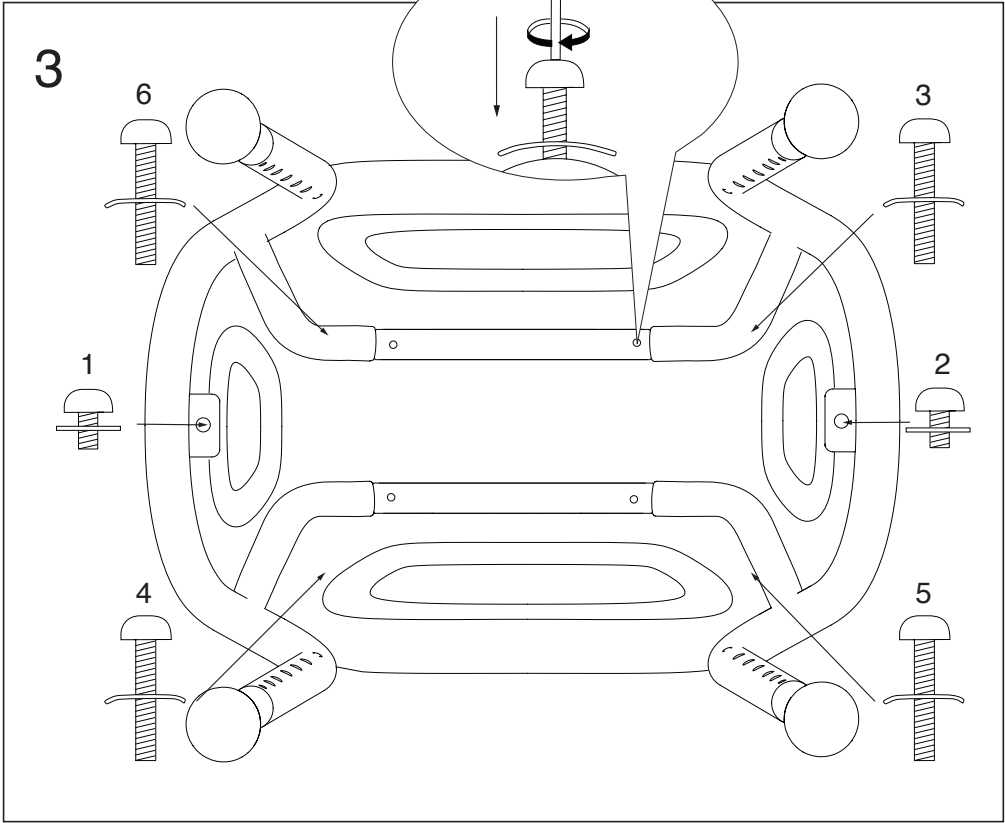Article No. LSG101

2

## SVENSKA

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## ENGLISH

Let's Sing is a new shower stool with a modern design to be used in the bathroom and shower. Let's Sing has a beautiful and stable construction for safe and secure use. The height of Let's Sing can easily be adjusted to the desired position with the buttons on the legs. A string basket is also available as an accessory and can be attached for storing soap, shampoo and a sponge.

**Warning:** Do not stand on the bath stool.

**Warning:** A defective product must not be used.

**Warning:** Always adjust all of the legs to the same height to avoid falling accidents because of the stool becoming unsteady.

**Max user weight:** Do not overload. Maximum user weight is 150 kg.

**Cleaning:** Use detergent without solvent, pH 5 – 9, or 70% disinfection spirit.

**Decontamination:** Use alkaline detergent pH min 6, in max 85°C/185°F for no more than 3 min.

**Spare parts:** Use only original accessories and original spare parts from Trust Care.

### Accessories

**Basket:** A small basket that is fastened under the seat.

### Technical data:

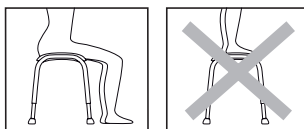
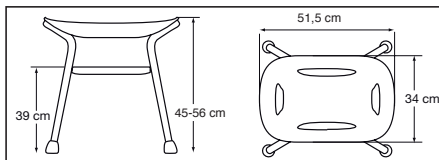
**Material:** Seat ABS, Tube frame aluminium, Feet TPE

**Weight:** 2,4 kg

**Max user weight:** 150 kg

**Height:** Adjustable from 45 to 56 cm.

**Colour:** Seat white, legs black, feet Black

The stool is intended to sit on. Do not stand on the stool because of the risk of falling accidents.

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