DK	Sliding foil	2
GB/US	Sliding foil	4
DE.....	Sliding foil	7
SE.....	Sliding foil	10
NO	Sliding foil	13
FR.....	Sliding foil	16
IT	Sliding foil	20

Manual – Version 1.00

DK... SMART GLIDE

Vers. 1.00

Varenr.:

284472 M Smart Glide 100 cm

284473 L Smart Glide 140 cm

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Item nos:

284472 M Smart Glide 100 cm
284473 L Smart Glide 140 cm

1.00	Purpose and use	4
1.01	Manufacturer	4
1.02	Description	4
1.03	Intended purpose	4
1.04	Area of use	4
1.05	Conditions of use	4
1.06	Important/Precautions	5
1.07	Use	5
2.00	Maintenance	5
2.01	Cleaning	5
2.02	Storage	5
3.00	Lifetime	5
4.00	Technical specifications	5
5.00	EC-Declaration of conformity	5
6.00	Environmental policy statement – V. Guldmann A/S	5
7.00	Warranty and service conditions	5
A.	Warranty	5
B.	Service or Repair	6
	Use of Guldmann Smart Glide in connection with the application of a sling	21

1.00 Purpose and use

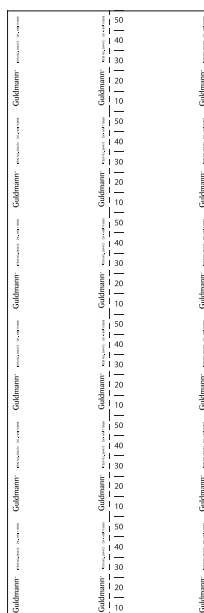
1.01 Manufacturer

V. Guldmann A/S
Graham Bells Vej 21-23A
DK-8200 Aarhus N
Tel. + 45 8741 3100
www.guldmann.com


1.02 Description

Guldmann Smart Glide is made of nylon with an embossed measuring tape as well as cutting lines, making it easy to cut the right size slide without wasting any material. The item number and HMI number is also embossed on the Smart Glide.

Smart Glide is available in various sizes and is available in rolls so that you can cut your own custom pieces.

Width: 100 cm

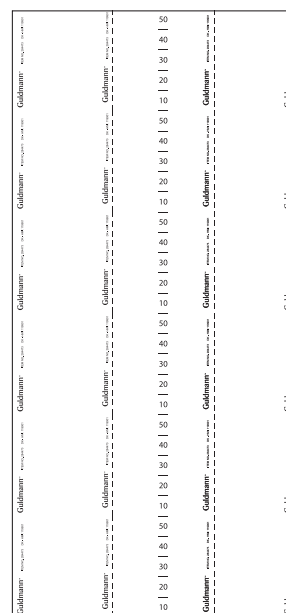Width: 140 cm

1.03 Intended purpose

Guldmann Smart Glide is intended for reducing friction when doing care tasks or moving and handling tasks.

1.04 Area of use

Guldmann Smart Glide is suitable for use in hospitals, nursing homes, institutions, rehabilitation centres and private homes.

1.05 Conditions of use

When using Smart Glide, it is assumed that:

- The staff is trained and has received instruction in use of Guldmann Smart Glide
- The helper/ staff is aware of the user's well-being during use
- Any restrictions with regards to the user are taken into account

1.06 Important/Precautions

- Read the entire user manual before starting to use the Guldmann Smart Glide
- Review the slides for defects before each use
- Make sure that the sliding ability is present before each use
- Never leave a user with the Guldmann Smart Glide underneath them
- Never leave the Guldmann Smart Glide on the floor

1.07 Use

Guldmann Smart Glide is intended to be a product that follows the individual user. When a piece of the Smart Glide material is worn or the person using it does not need it any longer, it is discarded. Smart Glide can be washed according to regulations and can be used again if necessary. The sliding ability must be tested on a continuous basis and a procedure for this should be prepared. You also need to take into account hygiene.

Think the situation through before using the Guldmann Smart Glide. If you are using Smart Glide in connection with the application of slings, the Smart Glide must be removed before the lift is made. There should never be a Smart Glide between the user and a sling during a lift or transfer.

Use of Guldmann Smart Glide in connection with the application of a sling see page 21.

2.00 Maintenance

2.01 Cleaning

Wipe down

Normal washing at the indicated temperature

Do not use bleaching agent

Tumble-drying at low temperature

Do not iron

Guldmann Smart Glide can be cleaned and disinfected using a wet cloth with soap and/ or Ethanol. You can use Wetwipe Ethanol and soap. Do not use any type of softener or bleach such as Chlorine.

2.02 Storage

Smart Glide is to be stored in a dry place with temperatures between -5 and +40 Centigrade. Do not store in direct sunlight.

3.00 Lifetime

The life span for a Guldmann Smart Glide depends on the use, wash a.s.o. Over time the fabric gradually loses its gliding ability.

4.00 Technical specifications

Max. weight 150 kg/ 330 lbs
Material Nylon with a 1% silicone coating

5.00 EC-Declaration of conformity

The product is manufactured in compliance with regulation (EU) 2017/745 of the European parliament and of the Council of 5 April 2017, as medical device class 1.

6.00 Environmental policy statement – V. Guldmann A/S

Guldmann is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

It is Guldmann's goal to:

- Comply with the current environmental legislation (e.g. WEEE and REACH directives)
- Ensure that we, at the widest possible range, use RoHS compliant materials and components
- Ensure that our products do not have an unnecessary negative impact on the environment regarding use, recirculation or disposal
- Ensure that our products contribute to a positive working environment in the places they are utilised

Inspections are made annually by the Department for Nature and Environment from the Municipality of Aarhus using the Danish Environmental Protection Act, section 42 as a reference.

7.00 Warranty and service conditions

A. Warranty

Guldmann warrants its equipment is free from material defects under normal use, and will perform substantially in accordance with the specifications set forth in documentation provided with the equipment.

This express warranty shall be in effect for one year from the date of original purchase and installation (the "Warranty Period"). If a valid claim is made during the Warranty Period for malfunction or equipment defect, Guldmann will repair or replace the equipment at no additional cost to you. Guldmann retains sole discretion as to whether the equipment will be repaired or replaced.

The warranty does not cover any part of the equipment that has been subject to damage or abuse by the user or others. The warranty does not cover any part of the equipment that has been altered or changed in any way by the user or others. Guldmann does not warrant that the lifting device functions will meet your requirements, be uninterrupted or error free.

The warranty set forth is in lieu of all other express and implied warranties, whether oral, written or implied, and the remedies set forth above are your sole and exclusive remedies. Only an authorized officer of Guldmann may make modifications to this warranty, or additional warranties binding on Guldmann. Accordingly, additional statements such as advertising or presentations, whether oral or written, do not constitute warranties by Guldmann.

This warranty shall be null and void if the equipment is operated and maintained in any manner inconsistent with its intended use or the instructions provided with the product. Further, in order for the warranty to remain in effect for the full Warranty Period, all service to the equipment must be provided by a Guldmann certified technician. Any parts or components repaired or replaced by a Guldmann certified technician will be guaranteed for the remainder of the Warranty Period.

Only for USA

This warranty shall be null and void if the equipment is operated and maintained in any manner inconsistent with its intended use

or the instructions provided with the product. Further, in order for the warranty to remain in effect for the full Warranty Period, all service to the equipment must be provided by a Guldmann Certified Technician. A Guldmann Certified Technician is a technician who has successfully completed Guldmann Service Training, and who holds a valid Service Training Certificate from Guldmann, and is in possession of a valid password to access Guldmann's Service and Information Console (SIC). A Guldmann Service Training Certificate and SIC password are valid for three years (only USA) from the date the technician is first certified. Thereafter, the technician must undergo re-certification training to obtain a new valid certificate and password. Any parts or components repaired or replaced by a Guldmann Certified Technician will be guaranteed for the remainder of the Warranty Period. In the event the warranty is rendered null and void, the purchaser shall indemnify and hold Guldmann harmless of and from any and all claims or liability arising as a result of equipment malfunction or misuse.

B. Service or Repair

Contact Guldmann Repair for an authorization to return any defective item during the Warranty Period. You will be provided with a return authorization number and address for returning the item for warranty service or replacement. Do not return items to Guldmann under warranty without receiving a Return Authorization Number.

If mailing the item, pack it carefully in a sturdy carton to prevent damage. Include your Return Authorization Number, a brief description of the problem and your return address and phone number. Guldmann does not assume the risk of loss or damage while in transit, so it is recommended you insure the package.

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Guldmann GmbH
Guldmann Sverige AB
Guldmann Srl
Guldmann Inc.