


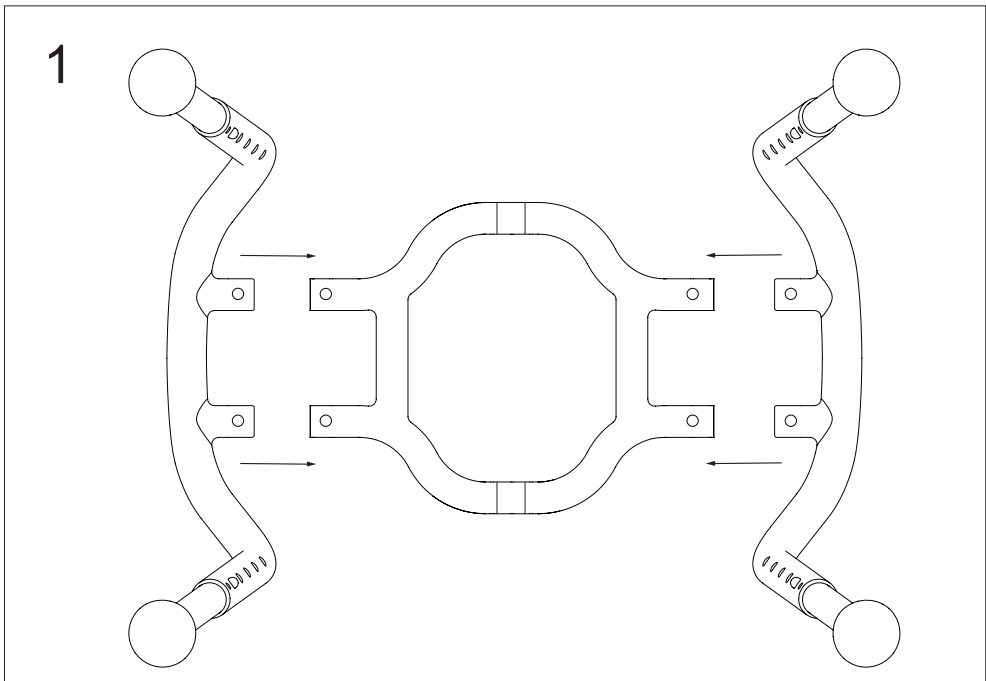
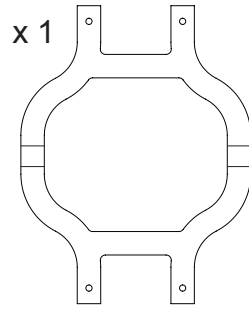
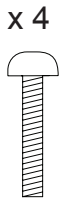
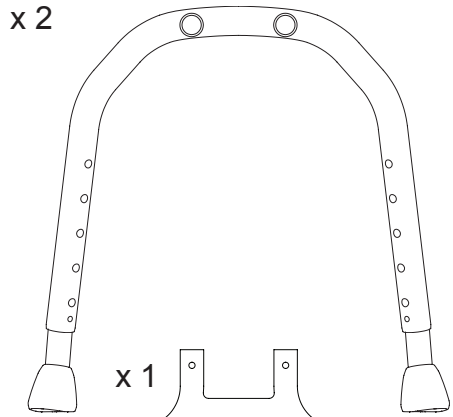
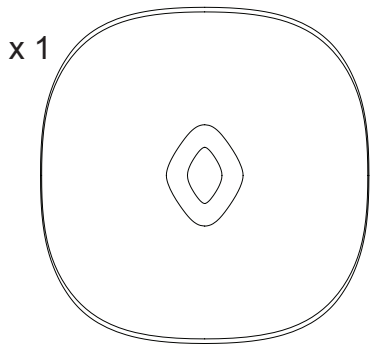
TRUST
CARE

LET'S SHOWER

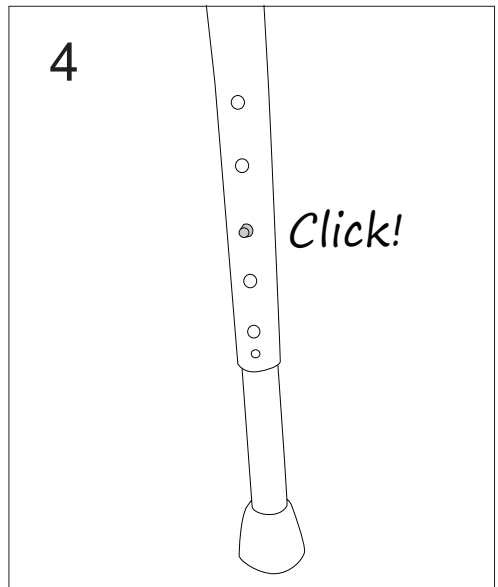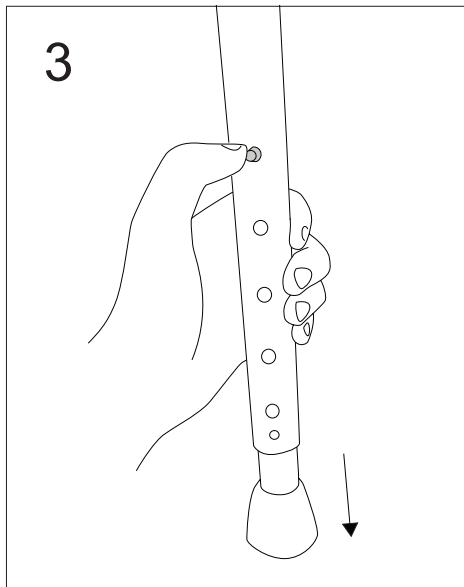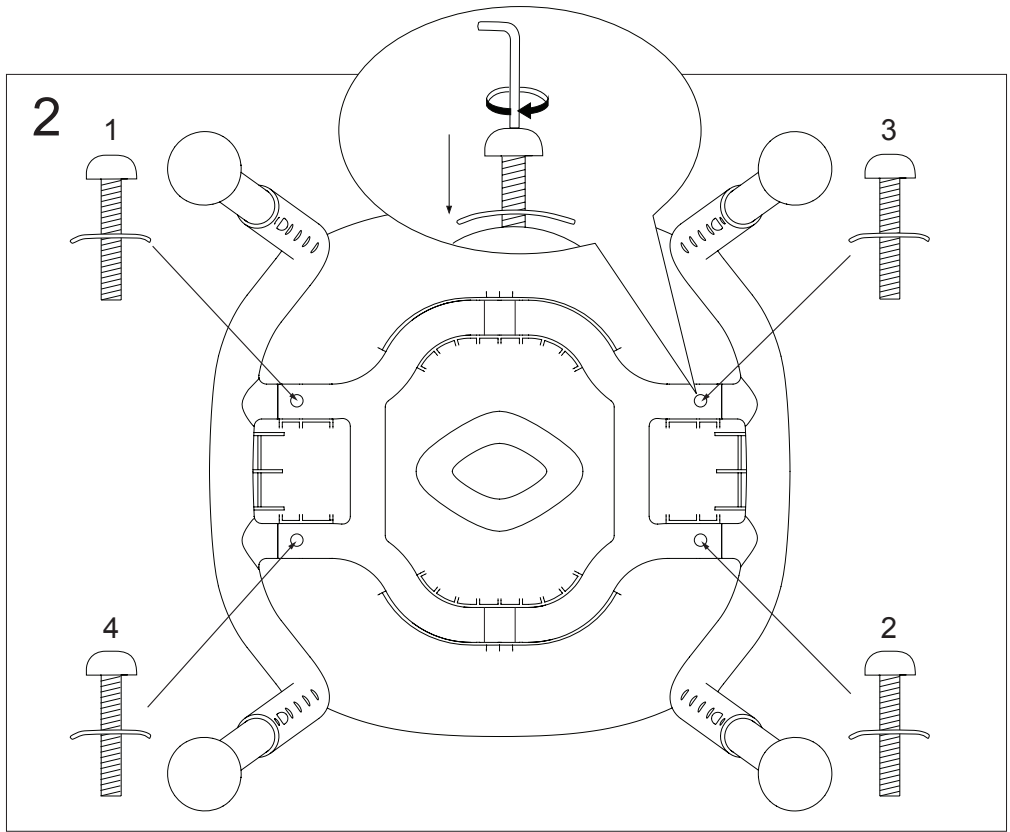Article No. LSH101

ENGLISH

Let's Shower is a new bathstool with a modern design to be used in the bathroom and shower. It has a beautiful and stable construction for safe and secure use. The height of Let's Shower can easily be adjusted to the desired position with the buttons on the legs.

Warning: A defective product must not be used.

Warning: Always adjust all of the legs to the same height to avoid falling accidents because of the stool becoming unsteady.

Warning: The stool is not intended for use in hot or humid places like a sauna. The stool should also not be exposed to direct sunlight for extended periods of time - this could damage the seat.

The seat can withstand temperatures up to 78°C. However the seat should not exceed 41°C when used in recommended settings. Extended exposure to direct sunlight and hot temperatures exceeding 41°C are not considered within the range of normal use.

Warning: The seat can be damaged if it is loaded with heavy or sharp objects. Do not stand on the bathstool. Do not sit on the edge of the bathstool. Do not overload the stool. Make sure the seat is not moving or rotating when standing up or sitting down. Do not remove the rubber feet. If any of the rubber feet are missing or in bad condition they need to be replaced before use again.

Warning: If there is any signs of corrosion on the product do not use it. Corrosion can make the stool unsafe to use. The stool should not be used if the frame is bent or damaged in any way. There must be no cracks or other damage to plastic parts. The height adjustment buttons must snap properly into the adjustment holes before use.

Persons with **cognitive impairment** should be assisted when using the Let's Enjoy bathstool.

A **.pdf version** of this user manual is available for download from the Trust Care website

www.trustcare.se This version can be scaled up if required and read on screen or printed out at a larger scale for use by **visually impaired** persons and those with **reading difficulties**.

The **expected lifetime** of this bathstool is 10 years.

Repair: Any required repairs must be carried out by skilled and qualified personal. Make bimonthly checks of the stool. Check for any damage and rust on all parts of the stool and its accessories and check for any missing parts. Use only original accessories and original spare parts from Trust Care. Spare parts can be ordered from Trust Care. Always make sure the legs are adjusted to the same height setting. If there are any missing parts when opening the box, please contact your reseller or Trust Care.

Technical data:

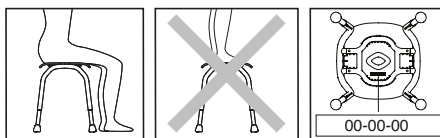
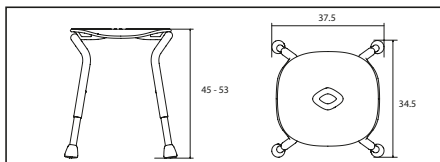
Material: Seat ABS, Tube frame aluminium, Feet TPE

Weight: 2,4 kg

Max user weight: 150 kg

Height: Adjustable from 45 to 53 cms.

Colour: Seat black, legs black, feet black

The stool is intended to sit on. Do not stand on the stool because of the risk of falling accidents.

Max user weight: Do not overload. Maximum user weight is 150 kg.

Cleaning: Use detergent without solvent, pH 5 – 9, or 70% disinfection spirit. Do not place any items in the dishwasher.

Decontamination: Use alkaline detergent pH min 6, in max 85°C/185°F for no more than 3 min.

Spare parts: Use only original accessories and original spare parts from Trust Care.

Återvinning / Genbrug / Recycling / Recycling / Recycling / Riciclaggio / Recyclage

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This product should be recycled in accordance with national guidelines.

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2 year warranty against defects in materials and manufacturing.

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ENGLISH

Let's Sing String basket.

An elegant string basket is available as an accessory for Let's Sing. Practical and easy to attach under the seat with a snap fastening. The string basket is good for storing schampoo or a bath sponge.

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