DA	Brugsanvisning.....	2
EN	Instructions for use.....	12
DE	Gebrauchsanweisung.....	22
SV	Bruksanvisningar.....	32
FI	Käyttöohjeet.....	42
NL	Gebruiksaanwijzing.....	52
EE	Kasutusjuhend.....	62

# Brugsanvisning

DA

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# Instructions for use

EN

## FIND PEACE AND BALANCE WITH THE RELOAD VEST

Congratulations on your Reload vest.

The Reload vest is a newly developed and well thought-out, high-quality aid in a stylish design and material, offered by ZiboCare.

Based on our many years of experience in the healthcare sector and sensory aids – and in close cooperation and dialogue with users, healthcare professionals and designers – we have successfully developed Reload - an innovation in the field of “sensory vests”. ZiboCare’s sensory products have all been developed based on internationally recognised theories of sensory integration and are well documented in research, documentation, testing and case studies

Always read the instructions for use carefully before using this product.

# IMPORTANT SAFETY RULES AND CONTRAINDICATIONS

Read all instructions before use.

The Reload vest may only be used for its intended purpose as described in these instructions for use. The product may not be combined, assembled or repaired with any other parts, accessories or anything other than is described in this manual or in other documentation provided by ZiboCare.

Always check that the product is intact before use. If the fabric is broken, do not use or wash the vest.

For the mentally handicapped or users who are unable to communicate their experiences with the Reload vest, the vest should only be used under supervision. Always be aware of the user's reaction to the vest.

The Reload vest may not be used near or in connection with fire sources/hot surfaces, such as an open fire or burning cigarettes. Smoking is not recommended when using the Reload vest.

The Reload vest may not be used for lying down and sleeping. Do not place the vest over your neck when lying on your back.

The Reload vest should not be used for children under 3 years old, as even the smallest size is too big for them.

Do not heat the Reload vest in a microwave or another heat source.

Be careful not to trip over the Reload vest if it is lying on the floor.

For users with weak lung capacity, the use of a tight, heavy vest may cause difficulty in breathing. For this reason, it is recommended that the use of the Reload vest is discussed with the responsible physician or another relevant healthcare professional before use.

Be aware of the risk of the vest getting stuck during play.

For complex issues, self-destructive and/or externalising behaviour, consult a doctor or a ZiboCare consultant before use.

For complex issues, self-destructive and/or externalising behaviour, consult a doctor or a ZiboCare consultant before use.

# HOW THE RELOAD VEST WORKS

## Purpose and use

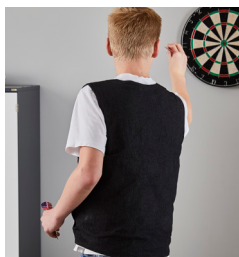
The Reload sensory-stimulating, unisex, weight and tactile vest is used to stimulate sensory and tactile nerve impulses, allowing the user to experience a good and enveloping effect as well as improved well-being and calmness.

The weight of the vest and the deep, dynamic stimuli from the ½-eggs affect and stimulate the user's tactile and proprioceptive senses, resulting in a calming and body-hugging effect. The ½-eggs are sewn in chequerboard squares with ½ of the arches bent towards the user, so that the stimuli change when moving.

The ½-egg-design gives the vest a smooth surface on the outside and the look of a regular vest.

The vest, equipped with internal adjustment straps, a zip closure and two pockets, is made in a smart and timeless design and in a practical and durable fabric.

## Examples of usage of the Reload vest:

# WHO WILL BENEFIT FROM THE RELOAD VEST?

Suitable for both children and adults, the vest can be used during daytime hours when you need to be able to gather yourself, find peace and concentrate. The vest can be used while sitting, standing or walking.

The straps on the inside of the vest allow you to customise the enveloping effect, so it can be precisely tightened and adjusted to suit your individual needs and preferences.

The Reload vest may help increase concentration and relieve motor restlessness and learning disabilities. This may be useful, for example, in institutions, at school during activities or seated work, during meals or communal activities, at home, during transport or in situations that may be overstimulating and feel unsafe or anxiety-provoking.

## **The vest can be used for:**

- Mental disorders, such as anxiety
- Motor or mental restlessness
- Concentration issues
- Racing thoughts
- Poor body image
- Attention disorders, such as ADHD and ADD Autism, Asperger's & Tourette's syndrome
- Dementias
- Brain Injury
- Failure to thrive
- Sleep issues
- Spasticity
- Neurological disorders
- Eating disorders
- General failure to thrive

The Reload vest can be used at any age (but not by children under 3) and by people with or without diagnoses. personer med eller uden diagnoser.

# HOW TO USE THE RELOAD VEST

For some, gradual adaptation to the vest may be necessary. Start by trying it on in a safe environment and situation and at shorter intervals. Gradually, the intervals at which the vest is used can be extended, and gradually the vest can be used in more challenging situations. For others, it will not take any getting used to and they immediately find using the vest comfortable from the get-go.

Healthcare professionals and educators assisting the user with the Reload vest should pay particular attention to the users' reaction if they are not able to clearly express their experiences with the vest. If the user is a child, be aware of the risk of the vest getting stuck in the immediate surrounding environment and in toys during playtime.

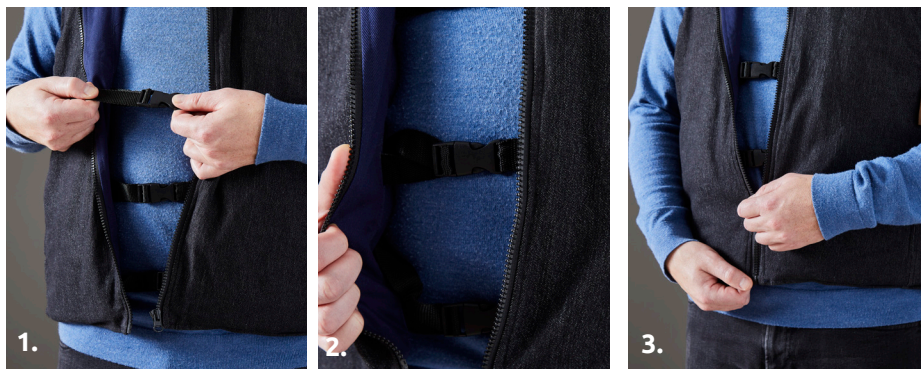
If a challenging situation is foreseeable, the vest may be put on before it arises, so that the effect will occur before the situation is at its most challenging.

The vest can be used before, during and after a challenging situation; for example, to calm down before a difficult and challenging task or situation is about to occur. The vest can be used during the challenging task or situation, or after the challenging task for its calming effect if the user is upset.

Some users feel that the effect of the vest wears off after about ½ hour of use, after which they take a break from wearing it. Others benefit from wearing the vest for several hours at a time.

Since it is a question of feeling and everyone has a different sensory system, the interval and frequency with which the vest can be used varies greatly among users. Individual users are recommended to test out the product and find out what works best for them.

### 1. Putting on the vest

Putting on and adjusting the vest. For users who, due to limited mobility, or other reasons, find it difficult to lift the vest over their shoulders and put their arms through the armholes, assistance may be required.

2. The vest is tightened by adjusting the inner straps and then closing the buckles. Maximum stimulation is achieved when the vest sits tightly around the body. However, as everyone has a different sensory system, preferences as to how tight and close the vest should be or feel are entirely individual.

3. The vest is then closed at the front with the zip.

The Reload vest can be worn under or over other clothing.

The Reload vest can be used in combination with the Reload collar, which adds additional weight and stimulation to the shoulder, the neck and the upper chest area. The collar is placed on the outside of the vest, as shown in the picture.

Further stimulation of the sense of touch and muscle joints can be added by using the O-Zone rug, which can be placed over the lap in the sitting position.

# RELOAD SIZE AND WEIGHT VARIANTS

Item	Variant	HMI no.	REF no.
Reload Vest	Child size S, 110/122, 1,1 kg	129541	7000247001
Reload Vest	Child size M, 128/140, 1,6 kg	129544	7000247002
Reload Vest	Child size L, 146/158, 2,1 kg	129547	7000247003
Reload Vest	Adult size XS/S, 2,5 kg	129551	7000247004
Reload Vest	Adult size M/L, 3,5 kg	129554	7000247005
Reload Vest	Adult size XL/XXL, 4,3 kg	129557	7000247006

## ACCESSORIES

### Transport and storage bag

The Reload vest comes in a practical and convenient bag, which makes it easy to take the vest with you everywhere; on weekend getaways, school camp, on holidays, etc. At the same time, the bag protects the vest from dirt and moisture.

### Wash bag

The Reload vest comes with a zipped wash bag, which you can use to put the vest in during washing.

# CLEANING, MAINTENANCE AND WASHING

## Washing instructions

The Reload vest is washed in the included wash bag in a regular washing machine. Need for washing depends on individual use and application. If the vest is passed on to another user, it should be washed and placed in a clean cover.

We recommend that you zip the vest before washing.

NB! Wash with similar colours as excess dye may affect other clothes. Excess colour fades over time as the colour changes and is covered by a patina after repeated washing.

- Always check that the Reload vest is intact before washing and that there is no damage or breakage of fabric or seams.
- In case of damage or breakage of fabric or seams, the vest is not washable.
- The Reload vest should be washed in a wash bag.
- The Reload cover does not require washing in a wash bag.
- The Reload vest should be washed on a gentle cycle at a maximum of 40° Celsius and must be dried flat.

## Maintenance

The vest should be aired regularly. Other than washing and maintenance as described above, the Reload vest requires no additional cleaning or maintenance.

# GENERAL INFORMATION

This product is a medical device and CE marked according to Regulation (EU) 2017/745 of 5 April 2017

The user and/or patient should report any serious adverse event that has occurred in connection with the device to the manufacturer and the competent authority of the Member State where the user and/or patient is established.

## Storage

For maximum protection of the Reload vest, store it in the enclosed bag, which protects against moisture and dirt. The Reload vest should always be stored in a dry place. Make sure the Reload vest is not lying on the floor where you might trip on it.

## Marking

The Reload vest is marked in accordance with EU legislation for Class I medical devices, including a product name, production date, item number, size/weight and CE marking. The label with relevant information, including care instructions, is sewn into the vest so that the information is always easily accessible.

## Warranty

The product is covered by a 2-year warranty for production defects pursuant to the Danish Sale of Goods Act. Ordinary wear and tear and malfunction are not covered by the warranty. The warranty is void in the following cases:

- Lack of or incorrect maintenance of the Reload vest
- Excessive use or misuse of the Reload vest
- Overload of the Reload vest

## Service life

The product has an expected service life of 5 years based on correct use, handling, washing and storage as described in the instructions for use. The service life may be shortened by frequent washing. It is the user's responsibility to continuously assess whether the product can be used beyond the stated service life.

## Recycling and disposal.

We care about the environment and work on the basis of the UN's world goals. In connection with disposal, the Reload vest can be split into the following individual parts, which can be recycled:

- Fabric
- Silicon dioxide

Waste is sorted and disposed of according to local regulations.

### Service, maintenance and repair

The Reload vest requires no service or maintenance other than as specified under cleaning, maintenance and washing.

If the vest has been damaged, please contact the retailer where the vest was purchased and they will assess and advise on potential repair.

### Returns and claims

In the event of a return or claim, please contact the retailer where the Reload vest was purchased. They will help you with the handling of the matter according to applicable business terms and conditions.

### Symbol legend

	Warning, see under important safety rules		Max. wash temperature 40°C. Delicate cycle
	Reference number		Do not bleach
	Manufacturer		Do not iron
	Date of production as indicated on the vest label		Do not tumble dry
	Lot no.		Do not dry clean
	Medical device		Do not heat in microwave, or similar device
	See instructions for use		Do not heat
	CE marked according to class 1, MDR 2017/745		Must be dried flat
	Sensory stimulant		

## TECHNICAL SPECIFICATIONS

Size and weight:	See product variations
Outer fabric:	46% cotton / 26% viscose / 26% polyester / 2% elastane
Filling:	100% polyester
Lining:	100% cotton
Weight elements:	100% silicon dioxide

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