
Mobilex Koala shopping trolley

Art. 319001

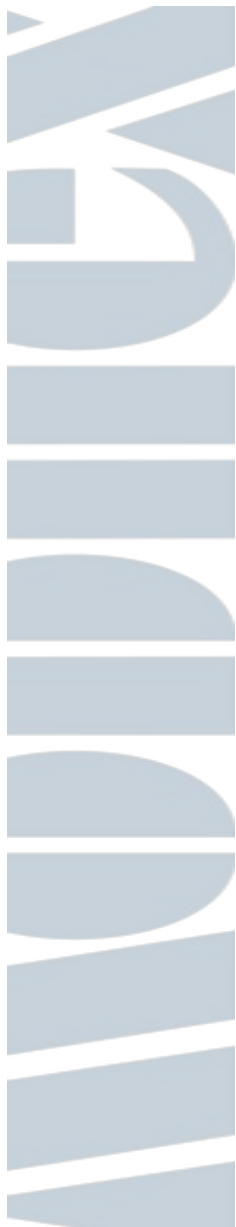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DEUTSCH

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DANSK

User Manual 8-10

ENGLISH

Manuale d'uso e manutenzione 11-13

ITALIANO

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1. Introduction

Thank you for buying a high-quality Mobilex product. This user manual contains a detailed product description and important guidelines for correct use of the product. You must read the instructions carefully before using the product. Please always observe all guidelines, in particular the security instructions, in order to use the product safely.

If you have any questions, please contact your dealer or Mobilex A/S

<p>Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark</p>	<p>Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk</p>
--	---

2. Security Instructions

- Use only the product if it is technically flawless
- In case of any product damage or faults please contact your dealer immediately
- Use the product only for the described purpose
- Please notice that there is a risk of injury at all joints on the Koala Shopping trolley
- Max. load of the trolley is 110 kg in total
- Max. load of the shopping bag is 30kg

3. Description

Product Height : 960mm
 Sitting Height : 650mm
 Product Weight : 2.5kg
 Weight Capacity : 110kg
 Material : Aluminum + Nylon (Trolley), Fabric (Bag)
 Color : Blu


Parts supplied:

- ✓ 1 Mobilex Koala Shopping trolley
- ✓ This user manual


Control upon delivery

After unpacking the rollator please check if all parts shown in the drawing have been delivered. If any parts are missing please contact your dealer immediately.

How to Install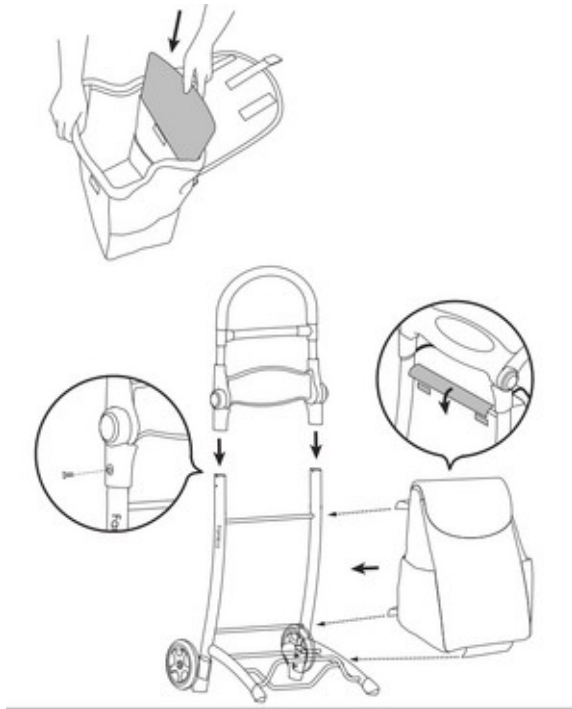

How to Use

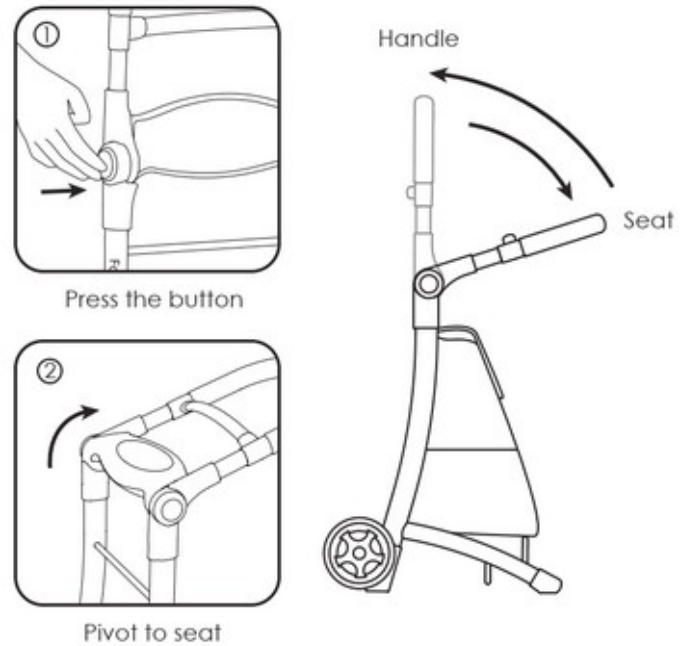4. WARNING

- Do not use the Mobilex Koala Shopping trolley as seat on slippery surfaces.
- Do not stand on the handle seat.
- Do not stand on handle whilst it's in the seated position.
- Check all screws are tight before using for the first time.
- Always keep the feet flat on the floor when sitting.
- Storage weight limit : 30kg.
- Seat user maximum weight : 110kg.
- Do not propel the Mobilex Koala Shopping trolley whilst seated.

5. Maintenance and care

Normal dirt on metal and plastic parts can be removed with normal cleaning agents. Check the contents of the cleaning agents and use only products suitable for cleaning and disinfection. Do not use solvents or abrasive cleaning agents

Despite the solid structure and use of resistant and anti-corrosive materials the trolley is subject to wear. It is therefore recommended to check the trolley regularly (once a year as a minimum). Such a check should comprise: A visual control of all parts of the frame: if they have been bent, damaged, worn or corroded and in particular a control of the following

- ✓ Rollator frame
- ✓ Push bars, handles
- ✓ Wheels
- ✓ Seat and locking device
- ✓ Shopping bag
- ✓ Check if any mounting parts are missing (e.g. nuts, screws, rivets ...) and if they function and are firmly tightened

leaning the wheels

It is recommended to clean the wheels in hot water with a mild cleaning agent. Lubricate the bearings with oil periodically or when the wheels start squeaking.

6. Disposal of the product

The product may not be disposed of with household waste but must be brought to the local recycling site.

7. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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Vid upptäckt av material- och produktionsfel, vänlig kontakta återförsäljaren omedelbart eller direkt till Mobilex A/S.