**Sicherheits- und Pflegehinweise**  
**Safety and care instructions**  
**Consignes de sécurité et d'entretien**  
**Indicazioni di sicurezza e manutenzione**  
**Indicaciones de seguridad y de cuidado**  
**Veiligheids- en onderhoudsinstructies**  
**Sikkerheds- og plejeanvisninger**  
**Säkerhetsanvisningar**  
**Sikkerheds- og vedlikeholdshenvisninger**  
**Turva- ja hoito-ohjeet**  
**Wskazówki dotyczące bezpieczeństwa i czyszczenia**  
**Bezpečnostní pokyny a instrukce k péči**  
**Руководство по эксплуатации**  
**安全上／お手入れ時のご注意**

acunis

 **ESCHENBACH**

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## English

### Safety instructions

- ▶ Danger of blinding or injury! This product does not protect your eyes when looking directly at the sun - it is not suitable for the direct look at the sun!
- ▶ The product does not protect against artificial light sources, such as used in tanning shops.
- ▶ The product must not be used as eye protection against mechanical hazards such as bumps, flying splinters etc..
- ▶ Protect filter glasses and clip-ons from knocks, strikes, moisture and excessive heat! Never place filter glasses and clip-ons on a heating element or in sunlight!

- ▶ Ensure that other people, especially children, are aware of these dangers!

## **Care instructions**

- ▶ Do not use any soap solutions that contain softeners, alcohol-based or organic solvents or abrasive cleaning products! These may damage the lenses.

Clean your filter glasses and clip-ons with a soft, lint-free cloth (e.g. spectacle cleaning cloth). Moisten the cleaning cloth to remove more persistent stains (e.g. fingerprints).

Filter glasses and clip-ons to protect eyes from sunlight comply with standard DIN EN ISO 12312-1 as well as regulations (EU) 2016/425<sup>(1)</sup> and 2017/745<sup>(1, 2)</sup> in their currently valid versions.

## acunis filter glasses<sup>(1)</sup>, filter clip-ons<sup>(2)</sup>

Filter

Filter category

acunis 25% (1615-325, -425, -625)

1

acunis 50% (1615-350, -450, -650)

1

acunis 75% (1615-375, -475, -675)

2

= Roadworthy by day. Not suitable for driving at night or in conditions with reduced brightness.

= limited attenuation of solar radiation

= good protection against solar radiation

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Eschenbach Optik GmbH