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MC 001-1732 [HMI: 49368] Master Turner Soft - **Large**

Størrelse / Dimensions / Größe:
L 215 x B/W 250

MC 001-2540 [HMI: 49370] Master Turner Two In One Soft - **Large**

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
Størrelse / Dimensions / Größe:
L 47 x B/W 4 cm


MC 001-2500 [HMI: 49873] Master Turner DUO Two In One Soft Bund / Bottom / Unterteil - **Large**

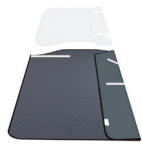Størrelse / Dimensions / Größe:
L 160 x B/W 145 cm

MC 001-2501 [HMI: 49872] Master Turner DUO Two In One Soft Top / Top / Oberteil - **Large**

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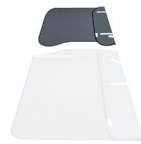Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantl

Do not use softener.

Use detergent without bleach and chlorine-containing compounds.

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WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care's** products with lowered safety rails

Never leave the user with master **care's** products attached to the lift or with lowered safety rails

IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- **Velcro straps must be closed by washing and drying**
- **During wash and drying, the washing straps MUST be applied to the velcro on the toppiece**

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These products comply with the standards for Class 1 products in the Medical Device Regulation (EU) 2017/745.

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