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1. Introduction

You have chosen a high-quality Mobilex product and we are especially pleased about that. This manual contains a detailed description of the product and important instructions for its correct use. Please read this manual carefully before using the product. Please observe all instructions, especially the safety instructions, and follow them at all times to ensure safe and continuous use of the product.

Intended use

The Mobilex rollator Tiger Outdoor is a walking aid for people who need support when walking due to their limited mobility but are still able to use the rollator without risking themselves or others. Thanks to the seat and the lockable brakes with parking function, the user can take a break and sit down at any time. Any other use other than pushing the rollator or sitting on it for a short time is prohibited. The rollator is tested and approved for both indoor and outdoor use.

Indication

The device is intended exclusively for people who require stable support when walking due to their state of health.

Contraindication

The rollator is not intended for people, e.g. with balance disturbances or insufficient muscle power. If you have any questions about the product, please contact your dealer or Mobilex A/S directly

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk
--	--

2. Safety instructions

- Only use the product if it is in perfect technical condition.
- If you find any damage or defects to the product, please contact your dealer immediately.
- Observe the information on the product sticker.
- Use the product only for the intended area of application.
- There is an increased risk of clamping and crushing at all joints of the rollator.
- Check the correct function of both wheel brakes before each start-up.
- Only use the rollator when it is fully open and with the folding mechanism locked.
- Only use the rollator seat with the brakes locked.
- Always pull or lock both brakes at the same time.
- The "Tiger Outdoor" rollator must not be loaded with more than 200 kg in total, the shopping bag with a maximum of 5 kg.
- While the rollator is being pushed, there must be no person, animal or other object on the seat.
- No changes may be made to the product without the written consent of the manufacturer.
- Make sure that the brakes are always in the parking position when you park and release the rollator before using it.
- The mounted back strap on the rollator is mainly for stability reasons and should always be mounted when the rollator is in use.

3. Description of the rollator

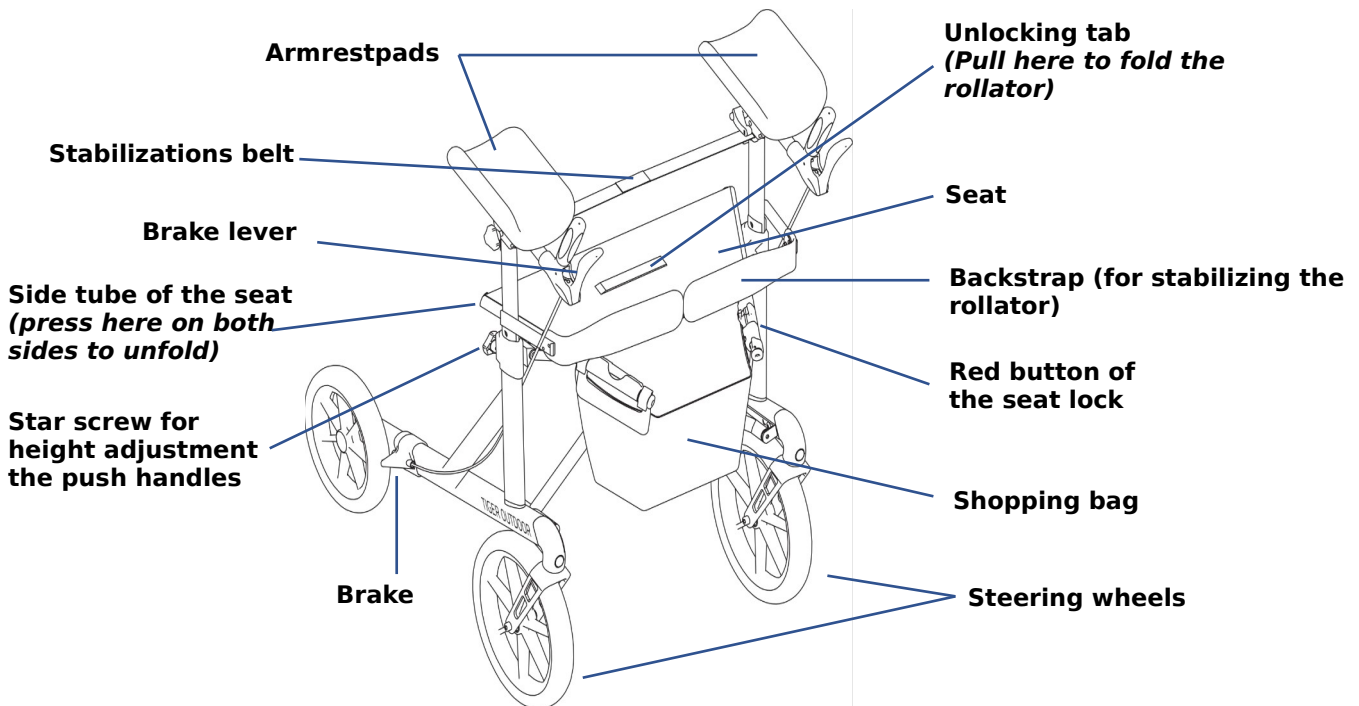
The Mobilex rollator "Tiger Outdoor" has a powder coated aluminium frame and is equipped with 4 wheels. Its aluminium frame is reinforced and allows folding, storage and transport. The rollator is easy to manoeuvre and offers the necessary stability. The push handles are height adjustable. In addition, it is equipped with a factory-fitted seat, a back strap and a shopping bag.

Scope of delivery and installation instructions

The Mobilex rollator "Tiger Outdoor" is delivered packed in a cardboard box.

Scope of delivery:

- ✓ 1 rollator, folded incl. seat, back strap and carrying bag
- ✓ This user manual

First check

After you have removed the device from its packaging, first check whether nothing is missing and whether all the parts visible on the drawing are present. If you notice that any parts are missing, contact your dealer immediately.

Unfolding the rollator

- Place the folded Rollator next to you.
- Press the red button on the seat tube lock (see picture on the right).
- Grip both seat tubes simultaneously and push them downwards until you hear a "click" sound from the locking system.
- The rollator is now unfolded and secured against unintentional folding.
- Close the belt that connects the two push handles (see picture below right) and the backstraps for stabilization. Check the opened walking aid to see if both belts are tightened.

Make sure that the seat tubes snap into place firmly on both sides when unfolding the Rollator. All parts must be firmly locked before the Rollator can be used. Failure to observe these warnings may result in injury.

Folding the rollator

- First loosen the belt that connects the two push handles.
- Pull up the release tab to collapse the Rollator.
- Fold the Rollator until it clicks into place. The Rollator is now folded.

Handles Assembly and height adjustment

- First loosen the belt that connects the two push handles.
- Loosen the screw for height adjustment.
- Position the handles at the desired height.
- Tighten the screw again to adjust the height.

Both handles should always be adjusted to the same height. Correctly adjusted handles should be approximately at the elbow height of the user when standing upright with loosely lowered arms.

Using the handbrakes

- To brake, pull both brake levers upwards.
- The more the brake levers are actuated, the greater the braking effect.
- To remain STOPPED, hold the brake levers tight.
- To MOVE the rollator release the brakes. They remain released automatically.

*Always activate the parking brakes when you are not using the Rollator!
 Never release the parking brakes when sitting on the Rollator!
 If you park the Rollator on uneven ground, the parking brakes must always be activated!*

Locking/releasing brakes

- To stop the Rollator, pull the brake lever upwards.
- To park, push the brake levers down until you hear a "click".
- The Rollator may only be used for sitting when the brakes are locked.
- Before you drive off again, stand in the direction of travel and release the brake levers by pulling upwards.

Always lock the brakes when parking.

Never use the seat without the brakes locked!

Never lean back when sitting. It is recommended that you only use the front seat area and do not sit on the bag or lean backwards.

Functional description

The folding frame allows the Rollator to be folded to save space. The push rods (handlebars) can be adjusted to your body height by the variable height adjustment. The seat allows you to take a rest.

Options / Accessories

Various accessories are available for the Mobilex Tiger Outdoor with the following item no. 312317, 312323, 31233D, 31234A, 31236A, 31236B. Please contact your dealer.

Handling during transport

It is recommended to transport the product folded. Its design allows easy folding and unfolding. During transport, make sure that the product is in a stable position without being able to move.

Application

Stand centrally behind the rear wheels, grasp the handles and release the brakes. The brakes are specially designed so that no extra force is required.

Mobilex arthritis rollator Tiger Outdoor	Antracite grey Item nr. 312205	Brown metallic Item nr. 312206
Height of the forearm support (adjustable)	87 - 110 cm	87 - 110 cm
Recommended user heights	140 - 185 cm	140 - 185 cm
Seat height	62 cm	62 cm
Total width opened / folded	71 cm / 31 cm	71 cm / 31 cm
Inner width between push handles	52 cm	52 cm
Length (depth)	77 cm	77 cm
Weight (+ bag 380 g)	12 kg	12 kg
Max. load (rollator / bag)	200 kg / 5 kg	200 kg / 5 kg
Brakes	2 lockable with parking function	
Wheel sizes	Front and back 200 x 35 mm	

Product label

The product label is placed on the lower part of the frame.

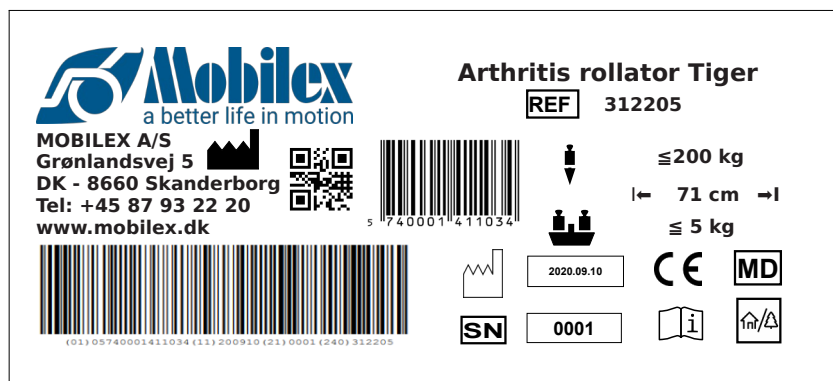
Illustration of the nameplate Tiger 312205

	QR-code link to homepage		Manufacturing date
	Manufacturer		Max load
	Observe the user manual		Max width
	To be used indoor and outdoor		serial no
	CE marked product		Item Number
	Max load basket		Medical Device
	Only to be used indoor		

Overview icon meaning

Materials

The rollator consists of corrosion-protected materials. The frame is made of powder-coated aluminium with various plastic parts, the handles are made of rubber and the seat and bag are made of tear-resistant nylon.

5. Maintenance and care

Normal dirt can be removed with standard cleaning agents and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (do not use alcohol-containing products and no solvents or abrasives).

The device basically does not need maintenance. Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following to be checked:

- ✓ Cleaning and disinfection of the product
- ✓ Frame/product should be checked if it has been bent, damaged, worn or corroded
- ✓ Wear parts are checked and replaced if necessary
- ✓ General retightening of the product

Recommended service life: Normally 10 years depending on use.

Cleaning the wheels

It is recommended to clean the wheels in hot water with a mild cleaning agent. Lubricate the bearings with oil periodically or when the wheels start squeaking.

Warning

If the product is exposed to very calcareous water, a binding and accumulation of lime on wheel bearings and other moving parts may occur. In this case, the product can be cleaned with a limescale remover and then lubricated with lubricating oil.

Warning

When used in highly chlorinated environments extra strong corrosion may occur and the warranty will be void.

Warning

Please note that if the frame is bent or broken, the product must not be recycled but must be replaced

Storage

We recommend storage in a dry place.

Transport recommendations

The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

Reuse

The product is suitable for reuse. Before reusing, where a new user is to take over the product, the product must be checked according to point 5.

6. Troubleshooting

<i>Problems</i>	<i>Possible Causes</i>	<i>Solutions</i>
The rollator is difficult to guide or push	Dirt between wheels and forks	Remove dirt or hair between wheels and forks
The brakes do not work or work only in one side	The brake shoes are worn, or the brakes are not adjusted correctly	Adjust the brakes. Replace the brake shoes if needed (→ service centre)
The seat does not fit in when the rollator is unfolded	The locking device is in a wrong position or defective	Check that the locking device is in a correct position. Replace it if needed (→ service centre)

7. Disposal of the product

The product may not be disposed of with household waste but must be brought to the local recycling site.

8. Guarantee

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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(Konformitätserklärung / Konformitetserklæring / Dichiarazione di conformità)

Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark,

Hereby declare under our sole responsibility as a legal manufacturer that the product specified on the product list below, meet the essential health and safety requirements and is in conformance with the provisions of the Regulation (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL on medical devices.

The product specified on the product list below is "technical aid for the disabled", classified as Class I, medical device. The classification is based on the requirements of Rule 1 of annex VIII, of the Regulation (EU) 2017/745.

The CE marking has been affixed on the product according to Annex V of the Regulation (EU) 2017/745.

PRODUCT LIST

Arthritis Rollator Tiger Outdoor- item no. 312205+312206

ACCESSORIES LIST

Item nr.	Accessories item nr.
312205-06	312317, 312323, 31233D, 31234A, 31236A, 31236B

Harmonized norms used during conformity estimation:

PN-EN ISO 11199-2:2008; EN ISO 14971:2011, EN 1041:2009

EU2017/745 for Medical Equipment
(über Medizinprodukte / for medicinalprodukter / del Consiglio Europeo)

In case of unintended incidents or suspected faults, failures or defects of the product, the dealer or Mobilex A/S must be contacted /

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