Rollator KUDU

Art. 312075, 312076 & 312077

Montage- und Gebrauchsanleitung
Montage- og brugervejledning
User Manual
Manuale d'uso e manutenzione
Installations- och bruksanvisning

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1. Introduction

Thank you for buying a high-quality Mobilex product. These user instructions contain a detailed product description and important guidelines for correct use of the product. You must read the instructions carefully before using the product. Please always observe all guidelines, in particular the security instructions, in order to use the product safely for a long time.

Intended use of the device

The rollator Kudu is a medical device which has been designed to increase the mobility of persons who have difficulties standing or walking. The rollator enables those persons to regain the ability to perform daily chores. The rollator is suitable for persons who as a result of various diseases need stable walking support. The rollator should only be used as a support for standing, walking and sitting. Any other use is prohibited.

Indication

The rollator is a medical device designed to help people with functional limitations. The indications for use are diseases that cause mobility limitations and disabilities, obstructing independent sitting, standing and walking. Therefore, there is a need for an additional stable support when performing the above activity. Indications cannot be related to one specific clinical picture and they depend on individual evaluation of potential user's health status by professional physician or physical therapist, but main target user groups are elderly people, persons with paresis, rheumatic diseases, and other people with limited mobility of the lumbar spine, knees or hips. Using the product for any purpose other than those described above is prohibited.

Contraindication

Rollators are not indicated for those persons who can't maintain proper body balance in a standing position. The device must not be used if the user has problems with impaired perception. Contraindications cannot be related to one specific clinical picture and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist. In addition, observe the safety information in Chapter 2.

If you have any questions, please contact your dealer or Mobilex A/S.

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Phone: +45 87 93 22 20
Telefax: +45 87 93 17 77
Email: info@mobilex.dk

2. Security instructions

- Use only the product if it is technically flawless
- In case of any product damage or faults please contact your dealer immediately
- Observe the data on the product label
- Use the product only for the described purpose
- Please notice that there is a risk of injury at all joints on the rollator
- Always check before use that the wheel brakes function perfectly
- Use the rollator only when it has been fully unfolded and the locking device has been activated
- The rollator should only be used on a flat stable surface, indoors or outdoors
- The seat may only be used when the brakes have been firmly locked
- Both brakes should always be activated or locked simultaneously
- Max. load of the rollator is 150 kg in total
- Max. load of the shopping bag is 5 kg
- It is not allowed to carry an extra person on the seat
- The product must not be changed without the manufacturer's written permission
- The rollator must always be braked when not in use

3. Description

The rollator Kudu have four wheels and is a light weight rollator with cross foldable frame. The product is made of aluminium, so it is light and easy to store or transport. The rollator offers great manoeuvrability, stability and it's push handles are height adjustable for a user fit. The product consists of a powder coated foldable structure frame made of aluminium tubing, and foldable cross. It is also equipped with fabric seat which allows user to rest and with shopping bag to carry small items.

The rollator is delivered in a box with contents:

- ✓ 1 Rollator
- ✓ 2 removable push handles with brake handle
- ✓ 1 user manual
- ✓ 1 shoppingbag
- ✓ 1 Crutch holder
- ✓ 1 Bike bell

Set up description

- Check with the list from above if nothing is missing. If you'd discover at that stage that some part is missing or damaged, do not continue assembly process. Contact your supplier of the product right away.
- Put the rollator with it's wheels on the floor.
- Grab both seat tubes and simultaneously push them down until you'll hear the "click" sound of locking mechanism of the frame in open position. Try to pull frame folding strip up to fold the frame. If it is impossible to do, the frame is opened and secured correctly. See the rollator in "OPEN" status in the figure above.
- Always ensure the frame is stable and fixed when the rollator is opened.
- Adjust both push handles to the same height. Usually the correct setting is adjusted if the handles are on the level of user's wrists when standing with arms down along the body.
- Loosen fixing levers located under the push handles. Then set the handles to required height. Both handles should be set to the same height. Tie the fixing knobs on the handle tubes.
- Make sure that the fixing levers (2 pieces) are securely locked so that the rollator is stable and parts can not move when forced.
- Always test to see that the rollator and attachments are properly and safely locked in place BEFORE using.
- Failure to follow these warnings can result in personal injury.

Folding description

- To fold the rollator press down with your right hand frame lock lever located under the right seat tube (see picture). At the same time pull the frame folding strip up. The rollator should fold almost automatically.
- You can secure the device in folded position with folded frame hook placed at the rear end of right seat tube. Attache loose end of the hook to the left seat tube. Before opening the rollator back you must unattach the folded frame hook.

Function description

- Adjust the height of the push handles so that when the user's arm is down along the body, the supports are at wrist height. Adjust the height by releasing the fixing knob located under the handle tube. Then adjust handles to your desired height, then tie the fixing knob securely. Repeat the steps for the other support, so both are at the same height.
- Care should be taken to ensure that ALL adjustments are secured and that castors and moving parts are in good working order before using.
- To stop the rollator simply pull the brake handles back towards hand grips and hold brake handles in the "BACK" position to remain in position. TO MOVE release the brake handles.
- To lock the brakes in "engaged" position push the brake handles forward until you can hear the "click" sound. To unlock the brakes pull the brake handles back until you can feel them loose.
- While using the rollator in a stationary position, the hand brakes MUST always be locked. You should always engage the breaks when you don't plan to move the device. Never leave device without breaks engaged when there is someone sitting in it! Never leave device on slopes without breaks engaged!

Transport recommendations

The rollator is an easily foldable product convenient for transportation. If it is necessary to transport it, we recommend to fold it as it is written in description above. Rollator should be transported in the way that it is stable and there is no risk of sudden and dangerous movements of the transported device.

Accessories

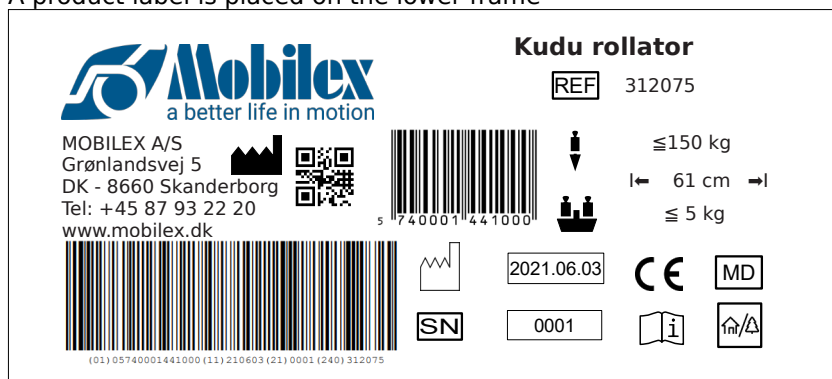
The rollator comes with two lockable brakes on the rear wheels, with tip-assist, a crutch holder and a bike bell. There are no other accessories provided for this device.

4. Technical data

Rollator Kudu	Standard Item no. 312075	Medium Item no. 312076	Small Item no. 312077
Height of the push handles (adjustable)	80 - 98,5 cm	79 - 88 cm	69 - 80 cm
Seat size (H x W x D)	62 x 42 x 20,5 cm	57 x 42 x 20,5 cm	50 x 42 x 20,5 cm
Size of the foldet rollator (H x W x D)	82 x 29 x 67 cm	79 x 27 x 68 cm	67 x 27 x 72 cm
Total width	61 cm	61 cm	61 cm
Inner width between push handles	45,5 cm	45,5 cm	45,5 cm
Length (depth) opened	67 cm	68 cm	70 cm
Weight	8,0 kg	7,9 kg	7,7 kg
Max. load (rollator / bag)	150 kg / 5 kg	150 kg / 5 kg	150 kg / 5 kg
Brakes	2 lockable with parking function		
Wheel sizes	Front and back 200 x 35 mm		

Product label

A product label is placed on the lower frame

Example of product label Kudu 312075

	QR-code link to homepage		Manufacturing date
	Manufacturer		Max load
	Observe the user manual		Max width
	To be used indoor and outdoor		serial no
	CE marked product		Item Number
	Max load basket		Medical Device
	Only to be used indoor		

Overview icon meaning

5. Maintenance and care

Normal dirt can be removed with standard cleaning agents and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (do not use alcohol-containing products and no solvents or abrasives).

The device basically does not need maintenance. Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following to be checked:

- ✓ Cleaning and disinfection of the product
- ✓ Frame/product should be checked if it has been bent, damaged, worn or corroded
- ✓ Wear parts are checked and replaced if necessary
- ✓ General retightening of the product

Recommended service life: Normally 10 years depending on use.

Cleaning the wheels

It is recommended to clean the wheels in hot water with a mild cleaning agent. Lubricate the bearings with oil periodically or when the wheels start squeaking.

Warning

 If the product is exposed to very calcareous water, a binding and accumulation of lime on wheel bearings and other moving parts may occur. In this case, the product can be cleaned with a limescale remover and then lubricated with lubricating oil.

Warning

 When used in highly chlorinated environments extra strong corrosion may occur and the warranty will be void.

Warning

Please note that if the frame is bent or broken, the product must not be recycled but must be replaced

Reuse

The product is suitable for recycling. Before recycling, where a new user is to take over the product, the product must be checked according to point 5.

6. Troubleshooting

<i>Problems</i>	<i>Possible Causes</i>	<i>Solutions</i>
<i>The rollator is difficult to guide or push</i>	Dirt between wheels and forks	Remove dirt or hair between wheels and forks
<i>The brakes do not work or work only in one side</i>	The brake shoes are worn, or the brakes are not adjusted correctly	Adjust the brakes. Replace the brake shoes if needed (→ service centre)
<i>The seat does not fit in when the rollator is unfolded</i>	The locking device is in a wrong position or defective	Check that the locking device is in a correct position. Replace it if needed (→ service centre)

7. Disposal of the product

The product may not be disposed of with household waste but must be brought to the local recycling site.

8. Guarantee

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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Hereby declare under our sole responsibility as a legal manufacturer that the product specified on the product list below, meet the essential health and safety requirements and is in conformance with the provisions of the Regulation (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL on medical devices. The product specified on the product list below is "technical aid for the disabled", classified as Class I, medical device. The classification is based on the requirements of Rule 1 of annex VIII, of the Regulation (EU) 2017/745.

The CE marking has been affixed on the product according to Annex V of the Regulation (EU) 2017/745.

PRODUCT LIST

Kudu Rollator Item number: **312075-77**

Kudu Rollator:

REF / item no.	312075	312076	312077
UDI-DI	5740001441000	5740001441017	5740001441024
BASIC-UDI-DI	57400014KUDUKN		

ACCESSORIES LIST

Item nr.	Accessories item nr.
312075-77	

Harmonized norms used during conformity estimation:

PN-EN ISO 11199-2:2005, EN ISO 14971:2012, EN 1041:2009

In case of unintended incidents or suspected faults, failures or defects of the product, the dealer or Mobilex A/S must be contacted /

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Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark, **SRN: DK-MF-000021885**