

**DANSK
ENGLISH**

**VEJLEDNING/
INSTRUCTIONS**

Elefanten The Elephant


FLEXIBLE HOOK SUPPORT
SYSTEM



100% DANSK
PRODUCERET I
DANMARK AF
GO OR STAY Aps



ENGLISH

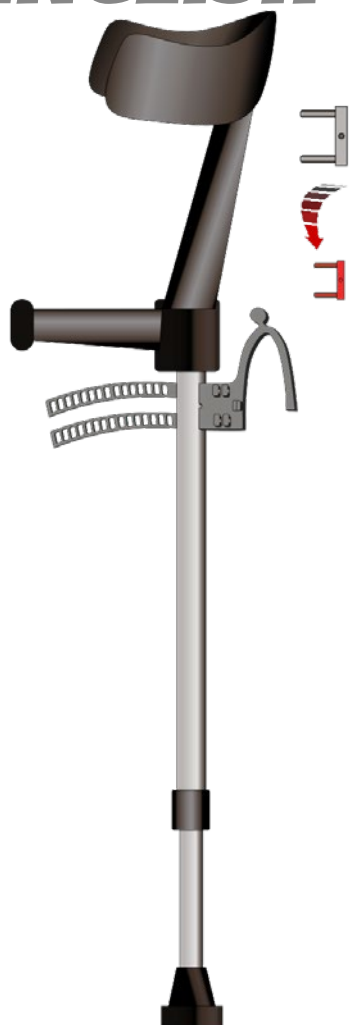


SCAN QR-CODE FOR
MORE INFO




www.goorstay.com

Patent Pending

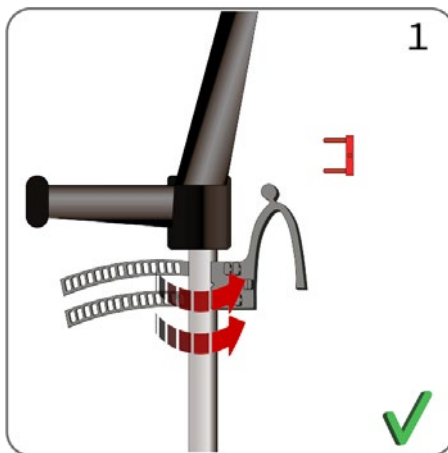


Step 1

Place the Elephant as shown and wrap the straps around the crutch/cane shaft so the holes align with the lock.

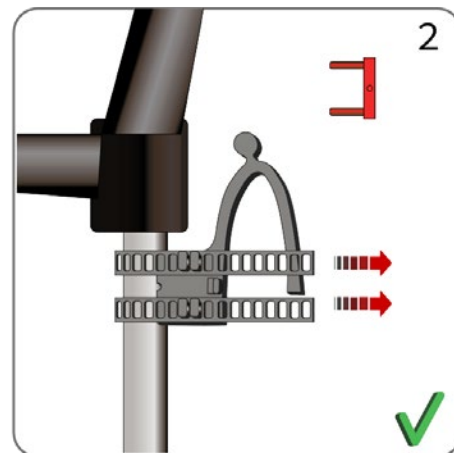


Step 2

Tighten both straps evenly by first placing the strap loosely and then pulling it 1 or 2 holes tighter.

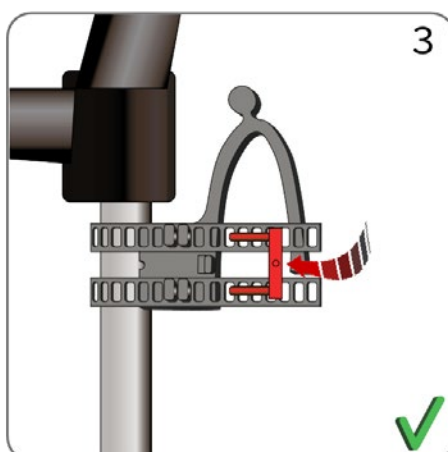


Step 3

Ensure the locking pin is inserted with the small nub facing upward.

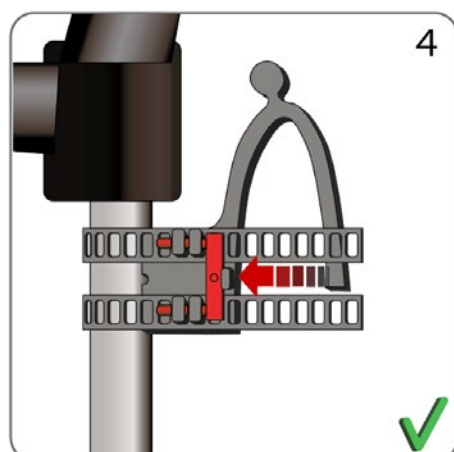


Step 4

When the pin is mounted as shown in the illustration, it is correctly positioned.

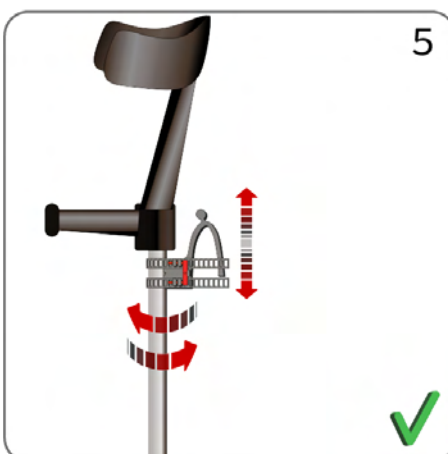

Step 5


Double-check that your Elephant is as tight as you want it- If it can rotate and slide up and down, it's positioned correctly. However, it should not be too loose. If it is too loose, repeat from step 2.

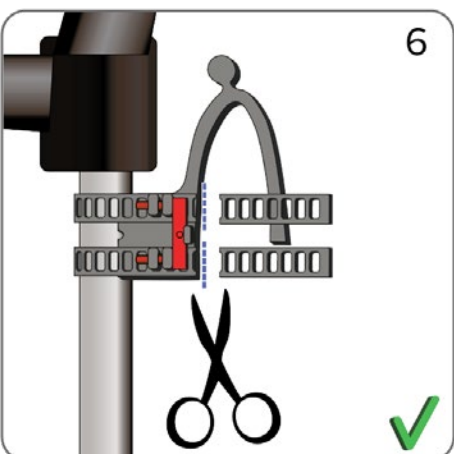

Step 6

When your hook is in the correct position, trim the excess ends. Cut through the hole in the strap as shown by the dotted line. The Elephant is now mounted on your cane or crutch.

DANSK

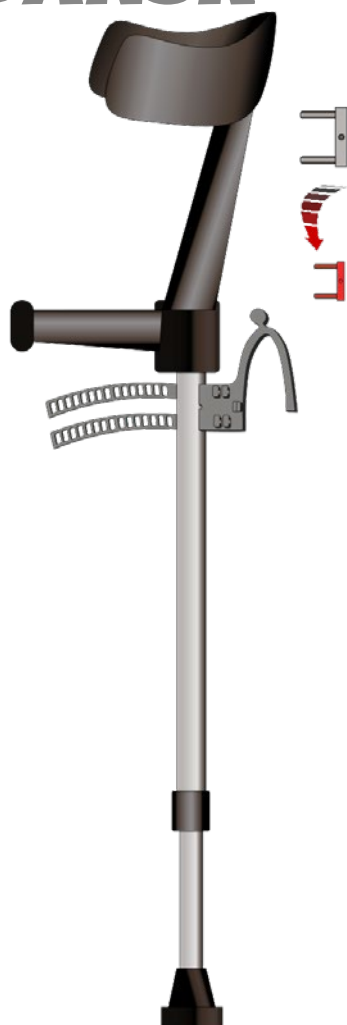


SCAN QR-KODEN FOR
FLERE
ONLINE VEJLEDNINGR




www.goorstay.com

Patent Pending

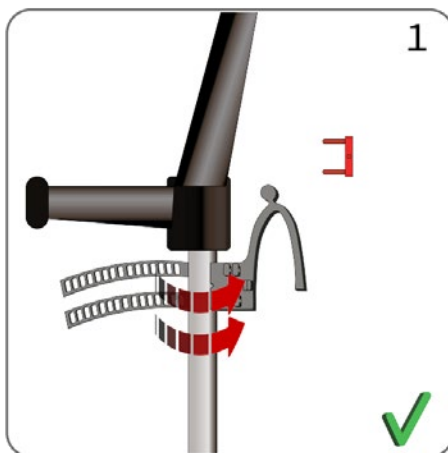


Trin 1

Sæt elefanten som vist og før remmene rundt om krykkens/stokkens stang, så hullerne passer ned over låsen.

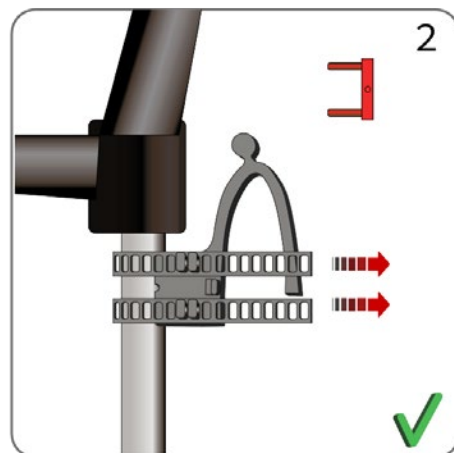


Trin 2

Stram begge remme lige meget, ved først, at sætte remmen løst på og dernæst trække det 1 eller 2 huller strammere.

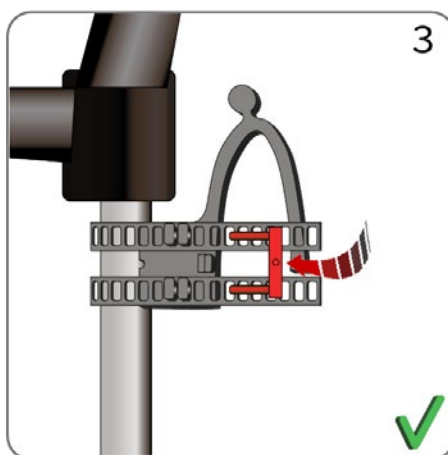


Trin 3

Sørg for at indsætte låsesplitten med den lille dut opad.

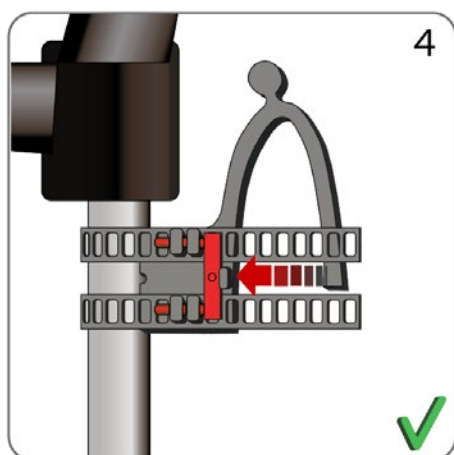


Trin 4

Når splitten er monteret som vist på illustrationen sidder den korrekt.

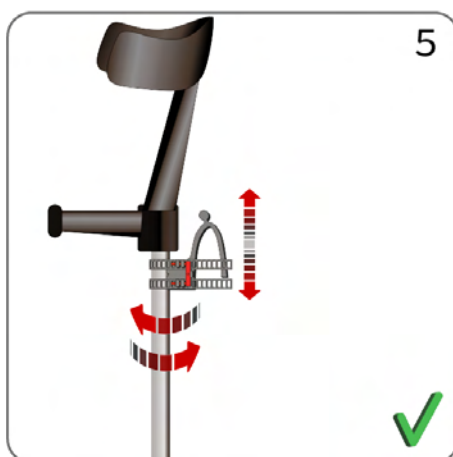


Trin 5

Dobbelt tjek om din Elephant sidder, så stramt som du ønsker. Kan den rotere og skubbes op og ned, sidder den som den skal. Den må dog ikke sidde for løs. Hvis den sidder for løs, gentag fra trin 2.

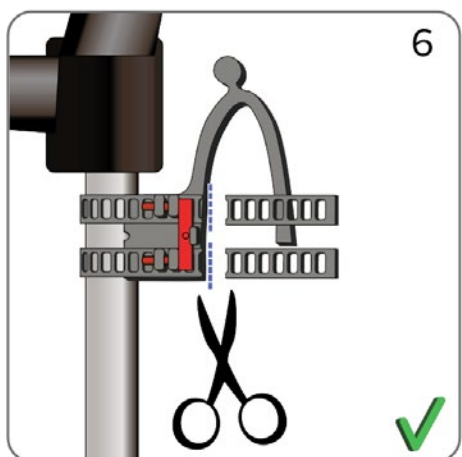


Trin 6

Når din krog sidder hvor den skal, klip det overskydende af enderne af, klip igennem hullet i remmen som vist med stiplede linje. Elefanten er nu monteret på din stok eller krykke.

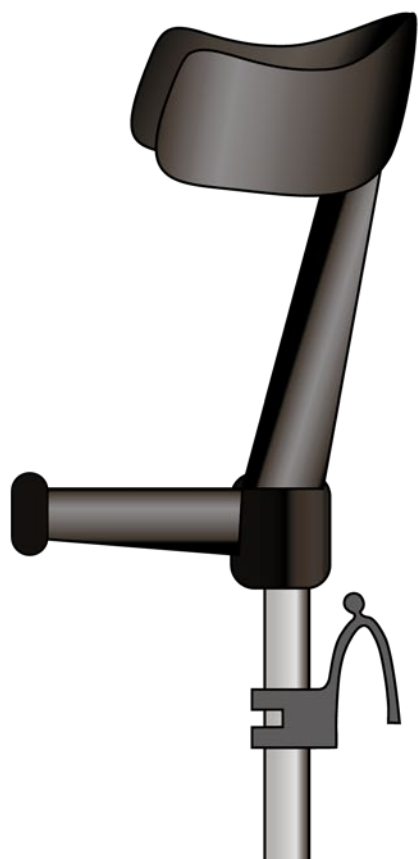

ALTERNATIV POSITION

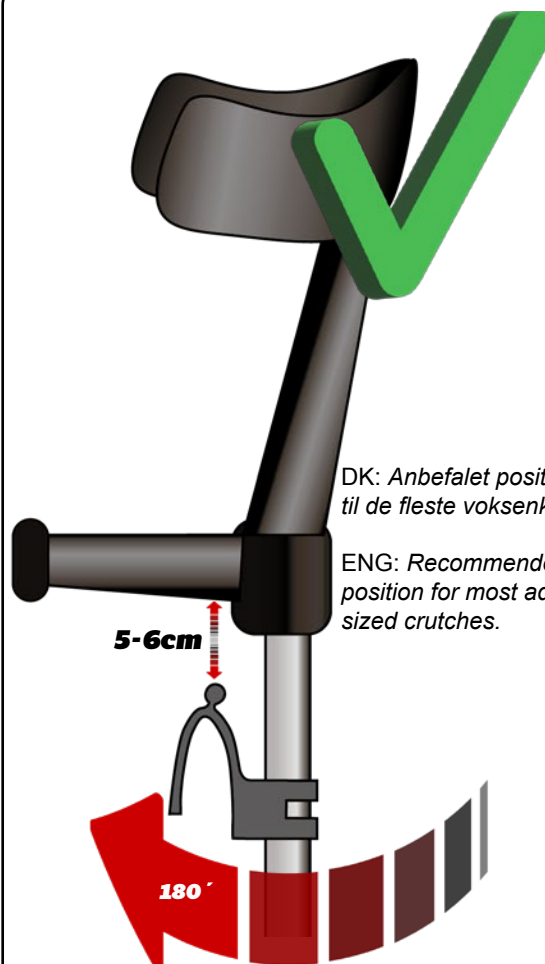

DK: Anbefalet position
til de fleste voksenkrykker.

ENG: Recommended
position for most adult
sized crutches.

5-6cm

180°

BEST POSITION

FUNKTIONER / FUNCTIONS



BORD ✓



VASK ✓



KØKKEN-
BORD ✓



STOL ✓



BILDØR ✓



Opsamlings-funktion
Pickup function

