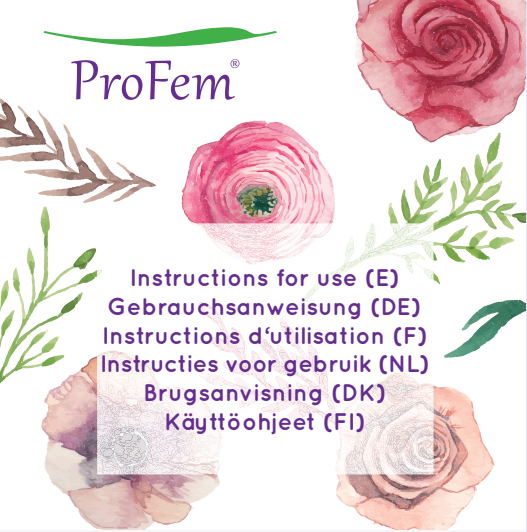
**Instructions for use (E)
for Silicone vaginal pessaries, type:**
Cube pessary, no Drains
Cube pessary, with Drains

Indications for use
Flexible silicone pessaries of the above types are used to alleviate symptoms of uterine prolapse (prolapse symptoms), cystocele or rectocele.

Contraindications
The use of pessaries is contraindicated in patients with acute infections of the genital tract, pelvic inflammatory disease or endometriosis.

Caution
This medical device should only be used on the instructions or prescription of a healthcare professional. Avoid any use of force when inserting or removing the pessary into the vagina. Improper use of pessaries can cause abrasion of the vaginal walls and ulceration. Pessaries are intended for use on one patient and are not supplied sterile.
The first time the pessary is used, the patient and doctor should insert it together. Clarify pessary use during sexual activity.

Application
Before fitting and insertion, the doctor will determine the correct pessary through a general pelvic examination. She/he should determine if the patient requires oestrogen therapy before prescribing a pessary.

Generally, it may be more comfortable for the patient to use a vaginal lubricant before inserting the pessary.
Pessary fitting kits are a valuable aid in choosing the right size and type of pessary.

When fitting a pessary, the doctor will ensure that the patient is instructed in cleaning, insertion and removal. The patient should have performed these tasks before the first follow-up visit. It should also be noted that any discomfort should be reported to the doctor immediately.

The doctor will ensure within 24 to 48 hours that the patient is not allergic to pessaries. This includes examining the vagina and asking if the patient has experienced any discomfort, irritation, pressure, tenderness or unusual vaginal discharge. It is determined whether there has been any improvement in the patient's personal symptoms.

Insertion
1. Familiarise yourself with the use of the pessary before using it for the first time.
2. Squeeze or fold the pessary (2) and gently insert it into the vaginal opening (3).
3. After the pessary has passed through the vaginal opening, release it so that it expands to its original shape.
4. Gently push the pessary towards the cervix (4).
5. When it is in the correct position, you should be able to insert a finger between the pessary and the vaginal wall. This distance ensures appropriate comfort and prevents pressure damage.
6. After insertion, the pessary should remain permanently in position when standing, sitting, kneeling or bending. It should not be uncomfortable for you to wear during any of these routine activities.
7. Should an allergic reaction, discomfort, irritation, pressure or unusual discharge occur after insertion of the pessary, the patient should consult her doctor immediately.
8. The cube pessary should be removed daily, preferably in the evening and re-inserted in the morning.

Remove
1. Loosen the pessary. Caution: When removing the cube pessary it is important to break the suction of the cube to easily remove it and to minimise risks to the vaginal wall. Use a finger to break the suction on each side of the cube. Then with one finger inserted between the vaginal wall and the cube, and, one finger through the loop, tilt the cube down to make it easier to remove.
2. To pull out the pessary, gently push it forward towards the vaginal entrance. 2. Schieben Sie zum Herausziehen des Pessars dieses vorsichtig nach vorne in Richtung Scheideneingang. Wenn das Pessar erreichbar ist, drücken Sie es zusammen und ziehen es vorsichtig heraus. (5)

Cleaning
For cleaning, the pessary should be removed regularly. To do this, clean the pessary only with soapy water that is not too hot and then rinse carefully with warm water.

Safety Control Check
Check the pessary regularly for cracks and damage! Replace the defective pessary with a new one immediately, as it can no longer be cleaned properly.

Recommendation
Patient and doctor should make appointments for follow-up examinations early.

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GUANGZHOU FAME MEDICAL CO., LTD. Second Floor A No. 8, Lianhua Port Industrial Zone, Lotus Mountain Bonded Area,