
Schine Walk user manual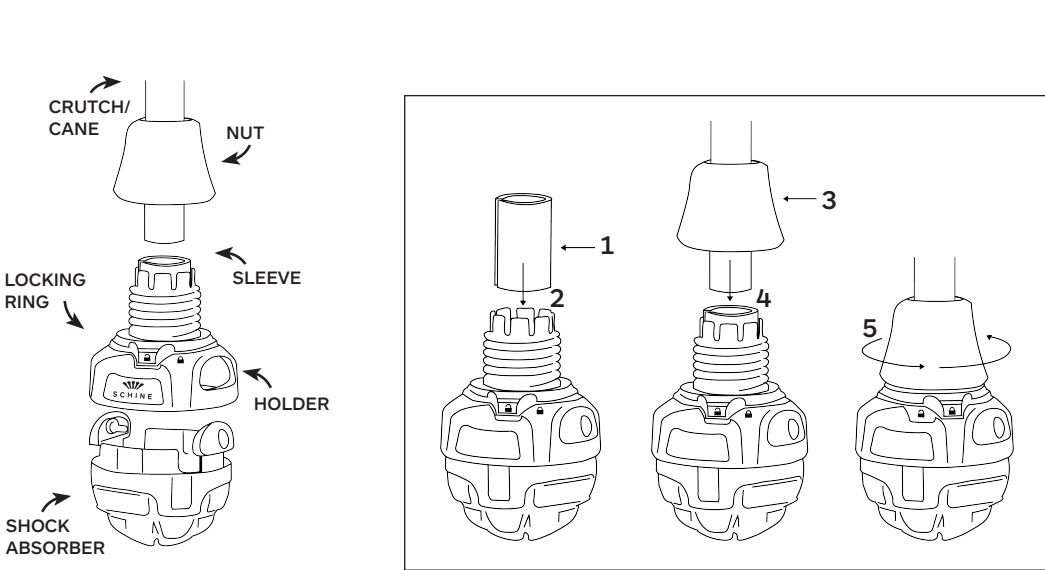


Schine Walk parts

A - Installation


Watch
installation
video:

More info: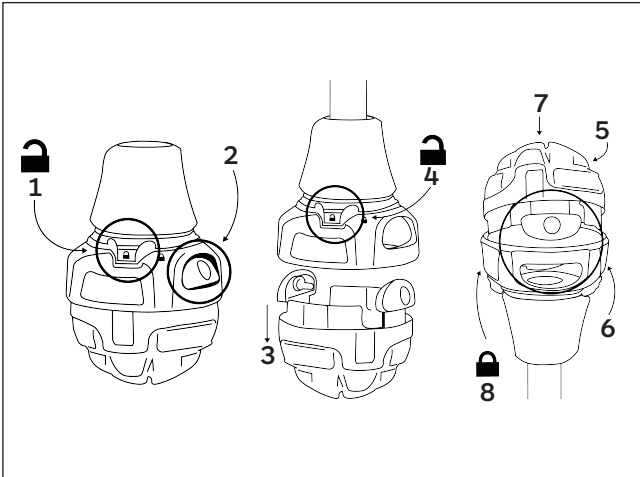

B - Shock absorber replacement

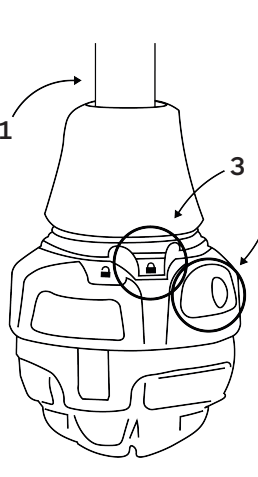C - Safety check

English

Read this instruction manual before using Schine Walk. It contains important information for you. Follow the instructions and keep the user manual. You may need to refer to it again. Schine Walk is a crutch tip with cushioning that can be mounted on most crutches and canes with shaft diameters of 17 mm, 19 mm, and 19.5 mm. It offers a smoother and more comfortable walking experience and can help reduce the impact on hands, shoulders, and neck. The shock absorber's wide contact surface and high friction offer an excellent grip on various surfaces. Also available with spikes for winter use.

A - Installation

Note that Schine Walk extends 6 cm from the end of the shaft to the ground. The length of the crutch/cane must be adjusted accordingly. A locking ring is located on the holder. Two padlock symbols indicate the positions for locked and unlocked. When the locking ring is open, the shock absorber can be replaced. Always turn the locking ring to the locked position before using Schine Walk. Measure the diameter of the shaft to choose the correct sleeve. (See dimensions on the packaging).

- Choose sleeve A (thin) for Ø 19.5 mm and Ø 19 mm or sleeve B (thick) for Ø 17 mm.
- Press the sleeve down to the bottom of the holder.
- Thread the nut onto the shaft.
- Push the crutch down to the bottom of the holder.
- Screw the nut onto the holder.

B - Shock absorber replacement

REMOVE THE SHOCK ABSORBER
It helps to turn Schine Walk upside down.
1. Turn the locking ring to the unlocked position.
2. Press the snaps in, one at a time.
3. Release the shock absorber by pulling it out of the holder.

INSERT THE NEW SHOCK ABSORBER
4. Note: The locking ring must be in the unlocked position.

- Insert the shock absorber at a slight angle.
- Center the color code in the hole of the holder.
- Press the shock absorber firmly from above.
- Turn the locking ring to the locked position. See the image C – Safety check, item 3.

BE CAREFUL!

Although the Schine Walk shock absorber provides high friction and good grip, be alert to potential hazards that may occur, especially on wet, slippery, or uneven surfaces. Shock absorbers with spikes should only be used outdoors, as there is a risk of slipping indoors. Before use, make sure that the crutch tip is intact, clean, and properly assembled. Do not store Schine Walk near a heat source. **Serious events related to the product must be reported to Milmas AB and the MHRA (Medicines and Healthcare products Regulatory Agency).**

C - Safety Check

- Make sure that the Schine Walk is securely attached to the crutch by pulling on the crutch tip.
- Ensure that both snaps have engaged in their designated holes.
- The locking ring must be in the locked position. Schine Walk is locked when the locking ring aligns with the ridge of the holder.

Check Schine Walk regularly. Replace the shock absorber when the damping effect has decreased or if it is worn or damaged. If the holder is loose, replace the sleeve.

PRODUCT FACTS

- The crutch tip consists of a holder and a replaceable shock absorber.
- Large contact surface.
- High-friction shock absorber.
- 2 shock absorber options, medium and firm.
- Studded shock absorber for winter surfaces.
- Reinforced bottom in the holder.
- Maximum user weight 140 kg.
- Clean with water and dry thoroughly before use.

SPECIFICATIONS

Weight: 89 g
Length: 95 mm
Width: 56 mm
Height: Mounted it extends by 60 mm and 19.5 mm
Fits shafts with a diameter of: 17 mm, 19 mm and 19.5 mm
Material: Recyclable plastic
Origin: Made in Sweden
Patent
Basic UDI-DI: 735007494SWUP

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