ThevoCare

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Dear Customer,

Thank you for choosing our product and for the trust you have placed in us. In order for you and the user to enjoy your new product safely, conveniently, and comfortably, we kindly ask that you read the Instructions for Use. Doing this will also help avoid damage, injury, and incorrect operation and handling of the product. Please ensure that a copy of the Instructions for Use remains with the product so that every user has access to it.

If you have any questions, please contact your authorised distributor or us directly.

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Phone: +49 (0) 4761 8860

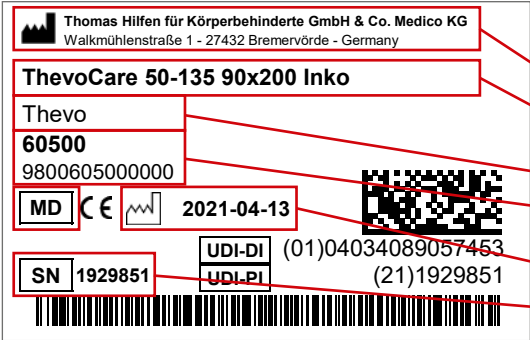
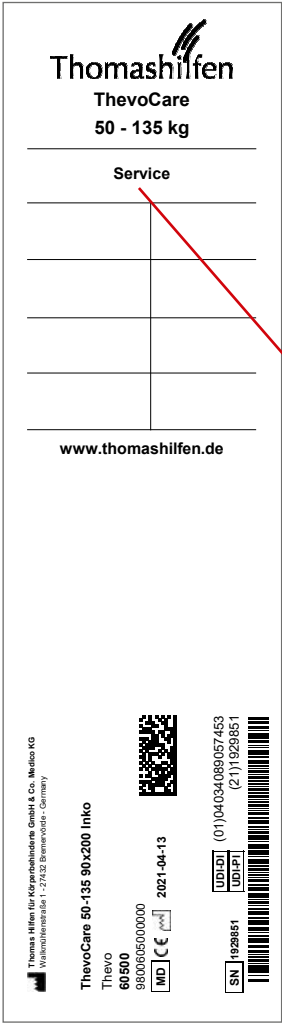
www.thomashilfen.de

We reserve the right to make technical changes. This can lead to differences between your product and how it is presented in brochures and other media.

Table of Contents

Figures	3
Preface / Manufacturer	12
Labels / Model Identification	13
General Remarks / Safety Instructions	14
Intended Purpose	14
– Intended Use / Application Areas	14
– Indication / Patient Target Group / Intended Users	14
– Risks Associated with Use and Contraindications	15
Product Description	15
– Scope of Delivery / Standard Equipment	15
– Options	15
– Putting into Service	15
– Teaching the User	15
Adjustment Options / Features	16
– Accessories	16
– Technical Data	16
– Material Specifications	16
Cleaning, Maintenance, and Repair	17
– Cleaning, Care Instructions, and Disinfection on a Daily Basis	17
– Maintenance and inspection by the distributor / Maintenance Schedule	17
– Spare Parts	18
– Storage / Disposal	18
EU Declaration of Conformity	18
Warranty	18
Period of Use / Service Life	19
Reuse	19
Distributor	19

Labels / Model Identification

Label	Meaning
	Barcode Label Position on the Product (Fig. 1) - Manufacturer - Product Name - Product Group - Item Number short / long - MD = Medical Device - Manufacture Date - Serial Number
	Maintenance Label + Barcode Label Position on the Product (Fig. 1) The maintenance label documents who did the regularly scheduled maintenance work and when. - Mattress (Fig. 1)

Remarks

General Remarks

- Read the Instructions for Use carefully before first use. If you have difficulty reading, please have them read to you. If the copy of the Instructions for Use is missing, you can download one from www.thomashilfen.de. The contents of the digital Instructions for Use can be enlarged.
- Check the packaging for any transport damage and to be sure the product is in flawless condition.
- Changes to the product are only permitted within the scope of the intended adjustment options.
- Cleaning and daily care can be provided by relatives and caregivers. Maintenance and inspection are done by the distributor.
- All labels have to remain on the product and should not be tampered with.
Exception: The maintenance label has to be updated and completed after each maintenance.

Safety Instructions

- Please do not overload the product and note the maximum user weight and the maximum load capacity (see technical data).
- The product should only be used by one user at a time. No other individuals should be placed in or on the product.
- Heavy perspiration, high humidity, and extreme changes in temperature can have an impact on the foam in the mattress. Pay attention to changes such as sagging in the mattress.
- If there is a serious and unexpected accident with the product, please inform our company immediately and the responsible authorities.

Intended Purpose

Intended Use / Application Areas

The ThevoCare micro-stimulation system is a class 1 medical device.

The system is suitable for use in home care and in professional care settings. It consists of a mattress with integrated wing suspensions which promote natural movement and the user's body awareness. The system allows users to lie a specific way; it can be complemented as required by other positioning aids. The foam core and its fine wave structure create a pleasant and soft lying sensation. Special air channels in the foam core allow for very good air circulation so that moisture can be removed.

The system replaces a standard mattress and goes into existing bed frames with the reclining surface and is also used in beds in care settings. The adjustment options of the existing bed frame are not limited by the micro-stimulation system. The system does not replace manually changing positions, but the system can lengthen the lying intervals. It is suitable for use in combination with digital assistance systems in care settings.

Any use that goes above and beyond this does not comply with the intended use.

Indication / Patient Target Group / Intended Users

The micro-stimulation system is suitable for adults and adolescents with limited mobility, a circumstance that adversely impacts changes or shifts to body position. Furthermore, the product is used if the user is at risk of having pressure sores on the skin and subcutaneous tissue, such as on the back, buttocks, or heels, and if pain is experienced when lying down.

This can definitely be used if there are illness or age-related difficulties falling asleep, sleep disturbances, and impaired body awareness during sleep.

Risks Associated with Use and Contraindications

If used properly, risks can be eliminated during use. There are no known contraindications.

Product Description

Scope of Delivery

Your product is delivered with the standard equipment and comes preassembled. A copy of the Instructions for Use is enclosed with the product.

Standard Equipment

Mattress core; wing suspensions; cover (jersey or incontinence)

Options

Upon request the product can be made in the following sizes:

- Width options from 70 cm to 140 cm (27.6" to 55.1")
- Length options from 180 cm to 220 cm (70.9" to 86.6")

The mattress is available for the following weight categories:

- **ThevoCare 50** – up to max. 50 kg patient weight (110.2 lbs)
- **ThevoCare 135** – from 50 kg to max. 135 kg patient weight (110.2 to 297.6 lbs)

If the patient is in the upper end of a weight category (+/- 2 kg / 4.4 lbs), the next higher weight category should perhaps be chosen.

If the patient's body has an atypical distribution of weight, the next weight category should perhaps be chosen.

Putting into Service

- Carefully remove the transport protection and the packaging.
- Keep the packaging material and small parts out of the reach of children to prevent swallowing or choking.
- The product can be used in any bed. Put the mattress with the red side up onto the slatted frame or your bed's supporting structure.
- The mattress can also be used without any slatted frame. In this case, there has to be a support surface of at least 13 cm / 5.1" in width on the long sides.
- Please note: For beds with side rails, side rail risers might be need.

Teaching the User

The user is taught how to use this device by trained staff who will reference these Instructions for Use.

Adjustment Options / Features

Accessories

Extra mattress cover (Fig. 2)

An extra mattress cover made of jersey or incontinence material is available if the original mattress cover has to be cleaned or discarded.

Rescue belt system (Fig. 3)

With the rescue belt system bedridden patients can be removed quickly from danger. The rescue belt system is integrated into the cover of the mattress. The patient is secured by two hook and loop safety belts (at the chest and thigh). In an emergency, the hook and loop belts are pulled over the mattress, bedding, and patient and are connected. This way the patient and the bedding are secured quickly and easily to the mattress. The patient's arms are under the covers so that they are not injured during transport.

Using the pull straps at the top and foot of the bed the mattress, patient, and bedding are pulled out of the bed at a 90° angle. This way the patient can be evacuated horizontally and vertically from danger. The mattress slides very easily on all floor types. There are two lifting straps on each long side which allow emergency personnel to take the patient without transferring them from the bed.

The fabric of the belts is 100% polyester and is free of chemicals. This makes them non-irritant and also suitable for patients with allergies.

Maximum load capacity: up to 150 kg / 330.7 lbs

All users have to be familiar with how the product works. This way in an emergency the user can be removed quickly from the product.

Technical Data

ThevoCare 50

Overall dimensions (w x l)	90 x 200 cm / 35.4 x 78.7"	100 x 200 cm / 39.4 x 78.7"
Height	15 cm / 5.9"	15 cm / 5.9"
Weight	depending on option approx. 20 - 25 kg / 44.1 - 55.1 lbs	
max. user weight	50 kg / 110.2 lbs	50 kg / 110.2 lbs

ThevoCare 135

Overall dimensions (w x l)	90 x 200 cm / 35.4 x 78.7"	100 x 200 cm / 39.4 x 78.7"
Height	15 cm / 5.9"	15 cm / 5.9"
Weight	depending on option approx. 20 - 25 kg / 44.1 - 55.1 lbs	
max. user weight	135 kg / 297.6 lbs	135 kg / 297.6 lbs

Material Specifications

The open-pored and flame retardant cold foam provides moisture absorption and release and thermal regulation making the foam highly durable. The covers are flame retardant.

Cleaning, Maintenance, and Repair

Cleaning, Care Instructions, and Disinfection on a Daily Basis

Foam Mattress

The mattress can be autoclaved making it suitable for reuse.

Jersey Cover

The jersey cover is breathable and can be easily unzipped and removed for washing. Please also note the following care instructions.

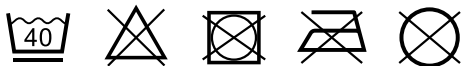
Incontinence Cover

The incontinence cover can be cleaned by wipe-down disinfection. The incontinence cover can be easily unzipped and removed for washing. Please also note the following care instructions.

Protective Cover

The black cover can be easily unzipped and removed for washing. Please also note the following care instructions.

Disinfection

The mattress and suspension framework can be cleaned by steam vacuuming. This has to be done to disinfect and before each reuse.

We recommend two disinfecting methods recognised by the Robert Koch Institute:

- the fractionated vacuum method (vacuum-steam vacuum method) up to a maximum temperature of 110° C.
- the steam flow method up to a temperature of 110° C.

Important note: Thermal treatment above 110° C leads to premature ageing and deforming of the plastic parts which is why it shouldn't be done. When using chemical disinfectants always follow the information from the manufacturer to ensure they are suitable for plastics.

Maintenance and inspection by the distributor

Preparation

A copy of the Instructions for Use has to be enclosed with the product. If one is not available, request a copy from the distributor. Familiarise yourself with how the product works. If you are not familiar with the product, read the Instructions for Use before testing it. Clean the product before starting to test it. Please note the care instructions in the Instructions for Use and product-specific test instructions. If you have further questions, please contact the staff member you dealt with or Thomashilfen directly.

Service and repairs to the product should only be done by qualified staff. It is the user's responsibility to ensure that the prescribed inspections / maintenance are done regularly and in a timely fashion. If damage is apparent, the user has to actively inform and have the damage fixed immediately by qualified staff.

Maintenance Schedule – Annual Inspection

	Area	Function		Damage		Replace	
1	Cover	👁 / 🔄		👁 / 🔄			
2	Foam core	👁 / 🔄		👁 / 🔄			
3	Wing suspensions	👁 / 🔄		👁 / 🔄			
4	Labels			👁 / 🔄			
done by:				on:			
👁 = Inspection 🔄 = Reuse ✔ = Test completed							

Spare Parts

Only original spare parts should be used for repairs. A list of the spare parts is available at www.thomashilfen.de. To receive the right spare parts or for possible technical recalls, we normally require the serial number / UDI-PI and the exact UDI-DI.

Removal / installation of spare parts should only be done by qualified staff! In the event of returns, please make sure the product is securely packaged for transportation.

Storage

Store the product in a safe place when it is not being used. Do not lay or put any heavy objects on the product while it is being stored. Protect the product from weather like rain, intense sunlight, and frost. If the product is wet, wipe the metal and plastic parts dry and let the padding dry before storing the product.

Disposal

If the product is no longer usable due to its condition or it has reached the end of its service life, it can be recycled at the local waste disposal company. Please refer to the local guidelines for recycling and disposal.

EU Declaration of Conformity

We hereby declare as the party solely responsible that this product and any versions and accessories comply with the basic regulations. The current EU Declaration of Conformity can be downloaded at www.thomashilfen.de or requested from us directly.

Warranty

Thomashilfen offers you the two-year warranty as required by law for this product. The warranty period starts when the item is handed over. The warranty includes all claims which impact function. Not included is damage which is the result of improper use (e.g. overloading) and natural wear and tear.

Warranty conditions

The company Thomashilfen vouches for the quality of the product delivered. The choice of material and the workmanship are based on the highest production standards. All ThevoCare products undergo a thorough final inspection. Thomashilfen grants a ten-year guarantee on its elastic winged springs.

This ten-year guarantee shall apply also to the ThevoCare mattress core. Consider the mattress core as being as good as new for the first two years. In the event of any guarantee claim thereafter, we shall then have to deduct from the purchase price a 10 % reduction in value for every year of use. Please understand, however, that the statutory warranty period shall apply to the mattress covers.

A three-year guarantee shall apply to all defects in material or manufacturing. Thomashilfen reserves the right to dimensional differences within a tolerance of up to ± 2 cm in the case of upholstered parts. Damage caused by improper handling, misappropriation, subsequent alterations or normal wear and tear shall be excluded from the warranty. Unfortunately, assembly and transportation costs incurred after the statutory warranty period has expired shall have to be charged.

Unfortunately, assembly and transportation costs incurred after the statutory warranty period has expired shall have to be charged. Therefore, please keep in a safe place all documentary records received in connection with ThevoCare.

Important hint: ThevoCare is autoclavable and may be re-used. ThevoCare should be cleaned and disinfected using one of the cleaning methods stated on page 17 with every change of user. Servicing by the manufacturer is not necessary.

Period of Use

The product has a period of use of up to 10 years if used correctly and serviced regularly and professionally as per the information from the manufacturer. The product can be used beyond this point if it is in safe and technically sound condition. Time spent in storage at the distributor does not count towards the period of use.

The theoretical period of use does not equate to a guaranteed service life. It depends on the intensity of use and always undergoes an individual assessment by the distributor.

The period of use does not apply to wear parts like covers, the mattress core and so on.

Service Life

The product has a service life of 10 years or 2 reuses (3 users) if the requirements stipulated for the period of use are met. The service life consists of the period of use and the storage times between reuses.

Reuse

The product is suitable to be reused. Before being passed on to the next user the product always has to be carefully cleaned and disinfected. The condition has to be checked by trained staff for any wear and damage. Any issues have to be remedied to ensure the continued safe use of the product. A copy of the Instructions for Use is a component of the product and has to be passed on to the new user.

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O'Neill Healthcare Ltd.
Thomas Hilfen für Körperbehinderte GmbH & Co. Medico KG