Nederlands: Functionele 4-Punts Knie Orthese
Handleiding

Indicaties

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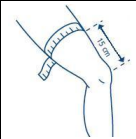
BraceID[®]

BraceID[®]

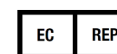
Functional Knee Orthosis 4-Point Principle ACL / PCL / MCL / LCL

		Code No. Artikelnr. Référence Art. Nr.	Size Maat Taille Größe
Left Links Gauche Links	39 - 47 cm	12-0022-SL	small
	48 - 52 cm	12-0022-ML	medium
Right Rechts Droite Rechts	53 - 26 cm	12-0022-LL	large
	61 - 67 cm	12-0022-XLL	x-large
Right Rechts Droite Rechts	39 - 47 cm	12-0022-SR	small
	48 - 52 cm	12-0022-MR	medium
Right Rechts Droite Rechts	53 - 26 cm	12-0022-LR	large
	61 - 67 cm	12-0022-XLR	x-large

check label on packaging or on the brace

Orthobroker bvba . Ter Stratenweg 11
2520 Ranst . www.orthobroker.com

English: Functional 4-Point Knee Orthosis
Patient Application Guide

Indications

- Instabilities and ruptures of ACL, PCL MCL or combined instabilities
- Complex knee instabilities from traumatic or degenerative origin
- Pre- /post-operative treatment of the above mentioned injuries
- Protection of the knee ligaments in sports

Intended use

Apply the device to provide support to the knee joint. This brace allows the user to perform functional activities with very little restrictions.

Precautions

- Don't apply the straps too tight that it interferes with circulation or that nerves experience too much compression.
- If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.
- In case you have doubts about the application of the product, you should contact an orthopaedic specialist (doctor or technician).

Application of the brace

1. Open all hook closures, pull the strap through the D-Ring and close the hook temporarily again by laying it down on the strap
2. Be seated on the edge of a chair with the knee bended for about 45°. Step with the foot through the brace and move the centre of the joint about 2 cm above the centre of the patella.
3. Fasten strap 1 (front - shin) and strap 2 (back - thigh)
4. Push the hinges backwards for about 1 cm. Note: these are now positioned slightly behind the knee, however, when all other straps are firmly closed & attached, the hinge moves forward and gets in the right position.
5. Fasten straps 3 through 5. Tighten strap 5 firmly!
6. Get in the upright position and tighten strap 6 (in case of PCL injury). Make a few steps and bend the knee when this is possible or allowed, and re-tighten straps when necessary for a better fixation or comfort
7. Check whether the aluminium cuffs fit correctly at thigh and calf without creating excessive pressure. The condylar pads should also ensure a correct fit of the hinges around the knee. The brace has been delivered with 3 different types of knee pads. These should enable you to adjust the width of the brace correctly to the width of the knee.
8. Only for technician or doctor: enclosed flexion and extension stops allow you to determine the patient's knee range of motion. Unscrew the black hinge cover and place the wedges with correct degrees at the front or back of the hinge.

Care & Cleaning

Remove the soft fabrics from the stays. Fasten closures and hand-wash with lukewarm water and mild soap. Rinse thoroughly. Do not wring. Dry at room temperature and be sure the foam is completely dry before re-applying it. Clean the metal frame with a wet cloth and dry it afterwards. Check screws for tight fit regularly and contact an orthopaedic technician in case parts should be replaced.

Warnings

The product should only be used in case of a medical indication or when prescribed by a physician. It should also be applied by a competent orthopaedic technician. This person has also the responsibility to provide the patient accurate information about correct and safe use, according its individual needs.

The patient should under no circumstances make any change or adjustment to the brace without the consent of physician or orthopaedic technician. (i.e. application or removal of hinge wedges)

This device is meant for single patient use

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