

DESIGNED AND DEVELOPED IN NORWAY

Letsip® K-Series Big pack

Letsip® K4 (55mm/2.25")

Letsip® K5 (65mm/2.56")

Letsip® K10L (53mm/2.1" wire)

Letsip® K10M (35mm/1.4" wire)




The K-series can contribute to a better lift of the tip of the tongue and can exercise lateral movements




K10L



K10M

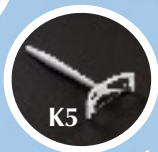

K5

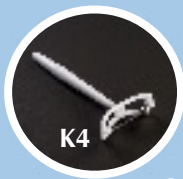

K4



Develop the fine motor skills of the tongue

www.letsip.com/v3



www.letsip.com



Scan me

Train the tip of the tongue to improve your fine motor skills



Letsip® K-Series Big pack

Letsip® K-Series:

The Letsip® K-series exercises and stimulates the tongue, through the movement of the ball. The ball is big and heavy and should therefore be used with caution, so that no damage is done to the teeth.

Cleaning and disinfecting:

We recommend that you lightly wash all products before use, excluding the Letsip® handle piece. All the pieces, excluding the handle, are dishwasher safe, up to 158°F / 70°C. The Letsip® handle piece is water resistant and can be disinfected with wipes or a light spray. Disinfecting your Letsip products is best done with LifeClean. We recommend using LifeClean, as it does not cause any visible damage to Letsip products. LifeClean administers the highest level of disinfection, coming into full effect within two minutes of proper application.

www.letsip.com

NOTE!

Cleaning Letsip products with spirits or washing the plastics in high temperatures, can change the shape and strength of the product over time. Disinfecting with alcohol may also over time, cause the discolouration of the handle. Follow the cleaning guide at www.letsip.com

WARNINGS & PRECAUTIONS

Keep out of the reach of unsupervised children. Caution is recommended when using Letsip products around the mucous membranes in the mouth, and on exposed and/ or irritated skin. Vibration on the throat is not recommended. Always check the product before use and discard the product at the first signs of damage or weakness.

The ball is big and heavy and should therefore be used with caution, so that no damage is done to the teeth!



DESIGNED AND DEVELOPED IN NORWAY

2 year guarantee



Letsip AS

Langelandsvegen 51, N-6010 Aalesund Norway
Phone +47 480 09 500 post@letsip.com www.letsip.com

PATENTED All products are protected by the international patent application

Material ABS Dow USA. Ball and string Surgical steel. Made in P.R.C.

