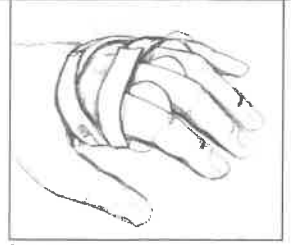
DeRoyal®

DeRoyal®

DeRoyal®

REF 202

HAND SPLINT
Ulnar Deviation Splint
FÉRULA PARA MANO
Férula para desviación cubital
ATELLE DE MAIN
Atelle pour déviation cubitale
TALA PARA MÃO
Tala de desvio cubital

Manufacturer
DeRoyal
200 DeBusk Lane
Powell, TN 37849 U.S.A.
888.938.7828

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- DIRECTIONS FOR USE**
1. To assess the fit, slide fingers into the finger dividers. Determine if the fingers are supported in the desired position.
 2. To adjust the splint, bend the palmar tabs at the base to accommodate for the radio-ulnar position of each finger.
 3. To secure the proximal strap, pull the strap across the dorsum of the hand, pulling the strap through the slot on the radial side. Fasten the proximal strap by folding the loop of the hook material back onto the loop material end.

Small female	B	2 1/4-2 3/4"	5.71-6.98 cm
Average female, small male	C	2 7/8-3 1/4"	7.28-8.26 cm
Average to large male	D	3 3/8-3 3/4"	8.57-9.53 cm

PRECAUTIONS
These products are to be fitted initially by a healthcare professional who is familiar with the purpose for which they are used. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare professionals, care providers involved in the patient's care, and the patient. If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.

CARE INSTRUCTIONS
Straps may be removed and laundered separately. Splint may be cleansed with a nonabrasive detergent and water. Air dry prior to wearing.

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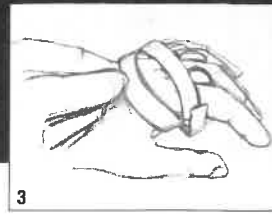
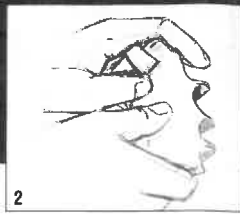
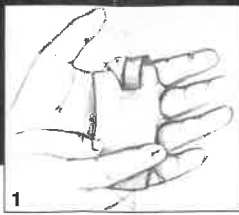
4. Align finger tabs between the fingers. Pull the distal strap in a ulnar-radial direction, through the slit at the radial side of the splint. Fold the hook material end of the strap onto the loop material strap. Adjust for comfort.

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©2001 DeRoyal. All Rights Reserved.