



# MADE IN EUROPE

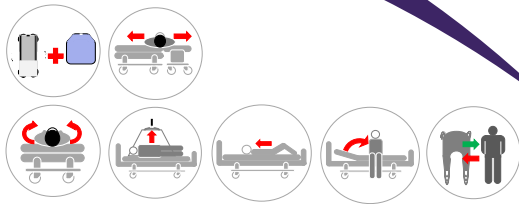

This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

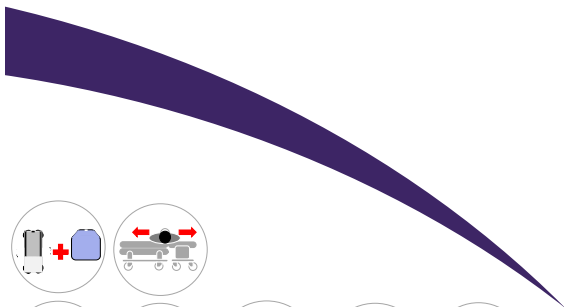

(DK) Forordning MDR 2017/745 for medicinsk udstyr klasse 1  
(UK) Regulation MDR 2017/745 for medical devices class 1



ISO 12 31 03 21

The Original Nordic  
Human Handling Concept



## Quick guide

(DK) **Master Turner Royal** - til madrascovers  
(UK) **Master Turner Royal** - for mattress covers

19\_20240910 DK/UK



## The Original Master Turner System



### Vare nr./Item no. HMI nr./no. Navn/Name - Størrelse/Size UDI-DI

<input type="radio"/>	MC 001-1713	131089	Master Turner Royal - S	5744001973912
<input type="radio"/>	MC 001-1723	131090	Master Turner Royal - M	5744001973929
<input type="radio"/>	MC 001-1734	137037	Master Turner Royal Combi - L	5744001974834
<input type="radio"/>	MC 001-1735	128452	Master Turner Royal - L	5744001973936
<input type="radio"/>	MC 001-1747	139893	Master Turner Royal - XL	5744001974988
<input type="radio"/>	MC 001-2535	131086	Master Turner Royal Inco - L	5744001973974

### Eftersyn:

Produktet skal kontrolleres jævnlig og gerne inden hver brug. Kontroller, at der ikke er skader på stof, stropper, sømme mm. Har produktet høj friktion eller på anden vis er i stykker, skal produktet kasseres.



### Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams and straps. Products with high friction or other deficiencies must be discarded.



(DK) Efterlad aldrig bruger på produktet på sengekanten.

(UK) Never leave the user on the product when lying on the edge of the bed.



(DK) Efterlad aldrig produktet liggende på gulvet.

(UK) Never leave the product on the floor.




(DK) Efterlad aldrig en bruger liggende på produktet, mens dette er koblet på en lift.

(UK) Never leave the user lying on the product whilst connected to a hoist.



(DK) Sørg altid for, at brugeren ikke hænger fast i stropperne.

(UK) Always make sure that the user is not caught in the straps.

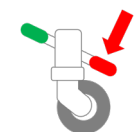

(DK) Sørg for, at alle relevante bremses er låst, inden forflytning foretages.

(UK) Make sure that all relevant brakes are activated before transferring the user.



(DK) Må ikke bruges som liftsejl – kan med fordel kombineres med lift til fx vendinger i seng.


(UK) Do not use as a sling – can be combined with a hoist for repositioning in bed.

(DK) For mere information, brugervejledning, videoer mm., se  
(UK) For more information, User Manual, videos etc. visit  
[www.master-care.dk](http://www.master-care.dk)


Developed & Manufactured by:



# The Original Master Turner

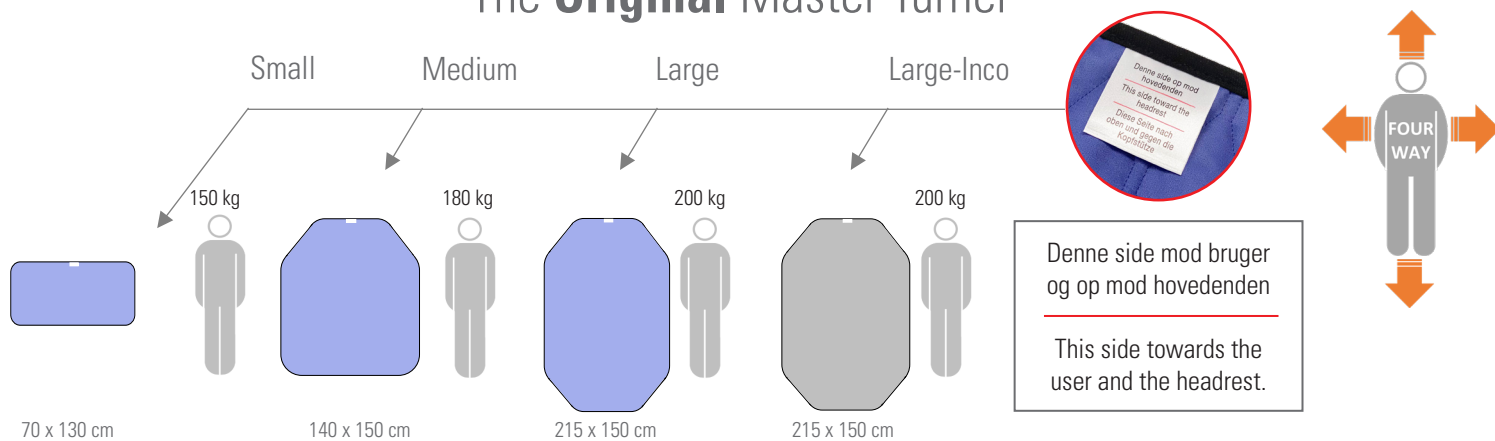

## (DK) Kombinationsmuligheder / (UK) Combinations

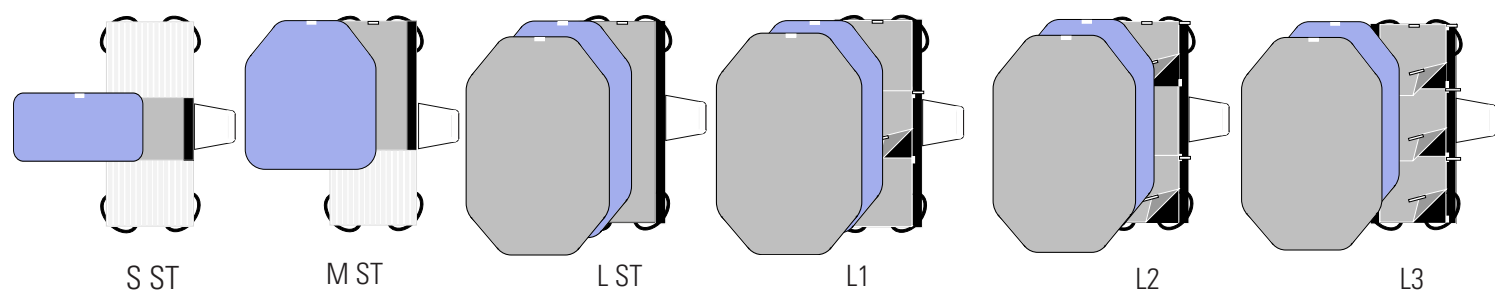

## (DK) Anvendelsesmuligheder / (UK) Applications



(DK) Skånsom vask og tørretumbling forlænger holdbarheden væsentligt.

(UK) Gentle washing and dry tumbling significantly prolongs the durability.

(DK) Anvend aldrig skyllemiddel, da dette forringer glideegenskaberne væsentligt.

(UK) Never use fabric softener as this will reduce the sliding properties significantly.



Find us on:



master care A/S, Sofienlystvej 3, DK 8340 Malling, Danmark  
Tel. +45 86 93 85 85, E-mail: info@master-care.dk