

KomfiTilt *Edition-3* New Features User Guide



KomfiTilt Lateral Tilt System

Normal ON/OFF function will activate Left-Supine-Right (A-B) function of the cells



MANUAL LOCK-OFF OF LEFT OR RIGHT CELL

To Lock-Off either the Right or Left cell, so that it doesn't inflate during the operational cycle:

On Start-Up, switch the power ON and the faceplate lights will light up.

Cell A only

Press the 'Reset Alarm' button and hold down

The 'Normal' green light will flash and the audible alarm will sound

Release the 'Reset Alarm' button and the KomfiTilt will function inflating cell A only

Cell B only

Press the 'Reset Alarm' button again and hold down

The 'Low' red light will flash and the audible alarm will sound

Release the 'Reset Alarm' button and the KomfiTilt will function inflating cell B only


Cell A & B in normal rotation

Press the 'Reset Alarm' button again and hold down

The 'Power' green light will flash and the audible alarm will sound

Release the 'Reset Alarm' button and the KomfiTilt will function inflating cell A & B in the normal rotation

Press & hold

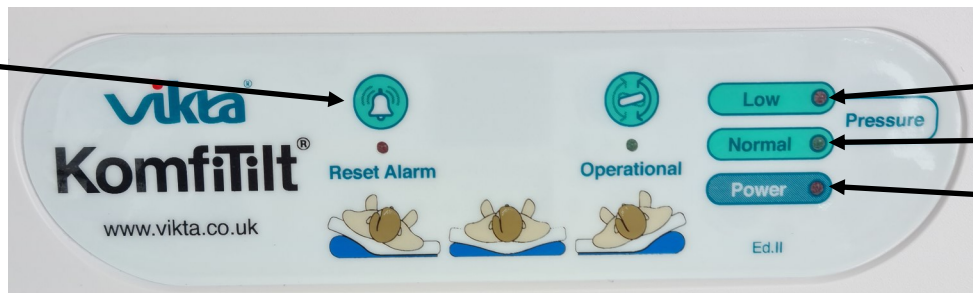

VIKTA LTD

Unit 2 Edison Gate

West Portway

Andover

SP10 3SE UK



T: +44(0)1264 366330



E: info@vikta.co.uk




W: www.vikta.co.uk

KomfiTilt *Edition-3* New Features User Guide

KomfiTilt APP now available for iOS and Android



iMat IPC

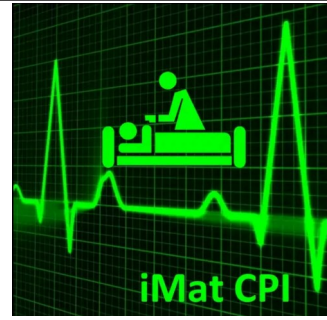

Download and connect to your KomfiTilt via Bluetooth

Input Model No: (e.g. SQ-BT1955) this model number can be found on the pump box

When connected a tracking record will shown on the graph (the two colours represent the different cells)

Cycle Time adjustment

Press the 'Clock' icon and an audible alarm will sound and the green light will move to the next setting (cycle time can be adjusted between 10 / 15 / 20 / 25 minutes)

Cell Lock-Off

Press the 'Cell Tilt' icon and an audible alarm will sound and the green light will move to the next setting (B / A & B / A)

History Records


Press the 'History Records' and select from the list to show the stored record

Connect

Access Stored History

Adjust Cycle Time

Adjust Cell function
A / B / A&B

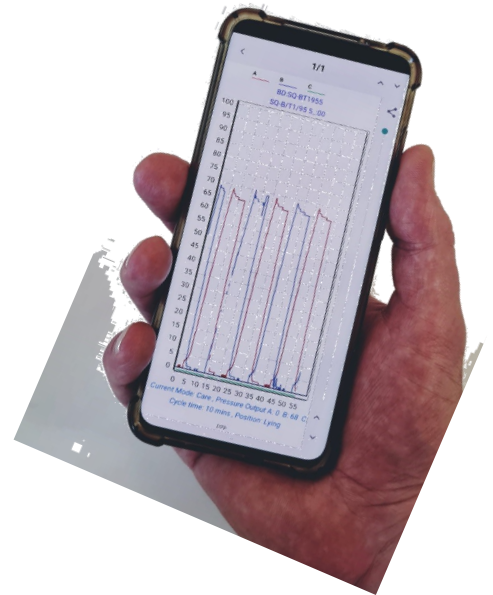

VIKTA LTD

Unit 2 Edison Gate

West Portway

Andover

SP10 3SE UK



T: +44(0)1264 366330



E: info@vikta.co.uk



W: www.vikta.co.uk