

Quick guide SIMPLE TIM letvægtskørestol

UNFOLDING AND ASSEMBLY

Step 1. Unfolding the wheelchair.

Place the wheelchair on the ground. Unfold the wheelchair's backrest using the hand grips which are at the back of the backrest. Then press the side edges together so that all parts of the cart fit together.

Step 2. Adjustment of footplates.

Adjust the right and the left footplate by tightening the adjustment screw at the appropriate height. Proper space should be left between the footplates and the ground as footplates set too low may cause difficulties in overcoming such obstructions as curb. Adjust the footplates so that the angle between trunk and thighs is 90° (optimum sitting position is provided to the user, then).

Step 3. Assembly of protective strap.

Put the protective strap on the footrest as it prevents user's legs from sliding down under the wheelchair from footplates.

Note! Do not use the wheelchair without protective strap.

Step 4. Adjustment of anti-tip wheels.

In order to adjust anti-tip wheels, which are located at the back of the wheelchair, pull the lock, move the wheels upwards or downwards, and select appropriate height. 3

Step 5. Checking the assembly of wheels.

Lift the wheelchair slightly and spin the right wheel and then the left one by pulling the handrims and check if the wheels spin properly.

Step 6. Assembly of cushion.

Fix the cushion to the seat with Velcro.

On completion of the above activities, the wheelchair is ready to use.