

### **3. Assembly**

#### **Unfolding and folding the rollator**

- To unfold the rollator, pull the frame apart at the rear wheels.
- Press the hinge located in the middle of the frame until the rollator is completely unfolded.
- Fix push handles into the mounting openings and block them with locking knobs.
- To fold the rollator, pull the hinge of the crossbar which is located in the middle of the metal crossbar and fold the rollator with the use of handles or frame.
- Hang basket with tray on small hooks in the front part of the frame.
- Shopping bag can be removed by unclasping the clips which are on both sides of the frame.

#### **Brakes**

- To stop the rollator, pull the brake handle upwards and hold.
- While stop or using the seat, lock the brake with locking mechanism by pushing down the brake handle.
- To release the brake, pull the brake handle upwards.

#### **Height adjustment of handles**

- To adjust the height of the handles, unscrew the locking knobs which are located below the push handles, select appropriate height and tighten the locking knobs.