

## Quickguide Basic badebænk med ryglæn

### Assembly

**Step 1.** Place seat upside down on a flat surface.

**Step 2.** Fix crossed base tubes with seat in the middle using one wing screw with washer.

**Step 3.** Join base tubes with legs.

**Step 4.** Screw backrest tubes to base tubes using four wing screws with washers.

**Step 5.** Place assembled shower chair on the legs and tighten the backrest using four wing screws with washers.

### HEIGHT ADJUSTMENT

- To adjust the height of the shower chair, press metal pin, select appropriate height of the leg, and lock the position of the tubes relative to each other with the use of metal pin. The legs are adjustable in 2.5 cm increments.

**Note!** The pins must be set on the same level in each leg.