

## Quickguide Basic badebænk med håndtag, ryglæn og hygiejneudskæring

### ASSEMBLY

**Step 1.** Place the seat upside down on a flat surface.

**Step 2.** Fix crossed base tubes with seat in the middle using one screw with washer.

**Step 3.** Join base tubes with legs.

**Step 4.** Screw backrest tubes to base tubes using four wing screws with washers.

**Step 5.** Place shower chair with fixed seat and backrest tubes on the legs and tighten the backrest using four wing screws with washers.

### HEIGHT ADJUSTMENT

- To adjust the height of the shower chair, press metal pin, select appropriate height of leg, and lock the position of the tubes relative to each other with the use of metal pin. The legs are adjustable in 2.5 cm increments.

**Note!** The pins must be set on the same level in each leg.