

## Quickguide RANGER AIR rollator, varenummer: RANGER

### Elementy podpórki:

1. Handgrip
2. Adjustable tubing handgrip
3. Parking brake
4. Adjustment clamp
5. Backrest
6. Seat with handle
7. Bag hook
8. Front wheel
9. Brake adjustment
10. Rear wheel
11. Safety clamp
12. Kerb climber
13. Crutch/cane holder
14. Locking pins
15. Cup holder





### 3. Assembling, unfolding, folding and adjustment

#### Assembling

- Unpack all components from the box and lay out on a flat surface (frame with 2 rear legs connected by brake lines, 2 front wheels, backrest, shopping bag).
- Pressing the locking pins, insert both rear legs with wheels into the holes in the frame of the rollator.  
(the right and left legs are connected to the correct side of the rollator's frame by brake cables).
- Pressing the locking pins, insert both front wheels into the holes in the front forks of the rollator's frame.
- Install the backrest into the clamps on the inside of the handles.
- Hang the bag on the hooks.



### Brake function

- To stop the rollator, pull the brake handle up toward the handle, then hold.
- When stationary or using the seat, lock the brake with the parking clamp by pulling the brake handle down.
- To release the brake, pull the brake handle upward.



### Brake adjustment

Unscrew the cable holder, pull it up to the desired distance and then lock it by tightening the clamping screw.





### Unfolding

- To unfold the rollator, grab both handgrips and pull them outward.
- Press down on the seat rails until it is fully unfolded (click of the locking mechanism).

### Folding

- To fold the rollator, lift the seat by the handle and pull it up.
- Grab both handgrips of the rollator and press them inward.

### Handgrips height adjustment

To adjust the height of the handgrip:

- Press the brake down to set the parking mode and immobilize the rollator.
- Pull and hold the adjustment clamp.
- Adjust the height of the handles
- Release the adjustment clamp to lock the adjusted height of the handles. Pull the handle up and down several times to check the correct height locking.
- Adjust the height of the second handgrip in a similar way.
- Both handgrips should be set at the same height.

