

1. Cover

The cover protects the frame and provides limited visibility, air circulation, and medical accessibility, with transparent mesh and special openings. **(photo 1)**

Properties:

- Transparent mesh
- Three medical access ports.
- Mounting points for the Quick Release system.
- Can be opened on all four sides, additional openings with a zip at the head and foot end for emergency care or calamities.



Photo 1

Material composition:

- Blue cover: 100% Polyamide (PA).
- Mesh: 100% polyester (PES).
- White socks: 100% polyester (PES).
- Weight: 6.6 kg (excl. frame, incl. a sheet).



Photo 2

2. Inflatable frame

A sturdy, lightweight frame with several air chambers, specially designed for stability and protection. **(photo 2)**

- The air chambers are interconnected.
- After inflating the frame, the connections must be closed with the tube clamps.
- Weight: 1.9 kg



Photo 3

3. Sheet

The device is supplied with a sheet that is zipped into the frame. The sheet can be replaced with another CloudCuddle Maxx sheet. **(photo 3)**

Dimensions: 196 cm x 86 cm.

Composition: 100% organic cotton (ORGO).

Weight: 0.4 kg



Photo 4

4. Pump

Hand pump, exclusively intended for inflating the CloudCuddle Maxx frame. **(photo 4)**

- Weight: 1.1 kg

5. Bag

A sturdy bag for the secure transport and storage of the CloudCuddle Maxx. **(photo 5)**

- Material: 65% polyester (PES), 35% cotton (CO).
- Weight: 0.5 kg



Photo 5

6. Accessibility set

Set for adapted access and safety. **(photo 6)**

- Four snap buckles on the left and right panels respectively; to be fastened.
- When using the snap buckles, attach the rings to the respective loops. Push the two sides of the snap buckles together to fasten them. You will hear a click when the snap buckle is closed. Always check that the snap buckles are securely locked after fastening them. The snap buckles are an additional safety measure, to prevent the patient from opening the CloudCuddle Maxx.
- A pull cord on the inside when the snap buckles are not used.



Photo 6