

System 2000



General information

Note. All article numbers starting with **85** indicates that more than one detail is included in the article number. [Blue text indicates changes compared to previous manual.](#)

How to order

Before ordering spare parts study the exploded view closely and make sure that you note all involved spare parts.

Please note that gaskets and o-rings normally are included in kits and can not be ordered separately.

When taking contact with Arjo Parts sales always have the following information at hand:

- Correct name of the product. See type label.
- The serial number of the product. See labels.

Serial Number From November 2016

REF PRODUCT CODE
SN SERIAL NUMBER

Consecutive number

(01)GTIN No.

Site (VEPL= Veplas)

(11)YYMMDD(21)Serial No.

YYYY-MM-DD 8618944_2

Serial Number From March 2014

REF XXXXXXXXXXXXXXXX

SN XXXXXXXXXXXXXXXX

Consecutive number

Site (VEPL= Veplas)

YYYY-MM

8618944 A

Serial Number From June 2008

The serial number consists of the following parts:

0806000001

Year Month Consecutive Number

Manufacturing country and Manufacturing Works can no longer be readout.


Contents

Overview	4
A Bath tub and drain	6
B1 Panel	8
B2 Panel Ferro	10
C1 Control panel P300	12
C2 Control panel P220	12
C3 Control panel P200+	14
D1 Mixer unit P300	16
D2 Mixer unit P220/P200+	18
E1 Backflow prevention; Disinfection, alt A	20
E2 Backflow prevention; Disinfection, alt B	22
F Backflow prevention; Shower, alt A	24
G Backflow prevention; Shower, alt B	26
H Lifting unit	28
I Disinfection system – standard/standard MPA.....	30
J1 Disinfection system – automatic P220/P300	32
J2 MPA Disinfection system – automatic P220/P300	34
K Pressure Vacuum Breaker	36
L Dispenser unit	38
M Power box	40
N Hydromassage pump	42
O Hydrosound unit	44
P Emergency lowering	46
Q Accessories	48
R Sound and Vision (S&V)	50
S MPA Control Box.....	52
T MPA Tank Unit.....	54
Preventive Maintenance Parts/Recommended Spares	56
Service Articles and Special Tools	59

Overview

Pos	Description	See page
A	Bath tub and Drain	6
B1	Panel	8
B2	Panel Ferro	10
C1	Control panel P300	12
C2	Control panel P220	12
C3	Control panel P200+	14
D1	Mixer unit P300	16
D2	Mixer unit P220/P200+	18
E1	Backflow prevention; Disinfection, alt. A	20
E2	Backflow prevention; Disinfection, alt. B	22
F	Backflow prevention; Shower, alt. A	24
G	Backflow prevention; Shower, alt. B	26
H	Lifting unit	28
I	Disinfection system – standard/standard MPA	30
J1	Disinfection system – automatic P220/P300	32
J2	MPA Disinfection system – automatic P220/P300	34
K	Pressure vacuum breaker	36
L	Dispenser unit	38
M	Power box	40
N	Hydromassage pump	42
O	Hydrosound unit	44
P	Emergency lowering	46
Q	Accessories	48
R	Sound and Vision (S&V)	50
S	MPA Control Box	52
T	MPA Tank Unit	54

Overview

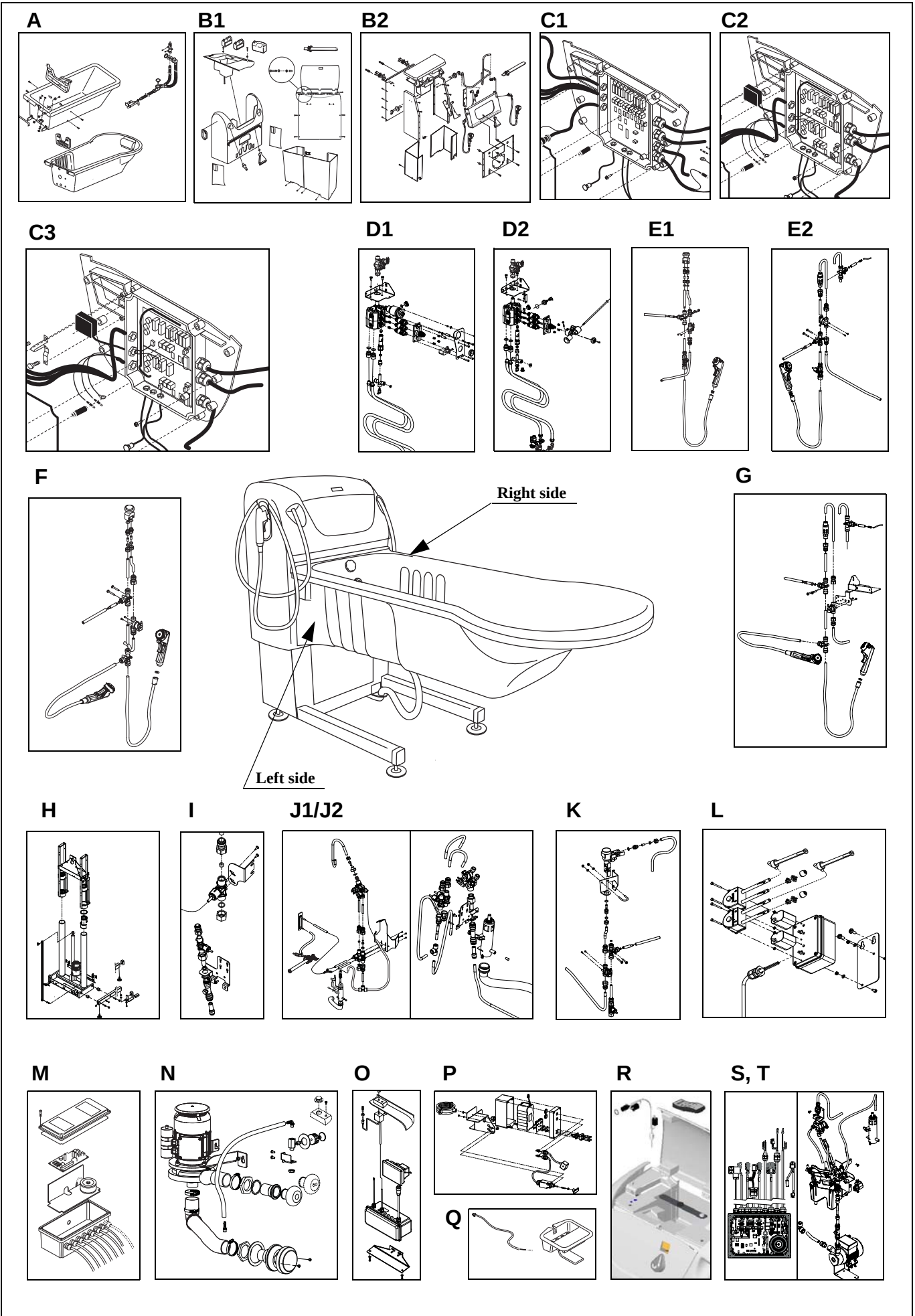


A <i>Bath tub and Drain</i>						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	8138678	1	Foot support			Accessory; New see-through version,
2	8331457-03	1	Pillow			Accessory
3	8440528	3/2	Sensor nipple			From 1403000106 2 pcs
4	6210014	2	Nut A4 M6M ISO 4032 M8 – 80			
5	6200005	3/4	Washer A4 BRB ISO 7089 8.4x16x1.6- 200hv			Old Primo Ferro; 4 washers
6	ARA1611-	1	Bathtub Rhapsody 1600			Colour (01,A,B,C,D) and Sound&Vision (S) coding for all tubs: -01=White / -01S=White S&V -A= Cream / -AS= Cream S&V -B= Rose / -BS= Rose S&V -C=Green / -CS= Green S&V -D= Blue / -DS= Blue S&V (For example: ARA2161-01S= Rhapsody 2150, white with S&V)
	ARA1612-	1	Bathtub Rhapsody 1600, hydromassage			
	ARA1613-	1	Bathtub Rhapsody 1600, hydrosound			
7	ARA1911-	1	Bathtub Rhapsody 1900			
	ARA1912-	1	Bathtub Rhapsody 1900, hydromassage			
	ARA1913-	1	Bathtub Rhapsody 1900, hydrosound			
8	ARA2161-	1	Bathtub Rhapsody 2150			
	ARA2162-	1	Bathtub Rhapsody 2150, hydromassage			
	ARA2163-	1	Bathtub Rhapsody 2150, hydrosound			
9	AAA1611-	1	Bathtub Primo 1600			
	AAA1612-	1	Bathtub Primo 1600, hydromassage			
	AAA1613-	1	Bathtub Primo 1600, hydrosound			
10	AAA1911-	1	Bathtub Primo 1900			
	AAA1912-	1	Bathtub Primo 1900, hydromassage			
	AAA1913-	1	Bathtub Primo 1900, hydrosound			
11	8154647-04	1	Foot support			Primo Ferro, Accessory
12	ASA1911-01	1	Bathtub Primo Ferro 1900			
	ASA1912-01	1	Bathtub Primo Ferro 1900, hydromassage			
	ASA1913-01	1	Bathtub Primo Ferro 1900, hydrosound			
13	ASA2161-01	1	Bathtub Primo Ferro 2150			
	ASA2162-01	1	Bathtub Primo Ferro 2150, hydromassage			
	ASA2163-01	1	Bathtub Primo Ferro 2150, hydrosound			
14	8411073	4	Spacer for foot support			Primo Ferro
15	8453180	2	Spacer for sensor		1403000105	Primo Ferro
16	6210037	2	Nut A4 ML6M ISO 4035 M16 - 035		1403000105	Primo Ferro
17	8442442	1	Nut for temp sensor		1403000105	
18	6262445	2	O-ring sensor nipple		1403000105	Primo Ferro
19	6262446	2	O-ring spacer		1403000105	Primo Ferro
20	6260053	1	O-ring			
21	S8454960	1	Knob			
22	8453083	1	Gasket, positive drain			
23	6907875	1	Drain plug, complete			
24	S8546519 8562307	1	Drain complete Drain complete, no overflow			For model AR37300EU1010
25	8555755-031	1	Drain holder			
26	S6319521	4	Hose clamp			
27	S8454924	1	Hose			
28	8150670	1	Positive drain			
29	6907909	1	Bottom connection			
30	6907926	1	Gasket			
31	6260100	4	O-ring			Primo Ferro
32	6190287	4	Screw A4 – K6S ISO 7380 M8 x 12 - 80			Primo Ferro
33	8547031-013	1	Decor stripe kit, blue			Trim 5,4m, tape and cleaner
	S8360308-034		Decor stripe kit, grey			Trim 5,2m, tape and cleaner
34	6316244	1	Plug			
35	S6319522	1	Hose clamp			
36	8663045	1	Bath sensor		1403000106	P200+/P220. For P300 see C1
37	8664364	1/6	Silicone gasket		1403000106	Rhapsody, Primo/Primo Ferro
38	6210254	1/3	Plastic nut PA M6M ISO 4032 M8		1403000106	Rhapsody, Primo/Primo Ferro
39	6210185	1	Nut A4 ML6M ISO 4035 M8 - 035		1403000106	For new bath sensor 8663045
40	6270421	2	Insulation sleeve		1403000106	For Primo Ferro only
41	6301221	2	Nylon washer 9,2x14,5x2		1403000106	For Primo Ferro only
42	AH1000677	1	Drain complete, no drain holder			New version VEPL00009169
43	6322090	1	Elbow connection D32x1 1/4"			
44	6370533	2	Cone TPE D32			
45	8664973	1	Pipe			
46	6210257	2	Nut D32 grey Purus			




<i>B1</i>		<i>Panel</i>				
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	8250255-034	1	Top box			
2	8144765	1	Shower handle holder			
3	8337059-03	1	Bending cover			
4	8344545	1	Water ejector			
5	6305091-1	1	Front (both with dispenser)			Grey trim instead of blue from March 2009
	6305091-2	1	Front (both)			
	6305091-3	1	Front (left, with dispenser)			
	6305091-4	1	Front (right, with dispenser)			
	6305091-5	1	Front (left)			
	6305091-6	1	Front (right)			
	6305091-7	1	Front (both with dispenser, hydrosound)			
	6305091-8	1	Front (both with hydrosound)			
	6305091-9	1	Front (left, with dispenser, hydrosound)			
	6305091-10	1	Front (right, with dispenser, hydrosound)			
	6305091-11	1	Front (left, with hydrosound)			
	6305091-12	1	Front (right, with hydrosound)			
6	8350633-031	1	Locking unit			
7	S8250449-034	1	Lid with badge			New version 1306000424
8	6318063-1	1	Back cover			
9	8358088	1	Fixation plate			
10	6210028	2	Nut A4-MHM DIN 1587 M6-80			
11	6190091	6	Screw A2 MFX ISO 7046 M5 x 12-80-Z			back cover
12	8546562	1	Screw kit (lid)			lid
13	8358134-036	1	Side plate, right			New version 1307000289 . See <i>H</i>
14	8258255-036	1	Side plate, left			New version 1307000289 . See <i>H</i>
15	6370358	1	Plastic cover, right			
	6370358-1	1	Plastic cover with holes for hydro adjustment, right			
16	6370359	1	Plastic cover, left			
	6370359-1	1	Plastic cover with holes for hydro adjustment, left			
17	6190406	2	Screw A4 – MRT-TT ISO 14583 M6 x 16			
18	8542170	1	Key kit			Two different keys
19	8450927	6	Angle attachment			
20	6314060	6	Rivet			
21	6262440	1	O-ring, ejector			Included in filling pipe
22	8418701	1	Arjohuntleigh badge	1306000424		
23	6270127	1	Hose			
24	6190337	8	Screw A4 MRT-TT ISO 14583 M6 x 12			
25	6200054	4	Washer A4-RBS ISO 7093 6.4x18x1.6-200HV			
26	8618933	1	Product label Harmonie			
	8618934		Product label Rhapsody			
	8618935		Product label Primo			
	8618936		Product label Rhadeero			

B1

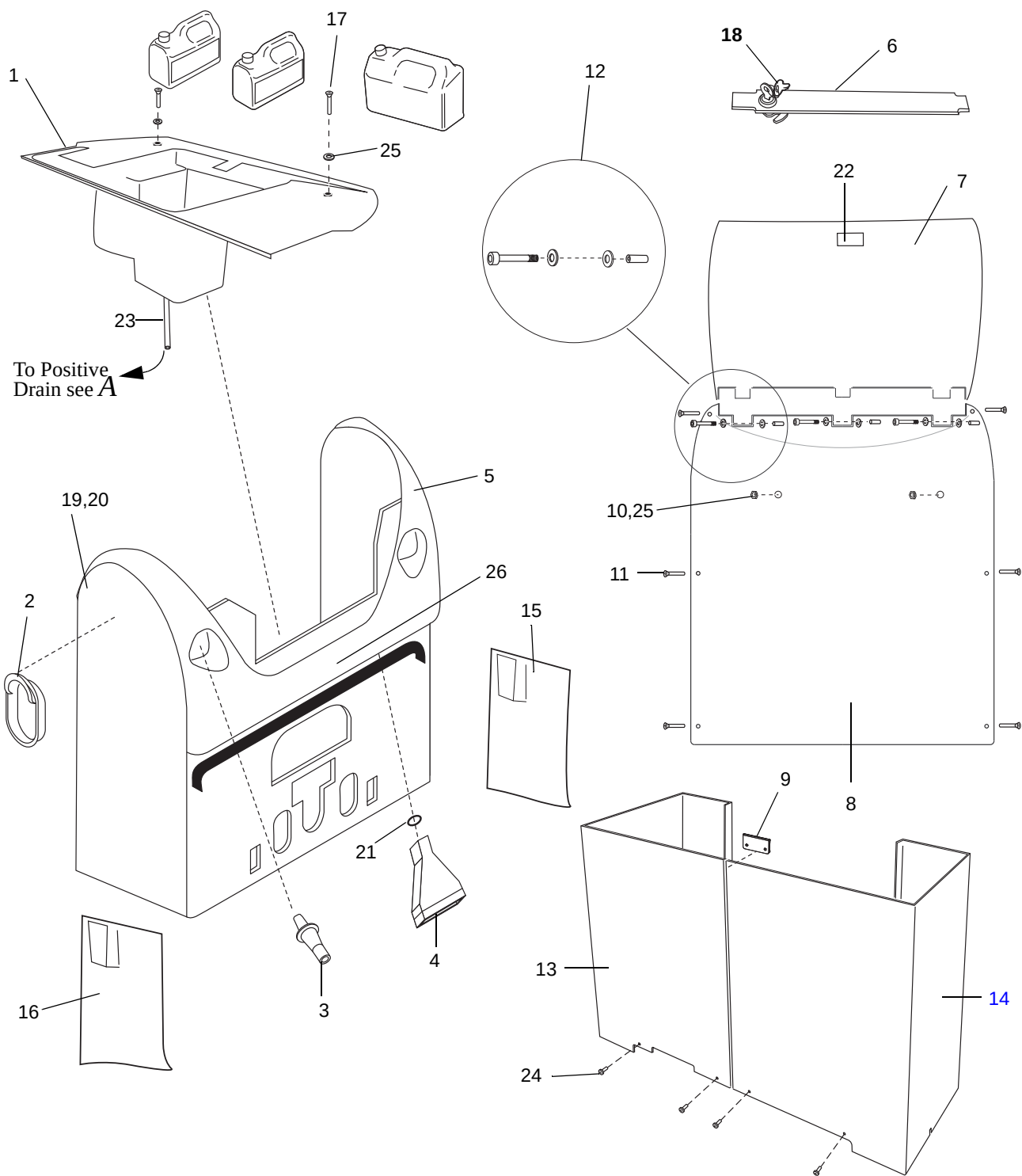


B2		Panel Primo Ferro				
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	8254696-034	1	Top box			
2	6190406	2	Screw A4 MRT-TT ISO 14583 M6 x 16			
3	8452873	2	Shower hose holder			
4	8441315-03	2	Shower handle hook			
5	6190276	8	Screw A4 RX PT K3.73x10			
6	8344545	1	Water ejector			
7	8350633-031	1	Locking unit			
8	8542170	1	Key kit			Two different keys
9	S8152950-034	1	Lid			Including screws
10	8245966	1	Panel back			
11	6210028	2	Nut A4-MHM DIN 1587 M6-80			
12	6200054	4	Washer A4-RBS ISO 7093 6.4x18x1.6-200HV			
13	8546723	2	Hinge kit			
14	6190269	10	Screw A2 K6S ISO 7380 M5 x 10 - 70			
15	8358088	1	Fixation plate			
16	8145964	1	Panel right			
17	8252790	1	Front			
18	8253227	1	Panel front			New version May 2009
19	8453235	4	Spacer			
20	8145965	1	Panel left			
21	8259517	1	Side plate, right			New version 1402000289 See <i>H</i>
22	8259516	1	Side plate, left			New version 1402000289 See <i>H</i>
23	8454933	2	Angle connection, shower hose			
24	6210154	2	Nut 1/2"x5mm NV28			
25	8359513	1	Pipe short	SEE0715343		
26	8359514	1	Pipe long	SEE0715343		
27	6270415	2	Shower hose			In production 1311001057
28	6262440	1	O-ring, ejector			
29	6310120	2	Plug-in coupling 1/2" 12mm	SEE0715343		
30	6200054	2	Washer A4-RBS ISO 7093 6.4x18x1.6-200HV			
31	6190337	12	Screw A4 MRT-TT ISO 14583 M6 x 12			
32	6270344	1	Hose short		SEE0712227	Old version
33	6270345	1	Hose long		SEE0712227	Old version
34	8459488	1	Adaptor Ø12 to Ø15		SEE0712227	Old version
35	6250196	1	T-coupling		SEE0712227	Old version
36	6306009	2	Washer (rubber)			

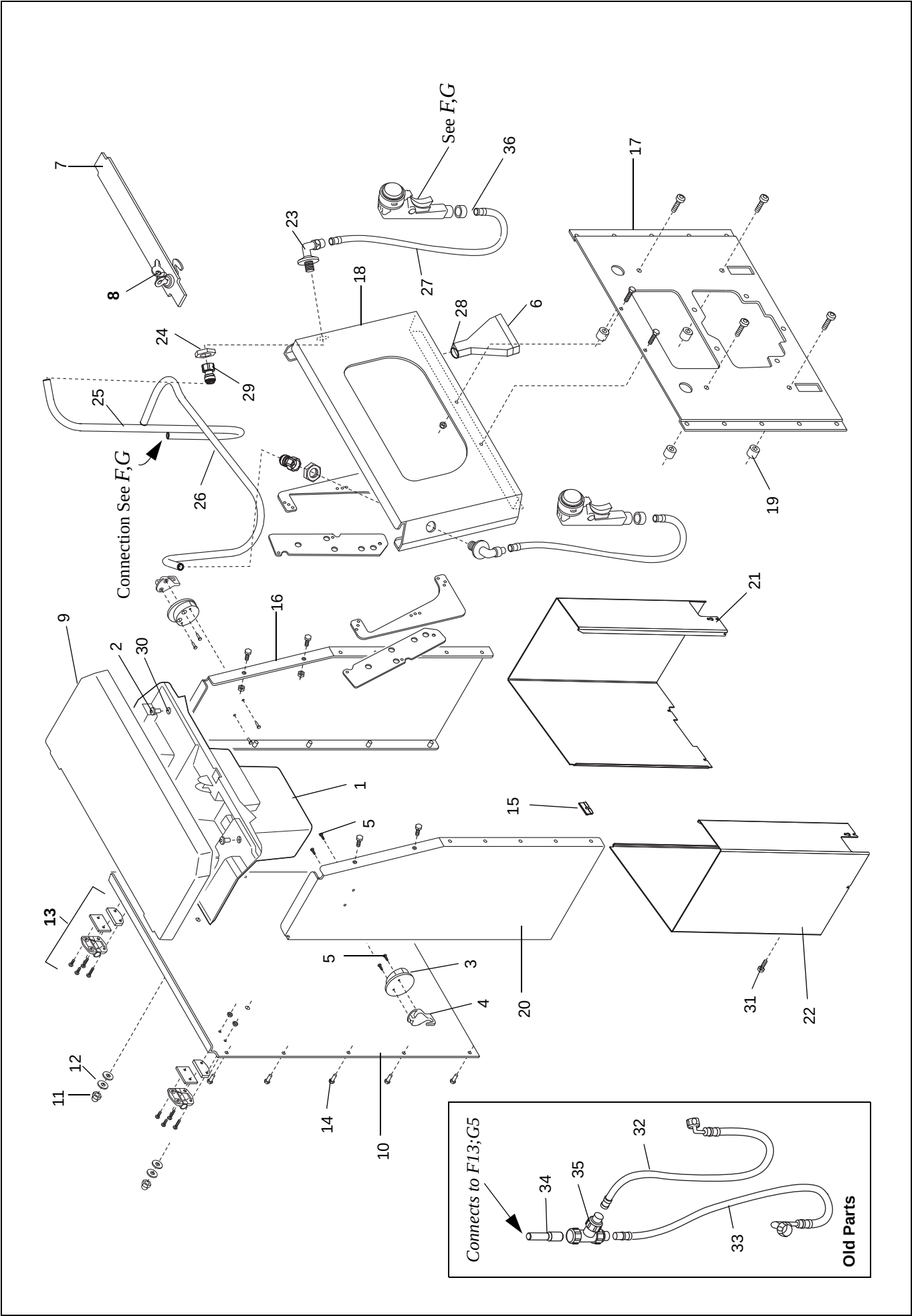


<i>C1</i> <i>Control panel P300</i>						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	8144753	1	Cover			
2	8546486	1	Panel with overlay		0905000277	See Technical Manual for illustrations.
	8561514		Panel with overlay	0905000278		
	8546776	1	Panel with overlay, fixed height		0905000277	
	8561515		Panel with overlay, fixed height	0905000278	June 2010	
3	8345000 8664499	1	Level cabling	1403000106	1403000105	On 834500 the ring terminal was a plug
4	8358372	1	Cabling power box/panel			
5	8258370	1	Cabling – solenoid valves			
6	8358382	1	Cabling – stepping motor			Incl. microswitch
7	8664365	1	Sensor – tub	1403000106		New version with nipple 2014-
8	S8445776	1	Sensor – shower			
9	S8445778	1	Sensor – filling			
10	6371027	1	PCB red LED, (EPROM included)		0905000277	
	8664079		PCB blueLED, (EPROM included)	0905000278		
11	6364286	1	EPROM			New version 1110001136
12	6130545	4	Screw A4 MC6S ISO 4762 M5x45-80			
13	8446507	4	Spacer			
14	8357559	1	Cabling-hydrosound			
15		1	Cabling-dispenser pump			See L Dispenser Unit
16	8258371	1	Cabling-auto disinfection			
17	6190365	6	Screw A2 R6B ISO 15480 ST4.8x16			
18	8444769	6	Washer			
19	S8663389	1	Seal			

<i>C2</i> <i>Control panel P220</i>						
Pos	Art. No.	Qty	Description	From ser.No.	Up to ser. No.	Remarks
1	8144753	1	Cover			
2	8546487	1	Panel with overlay		0905000277	See Technical Manual for illustrations.
	8561518		Panel with overlay	0905000278	1403000105	
	8563053		Panel with overlay (One display)	1403000106		
	8546775	1	Panel with overlay, fixed height		0905000277	
	8561517		Panel with overlay, fixed height	0905000278	June 2010	
3	8353132 8664522	1	Cabling, level, 3 cables	1403000106	1403000105	On 8353132 the ring terminal was a plug
4	8358372	1	Cabling, power box/panel			
5	8258370	1	Cabling , solenoid valves			
6	8451745	1	Spring			
7	6190276	2	Screw A4 RX PT K3.73x10			
8	8559259 8663047	1	Thermometer, complete	1403000106	1403000105	Version: no battery, illuminated 8a) For tub sensor 2014-, See A:36
9	8664081	1	PCB (EPROM included)			Note. When replacing PCB or EPROM on tubs with hydrosound or hydromassage up to SEE 0736162 cabling level with 2 cables must be replaced by cabling with 3 cables 8353132
10	6364285	1	EPROM (new version 1203000614)			
11	6130545	4	ScrewA4 MC6S ISO 4762 M5x45-80			
12	8446507	4	Spacer			
13	S8445778	1	Sensor – filling			
14	S8445776	1	Sensor – shower			
15	8664080	1	PCB, hydrocleaning			With hydroclean. Not used with MPA.
16	6364247	1	EPROM, hydrocleaning			With hydroclean. Not used with MPA.
17		1	Cabling-dispenser pump			See L Dispenser Unit
18	8258371	1	Cabling-auto disinfection			
19	8354567	1	Cabling PCB-PCB hydroclean			With hydroclean. Not used with MPA
20	6190365	6	Screw A2 R6B ISO 15480 ST4.8x16			
21	8444769	6	Washer			
22	S8663389	1	Seal			
23	See A:36	1	Tub sensor			

C1

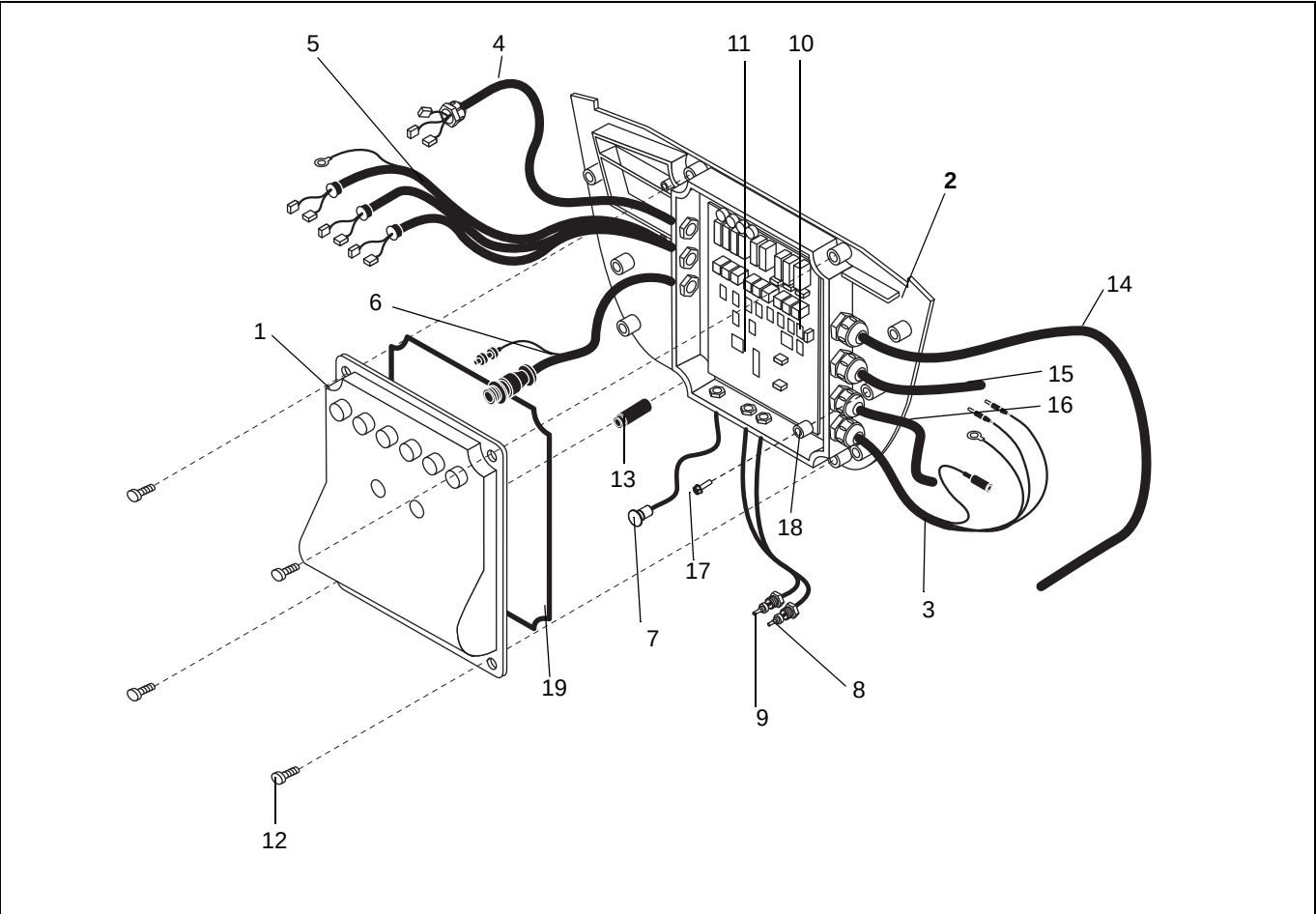


C2

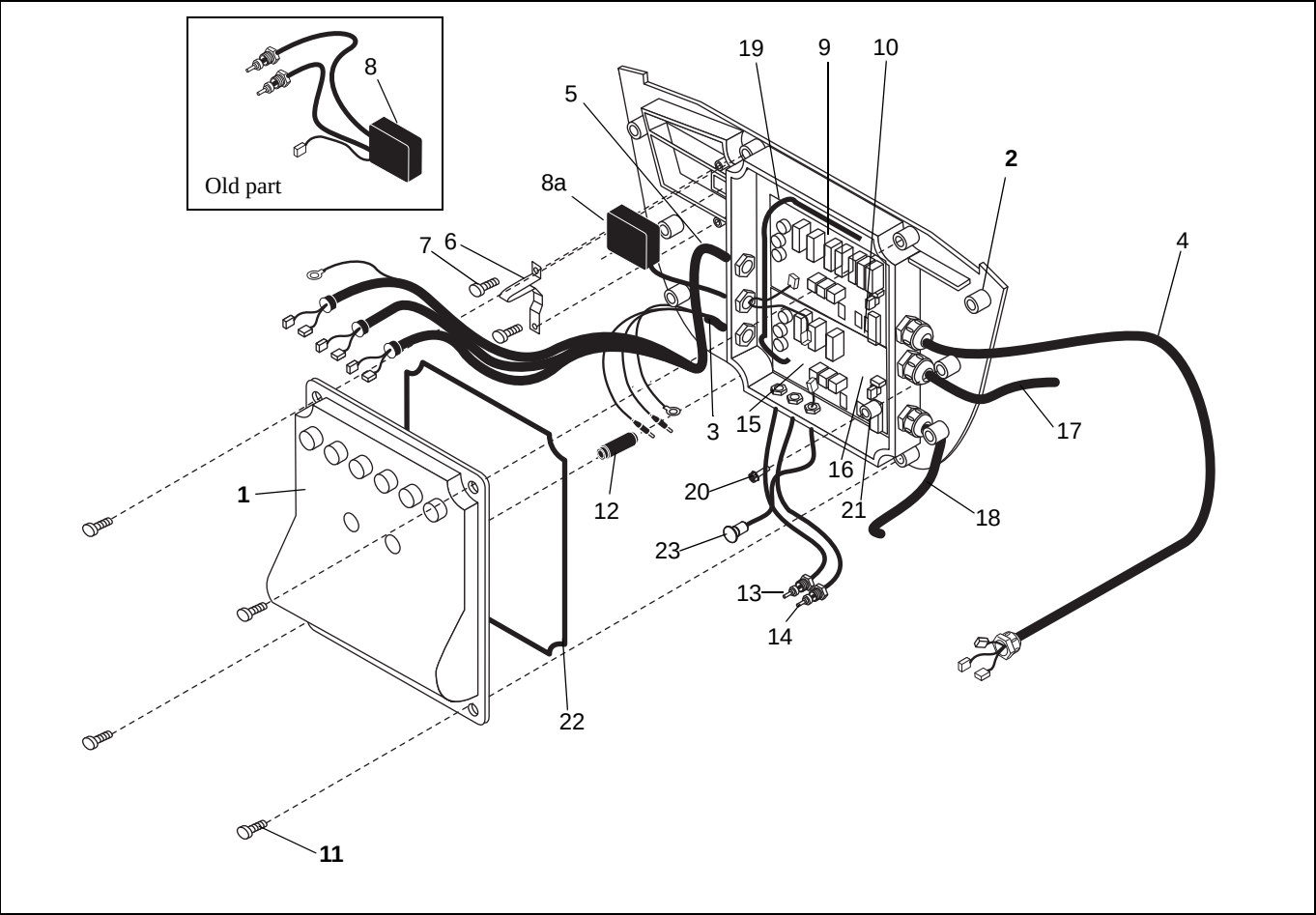


C3 <i>Control panel P200+</i>						
Pos	Art. No.	Qty	Description	From ser.No.	Up to ser. No.	Remarks
1	8144753	1	Cover			
2	8557733 8561516 8563051	1	Panel with overlay Panel with overlay Panel with overlay (One display)	0905000278 1403000106	0905000277	Same panel for fixed height and HI/LO. See Technical Manual for illustrations.
3	8353132 8664522	1	Cabling, level, 3 cables	1403000106	1403000105	On 8664522 one sensor is replaced by a ringterminal
4	8358372	1	Cabling, power box/panel			
5	8258370	1	Cabling , solenoid valves			
6	8451745	1	Spring			
7	6190276	2	Screw A4 RX PT K3.73x10			
8	8559259 8663047	1	Thermometer, complete Thermometer, complete	1403000106	1403000105	Version: no battery, illuminated 8a) For new tub sensor 2014-
9	8664081	1	PCB (EPROM included)			Note. When replacing PCB or EPROM on tubs with hydrosound or hydromassage up to SEE 0736162 cabling level with 2 cables must be replaced by cabling with 3 cables 8353132
10	6364285	1	EPROM (new version 1203001154)			
11	6130545	4	Screw A4 MC6S ISO 4762 M5x45-80			
12	8446507	4	Spacer			
13		1	Cabling-dispenser pump			See L Dispenser Unit
14	6190365	6	Screw A2 R6B ISO 15480 ST4.8x16			
15	8444769	6	Washer			
16	S8445778	1	Sensor – filling	1109001393		
17	S8445776	1	Sensor – shower	1109001393		
18	S8663389	1	Seal			
19	See A:36	1	Tub sensor			

C3

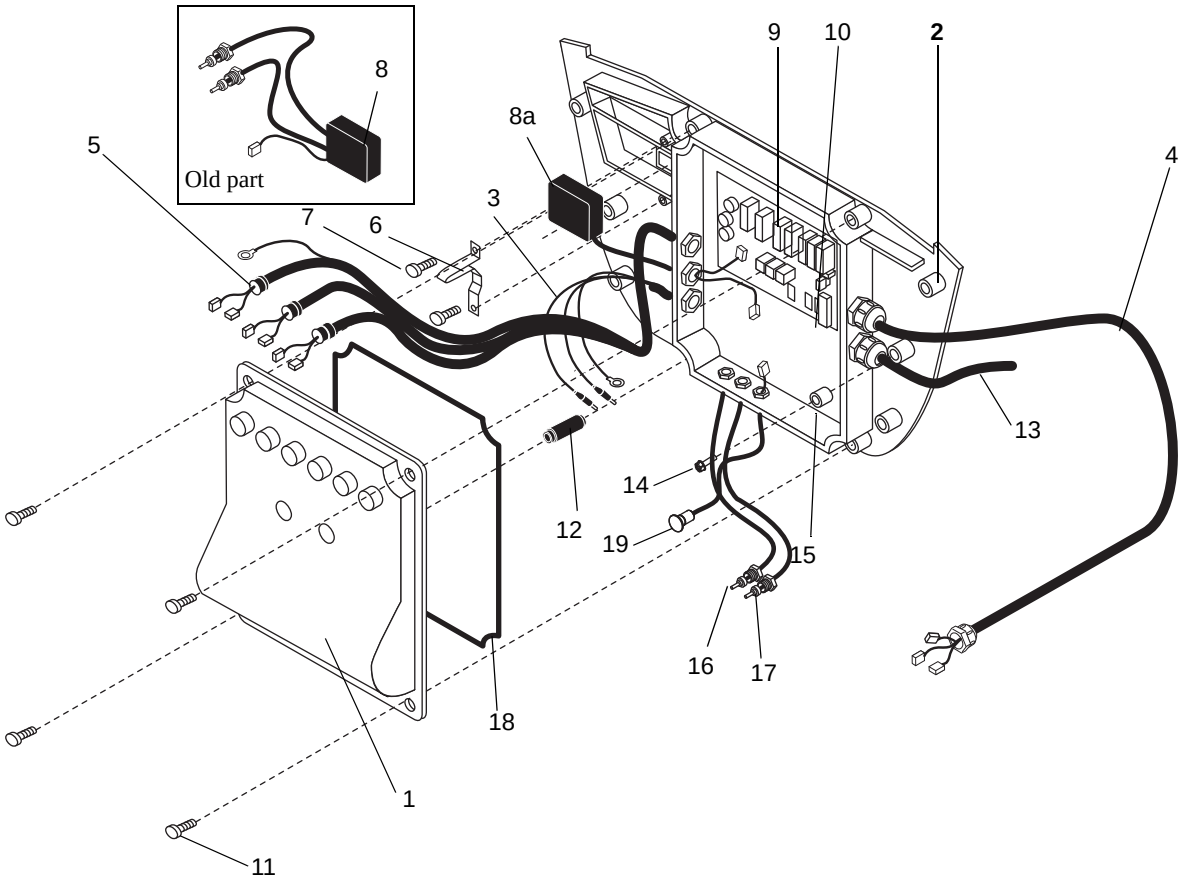


<i>D1</i>		<i>Mixer unit P300</i>				
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	6307185	1	Cog belt			
2	8344803	1	Belt wheel			
3	6190282	2	Stop screw A4 MS6SS ISO 4027 M6x8-21H			
4	8458141	1	Bearing			
5	6190132	4	Screw A4 MFX ISO 7046 M5 x 40-80-z			
6	8458260	1	Attachment, motor			
7	6303137	3	Spacer			
8	8458109	1	Attachment, microswitch			
9	6120312	1	Screw A4 MCS ISO 1207 M3x12-80			
10	6907715	4	Nut			
11	6907718 6907717	1 1	Adjustment plate Adjustment plate, UK		1103000299	From 1103000300 replacing 6907718
12	8547097	1	Repair kit mixer			3 cartridges with springs and 3 non-return valves, all o-rings
13	6907721	1	Outlet sleeve with o-rings; filling			
14	8345959	1	Stepping motor, complete			
15	6190399	2	Screw A4 MRX ISO 7045 M5 x 16-80			
16	6200002	2	Washer A4 BRB ISO7089 5.3x10x1-200HV			
17	6210042	2	Locking nut M5 A4 DIN 985-80			
18	8547094	1	Mixer complete			Incl pos 10-13,19,25,30,43
19	8546655	1	O-rings kit for mixer			New version
20	8358210	1	Mixer attachment			
21	6311411	5	Lid			
22	8665044	1	Solenoid valve RPE 3/4" 24VAC			Including coupling. Only parts pos 41-42 for this valve
23	S6907723 S6907794	1 1	Repair kit, Solenoid v. 3/4" Standard Repair kit, Solenoid v. 3/4" WRC, UK/JPN		1207000159	Membrane + anchor + anchor housing
24	6907727	1	Coil, solenoid valve		1207000159	
25	6907719	1	Outlet sleeve with o-rings, shower			
26	6250110	1	Compression ring			
27	8458268	1	Nipple conex			
28	8458215	1	T-pipe with couplings			
29	6260100	3	O-ring			
30	8457456	2	Nipple			
31	8546992	1	Kit Sensor attachment			Incl. o-ring
32	6316288	1	Plug			When no disinfection or MPA system.
33	8559596	1	Inlet pipe hot water, complete			
34	8559597	1	Inlet pipe cold water, complete			
35	8458217	2	Pipe connection for tempsensor			
36	6305121	1	Shut-off valve, hot water, complete			
37	6305122	1	Shut-off valve, cold water, complete			
38	6907788	2	Filter			
39	6270394	2	Connection hose 3/4" DVGW			l=618mm
40	8563020-618	2	Connection hose 3/4", WRAS GB			l=618mm
41	8563010	1	Filling pipe, complete			New version only for RPE fill valve
42	8563009	1	Filling pipe (hydrosound), complete			New version only for RPE fill valve
	8546943	1	Hot Water Flush kit P300			

D1

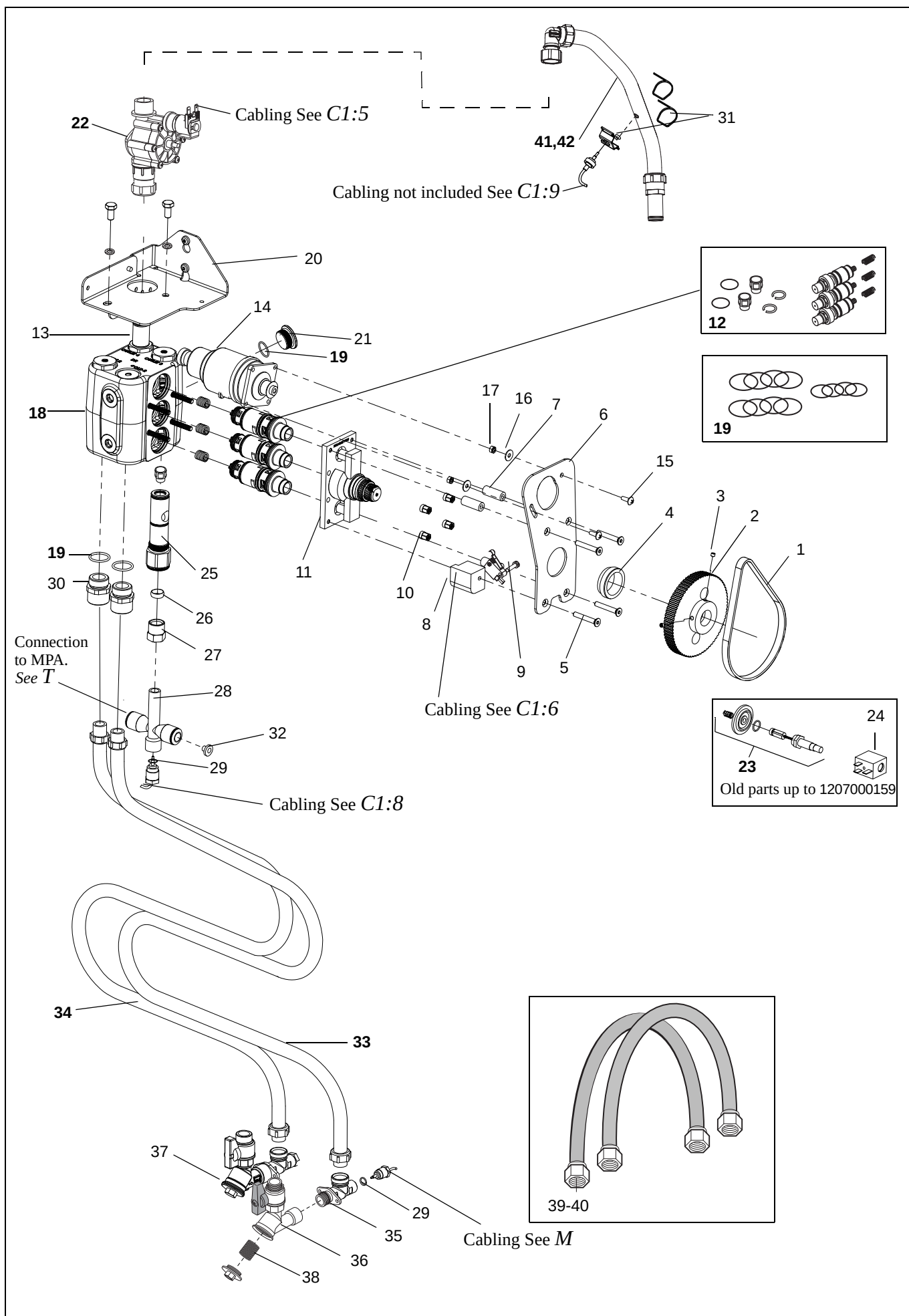

D2		Mixer unit P200+/P220			
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No. Remarks
1	6316120 8461219-036	1	Cover	0905000278	0905000277
2	8239131-031 8547048-034	1	Control knob	0905000278	0905000277 Incl. screw Incl. cover and screw (CB 8261220)
3	6319334	1	Stop ring		
4	6260018	1	O-ring		
5	6319338	1	Bushing-top part		Plastic from 20090831
6	6260011	1	O-ring		
7	8445221	1	Bushing		
8	8446524	1	Shaft		
9	8343863	1	Gear housing		
10	8345802	1	Cog wheel Z 25		
11	8343859	1	Cog wheel Z 30		
12	6190258	1	Screw A4 - MRX ISO 7045 M4 x 18 - 80		
13	6190403	1	Stop screw A2 – T6SS ISO 4028 M6 x 16 –21H		
14	6210013	1	Nut A4 M6M ISO 4032 M6-80		
15	6907715	4	Nut		
16	6907718 6907717	1 1	Adjustment plate Adjustment plate, UK	1103000299	From 1103000300 replacing 6907718
17	8547097	1	Repair kit mixer		3 cartridges with springs and 3 non-return valves, all o-rings
18	6907721	1	Outlet sleeve with o-rings; filling		
19	8547094	1	Mixer complete		Incl.pos.15-18, 23,24,29-30
20	8358210	1	Mixer attachment		
21	8443856	1	Bracket		
22	6190337	1	Screw A4 MRT-TT ISO 14583 M6 x 12		
23	8546655	1	O-rings kit for mixer		New version
24	8443862	1	Bushing temperature sensor	1403000105	
25	6311411	5	Lid		
26	8665044	1	Solenoid valve RPE 3/4" 24VAC		Including coupling.Parts pos 46-49 only for this valve
27	S6907723 S6907794	1 1	Repair kit, Solenoid valve 3/4" Standard Repair kit, Solenoid valve 3/4" WRC, UK/JPN	1207000159	Membrane + anchor + anchor housing
28	6907727	1	Coil, solenoid valve		
29	6907719	1	Outlet sleeve-shower with o-rings		
30	8457456	2	Nipple		
31	6316288	1	Plug		When no disinfection or MPA system.
32	6260100	1/2	O-ring	1403000105	For nipple
33	6250110	1	Compression ring		
34	8458268	1	Nipple conex		
35	8458215	1	T-pipe with couplings		
36	6260099	1	O-ring	1109001392	P200+ (For plug)
37	8458274	1	Plug	1109001392	P200+
38	8559596	1	Inlet pipe hot water		Red marking
39	8559597	1	Inlet pipe cold water		Blue marking
40	6250741	2	Connection		
41	6305121	1	Shut-off valve, hot water, complete		
42	6305122	1	Shut-off valve, cold water, complete		
43	6907788	2	Filter		
44	6270394	2	Connection hose 3/4" DVGW		l=618mm
45	8563020-618	2	Connection hose 3/4", WRAS GB		l=618mm
46	8547113	1	Filling pipe, complete, old P200+	1109001392	Only for RPE fill valve
47	8547114	1	Filling pipe hydrosound, complete, old P200+	1109001392	Only for RPE fill valve
48	8563010	1	Filling pipe, complete		Only for RPE fill valve
49	8563009	1	Filling pipe (hydrosound), complete		Only for RPE fill valve
50	8546992	1	Kit Sensor attachment		Incl. o-ring
	8546999		Hot Water Flush kit /P220		

D2

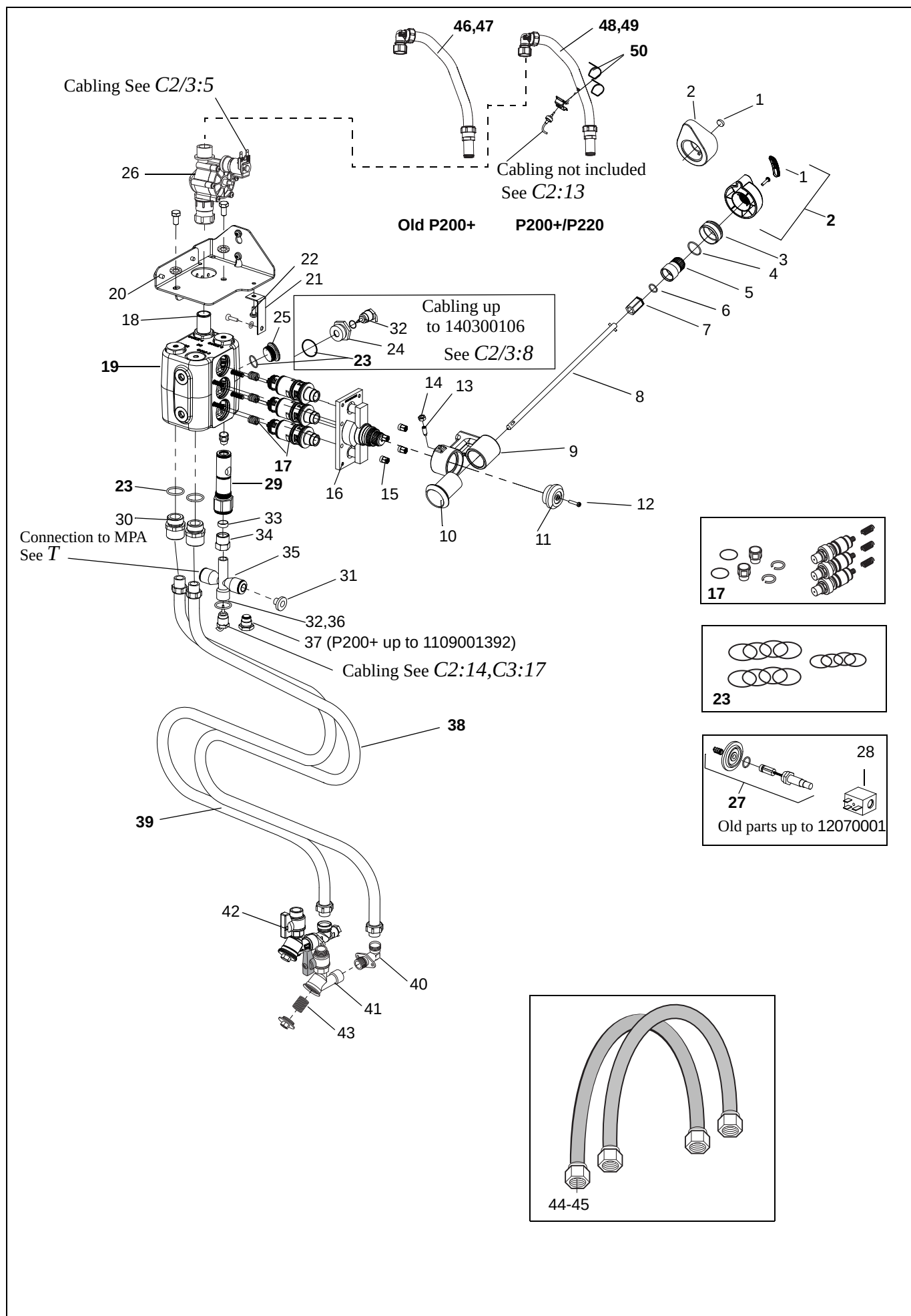


<i>E1 Backflow prevention; Disinfection, alt. A</i>						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	6326015	1	Vacuumbreaker DA			
2	6907932	1	Repair kit Vacuumbreaker DA			
3	6260108	2	O-ring			
4	8458181	2	Nipple			
5	6319493	2	Support Sleeve 10x1			
6	8458182	2	Nut, conex			
7	6250312	2	Compression ring Conex 12mm			
8	8458199	1	Pipe			
9	8458188	1	Pipe – feeding DA			
10	8459472	1	Nipple Ø 12			
11	6305076	1	Flow restrictor, drain			Black 1.9l/min
12	6270176	1	Hose 1,0m			
13	6310120	2	Plug-in coupling 1/2" 12mm			
14	8458198	1	T-connection			
15	6190352	2	Screw A4 MRT-TT ISO 14583 M6 x 40			BUMAX HARD
16	8358233	1	Pipe – feeding disinfection			Black marking
17	6312169	1	Solenoid valve, RPE 1/2"24VAC			
18	6190390	2	Screw FZB RXK ISO 7049 ST 4.2x9.5 -8.8 - FH			
19	S6303097	1	Shower handle yellow; complete			
20	6303107	1	Disinfection hose, grey; complete			
21	6316139	2	Washer, fibre Ø19Ø13x2			
22	8543696	1	Strainer kit, yellow			Strainer handle, o-ring, screw, plug
23	8545142	1	Repair kit, yellow			2 pins, sleeve, spring, 3 o-rings, 2 complete valves with o-rings
24	6907870	1	Swivel complete, yellow handle			Incl. plastic washer




<i>E2</i>		<i>Backflow prevention; Disinfection, alt. B</i>				
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	8458186	1	Pipe, DB			Pipe for old GB must be cut
2	8358232	1	Vacuumbreaker DB			
3	6310120	3	Plug-in coupling 1/2" 12mm			
4	8458199	1	Pipe			
5	8459472	1	Nipple Ø 12			
6	8458198	1/2	T-connection			Old GB =2
7	6190352	2	Screw A4 MRT-TT ISO 14583 M6 x 40			Bumax hard
8	6305076	1	Flow restrictor, drain			Black 1.9l/min
9	6270176	1	Hose 1,0m			
10	6312169	1	Solenoid valve, RPE 1/2"24VAC			
11	6190390	2	Screw FZB RXX ISO 7049 ST 4.2x9.5 -8.8 - FH			
12	8358233	1	Pipe, feeding			
13	6303107	1	Disinfection hose, grey; complete			
14	6316139	2	Washer, fibre Ø19Ø13x2			
15	6907870	1	Swivel complete, yellow handle			Incl. plastic washer
16	S6303097	1	Shower handle, yellow; complete			
17	8543696	1	Strainer kit, yellow			Strainer handle, o-ring, screw, plug
18	8545142	1	Repair kit, yellow			Pins, sleeve, spring, 3 o-rings, 2 complete valves with o-rings
19	8458143	1	Adapter			Old GB only
20	6310095	1	Quick coupling			Old GB only
21	6270301	1m	Hose			Old GB only

E2

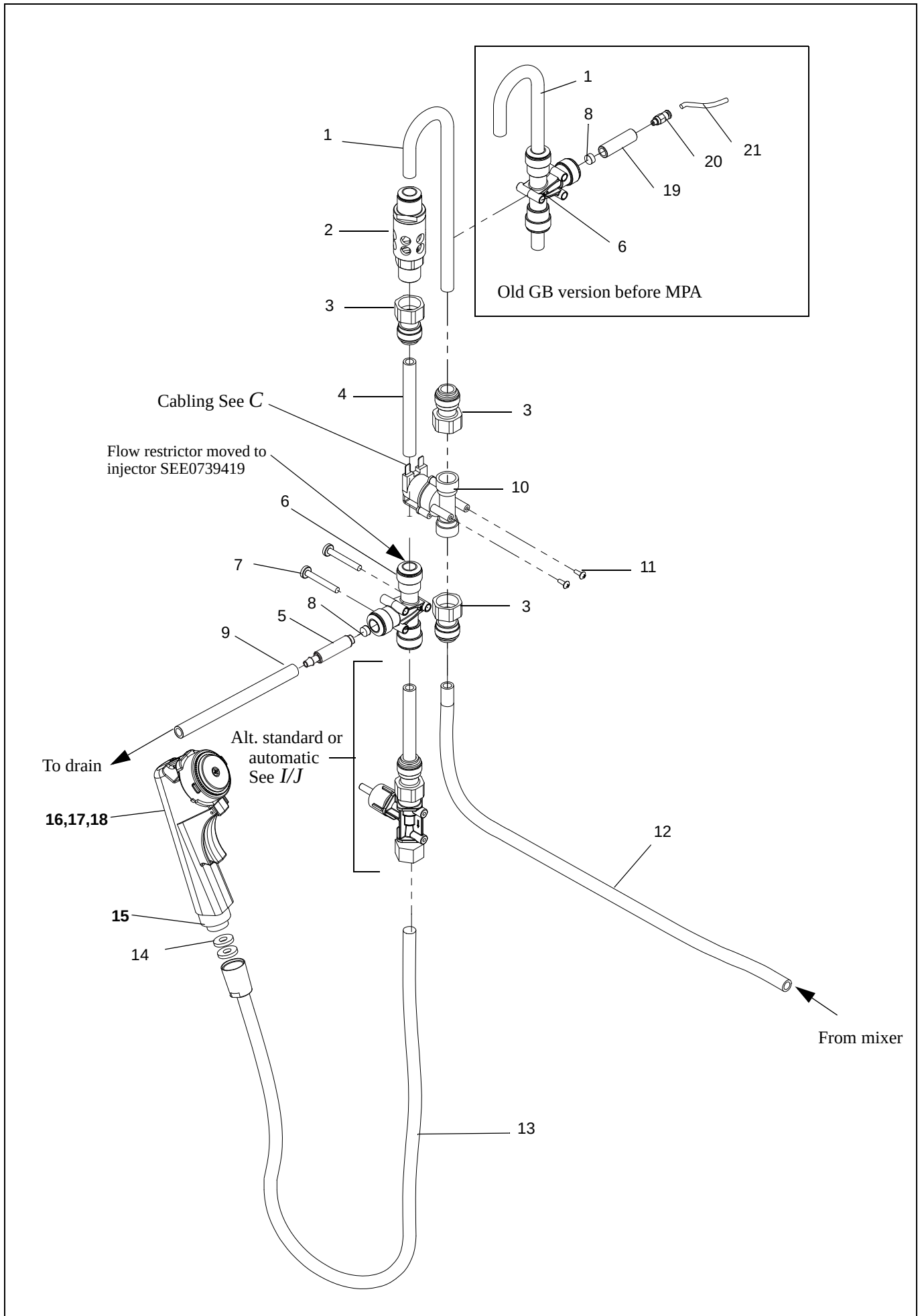

<i>F Backflow prevention ; Shower, alt. A</i>						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	6326015	1	Vacuumbreaker DA			
2	6907932	1	Repair kit Vacuumbreaker DA			
3	6260108	2	O-ring			
4	8458181	2	Nipple			
5	6319493	2	Support Sleeve 10x1			
6	6250312	2	Compression ring Conex 12mm			
7	8458182	2	Nut, conex			
8	8458199	1/2*	Pipe			
9	8458188	1	Pipe – feeding DA			
10	6305076	1	Flow restrictor, drain			Black 1.9l/min
11	6270176	1	Hose 1,0m			
12	6310120	2	Plug-in coupling 1/2" 12mm			
13	8458198	1	T-connection			
14	6190352	2	Screw A4 MRT-TT ISO 14583 M6 x 40			Bumax hard
15	6312169	1	Solenoid valve, RPE 1/2"24VAC			
16	6190390	2	Screw FZB RXK ISO 7049 ST 4.2x9.5 -8.8 - FH			
17	8459472	1	Nipple Ø 12			
18	8458231	1	Pipe – feeding shower			
19	8458090	1*	T-connection, shower			
20	S6303096 S6303122	1 1	Shower handle, grey; complete Shower handle, UK; complete			
21	6270413	1	Shower hose			Primo Ferro see B2 November 2013
22	6306009	1	Washer (rubber)			
23	8543697	1	Strainer kit, grey			2 strainers handle, 2 strainers hose, o-ring, screw, plug
24	8543698	1	Repair kit, grey			2 pins, sleeve, spring, 3 o-rings, 2 complete valves with o-rings
25	6907869	1	Swivel, complete grey handle			Incl. strainer


*=With two showers

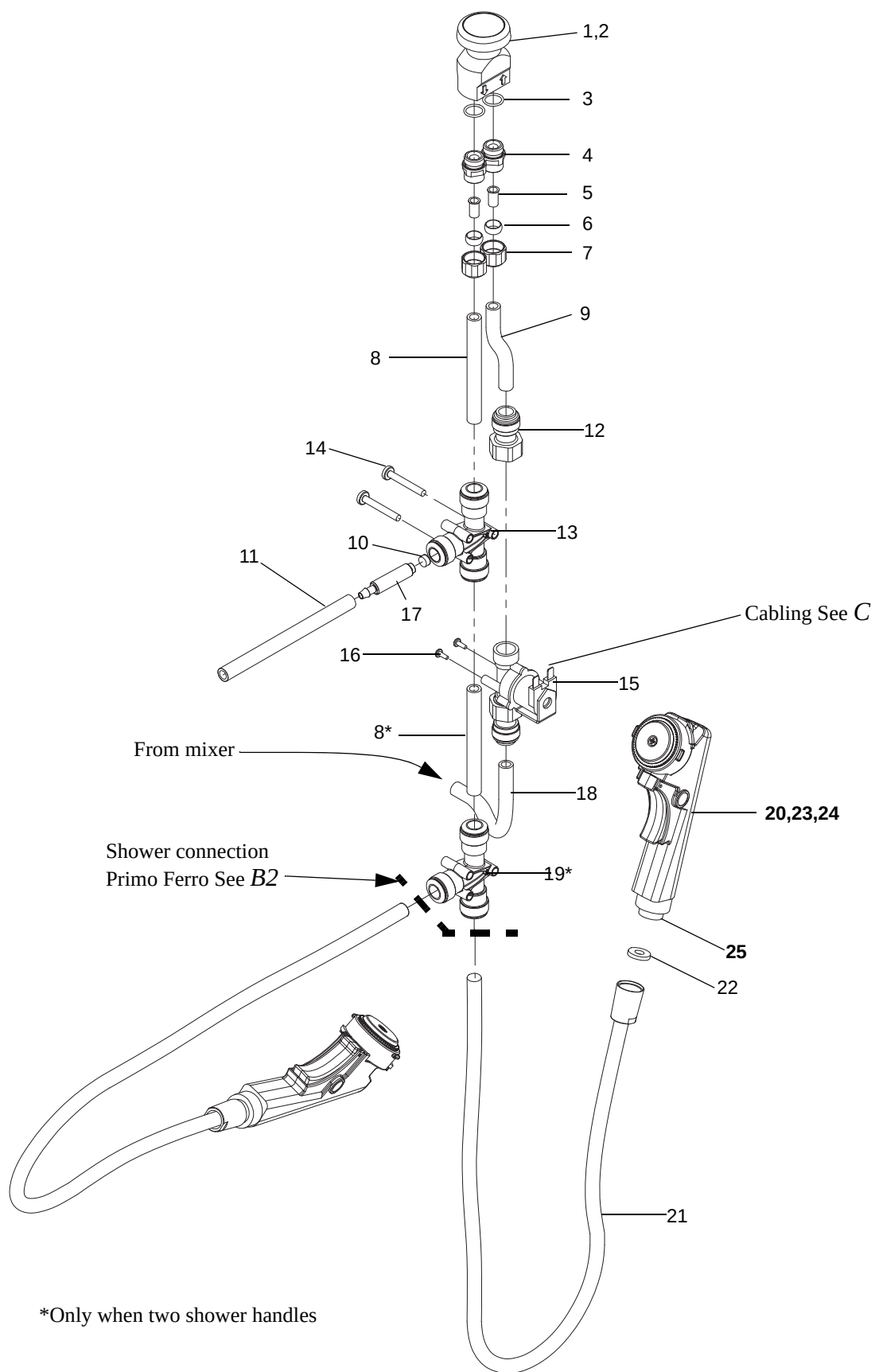F

<i>G Backflow prevention ; Shower, alt. B</i>						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	8458186	1	Pipe, DB			Pipe for old GB must be cut, black mark
2	8358232	1	Vacuumbreaker DB			
3	6310120	3	Plug-in coupling 1/2" 12mm			
4	8458199	1/2*	Pipe			
5	8458198	1/2	T-connection			Old GB =2
6	6190352	2	Screw A4 MRT-TT ISO 14583 M6 x 40			Bumax hard
7	6305076	1	Flow restrictor, drain			Black 1.9l/min
8	6270176	1	Hose 1,0m			
9	6312169	1	Solenoid valve, RPE 1/2"24VAC			
10	6190390	2	Screw FZB RXK ISO 7049 ST 4.2x9.5 -8.8 - FH			
11	8459472	1	Nipple Ø 12			
12	8458231	1	Pipe – feeding shower			
13	8458090	1*	T-connection, shower			
14	S6303096 S6303122	1/2* 1/2*	Shower handle, grey; complete Shower handle, UK; complete			
15	6270413	1/2*	Shower hose			Primo Ferro see B2 November 2013
16	6306009	1/2*	Washer (rubber)			
17	8543697	1/2*	Strainer kit, grey			2 strainers handle, 2 strainers hose, o-ring, screw, plug
18	8543698	1/2*	Repair kit, grey			2 pins, sleeve, spring, 3 o-rings, 2 complete valves with o-rings
19	6907869	1/2*	Swivel, complete grey handle			Incl. strainer
20	8458143	1	Adapter			Old GB only
21	6310095	1	Quick coupling			Old GB only
22	6270301	1m	Hose			Old GB only


*=With two showers

G

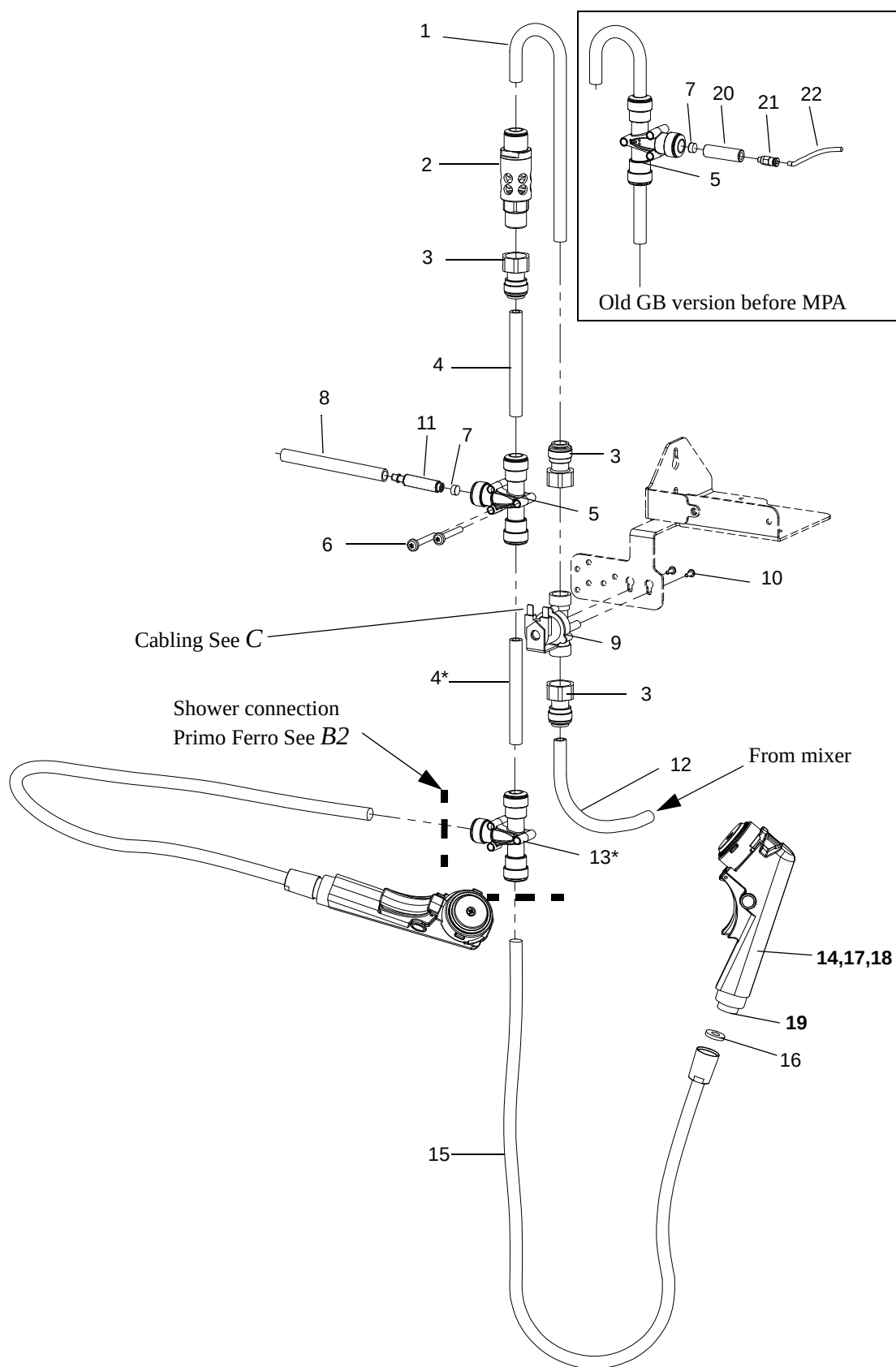


H Lifting unit						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	8258227-031	1	Frame			
2	8458091	1	Actuator			
3	6307186	1	Bolt HCB 10x35 zinc plated			
4	6190531	2	Split pin			
5	8257977	2	Clamp			New version SEE0723390
6	8546750	1	Sliding bush, kit			One for each side
7	8546991	1	Roller blind, kit			
8	8258207-031	1	Actuator attachment, back			
9	6190511	8	Screw A4 MRT-TT ISO 14583 M6 x 25			Bumax hard
10	6210043	8	Locking nut A4-DIN 985 M6-80			
11	8459309	1	Potential equalizer, male			
12	6370284	1	Potential equalizer, female			
13	6131260 6101260	2	Screw A4 MC6S ISO 4762 M12 x 60 Screw A4 M6S ISO 4014 M12 x 60 -80	SEE0723389 SEE0723390	SEE0723389	Bumax 88
14	6200077	2	Toothed washer A4-AZ DIN 6798 M12			
15	8258098-031	1	Actuator attachment, front			
16	8443618	4	Leg attachment bushing			
17	6131040	4	Screw MC6S 10x40 A4 DIN 912			
18	8457562	2	Foot			Incl adapter to fit hole Ø20mm
19	8344073-032	2	Plug			For 1900mm legs
20	8457497-031	2	Foot			For 1600/2150mm legs
21	8346503	2	Cover			For 1600/2150mm legs
22	AAA1600-03 AAA1900-03 AAA2150-03	1 1 1	Legs for 1600 mm bath tub, complete Legs for 1900 mm bath tub, complete Legs for 2150 mm bath tub, complete			New version with screw supporting side plates.
23	8338947-032	1	Plug			For 1600/2150mm legs
24	6101230	2	Screw A4 M6S ISO 4017 M12 x 30 -80			For 1600/2150mm legs
25	6200169	2	Washer A4-RBS ISO 7093 13 x 37 x 3 200HV			
26	6200009	1	Washer A4 BRB ISO 7089 13 x 24 x 2.5-200HV			
27	See P	1	Emergency lowering			
28	6907752	1	Solenoid (lowering valve)			
29	6190545	1	Bolt HCB 10x40 Fzb			
30	6200164 6200019	4	Washer A2 TRBDIN7349 10,5x25x4-100 Washer A4 TBRSE-SS1466 10,5x28x6-200 HV		1307000288 1307000289	
31	8358122-031	1	Fixed height dummy			
32	6303185	2	Spacer	1307000289		
33	6190511	2	Screw A4 – MRT-TT ISO 14583 M6 x 25 BUMAXHARD	1307000289		
34	8458110	2	Support		1307000288	

H

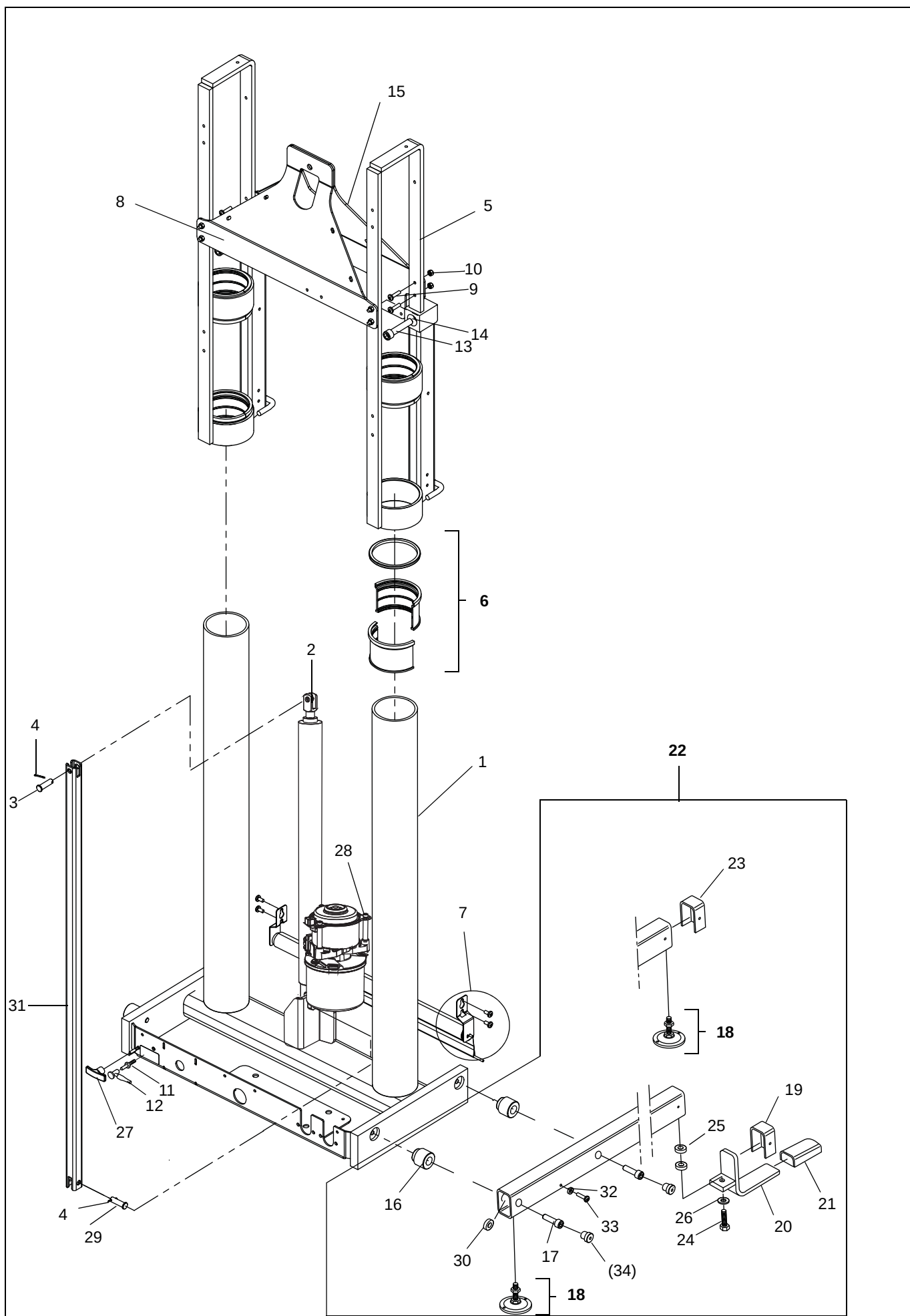


<i>I Disinfection system – Standard</i>						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	8343434	1	Flow meter			
2	8662234-03 8661802-03	1	Bottle connection, standard Bottle connection, P300			
3	6270116	1	Disinfection hose 1 m			
4	8458199	1	Pipe			
5	6310120	1	Plug- in coupling 1/2"x12mm			
6	8543494	1	Injector, complete			
7	6190390	2	Screw FZB RXX ISO 7049 ST 4.2x9.5 -8.8 - FH			
8	8458102	1	Attachment			
9	6260107	1	O-ring			
10	8446147	1	Nut			
11	6312197	1	Flow restrictor, disinfection			Grey, 4l/min, new part replaces previously used flow restrictor, bushing and o ring.from 1202001092. Older tubs before SEE0739419 should also use this new position if the old red flow restrictor is replaced.
12	AH1000441	1	Disinfection bracket			MPA only
13	AH1000879	1	Nipple Male 3/4"- female 1/2"			MPA only
14	6210154	1	Nut 1/2"x5mm NV28			MPA only
15	6313139	1	Washer Ø19xØ13x2			MPA only
16	6301199	1	Double Check Valve			MPA only
17	6250861	1	Reducer (15-10)			MPA only
18	6130337	2	Screw MRT-TT 6x12 A4			MPA only
19	AH1000467	1	Connection pipe disinfection			MPA only
20	PY7043	2	Coupling 1/2"x10mm			MPA only
21	PP0841	1	Solenoid valve 1/2" 24VDC ; MPA			MPA only
22	AH1000450	2	Pressure pipe			MPA only
23	6250890	1	Y-tube-to-tube 10-10			MPA only
24	See E1/E2	1	Disinfection Handle and hose			

I

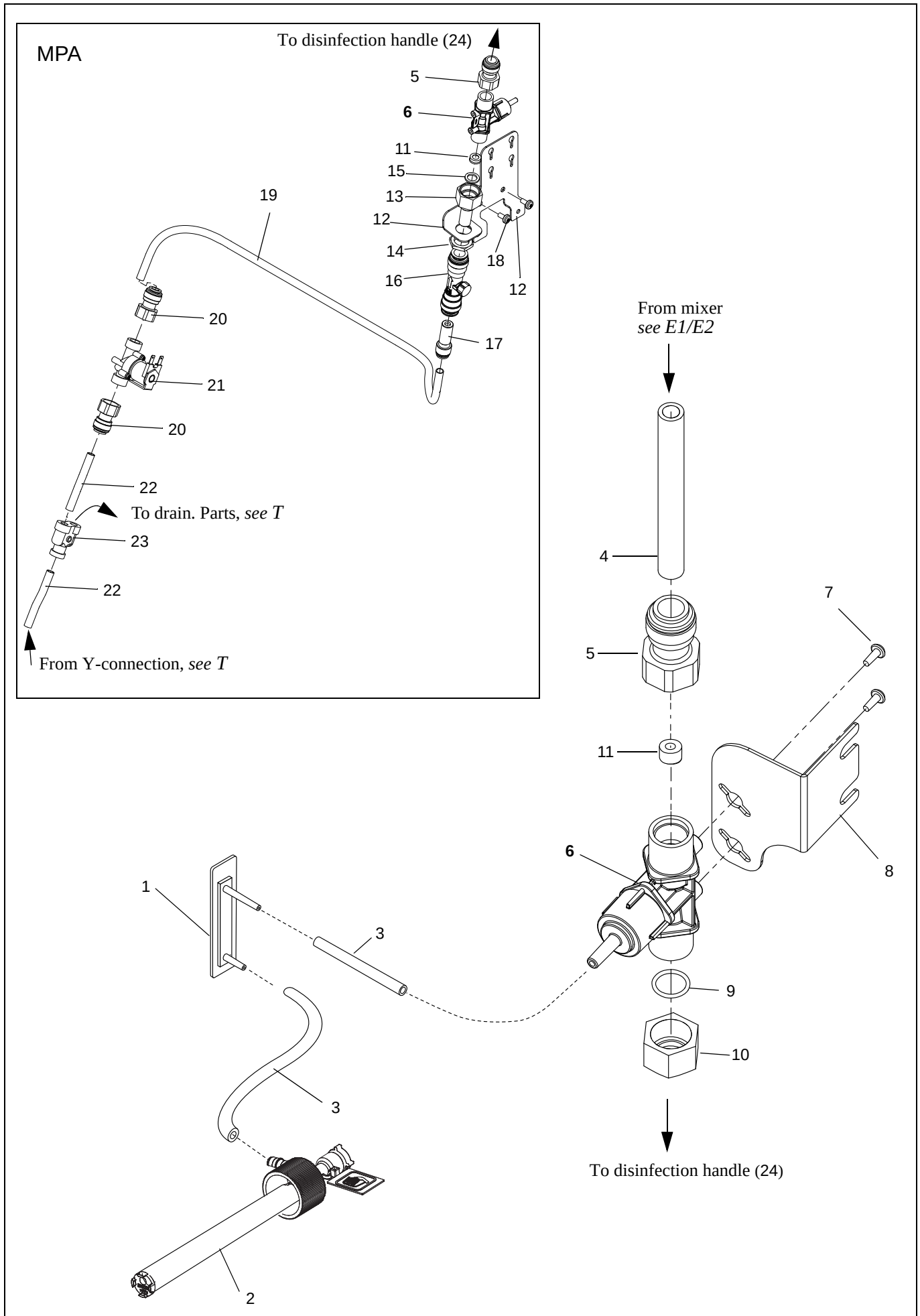

J1 <i>Disinfection system – Automatic P220/P300</i>						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	8458252	1	Pipe			
2	6312170	1	Nut Conex 12mm x G 3/8			
3	6250312	1	Compression ring Conex 12mm			
4	6319493	1	Support Sleeve 10x1			
5	8458249	1	Nipple, 3/4" 12mm			
6	6316138	1	Washer, 24/18x2			
7	6312171	1	Solenoid valve automatic 24VAC			Complete with 3 valves
8	6270363	1m	Hose			a=50mm b=80mm c=380mm d=660mm
9	8663012	1	Air gap hos assembly			
10	6316244	1	Plug			
11	See C1,C2	1	Cabling			
12	PY7043	3	Coupling 1/2"x10mm			
13	8543494	2	Injector, complete			
14	6270116	1m	Hose			
15	6319317	1	Hose nipple			
16	8343434	1	Flow meter			
17	8662234-03 8661802-03	1	Bottle connection, standard Bottle connection, P300			New versions november 2011 P300 illustrated
18	8358243	1	Attachment			
19	6190390	4	Screw FZB RXK ISO 7049 ST 4.2x9.5 - 8.8 - FH			
20	8150670	1	Positive drain			
21	6190404	2	Screw A4 RX PT K4.6x25			
22	8662236	1	Bracket, airgap			
23	6260107	1	O-ring			
24	8446147	1	Nut			
25	PY7056	1	Pipe			
26	PY7038	1	Female stud			
27	6301192	1	Check valve	SEE0705182	1211000217	
28	6305057	1	Flow restrictor, disinfection	SEE0739419		Red, 5l/min
29	8446139	1	Bushing – Flow control			
30	6260105	1	O-ring			
31	6312197	1	Flow restrictor, disinfection			Grey, 4l/min, new part replaces previously used flow restrictor, bushing and o ring from 1202001092
32	PY7037		Male stud			
33	See E1/E2	1	Disinfection Handle and hose			


J1

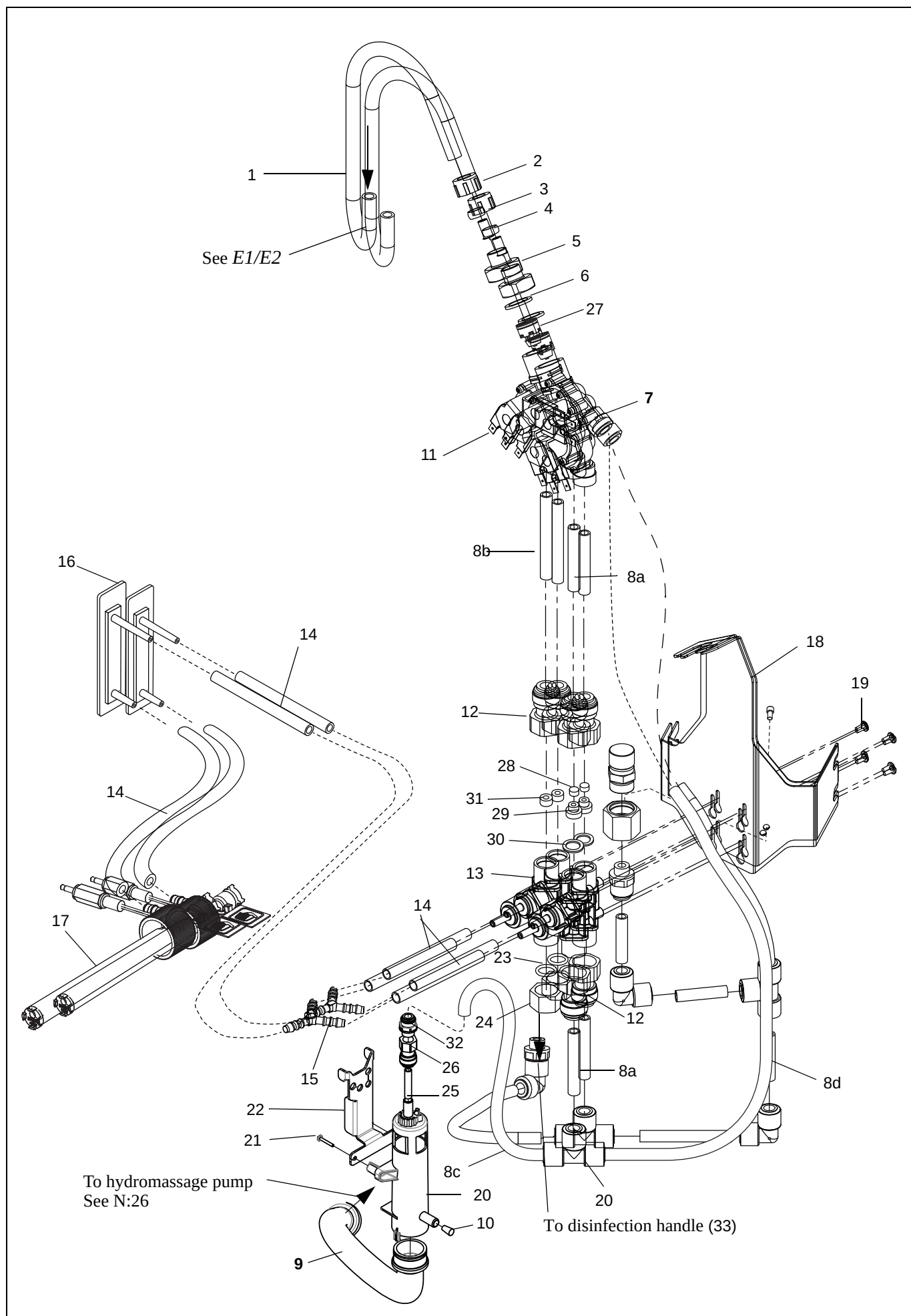

J2 MPA Disinfection system – Automatic P220/P300						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	AH1000441	1	Disinfection bracket			
2	AH1000879	1	Nipple Male 3/4"- female 1/2"			
3	6210154	1	Nut 1/2"x5mm NV28			
4	6301199	1	Double Check Valve			
5	6250861	1	Reducer (15-10)			
6	6190337	2	Screw A4 MRT-TT ISO 14583 M6 x 12			
7	6312178	1	Solenoid Valve 3/4" 3x10mm 24VDC			
8	6316138	1	Washer, 24/18x2			
9	8543494	2	Injector			
10	6190390	4	Screw FZB RXK ISO 7049 ST 4.2x9.5 - 8.8 - FH			
11	PY7043	5	Coupling 1/2"x10mm			
12	AH1000442	1	Constant leak drain			
13	6190404	2	Screw A4 RX PT K4.6x25			
14	AH1000450	3	Pressure pipe			
15	PP0860	1	Equal Tee 10mm John Guest PM0210E			
16	6316244	1	Plug			
17	8663012	1	Air gap assembly			
18	AH1000460	1	Filling Pipe autodisinfection			
19	AH1000463	1	Pipe Clean water autodisinfection			
20	AH1000464	1	Injector Pipe autodisinfection			
21	AH1000465	1	Injector Pipe disinfection hose			
22	AH1000467	1	Connection Pipe disinfection			
23	6310120	1	Plug-in coupling 1/2" 12mm			
24	See E.2	1	Disinfection Handle			
25	6312197	2	Flow restrictor, disinfection			Grey, 4l/min,
26	6250890	1	Y-tube-to-tube 10-10			
27	PP0841	1	Solenoid valve 1/2" 24VDC ; MPA			


J2

Disinfection hose, handle see E2.
Bottle connection see J1.

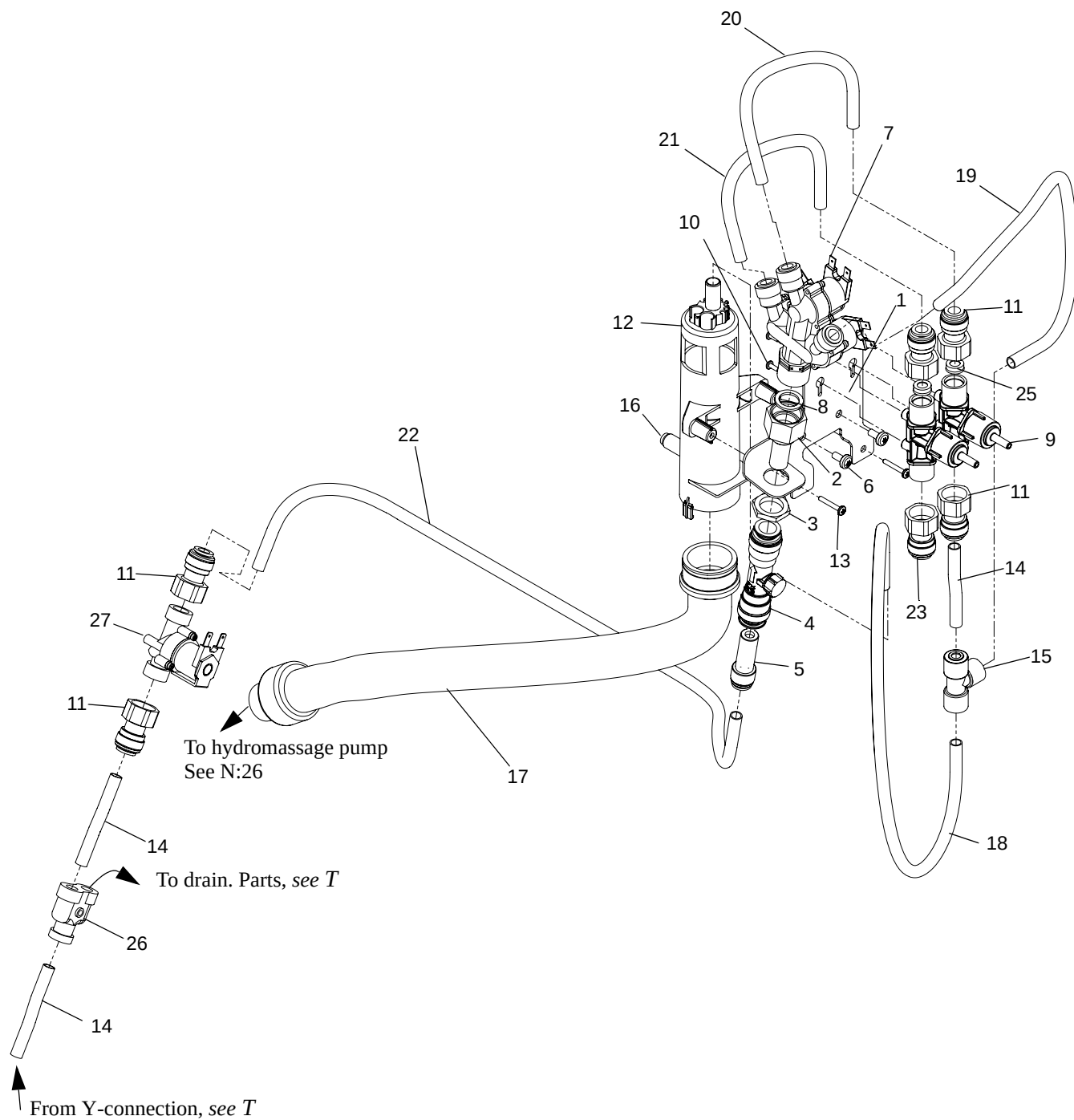


K Pressure Vacuum Breaker						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	8547084		Vacuum breaker kit			Incl. valve and attachment
2	6907933	1	Repair kit Vacuum breaker			
3	6190352	2	Screw A4 MRT-TT ISO 14583 M6 x 40			Bumax hard
4	6190337	2	Screw A4 MRT-TT ISO 14583 M6 x 12			
5	6210028	2	Nut A4-MHM DIN 1587 M6-80			
6	6200003	2	Washer A4 BRB ISO 7089 6.4 x 12 x 1.6-200HV			
7	8458127	1	Clamp			
8	6260108	1	O-ring			
9	6319493	2	Support Sleeve 10x1			
10	8458181	2	Nipple			
11	6250312	2	Compression ring Conex 12mm			
12	6312170	2	Nut Conex 12mm x G 3/8			
13	8458124	1	Feeding pipe, PVB			
14	8459472	1	Nipple Ø 12			
15	6305076	1	Flow restrictor, drain			Black 1.9l/min
16	8458198	1	Leak connector			
17	6270176	1m	Hose			
18	6190390	2	Screw FZB RXK ISO 7049 ST 4.2x9.5 - 8.8 - FH			
19	6312169	1	Solenoid valve, RPE 1/2"24VAC			
20	6310120	2	Plug-in coupling 1/2" 12mm			
21	8358233	1	Feeding pipe disinfection			
22	8458123	1	Pipe			

K

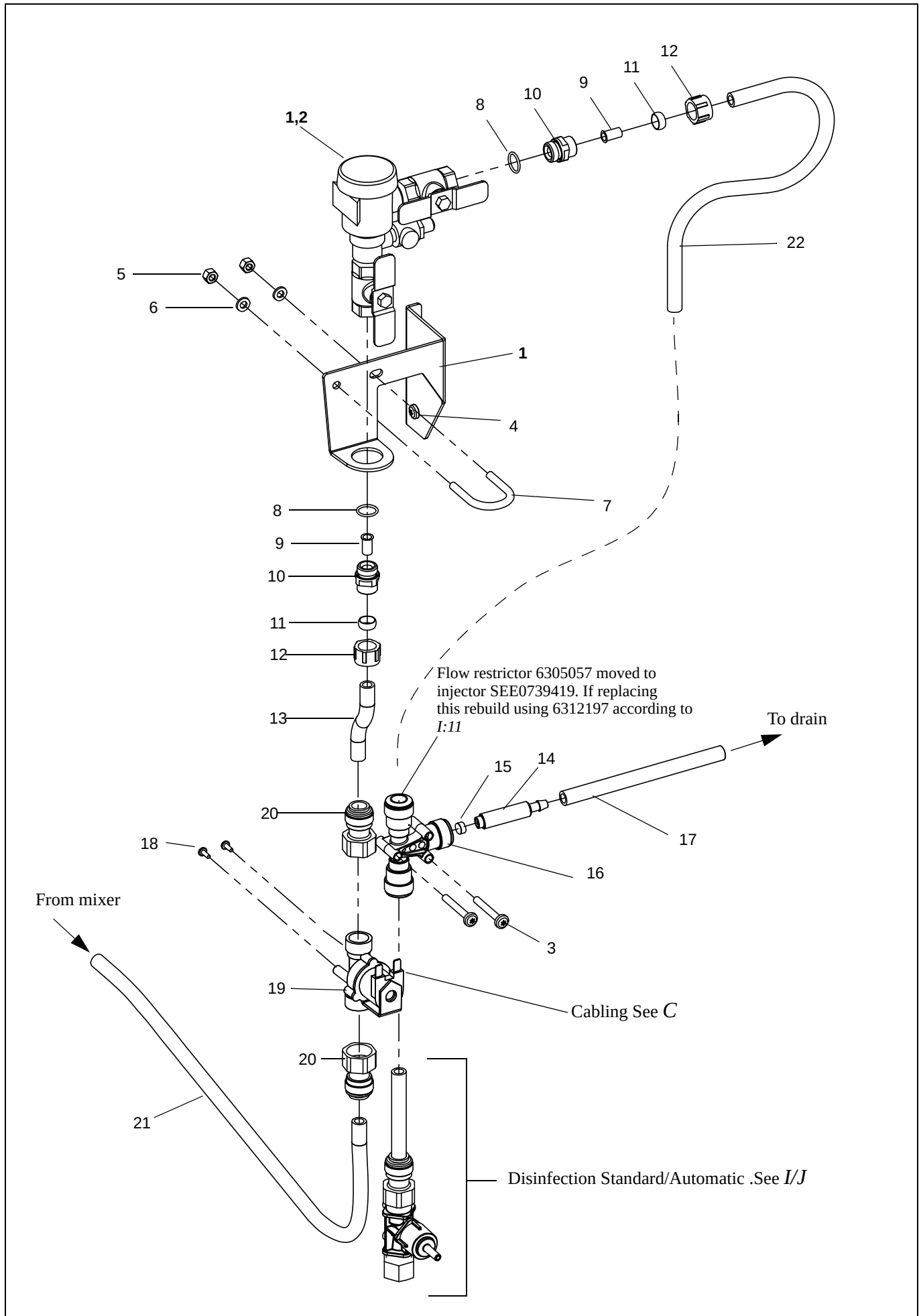



<i>L</i> <i>Dispenser unit</i>						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
	8558438	1	Dispenser unit,complete			
1	8358434	1	Cable – disinfection pump			
2	6120408	2	Screw A4 MCS ISO 1207 M4x8-80			
3	6200080	2	Washer A4 – AZ DIN 6798 M4			
4	6210012	2	Nut A4 M6M ISO 4032 M5 – 80			
5	8458113	1	Bracket			
6	6190337	2	Screw A4 MRT-TT ISO 14583 M6 x 12			
7	6371056	1	Pump box			
8	6210011	4	Nut M6M M4 A4 DIN 934			
9	6316290	2	Pump house with hose			
10	6190463	2	Screw A4 MFX ISO 7046 M3 x 30–80Z			
11	8661804-03	1	Bottle connection – oil			New version1111000346
12	8661803-03	1	Bottle connection – shampoo			New version1111000346
13	6319228	2	Hose clamp			
14	S8450983	2	Nozzle			
15	6312174	2	Motor			

L

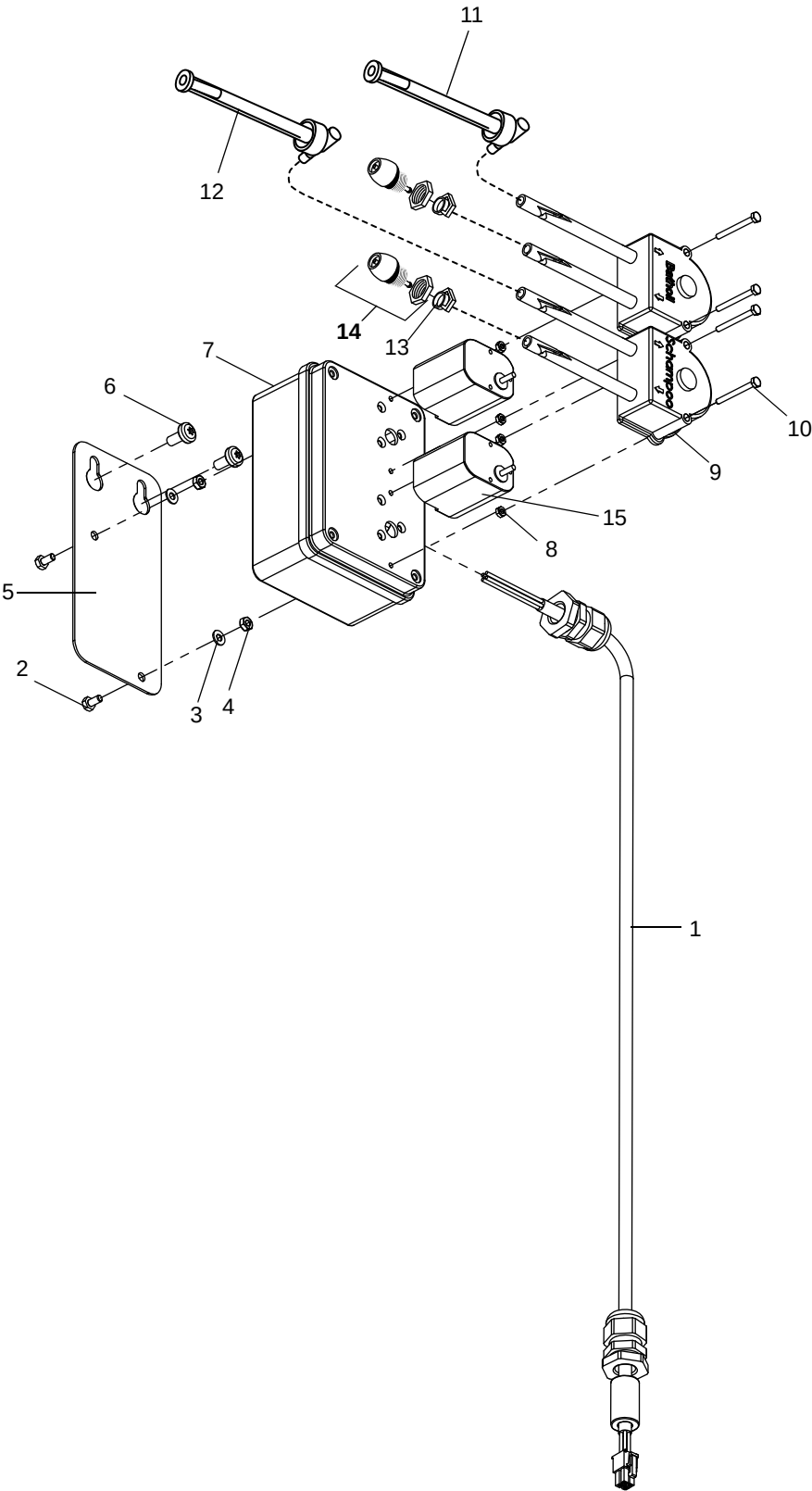

<i>M</i> <i>Power box</i>						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
	8558410-230	1	Complete unit 230V			
	8558410-120	1	Complete unit 120V			
	8558410-100	1	Complete unit 100V			
1	8239288	1	Lid			
2	8664078	1	PCBA Power Board: P220, P300			Fuses F1 and F2 not included
3	6320221 6320227	1	Transformer 120/230V Transformer 100V			
4	6190525	4	Screw FZB MC6S ISO 4762 M5x10-12.9			
5	8239290	1	Box			
6	6379122	1	Fuse holder			
7	6379007	1	Fuse T315mAL (5x20), all versions			F5
	6379121	1	Fuse T630mAL (5x20), all versions			F3
	6379112	1	Fuse T1AL (5x20), all versions			F6
	6379106	1	Fuse T3,15AL (5x20), all versions			F4
	6379103	1	Fuse T4AL (5x20), 230V			F2 (230V)
	6379095	1	Fuse T6,3AL (5x20), 120V,230V			F2 (120V), F1 (230V)
	6379014	1	Fuse T8AL (5x20), 100V			F2
	6379153	1	Fuse T10AL (5x20), 100,120V			F1
	6379240	1	Fuse T12AH (6,3x32), all versions			F7
8	8318616	1	Label 230V			
	8318615	1	Label 120V			
	8318614	1	Label 100V			
9	6210043	1	Locking nut A4-DIN 985 M6-80			
10	8444758	1	Cable lowering valve			
11		1	Cable, power box-hydro			See O
12	8358372	1	Cable, power box-front panel			
13	8358705	1	Cable- actuator			
14	S8445780	1	Sensor cold water			P300 only
15	S8445781	1	Sensor hot water			P300 only
16	8442059	1	Power cable 100V/230V			
	8358609		Power cable 120V			
17	8418563	1	Label, warning			
18	6319496	2	Reduction			P300 only
19	8459595	1	Seal			
20	8360604	1	Power cable adapter			Used with S&V and MPA

M

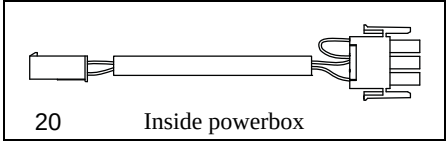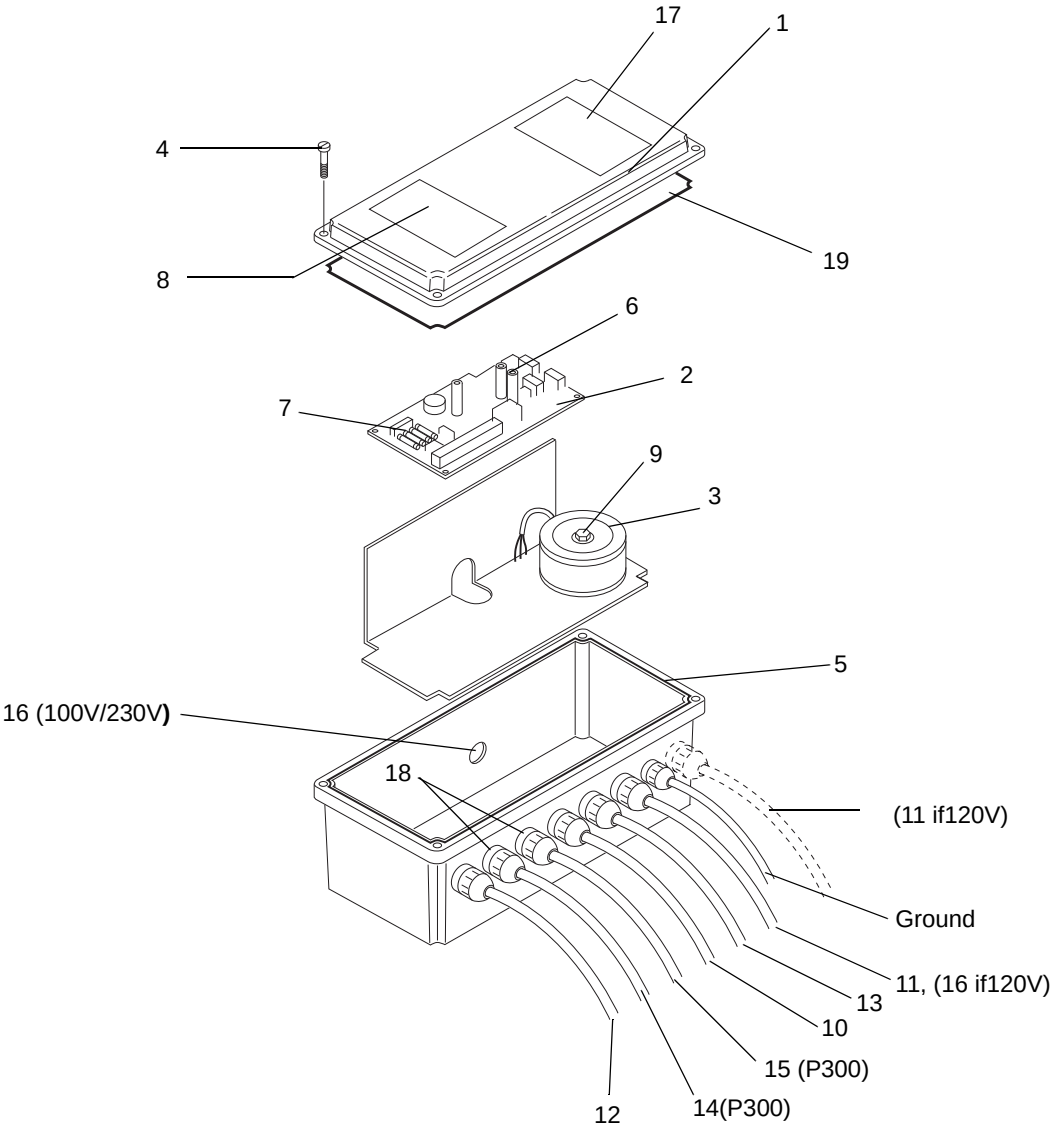

N <i>Hydromassage pump</i>						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	8440098	1	Sieve			
2	8258264	1	Inlet nozzle			
3	8442267	1	Gasket			
4	8436760	1	Nut 32			
5	6190337	2	Screw A4 MRT-TT ISO 14583 M6 x 12			
6	8358189	1	Hydro bracket			
7	8444921	1	Angle bracket			
8	8361830-120 8361830-220 8361830-230	1 1 1	Hydromassage pump, US/JP Hydromassage pump, Korea Hydromassage pump, EU			New version 0911001242.Hydro bracket included
9	6190151 6100512	2 2	Screw A2 - MFX ISO 7046 M5x10-Z Screw A4 - M6S ISO 4017 M5 x 12-80			System 2000 Primo Ferro
10	6319259	1	Hose nipple, straight			
11	6270218	1m	Hose 1 m			
12	8344550	1	Hydro nozzle			
13	8445782	1	Gasket			
14	S8559282	1	Hydro hose inlet, complete			Incl. 4 hose clamps and angle
15	8341958	1	Hydro outlet, double			
16	8440407	1	Hydro outlet, single			
17	6210159	1	Hydro hose nut			
18	6319110	6	Hose clamp SS			
19	6907910 6907911	1	Pump housing US/JP, Korea Pump housing EU			
20	6312082	1	Nut R1/4" t=4.5mm			
21	8546498	1	Ball valve complete			Incl. parts for use with old knob
22	8565046	1	Knob with label			New version VEPL00006462
23	6319296	1	Hose nipple			
24	6301124	1	Plug			
25	8359580	1	Hydro hose outlet, complete		1211000216	
26	8663018		Hydro hose outlet, with disinfection intake	1211000217		For autodisinfection system
27	6190300B	2	Screw A4 FXS ISO 7050 ST3.5x16-HZ			Introduced 1401000498
28	6319042	1	Hose clamp SS 26-38			For autodisinfection system
29	6190463	1	Screw A4 MFX ISO 7046 M3x30-80-Z			




O

Hydrosound unit



Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	8556006	1	Transducer			
2	8358190	1	Hydro plate			
3	6190337	2	Screw A4 MRT-TT ISO 14583 M6 x 12			
4	6100620	2	Screw A4-M6S ISO 4017 M6 x 20-80			
5	6200003	2	Washer A4 BRB ISO 7089 6.4 x 12 x 1.6-200HV			
6	8546985	1	Internal switch			Incl. Knob and Label Not for P300
7	8359276	1	Switch box, complete			Not for P300
8	S8357145	1	Gasket			
9	8446500	1	Lens			Not for P300
10	8458696	1	Label			Not for P300
11	6210016	1	Nut M6M M12 A4			Not for P300
12	S8556010-230 S8556010-120 S8556010-100	1	Hydrosound unit without transducer -EU P300,P220, P200+, Hydrosound unit without transducer -US P300, P220, P200+ Hydrosound unit without transducer -JP P300, P220, P200+			
13	8256981	1	Cover switchbox			Primo Ferro only
14	6190157	2	Screw MFX 5x16 A2			
15	8356014	1	Transducer cable			
16	8359277	1	Cable to powerbox			

O

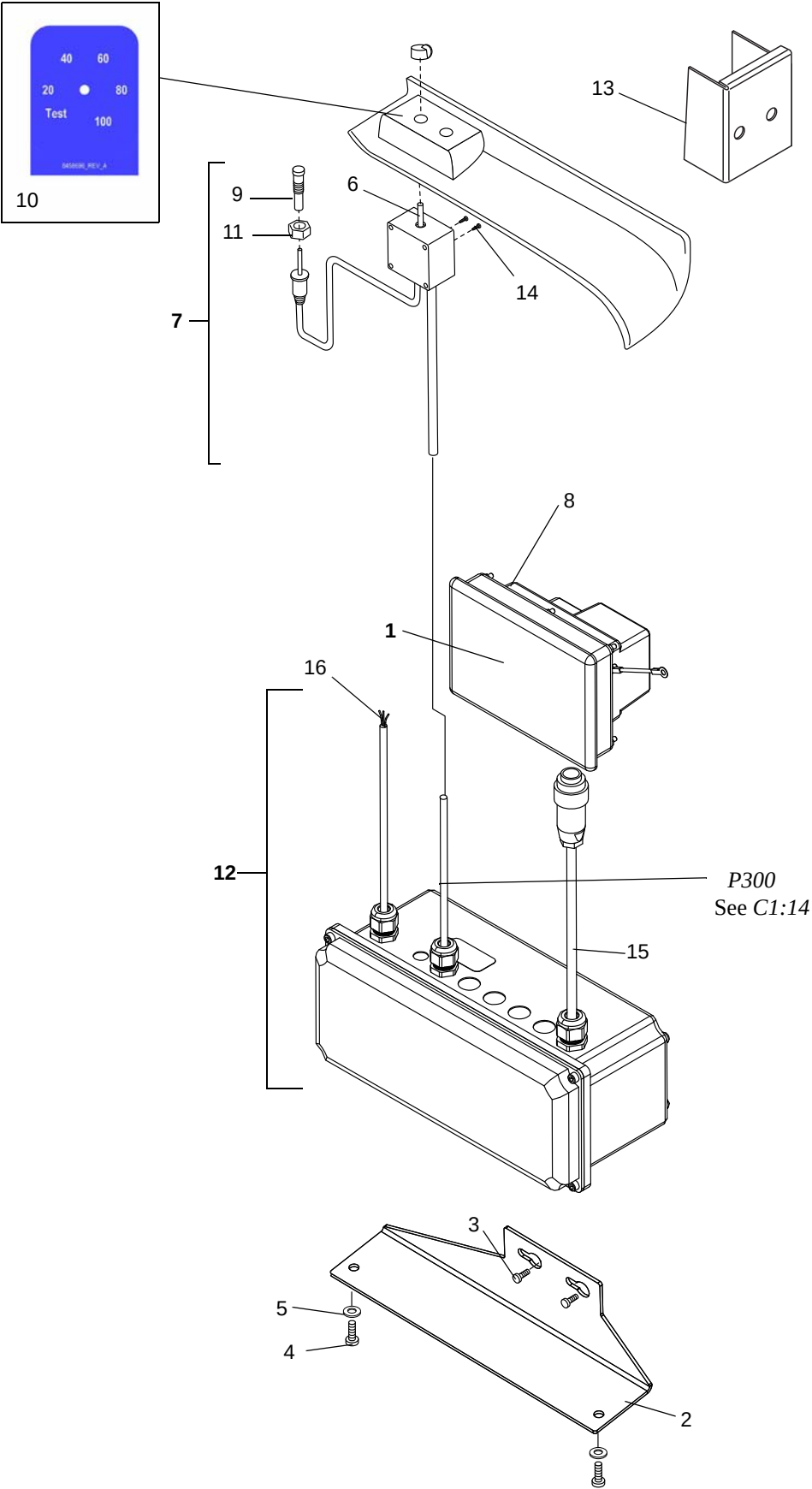





<i>P</i> <i>Emergency lowering</i>						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	8359475	1	Attachment			
2	8359480	1	Lowering switch with handle			
3	8459482	1	Box			
4	6190099	2	Screw A4 RXS ISO7049 ST5.5x16-80 CZ			
5	6370182	2	Skintop nut PG9			
6	8359481	1	Cable, lowering valve			
7	8546995	1	Battery pack			
8	6200001	2	Washer A4 BRB ISO 7089 4.3 x 9x 0.8- 200HV			
9	8459494	1	Internal wiring			
10	6316293	1	Gasket			
11	6210041	2	Locking nut M4 A4 DIN 985			
12	6120416	4	Screw A4 MCS ISO 1207 M4x16-80			
13	6130320	4	Screw A4 MC6S ISO 4762 M3x20- BUMAX			
14	6210040	4	Locking nut A4 DIN 985 M3-80			
15	6210206	8	Nut A4 ML6M ISO 4035 M3- 035			
16	8664325	1	PCB			
17	6250854	1	Bracket			
18	6210017	1	Nut A4 M6M ISO 4032 M16-80			
19	6301018	1	Bumper			

P

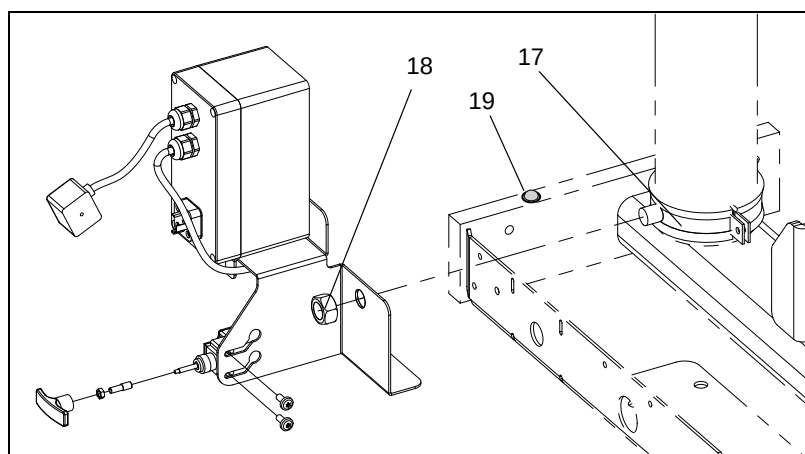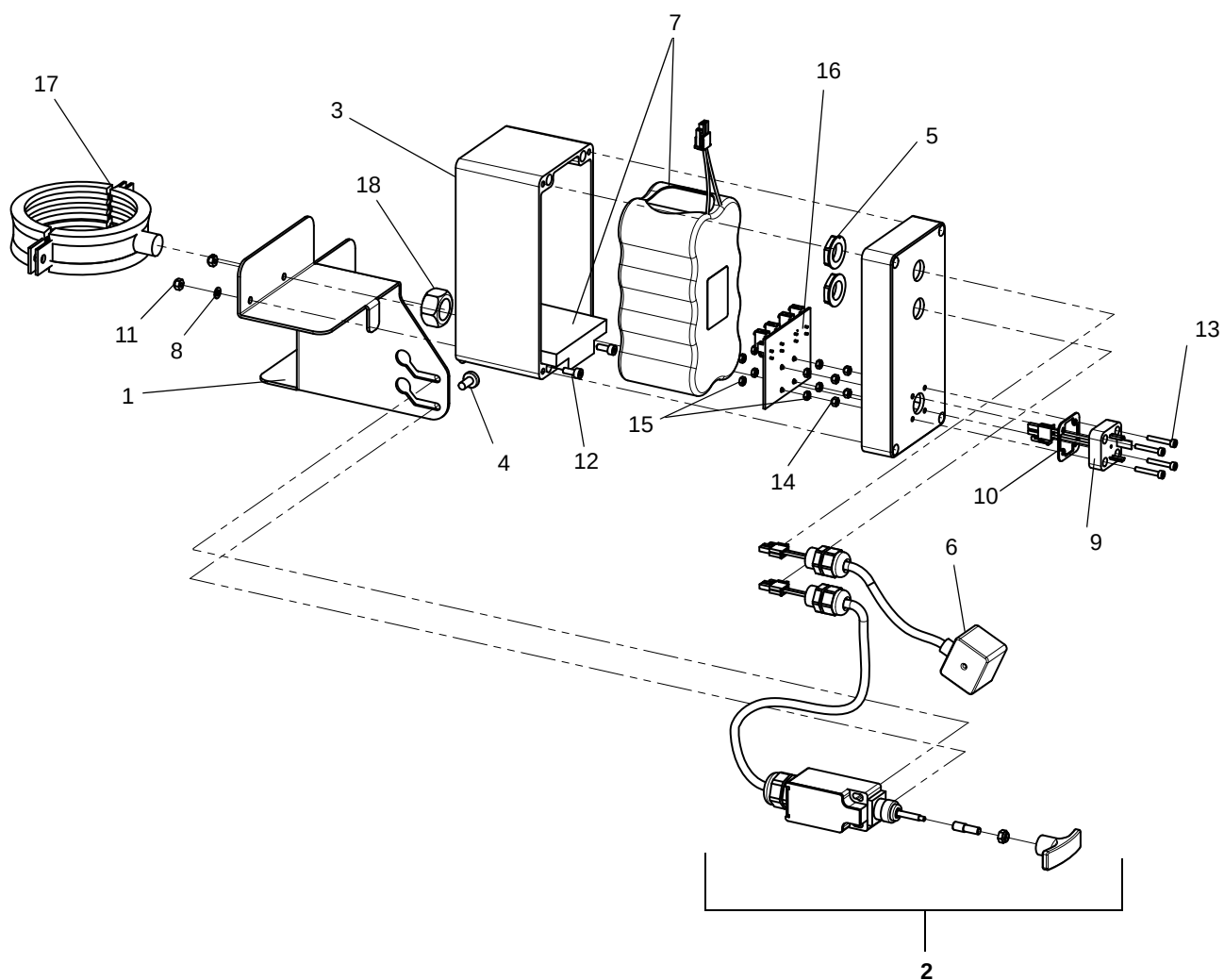


Q Accessories						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	ARA0500-03	1	Treatment hose, Hydromassage			Including holder
	6307067-04	1	Holder for treatment hose			Not. illustrated
2	ARA0043-04	1	Booster pump (UK market only)			
3	ARA 0300-031	1	Foundation frame			
4	ARA 0007-01	1	Toiletry tray			

Q

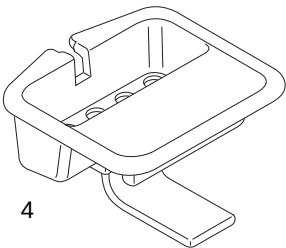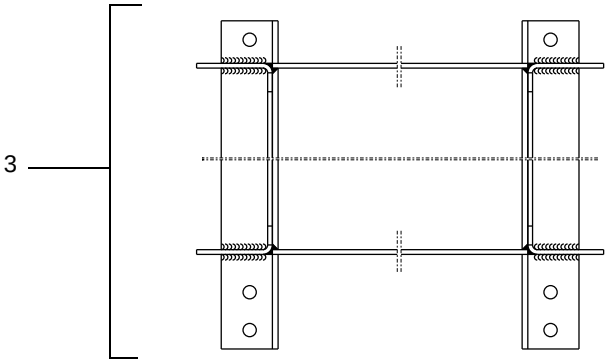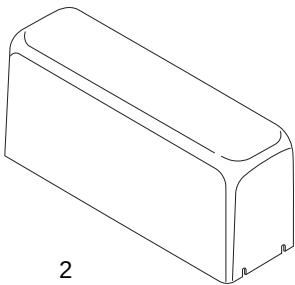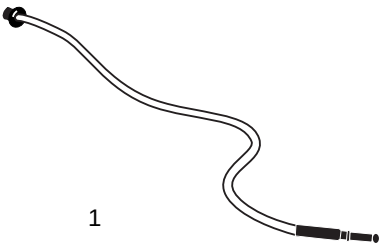


R Sound and Vision (S&V)						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	S8561139	1	Control Box, complete			Including pos 2-6,11,21
2	8663757	1	PCB			In production 1310001236
3	6320235	1	Transformer			
4	6379038	1	Fuse T500mAL 5x20mm			
5	6379093	1	Fuse T2AL 5x20mm			
6	6130416 6130420	4	Screw A4 MC6S ISO 4762 M4 x 16-70 Screw A4 MC6S ISO 4762 M4 x 20-70	131000123 6	131000123 5	
7	6210041	4	Locking nut M4 A4 DIN 985			
8	6200001	4	Washer A4 BRB ISO 7089 4.3x9x0.8-200HV			
9	8360886	1	Bracket			
10	6190337	2	Screw A4 MRT-TT ISO 14583 M6 x 12			
11	8360697	1	USB Cable			
12	6210256	1	Nut			New part number
13	6200171	1	USB cover			
14	6316314	1	Seal			
15	6190234	1	Screw A2-K6S ISO 7380 M5 x 12-70			
16	6200002	1	Washer A4 BRB ISO7089 5.3x10x1-200HV			
17	6210042	1	Locking nut M5 A4 DIN 985			
18	8360872	1	IR receiver cable			
19	8461202	1	IR receiver clip			
20	6190276	1	Screw A4 RX PT K3.73x10			
21	8360506	1	Power cable			
22	8661895	1	Gasket			
23	8360890	1	Remote control			Incl. holder
24	8663637	1	Remote control holder			
25	8261138	2	LED and cabling			
26	6371062	2	Speaker incl cabling			
27	8460892	1	Cable duct 145 mm			
28	8460893	1	Cable duct 220 mm			
29	8461314	1	USB memory			Incl. Nine Classic Works

R

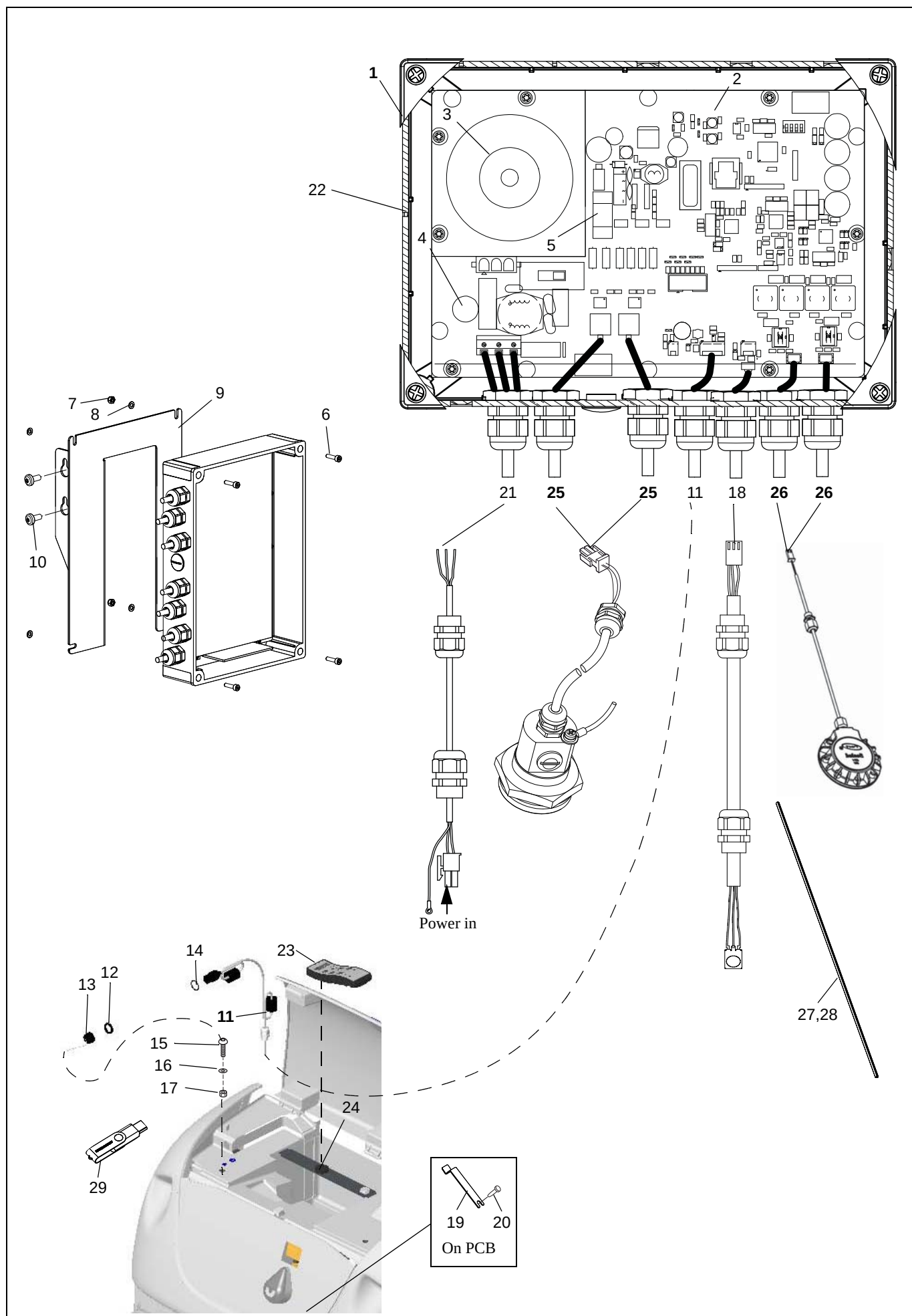


S <i>MPA Control Box</i>						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	AH1000425	1	Control Box Bracket			Not illustrated.
2	AH1000430	1	Box			
3	6190337	2	Screw A4 MRT-TT ISO 14583 M6 x 12			For bracket to tub.Not illustrated.
4	8661895	1	Gasket			
5	6200057	8	Washer A4- RBS ISO 7093 4.3x12x1-200HV			For bracket to box
6	6130420	4	Screw A4 MC6S ISO 4762 M4 x 20–70			For bracket to box
7	6210041	4	Locking nut M4 A4 DIN 985			For bracket to box
8	6130516	4	Screw A4 MC6S ISO 4762 M5x16-80			For lid
9	AH1000550	1	PCBA MPA/S2K			
10	6190276	7	Screw A4 RX PT K3.73x10			
11	6100412	1	Screw A4 - M6S ISO 4017 M4 x 12 – 70			
12	6320232	1	Transformer AA-061445-ME			
13	6200054	1	WasherA4-RBS ISO 7093 6.4x18x1.6-200HV			
14	6100640	1	Screw A4 - M6S ISO 4014 M6 x 40 – 80			
15	6200159	2	Toothed washer A4 – AZ DIN 6798 M4			
16	6303116	1	Spacer screw DSM M4 x 25			
17	6210011	1	Nut A4 M6M ISO 4032 M4 – 80			
18	6301084	1	Cable tie 92 X 2,4 mm			
19	6370182	3	Skintop nut PG9			
20	6370056	6	Jam nut GMP 11			
21	AH1000529	1	Cable Pump			
22	6307074	1	Plug Univ Mate N Lok Plug 2			To Cable pump
23	AH1000530		MPA Mains cable			
24	AH1000526	1	Cable Fill valve 706			Only with Autodisinfection
25	AH1000525	1	Cable Fill valve 708			
26	AH1000537	1	Control Cable S2000			
27	AH1000522	1	Cable Fill valve 709			
28	AH1000527	1	Cable Level sensor			
29	AH1000795	1	Cable Pressure Switch			
30	AH1000535	1	Adaptor Power Cable			Only with S&V (Inside MPA box)
31	8261611	1	Lid			Not illustrated
32	AH1000536	1	Power Cable MPA to S&V			Only with S&V
33	8547101	1	Jumper			10 Pcs.

S

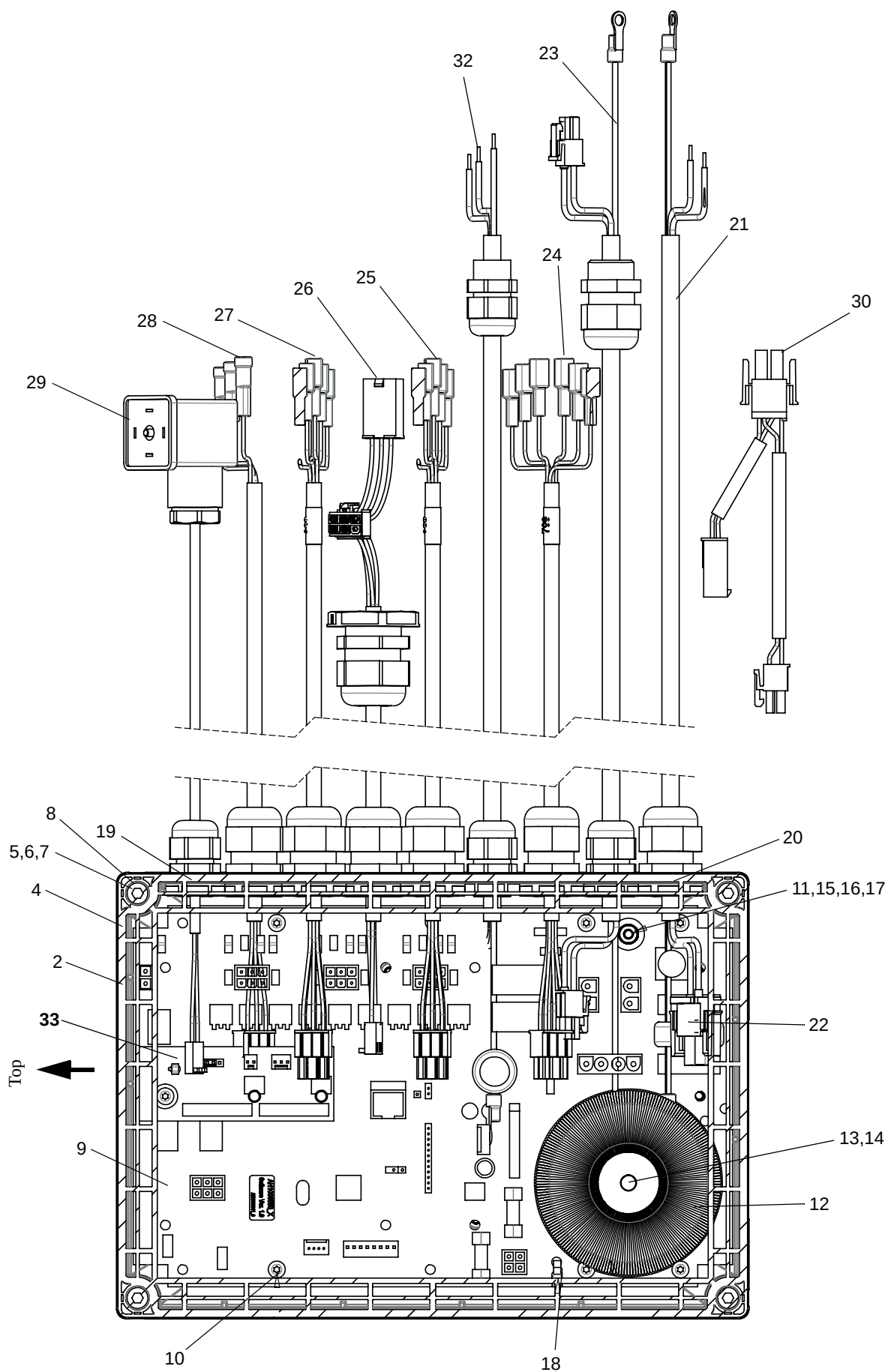

T MPA Tank unit						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	AH1000416	1	Solenoid Valve Holder			
2	6190337	6	Screw A4 MRT-TT ISO 14583 M6 x 12			
3	6312179	1	Solenoid Valve 3/4" 3x10mm "24 VDC			3 Flow restrictors (Fill valve 709)
4	AH1000414	1	Nut 3/4"			
5	8458249	1	Nipple, 3/4" Conex 12mm			
6	6316138	1	Washer, 24/18x2			
7	6250312	1	Compression ring Conex 12mm			
8	6312170	1	Nut Conex 12mm x G 3/8"			
9	See G	1/2	Shower handles			
10	AH1000413	1	Tank Filling Pipe			
11	6319493	1	Support Sleeve 10x1			
12	AH1000415	1	Tank Bracket			
13	8359821	1	Tank			
14	8459823	1	Tank Partition			
15	8359822	1	Tank Lid			
16	6190551	3	Screw Fe/Zn-Ni-PT WN1451 K50x16			
17	6316294	3	Plastic short cartridge			
18	8459825	1	Sensor rod short			
19	8459824	2	Sensor rod long			
20	6322107	1/2	Elbow 15mm 90°			With single shower 2
21	AH1000419	1	Pump Bracket			
22	AH1000418	1	Pressure Pump			
23	6100620	4	Screw A4-M6S ISO 4017 M6 x 20-80			
24	6200003	4	Washer A4 BRB ISO 7089 6.4 x 12 x 1.6-200HV			
25	6210043	1	Locking nut A4-DIN 985 M6-80			
26	AH1000411	1	Male Connector 15x1/2"			
27	AH1000449	1	Suction Hose			
28	6250861	2	Reducer (15-10)			
29	PP0860	1	Equal Tee 10mm John Guest PM0210E			
30	6310123	1	Stem adapter			
31	6301199	1	Double Check Valve			
32	AH1000446	1	Pressure Sensor Nipple			
33	AH1000450	5	Pressure Pipe			
34	AH1000442	1	Constant leak Drain			
35	AH1000453	1	Pressure sensor			
36	6361008	2	Cable tie			
37	AH1000468	1	Tank Fill Pipe 1			
38	AH1000469	1	Tank Fill Pipe 2			
39	AH1000470	1	Tank Fill Pipe 3			
40	AH1000471	1	Tank Water Circulation Pipe			
41	PP0841	2	Solenoid valve 1/2" 24VDC ; MPA			
42	6250890	1	Y-tube-to-tube 10-10			
43	PY7043	4	Coupling 1/2"x10mm			
44	PP0867	1	Flow restrictor 6l/m			
45	100061926	1/2	Shower hose			1 Single/2 Double
46	AH1007014	1	T-connection			Double shower

T

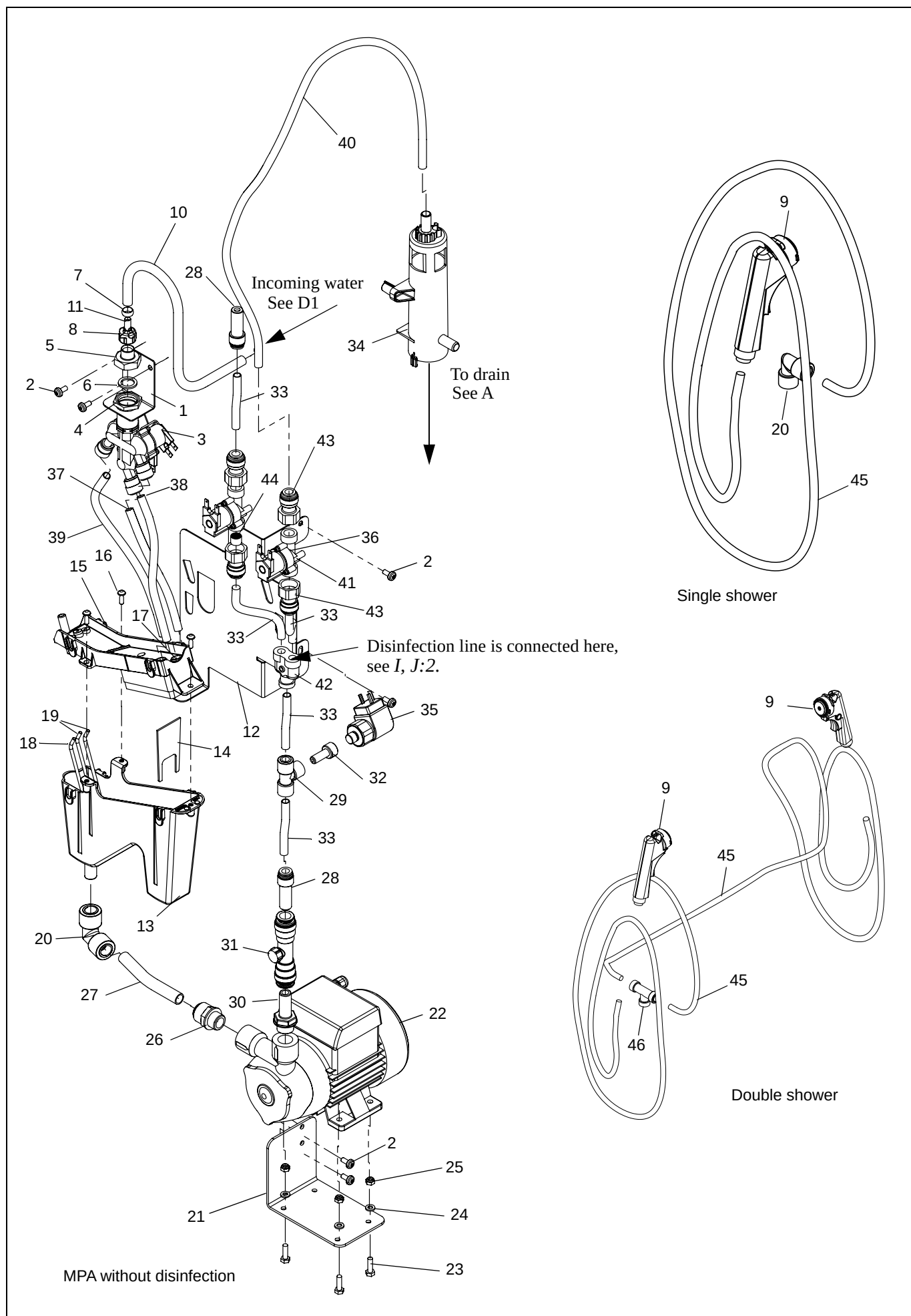

P200+/P220/P300

Preventive Maintenance Parts/Recommended Spares

Preventive Maintenance Parts	Number	Parts needed P200+	Parts needed P220	Parts needed P300	Page
Battery for thermometer; Lithium CR 2032 (Up to SEE0714190)	-	1	1	-	
Battery pack, with optional emergency lowering	8359487	1	1	1	P7
Cog belt	6307185	-	-	1	D1:1
Connection hose DVGW, l=618mm	6270394	2	2	2	D1:40 D2:45
Connection hose WRAS GB, l=618mm	8563020-618	2	2	2	D1:41 D2:46

P200+/P220

Preventive Maintenance Parts/Recommended Spares

Recommended spares	Part number	Number of baths		Page
		1	5	
Bottle connection, disinfection	8662234-03	1	2	I2 J:18
Bottle connection, oil	8661804-03	1	2	L11
Bottle connection, shampoo	8661803-03	1	2	L12
Cabling, level 3 pin	8353132	1	1	C2:3 C3:3
Control knob, mixer	8239131-031	1	1	D2:2
Disinfection hose, grey	6303107	1	2	E1:20, E2:13
Filter	6907788	2	4	D2:27
Flow restrictor disinfection, red 5l/min.	6305057	2	4	J28
Flow restrictor disinfection, grey 4l/min	6312197	1	2	I11
Flow restrictor drain, black 1.9l/min	6305076	2	4	G7, F10 E1:11, E2:8 K15
Flowmeter	8343434	1	2	I1 J17
Injector, complete	8543494	1	2	I6 J14
O-rings kit mixer	8546655	1	2	D2:24
Pump house with hose	6316290	2	4	L9
Repair kit mixer	8547097	1	2	D2:17
Repair kit shower handle grey	8543698	2	3	G18, F24
Repair kit shower handle, yellow	8545142	1	2	E1:23, E2:18
Repair kit solenoid valve 3/4" Repair kit solenoid valve 3/4" UK/JPN	S6907723 S6907794	1	2	D2:28
Shower handle yellow	S6303097	1	2	E1:19, E2:16
Shower handle, grey	S6303096	2	4	G14, F20
Shower hose Ferro, connection (1:st shower)	6270344	1	3	B2:32
Shower hose Ferro, connection (2:nd shower)	6270345	1	3	B2:33
Shower hose Ferro, grey	6270415	1	3	B2:27
Shower hose, grey	6270413	1	2	G15 ,F21
Shower hose, grey, MPA	100061926	1	2	T43
Solenoid valve automatic 24VAC(P220 only)	6312171	1	1	J7
Solenoid valve RPE 3/4" 24VAC	8565044	1	2	D2:26
Solenoid valve, RPE 1/2" 24VAC	6312169	1	2	E1:17, E2:10 F15, G9, K19
Solenoid Valve 3/4" 3x10mm 24VDC; MPA	6312178	1	2	J2:7
Solenoid Valve 3/4" 3x10mm "24 VDC;MPA	6312179	1	2	T3
Solenoid valve 1/2" 24VDC ; MPA	PP0841	1	2	J2:27, T41
Strainer kit shower handle grey	8543697	1	2	G17, F23
Strainer kit shower handle, yellow	8543696	1	2	E1:22, E2:17
Thermometer up to 1403000105	8559259	1	2	C2:8, C3:8
Thermometer from 1403000106	8663047			

Recommended spares	Part number	Number of baths		Page
		1	5	
Bottle connection, disinfection	8661802-03	1	2	I2, J18
Bottle connection, oil	8661804-03	1	2	L11
Bottle connection, shampoo	8661803-03	1	2	L12
Cabling, level	8345000	1	1	C1:3
Disinfection hose, grey	6303107	1	2	E1:20, E2:13
Filter	6907788	2	4	D2:44
Flow restrictor disinfection, red 5l/min	6305057	2	4	J28
Flow restrictor disinfection, grey 4l/min	6312197	1	2	I11, J31
Flow restrictor drain, black 1.9l/min	6305076	2	4	G7, F10 E1:11, E2:8 K15
Flowmeter	8343434	1	2	I1, J17
Injector, complete	8543494	1	2	I6, J14
O-rings kit mixer	8546655	1	2	D1:20
Pump house with hose	6316290	2	4	L9
Repair kit mixer	8547097	1	2	D1:12
Repair kit shower handle grey	8543698	2	3	G18, F24
Repair kit shower handle, yellow	8545142	1	2	E1:23, E2:18
Repair kit solenoid valve 3/4" Repair kit solenoid valve 3/4" UK/JPN	S6907723 S6907794	1	2	D1:24
Shower handle yellow	S6303097	1	2	E1:19, E2:16
Shower handle, grey	S6303096	2	4	G14, F20
Shower hose Ferro, connection (1:st shower)	6270344	1	3	B2:32
Shower hose Ferro, connection (2:nd shower)	6270345	1	3	B2:33
Shower hose Ferro, grey	6270415	1	3	B2:27
Shower hose, grey	6270413	1	2	G15, F21
Shower hose, grey, MPA	100061926	1	2	T43
Solenoid valve automatic 24VAC	6312171	1	2	J7
Solenoid valve RPE 3/4" 24VAC	8565044	1	2	D1:22
Solenoid valve, RPE 1/2" 24VAC	6312169	1	2	E1:17, E2:10 F15, G9, K19
Solenoid Valve 3/4" 3x10mm 24VDC; MPA	6312178	1	2	J2:7
Solenoid Valve 3/4" 3x10mm "24 VDC; MPA	6312179	1	2	T3
Solenoid valve 1/2" 24VDC ; MPA	PP0841	1	2	J2:27, T41
Strainer kit shower handle grey	8543697	1	2	G17, F23
Strainer kit shower handle, yellow	8543696	1	2	E1:22, E2:17

System 2000

Service Articles and Special Tools

Service Articles	Type	Quantity	Part number
Shell Cassida RLS2	Lubrication	400ml cartridge	6907792
Berulub Sihaf 2	Lubrication	2kg	6907944
Loctite 243	Locking	10ml	6950607
Loctite 2701	Locking	5ml	6950609
Loctite 5776	Locking	50ml	6311460
Teflon Tape TBA	Thread sealant (Hydropump)	-	6950701
Essve 193	Sanitary Silicone	310ml, cartridge	6319106
Electrolube HTC	Conductivity paste	60g	6319270

Special Tools	
Logger	8559816
Tool Conex 303-22x3/4"	8665047

At Arjo, we are committed to improving the everyday lives of people affected by reduced mobility and age-related health challenges. With products and solutions that ensure ergonomic patient handling, personal hygiene, disinfection, diagnostics, and the effective prevention of pressure ulcers and venous thromboembolism, we help professionals across care environments to continually raise the standard of safe and dignified care. Everything we do, we do with people in mind.



ArjoHuntleigh AB
Hans Michelsensgatan 10
211 20 Malmö, Sweden
www.arjo.com

arjo