



ease

by eloflex



For other languages,
use the
QR code

Target groups, combinations and performance

For users that have issues with pain and warmth. Who rely on a wheelchair for movement and who spend most of their time seated. Aids users who have difficulties to vary their position and have a medium to high risk of developing pressure sores. The cushion is advantageous for use in an electric wheelchair since it is somewhat resilient and easy on the back and coccyx during transport over uneven foundations.

CE The cushion is a medical product. It bears the CE-marking in accordance with MDR (EU) 2017/745.

Eudamed-registration number / SRN: SE-MF-000001678.
ISO-kod: 043303.

The cushion is labelled with a EAN-13 barcode in accordance with 73 400473 AAAA K.

Materials and any production errors are covered by a two-year warranty. The cushion meets the fire standards ISO 7176-16.

Developed and produced in Sweden by BIMA Plastteknik AB for Eloflex. For information, please contact customer service by calling: 010-183 31 00 or via e-mail: info@eloflex.se.



Handling



Make sure that the packaging is intact and free from any damage.

Remove the plastic bag and note the white label on the bottom of the cushion. This side should face down and the arrow indicates the back of the cushion.

The black tag with the yellow text "BACK" also indicates the back of the cushion.

The handle should face forward. The Velcro loop textile should face down.



Do not move the cushion by the handle when it is being used.



Cleaning and care

The Ease cushion does not require any setup or service. Cleaning and proper hygiene is all that's needed. The foam base is cleaned by wiping down with a surface disinfectant or with soap and water. It can also be steam cleaned. Dry in a warm and well-ventilated spot or a drying cabinet. The cover can be machine washed up to 60°C with the zip closed. Afterwards it can be put through a spin cycle and tumble dried or left to drip-dry. It is important to put the cover back on the base according to the markings and labels. Make sure to poke the corners of the cover out so that it fits the foam base nicely. Store in a dry and preferably in room temperature. Do not squeeze or fold.

Quality assurance

In order to ensure that the cushion is working properly and safely during its intended life span (5 years) Check the surface of the foam base for cracks or divots. The cover should be intact. Divots or sunken parts indicate that the cushion has passed its use by date and needs replacing. The life span of the cushion can vary due to use and care.

Additional covers

To order an additional cover; provide COV+ article number, for example: COV7350006084225.

To order a protective hygiene/incontinence cover; provide INKO+ article number, for example INKO7350006084225

Recycling

Foam base and cover is disposed of as combustible waste and can be used for energy recovery.

Cautions

1. The cushion is intended for use in wheelchairs.
2. The size of the cushion should match the size of the wheelchair.
3. Whilst not in use the cushion should be stored in a dry space and preferably in room temperature.
4. Avoid squeezing or folding the cushion in storage.
5. Do not use the cushion in a car seat due to the risk of sliding forward during hard stops.
6. The cushion meets the appropriate fire standards but can take damage from fire, cigarette embers or other sources of ignition.
7. Avoid exposing the cushion to rain or dew. Sharp objects or strong chemicals can damage the cushion.
8. Minor adjustments such as rounding of the edges or adjusting the cushion for a difference in leg length can normally be done without negatively impacting the properties of the cushion. Changes should always be documented.
9. Through our non-compliance report flaws can be detected so that appropriate measures can be taken. Any incidents or accidents that may occur during use should be reported to Eloflex AB. Serious accidents or incidents should also be reported to the appropriate authority through a MIR – form: www.lakemedelsverket.se

Article number, dimensions and maximum user weight

Ease one article number	Seat width (cm)	Seat depth to back canes (cm)	Height (cm)	Max weight (kg)
7350006084225 Standard cushion	42,5	40	10	135
7350006084256 With extended +5cm width of the seating area	47,5	40	10	135
7350006084263 With extended +10cm width of the seating area	52,5	40	10	150