

## Serviceanvisning Basic badebænk med håndtag

### Maintenance

- Clean the shower chair regularly using only water, sponge and mild detergents for plastic materials such as soap. Dry thoroughly.<sup>3</sup>
- **Do not use** for cleaning: strong detergents, abrasive agents, bleach, petrol or solvents (it may cause frame damage, visible defects and in consequence - loss of warranty rights).
- At least once every three months check:
  - tightening of wing screws
  - condition of locking pins
  - condition of tubes (if cracks)

It is recommended to check the rubber tips regularly. In case of worn-out, replace the tips with the new ones.

### 7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.