

## Serviceanvisning Klassisk badestol firkantet

### **MAINTENANCE INFO:**

The products maintenance consists of the cleaning and sanitation procedures. In order to guarantee a safe use and adequate standard of hygiene, the user should perform these procedures before every use.

The user has to make certain of the structural integrity of the device and its components. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
2. Remove the evident dirt before to proceed with the most accurate disinfection.
3. Use water or non aggressive detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth.

**Alternatively, the product can be cleaned in domestic dishwashers at the maximum temperature.**

### **REUSE REQUIREMENTS:**

The product is suitable for reuse by other users. Before reuse by others, the product must be correctly cleaned and sanitized. Before

the transfer of the product to others, the product must be checked by qualified dealers.

### **CORRECT DISPOSAL PROCEDURE:**

Do not dispose of the product in the normal household waste. Ask your local authority beforehand for the correct and environmentally friendly way to dispose it.