

Periodic Inspection

Liko™ Textile Products



7EN160834 Rev. 4

Valid for Liko: UniversalSling Mod. 000/002, UltraSling Mod. 06, OriginalSling Mod. 10/11, Original HighBack Sling Mod. 20/21, 200/210, SilhouetteSling Mod. 22, Soft Original HighBack Sling Mod. 26, ComfortSling Plus Mod. 300/310/350/360, HygieneSling Mod. 40/41/45/46, HygieneVest Mod. 50/55, MasterVest Mod. 60/64, Vest for StandingShell Mod. 67, AmputeeSling Mod. 70/75, LiftPants Mod. 92, Ultra LiftPants Mod. 920, Liko Straps Mod. 90, SupportVest Mod. 91/99, SafetyVest Mod. 93/94, ComfortVest Mod. 95, LiftSheet Octo, LiftStrap Octo, CrossStrap Octo, LiftSheet 1900, LiftSheet 800/1800, LiftSheet XL, LiftStrap XL, RepoSheet™ Original, MultiStrap, HeadSupport, Leg Harness, Extension Loop, FixStrap, Straps Mod 600 IC

Product information		Customer reference	
Product:		Agreement No:	
Product Number:		Name:	
Serial Number: See page 2		Address:	
Production Year: See page 2		Post Code:	

⚠ Liko textile products must be thoroughly inspected at least once every six months. More frequent inspections might be required if the products are used or washed more often than normal. The inspection should be performed by a person who is trained or has experience and knowledge of design, use, and maintenance of the textile product.

Periodic Inspections are also available for: Liko Accessories (3EN601001), Overhead Lifts (3EN191001), Overhead Rail System (3EN111001), Freestanding Systems (3EN301001) and Mobile Lifts (3EN371001).

If printed: Preferable to make a colour print of this instruction.

Inspection check points

Liko textile products

Inspection check points		Approved*	Not approved
1	Product label		
2	Reinforcement seams (bartacks)		
3	Loops/Straps		
4	Edging		
5	Seams		
6	Fabric		
7	Handles/Guide loops		
8	Buckles		
9	Laths (orderable parts)		
10	Documentation (orderable parts)		

* If check point not available on product, write NA (not applicable).

Inspection sign off

Complete inspection according to the instructions page 2-5, fill in this page and sign below.

Approval to use the product: Approved Not approved

If the textile product has one or more inspection points "Not approved", the product must be taken out of use.

If anything is unclear or if you have questions, please contact Hill-Rom.

Inspection performed by:		Date:	
Next inspection:			

Inspection performed in accordance with ISO 10535:2006 Annex B- Periodic inspection


Enhancing outcomes for patients and their caregivers:



Instructions for the check points

Liko™ textile products

 In this document, this warning symbol indicates that special care should be taken. If instructions are not followed there is a risk of serious injury.

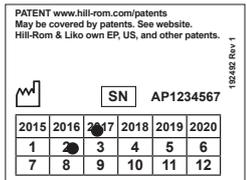Year of Production/Serial Number on textile products																						
The textile products has year of production and serial number placed in different ways on the product, see alternatives below.																						
<p>Alt 1. The product has serial number and year of production on the back of the product label. The serial number in the shape of a letter followed by a number series and year/month of production as y/m: YYYYMM.</p>	<p>Example:</p> 																					
<p>Alt 2. The product has serial number and year of production on two <i>separate</i> labels under the product label. The serial number in shape of a letter followed by a number series and year/month of production as y/m: YYYYMM.</p>	<p>Example:</p> <div style="display: flex; justify-content: space-around; align-items: center;"> <div style="border: 1px solid black; padding: 5px; margin: 5px;">L123456</div> <div style="border: 1px solid black; padding: 5px; margin: 5px;">2017-02</div> </div>																					
<p>Alt 3. The product has serial number and year of production on two <i>separate</i> labels under the product label. The serial number in shape of a letter followed by a number series and the year/month label is in shape of a table with numbers where year and month are marked.</p>	<p>Example:</p> <div style="display: flex; justify-content: space-around; align-items: center;"> <div style="border: 1px solid black; padding: 5px; margin: 5px;">L123456</div> <table border="1" style="border-collapse: collapse; text-align: center; margin: 5px;"> <tr> <td></td> <td>2017</td> <td>2018</td> <td>2019</td> <td>2020</td> <td>2021</td> <td>2022</td> </tr> <tr> <td>1</td> <td>2</td> <td>3</td> <td>4</td> <td>5</td> <td>6</td> <td></td> </tr> <tr> <td>7</td> <td>8</td> <td>9</td> <td>10</td> <td>11</td> <td>12</td> <td></td> </tr> </table> </div>		2017	2018	2019	2020	2021	2022	1	2	3	4	5	6		7	8	9	10	11	12	
	2017	2018	2019	2020	2021	2022																
1	2	3	4	5	6																	
7	8	9	10	11	12																	
<p>Alt 4. The product has serial number and year of production on one <i>separate</i> label under the product label. The serial number is in shape of two letters followed by a number series. Year/month of production is in shape of a table with numbers where year and month are marked.</p>	<p>Example:</p> 																					
<p>Alt 5. The product has serial number, year of production, and GS1 Data Matrix Barcode on the product label. The serial number in shape of a number series, the year/month/day of production in shape of YYYY-MM-DD, and the GS1 in shape of QR barcode and numbers.</p> <p>All products do not have serial number.</p>	<p>Example:</p> 																					

Instructions for the check points

Liko™ textile products

1 Product label


⚠ Check that the following can be read on the product label:

- product name
- product number
- max load
- serial number (if applicable)
- year of production
- washing instructions

See page 2 for instructions where on the product the serial number and year of production is to be found.

Note: Extension Loop, Leg Harness, and FixStrap only have product name and product number.

Example:






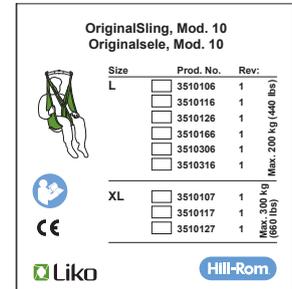


2 Reinforcement seams (bartacks)

⚠ Check to ensure that the threads and seams are intact. Missing stitches or loose thread are not approved.



Example:

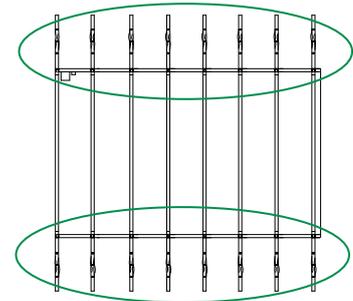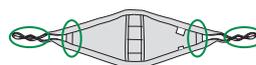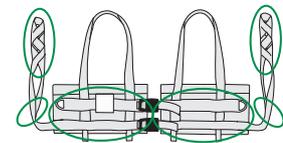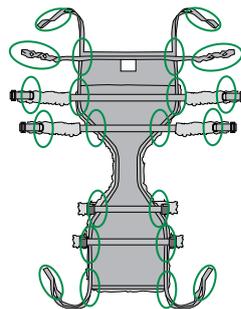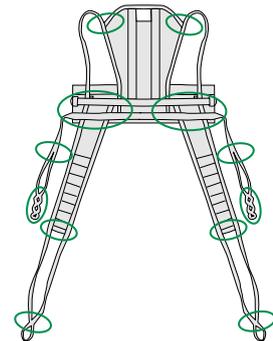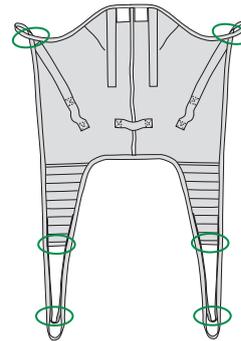


Instructions for the check points

Liko™ textile products

3 Loops/Straps

Check to ensure that the loops and straps have not been damaged - this often appears as visible broken threads on the webbing.


Example:

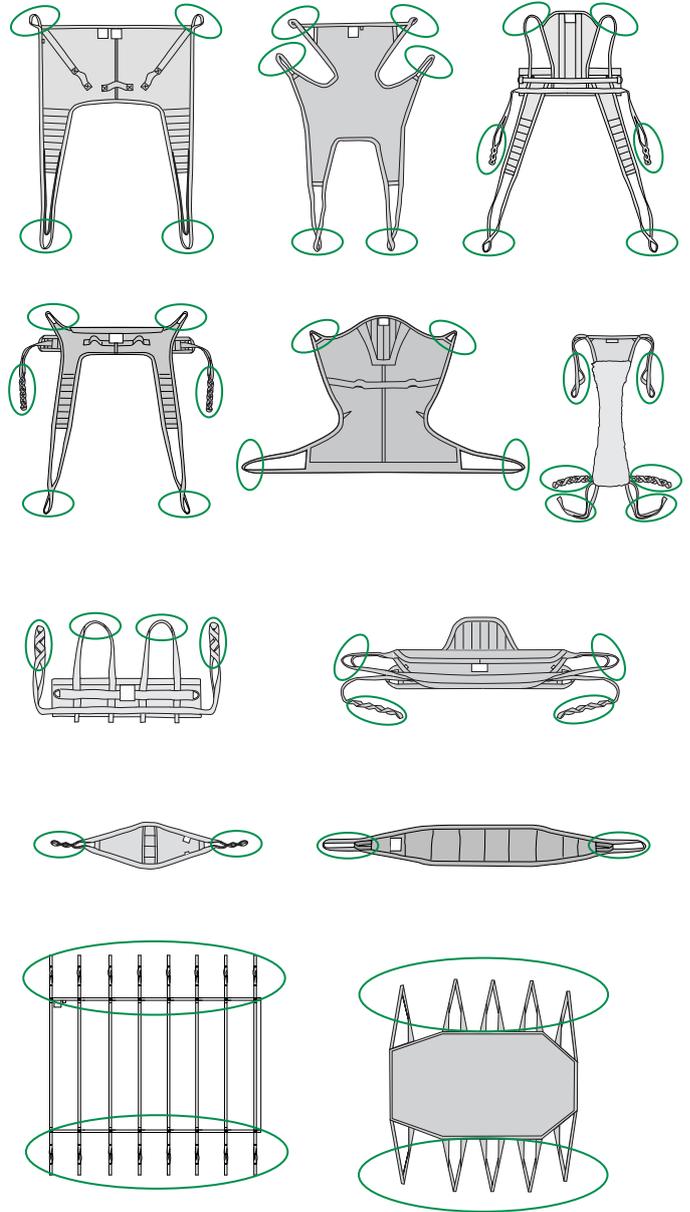

4 Edging

Check that the edging is not damaged.

Example:

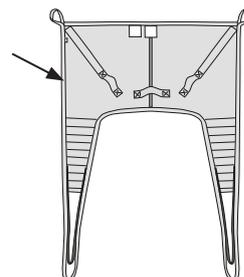

5 Seams

Check to ensure that the threads and seams are intact. Missing stitches or loose thread are not approved.

Instructions for the check points

Liko™ textile products

6 Fabric


Check to ensure that:

- the fabric does not have holes and is not thin or appear worn
- the fabric near the edging not appear worn or has holes
- the fabric under the handles is intact

7 Handles/Guide loops

Check to ensure that the seams on the handles/guide loops are intact.


Example:

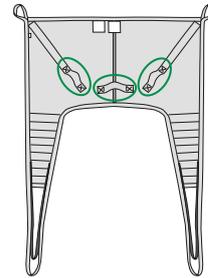

8 Buckles

Check the function of the buckle by locking the buckle, pulling on the belt and checking that the buckle does not open. Check for sharp edges.


Example:

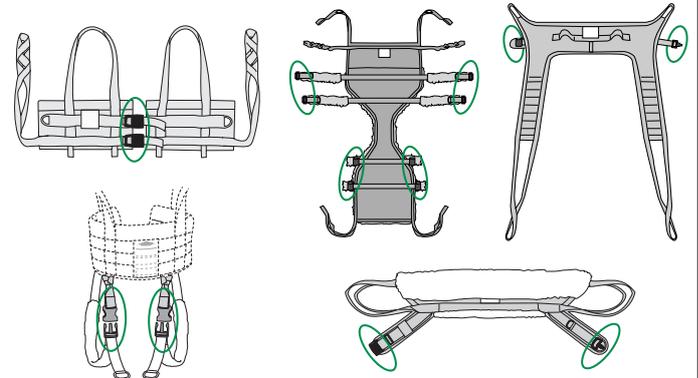

9 Laths

If needed for the function, check that the product has laths.

If laths are missing, new ones can be ordered from your local Hill-Rom representative.

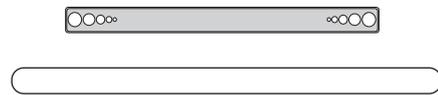

10 Documentation

Check that a valid Instruction Guide and Care and Maintenance instruction are available.

If Instruction Guide and/or Care and Maintenance instruction are missing, new ones can be ordered from your local Hill-Rom representative.

Example:





www.hillrom.com



Liko AB
Nedre vägen 100
975 92 Luleå, Sweden
+46 (0)920 474700

Liko AB is a subsidiary of Hill-Rom Holdings Inc.

Enhancing outcomes for
patients and their caregivers:

